

46781-10

11/21/2001

1/4

[Center Panel]

**PREMICIDE**  
Sanitizer/Solidifier

The efficacy of this product has been demonstrated in the presence of 100% blood serum.

**ACTIVE INGREDIENTS:**

Glutaraldehyde ..... 9.61%

**OTHER INGREDIENTS:** ..... 90.39%

**Total** ..... 100.00%

**KEEP OUT OF REACH OF CHILDREN**  
**DANGER**

FIRST AID	
<b>If in eyes</b>	<ul style="list-style-type: none"> <li>• Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.</li> <li>• Call a poison control center or doctor for treatment advice.</li> </ul>
<b>If on skin or clothing</b>	<ul style="list-style-type: none"> <li>• Take off contaminated clothing.</li> <li>• Rinse skin immediately with plenty of water for 15-20 minutes.</li> <li>• Call a poison control center or doctor for further treatment advice.</li> </ul>
<b>If swallowed</b>	<ul style="list-style-type: none"> <li>• Call a poison control center or doctor immediately for treatment advice.</li> <li>• Have person sip a glass of water if able to swallow.</li> <li>• Do not induce vomiting unless told to do so by a poison control center or doctor.</li> <li>• Do not give anything by mouth to an unconscious person.</li> </ul>
<b>If inhaled</b>	<ul style="list-style-type: none"> <li>• Move person to fresh air.</li> <li>• If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably mouth-to-mouth, if possible.</li> <li>• Call a poison control center or doctor for further treatment advice.</li> </ul>
For emergency information on product, call Metrex at 1-800-841-1428, Monday through Friday, between 6 a.m. and 4 p.m. Pacific Time. After 4 p.m., call Chemtrec 24-Hour Emergency Service at 1-800-424-9300.	
Have the product container or label with you when calling a poison control center or doctor, or going for treatment.	
<b>NOTE TO PHYSICIAN:</b> Probable mucosal damage may contraindicate the use of gastric lavage.	

See side panel for additional precautionary statements.

Net Contents: XX  
Solidifies up to XXXcc

**ACCEPTED**

NOV 21 2001

Approved for the Federal Insecticide, Fungicide, and Rodenticide Act as amended, for the purpose of registration, 1990  
EPA Reg. No. 46781-10

[Side Panel]

**PRECAUTIONARY STATEMENTS**

Hazard to Humans and Domestic Animals

**DANGER**

Corrosive. Causes irreversible eye damage. Causes skin irritation. Harmful if inhaled. Harmful if swallowed. Prolonged or frequently repeated skin contact may cause allergic reactions in some individuals. Causes asthmatic signs and symptoms in hyper-reactive individuals.

- Do not get in eyes, on skin or on clothing.
- Avoid breathing vapor.
- Do not swallow.
- Wear goggles, protective clothing and rubber gloves.
- Wash thoroughly with soap and water after handling.
- Remove contaminated clothing and wash before reuse.

**STORAGE AND DISPOSAL**

Do not contaminate water, food or feed by storage or disposal.

**Pesticide Storage:** Store in a cool, dry, well-ventilated area.

**Pesticide Disposal:** Dispose of waste according to Federal, State and local regulations.

**Container Disposal:** Do not reuse container. Triple rinse. Then offer for recycling or reconditioning, or puncture and dispose of in a sanitary landfill, or incineration or, if allowed by state and local authorities, by burning. If burned, stay out of smoke.

**FOR SINGLE USE ONLY**

**ACCEPTED**  
NOV 21 2001  
Under the Federal Insecticide, Fungicide, and  
Rodenticide Act as amended, for the  
pesticide, registered under  
EPA Reg. No. 46781-10

[Side Panel A -The following verbiage will appear only on the 500cc, 1200cc, 1500cc, 2000cc and 3000cc bottles]

Fast-acting encapsulant which contains scented dry glutaraldehyde to sanitize and solidify liquid medical waste and laboratory, human and animal wastes. Antibacterial. Controls odor formation by destroying odor-causing bacteria.

**The efficacy of this product has been demonstrated in the presence of 100% blood serum.**

**DIRECTIONS FOR USE**

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

To help prevent the blood from clotting inside the suction canister, immediately dispense PremiCide following the end of surgery or after the suction canister has been completely filled.


Wear hand, eye and face protection when handling infectious waste and using PremiCide.

- Shake and squeeze bottle to loosen any settled material.
- Remove plastic safety seal.
- Prior to or following surgery, open the appropriate port and place the PremiCide cap onto the open port and push down to secure proper fitting.
- Following surgery, turn off suction and remove the patient tubing. Close all ports and activate suction. Firmly grasp the base of the cap and pull upward on the bottle. Suction will pull the PremiCide powder into the canister.
- Once the PremiCide bottle is empty, tap the side of the bottle to remove any remaining powder. Push down on the bottle to close the system. Once the liquid waste is solidified, remove the bottle by gripping the base of the PremiCide cap and lift up. Replace the port cap on the suction canister. To assure thorough treatment of the suction canister, invert solidified canister and dispose of according to institutional procedures and Federal, State and local regulations.

For product or technical information, contact Metrex Customer Care at **800-841-1428** or visit our website at **www.metrex.com**.

EPA Reg. No. 46781-10      EPA Est. No. 72075-WI-001    Reorder No: 00-0000

Packed For:  
Metrex Research Corporation  
28210 Wick Road  
Romulus, Michigan 48174

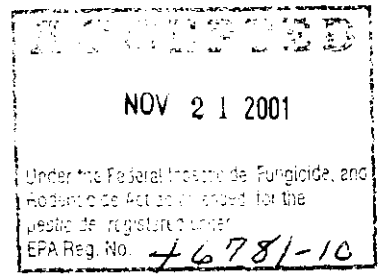

[Side Panel B -The following verbiage will appear only on the 10000cc multi-dose bottle]

Fast-acting encapsulant which contains scented dry glutaraldehyde to sanitize and solidify liquid medical waste and laboratory, human and animal wastes. Antibacterial. Controls odor formation by destroying odor-causing bacteria.

**The efficacy of this product has been demonstrated in the presence of 100% blood serum.**

**DIRECTIONS FOR USE**

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

To help prevent the blood from clotting inside the suction canister, immediately dispense PremiCide following the end of surgery or after the suction canister has been completely filled.

- Wear hand, eye and face protection when handling infectious waste and using PremiCide.
- Shake and squeeze bottle to loosen any settled material.
- Unscrew cap, remove safety seal and recap.
- Open a port in the top of the suction canister or provide access to the liquid waste if other device is used. Remove red cap.
- Insert PremiCide nozzle into opening. Gently squeeze the bottle, injecting PremiCide into the canister. Do not allow the tip of the bottle to touch the liquid waste. Recap port and replace red cap on bottle. Each graduation solidifies 1,000cc's.
- Allow sufficient time for encapsulation before handling.

For product or technical information, contact Metrex Customer Care at **800-841-1428** or visit our website at **www.metrex.com**.

EPA Reg. No. 46781-10      EPA Est. No. 72075-WI-001      Reorder No: 00-0000

Net Contents: XX

Packed For:  
Metrex Research Corporation  
28210 Wick Road  
Romulus, Michigan 48174

