

46781-10

1/28/2003

Page 182

Metrex Research Corporation
1717 West Collins Avenue
Orange, CA 92867

JAN 28 2003

Attention: Ms. Wendy Urtel


Subject: **Premicide**
EPA Registration No. 46781-10
Amendment Application Date: January 7, 2003
EPA Received Date: January 7, 2003

This will acknowledge receipt of your notification, submitted under the provisions of FIFRA section 3 (c) 9. Based on a review of the material, the following comment apply.

- Alternate Brand Name (Premicide CA)

This application is acceptable and has been made a part of the records for this file.

Sincerely,

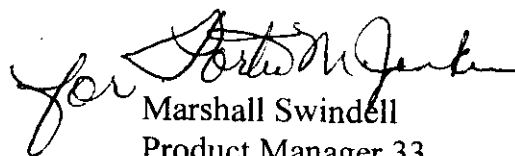

Marshall Swindell
Product Manager 33
Regulatory Management Branch I
Antimicrobials Division (7510C)

CONCURRENCES

SYMBOL							
SURNAME							
DATE							

PRECAUTIONARY STATEMENTS
Hazard to Humans and Domestic Animals

- DANGER**
- Corrosive. Causes irreversible eye damage. Causes skin irritation. Harmful if inhaled. Harmful if swallowed. Prolonged or frequent repeated skin contact may cause allergic reactions in some individuals. Causes antihistemic signs and symptoms in hyper-reactive individuals.
 - Do not get in eyes, on skin or on clothing.
 - Avoid breathing vapor.
 - Do not swallow.
 - Wear goggles, protective clothing and rubber gloves.
 - Wash thoroughly with soap and water after handling.
 - Remove contaminated clothing and wash before reuse.

SOLIDIFIES UP TO 500cc's
Sanitizer/Solidifier

The efficacy of this product has been demonstrated in the presence of 20% blood serum.

ACTIVE INGREDIENT:

- Glutaraldehyde 9.61%
- OTHER INGREDIENTS: 90.39%
- Total 100.00%

KEEP OUT OF REACH OF CHILDREN
DANGER

Fast-acting encapsulant which contains scented dry glutaraldehyde to sanitize and solidify liquid medical waste and laboratory, human and animal wastes, anatomical. Controls odor formation by destroying odor-causing bacteria.

The efficacy of this product has been demonstrated in the presence of 20% blood serum.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

To help prevent the blood from clogging inside the suction canister, immediately dispense Premicide following the end of surgery or after the suction canister has been completely filled.

Wear hand, eye and face protection when handling infectious waste and using Premicide.

- Shake and squeeze bottle to loosen any settled material.
- Remove plastic safety seal.
- Prior to or following surgery, open the appropriate port and place the Premicide cap into the open port and push down to secure proper fitting.

Following surgery, turn off suction and remove the patient tubing. Close all ports and activate suction. Firmly grasp the base of the cap and pull upward on the bottle. Suction will pull the Premicide powder into the canister.

Once the Premicide bottle is empty, tap the side of the bottle to remove any remaining powder. Push down on the bottle to close the system.

Once the liquid waste is solidified, remove the bottle by grasping the base of the Premicide cap and lift up. Replace the port cap on the suction canister. To assure thorough treatment of the suction canister, invert solidified canister and dispose of according to institutional procedures and Federal, State and local regulations.

For product or technical information, contact Metrex Customer Care at 800-841-1428 or visit our website at www.metrex.com.

ETA Reg. No. 48781-10 • EPA Est. No. 72075-WI-001

Packed For:
METREX RESEARCH CORPORATION

28210 Wix Road
Rumicuts, MI 48174



Recorder No: PG500 • Solidifies up to 500cc's
Net Contents: 0.72 ounces



FIRST AID	
If in eyes	Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eyes. Call a poison control center or doctor for treatment advice.
If on skin or clothing	Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for further treatment advice.
If swallowed	Call a poison control center or doctor immediately for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to do so by a poison control center or doctor. Do not give anything by mouth to an unconscious person.
If inhaled	Move person to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably mouth-to-mouth, if possible. Call a poison control center or doctor for further treatment advice.
For emergency information on product, call Metrex at 1-800-841-1428, Monday through Friday, between 8 a.m. and 4 p.m. Pacific Time. After 4 p.m., call Chemtrec 24-Hour Emergency Service at 1-800-424-9300.	
Have the product container or label with you when calling a poison control center or doctor, or going for treatment.	
NOTE TO PHYSICIAN: Probable increased damage may contraindicate the use of gastric lavage.	

See side panel for additional precautionary statements.

FOR SALE AND DISTRIBUTION IN CALIFORNIA ONLY.

Draft CA PG60-500-3

282