



UNITED STATES ENVIRONMENTAL PROTECTION AGENCY
WASHINGTON, DC 20460

OFFICE OF CHEMICAL SAFETY
AND POLLUTION PREVENTION

March 27, 2017

David Swain
Agent
Metrex Research
28210 Wick Road
Romulus, MI 48174

Subject: PRIA Label Amendment – Adding Public Health Organisms
Product Name: CaviWipes 1
EPA Registration Number: 46781-13
Application Date: 11/04/2016
Decision Number: 523339

Dear Mr. Swain:

The amended label referred to above, submitted in connection with registration under the Federal Insecticide, Fungicide and Rodenticide Act, as amended, is acceptable. This approval does not affect any conditions that were previously imposed on this registration. You continue to be subject to existing conditions on your registration and any deadlines connected with them.

Because you have opted to add statements pertaining to emerging viral pathogens to your label as described in the [August 19, 2016, Guidance To Registrants: Process For Making Claims Against Emerging Viral Pathogens Not On EPA-Registered Disinfectant Labels \(“Guidance”\)](https://www.epa.gov/sites/production/files/2016-09/documents/emerging_viral_pathogen_program_guidance_final_8_19_16_001_0.pdf), https://www.epa.gov/sites/production/files/2016-09/documents/emerging_viral_pathogen_program_guidance_final_8_19_16_001_0.pdf, you are subject to the following additional terms of registration:

1. You may make statements pertaining to emerging viral pathogens only through the following communications outlets: technical literature distributed exclusively to health care facilities, physicians, nurses and public health officials, "1-800" consumer information services, social media sites and company websites (non-label related). These statements shall not appear on marketed (final print) product labels.
2. Your statements pertaining to emerging viral pathogens must adhere to the format approved on the Agency-accepted master label.
3. You may make statements pertaining to emerging viral pathogens only upon a disease outbreak that meets all the following criteria:
 - a. The causative organism must be a virus that causes an infectious disease that has appeared in a human or animal population in the U.S. for the first time, or that may have existed previously but is rapidly increasing in incidence or geographic range.
 - i. For human disease, the outbreak is listed in one of the following Centers for Disease Control (CDC) publications:

- A. CDC Current Outbreak List for “U.S. Based Outbreaks” (www.cdc.gov/outbreaks),
 - B. CDC Current Outbreak List for “Outbreaks Affecting International Travelers” with an “Alert” or “Advisory” classification (www.cdc.gov/outbreaks) (also released through the CDC’s Health Alert Network (HAN) notification process)
 - C. Healthcare-Associated Infections (HAIs) Outbreaks and Patient Notifications page (www.cdc.gov/hai/outbreaks)
- ii. For animal disease, the outbreak is identified as an infectious disease outbreak in animals within the U.S. on the World Organization for Animal Health (OIE) Weekly Disease Information page (www.oie.int/wahis_2/public/wahid.php/Diseaseinformation/WI).
- b. The CDC or OIE has identified the taxonomy, including the viral family and/or species, of the pathogen and provides notice to the public of the identity of the emerging virus that is responsible for an infectious disease outbreak. Based on the taxonomy of the outbreak pathogen identified by the CDC or OEI, the pathogen's viral subgroup is [small non-enveloped, large non-enveloped, enveloped].
 - c. The virus can be transmitted via environmental surfaces (non-vector transmission), and environmental surface disinfection has been recommended by the CDC, OIE or EPA to control the spread of the pathogen.
4. You may begin communicating statements pertaining to emerging viral pathogens only upon CDC or OIE’s publication per term B.3.a. of an outbreak of an emerging viral pathogen meeting all of the criteria of term B.3. You must cease and remove all such non-label communications intended for consumers no later than 24 months after the original publication of the outbreak per term B.3.a., unless the Agency issue written guidance to the contrary due to continued public health concerns. The emerging pathogen claim language may remain on the master label.
5. Terms B.1 through B.4 above shall become immediately void and ineffective if registration for use against [identify label viruses] is suspended or cancelled or no longer meets the criteria for a disinfectant claim (see EPA Product Performance Test Guideline 810.2200). In addition, terms B.1 through B.4 above shall become immediately void and ineffective upon your receipt of evidence of ineffectiveness against any pathogen in a less-resistant Spaulding category.

A stamped copy of your labeling is enclosed for your records. This labeling supersedes all previously accepted labeling. You must submit one copy of the final printed labeling before you release the product for shipment with the new labeling. In accordance with 40 CFR 152.130(c), you may distribute or sell this product under the previously approved labeling for 18 months from the date of this letter. After 18 months, you may only distribute or sell this product if it bears this new revised labeling or subsequently approved labeling. “To distribute or sell” is defined under FIFRA section 2(gg) and its implementing regulation at 40 CFR 152.3.

Should you wish to add/retain a reference to the company's website on your label, then please be aware that the website becomes labeling under the Federal Insecticide Fungicide and Rodenticide Act and is subject to review by the Agency. If the website is false or misleading, the product would be misbranded and unlawful to sell or distribute under FIFRA section 12(a)(1)(E). 40 CFR 156.10(a)(5) list examples of statements EPA may consider false or misleading. In addition, regardless of whether a website is referenced on your product's label, claims made on the website may not substantially differ from those claims approved through the registration process. Therefore, should the Agency find or if it is brought to our attention that a website contains false or misleading statements or claims substantially differing from the EPA approved registration, the website will be referred to the EPA's Office of Enforcement and Compliance.

Your release for shipment of the product constitutes acceptance of these conditions. If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA section 6. If you have any questions, you may contact Aline Heffernan at 703-347-8602 or via email at heffernan.aline@epa.gov.

Sincerely,

A handwritten signature in black ink, appearing to read "John Hebert" with a stylized flourish at the end.

John Hebert, Chief
Regulatory Management Branch I
Antimicrobials Division (7510P)
Office of Pesticide Programs

Enclosure: Stamped Label

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

CaviWipes 1™

ACCEPTED

Mar 27, 2017

Under the Federal Insecticide, Fungicide and Rodenticide Act as amended, for the pesticide registered under EPA Reg. No. 46781-13

**[TUBERCULOCIDAL] [BACTERICIDAL] [VIRUCIDAL*[^]]
[FUNGICIDAL^^]**

Active Ingredient:

Didecyldimethylammonium chloride0.76%
Ethanol7.50%
Isopropanol15.00 %
[Inert] Other Ingredients†.....76.74%
Total.....100.00%

[†Does not include the weight of the towelette]

KEEP OUT OF REACH OF CHILDREN

WARNING

[See other panel for additional precautionary statements.]

EPA Reg. No. 46781-13
EPA Est. No. [56952-WI-001] [075272-WI-001]
[71979-SC-001] [51435-WI-004]
[56952-CHN-001] [73027-WI-001]

Made in USA

[CaviWipes 1 is available in the following packaging configurations]:

- o 12 canisters/CS
- o 20 packs/CS
- o 50/Box, 6 Boxes/CS, Single Use Towelettes]
- o Starter Kit – 1 canister & 12 refill packs
- o 12 Refill packs/CS
- o 12 Empty Canisters/CS

[1 minute]
[1 Step Germicidal Wipes]
[Fragrance Free]

Reorder No.
Net Contents: #[[lbs.] [oz]
[X lbs. X oz. / x kg.]
Pre-Saturated Towelette # x # in
[X Pre-saturated Towelette[s] (X" x X")]

[Packed For:]
[Distributed By:]
[Sold By:]
[Manufacturer By [or for]:]

Metrex Research
28210 Wick Road
Romulus, MI 48174, USA

[To reorder] [,] [For product or technical information] contact Metrex at 800-841-1428 or visit our web site at

www.metrex.com

[For SDS information please visit www.metrex.com]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

WARNING Causes substantial but temporary eye injury. Do not get in eyes or on clothing. Wear appropriate protective eyewear such as goggles, face shield, or safety glasses. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. Remove and wash contaminated clothing before reuse. [See other panel for additional precautionary statements.]

Physical or Chemical Hazards: Combustible. Do not use or store near heat or open flame.

FIRST AID

If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, and then continue rinsing. Call a poison control center or doctor for treatment advice.

For Chemical Emergency ONLY (spill, leak, fire, exposure or accident), call CHEMTREC at 1-800-424-9300.

For [ALL] [other]]inquiries about this product, call Metrex Customer [Service] [Care] at 1-800-841-1428, Monday through Friday, between 6 a.m. and 4 p.m. Pacific Time.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

Packaging Statements and Wipe Dispensing Instructions:
[Use A, B, or C as applicable depending on the wipe container/dispenser type.]

[A – Refillable, Non-refillable canister, buckets, tubs or pails]

[To dispense towelettes] [container] [canister][pail] [bucket] [tubs] [Dispensing Directions]:
[To start feed, remove cover and discard seal. From center of towelette roll, pull up a towelette corner, twist it into a point and thread it through the hole in the container cover. Pull through about one inch. Replace cover. Pull out first towelette and tear off at [(any angle)] [45°-90° angle]. Remaining towelettes feed automatically, ready for next use. When not in use, keep center cap closed to prevent solution loss.]

[B – Flat Pack]

[To dispense towelettes] [from flat pack:] [from Flat Pack wipes container:]
[Grasp the white tab that states "PULL HERE". Gently swing the flap open and grasp a towelette. Remove towelette from package and then swing flap closed. Gently tap the flap to ensure seal is made between it and the towelette package.]

[C – Single use packet]:

[Tear open packet. Unfold towelette and use as directed. Discard as directed.]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

This product meets the criteria for use against the following categories of emerging viral pathogens when used in accordance with the use directions for Adenovirus Type 5:
- Enveloped Viruses

*Per Draft Guidance: **Process for Making Claims Against Emerging Viral Pathogens Not on EPA-Registered Disinfectant Labels.** (The statement shall be made only through the following communication outlets: technical literature distributed exclusively to health care facilities, physicians, nurses and public health officials. "1-800" consumer information services, social media sites and company websites (non-label related). Information is NOT to appear on marketed product labels. One or both statements below may be used.)*

[Product name] has demonstrated effectiveness against viruses similar to **[name of emerging virus, which must be an enveloped virus]** on **[eligible surface type(s)]**. Therefore, **[product name]** can be used against **[name of emerging enveloped virus]** when used in accordance with the directions for use against **[name of supporting virus(es)]** on **[hard, non-porous surfaces]**. Refer to the **[CDC or OIE]** website at **[pathogen-specific website address]** for additional information.

[Name of illness/outbreak] is caused by **[name of emerging virus, which must be a enveloped virus]**. **[Product name]** kills similar viruses and therefore can be used against **[name of emerging virus, enveloped virus]** when used in accordance with the directions for use against **[name of supporting virus(es)]** on **[hard, non-porous surfaces]**. Refer to the **[CDC or OIE]** website at **[website address]** for additional information.

[Note: Adenovirus Type 5 a large non-enveloped virus requires a Contact time of 3 minutes]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal.

[Use A, B, C or D as applicable depending on the wipe container type.]

[A – Nonrefillable canister]

Product Storage: Store at room temperature. When not in use, keep center cap of lid closed to prevent moisture loss

Towelette Disposal: Do not re-use towelette. Dispose of used towelette in trash. Do not flush in toilet.

Container Handling: Nonrefillable container. Do not reuse or refill this container. Offer for recycling, if available. If recycling is not available, discard in trash.

[B – Refillable canister]

Storage: When not in use, keep center cap of lid closed to prevent moisture loss

Towelette Disposal: Do not re-use towelette. Dispose of used towelette in trash. Do not flush in toilet.

Container Handling: Refillable Container. Do not reuse or refill except as described in direction for use. Refill only with this product. Triple rinse cracked or broken containers and offer for recycling. Triple rinse as follows: Fill container 1/4 full with water and recap. Shake for 10 seconds. Drain for 10 seconds after the flow begins to drip. Repeat procedure two more times. Rinsate can be disposed down the drain. If recycling is not available, discard in trash

[C - packet]

STORAGE AND DISPOSAL

Storage: When not in use, keep flap closed to prevent moisture loss

Dispenser Disposal - Nonrefillable container. Do not reuse or refill this container. Offer for recycling, if available. If recycling is not available, discard in trash

Towelette Disposal - Do not reuse towelette. Dispose of used towelette and empty packet in trash.

[D – Flat Pack]

STORAGE AND DISPOSAL

Storage: When not in use, keep flap closed to prevent moisture loss.

Dispenser Disposal - Nonrefillable packaging. Do not reuse or refill this container. Offer for recycling, if available. If recycling is not available, discard in trash.

Towelette Disposal - Do not reuse towelette. Dispose of used towelette in trash.

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

FOR CANISTERS, FLAT PACKS, REFILLS AND SINGLE USE WIPE PACKETS

For use as a disinfectant:

Use CaviWipes 1 towelette to thoroughly wet the surface. Repeated use of the product may be required to ensure that the surface remains visibly wet for [1 minute] at room temperature (68°F/20°C) for Tuberculocidal and bactericidal, and virucidal*^[A] activity. [3 minutes for Adenovirus 5] Let air dry. Gross filth and heavy soil loads must be removed prior to disinfecting. [If streaking is observed, wipe with a clean, damp [cloth or] paper towel after [1 minute] [3 minutes for Adenovirus] contact time has expired.]

For use as a Fungicide [against [*Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*)] Pre-clean surface with a [CaviWipes1] or [towelette]. Then use CaviWipes 1 to thoroughly wet the surface. Repeated use of the product may be required to ensure that the surface remains visibly wet for 3 minutes at room temperature for fungicidal activity. Let air dry. [If streaking is observed, wipe with a clean, damp [cloth or] paper towel after 3 minutes contact time has expired.]

FOR CLEANING INSTRUMENTS PRIOR TO DISINFECTION:

Remove gross filth and heavy soil loads.

[The following direction is to be included when hard, non-porous exterior surfaces of ultrasound transducers or probes are listed as a use site.]

[To disinfect external surfaces of ultrasound transducers: Pre-clean ultrasound gel from external transducer to be disinfected [by using a clean washcloth]. Then thoroughly wet the probe with a **CaviWipes 1** towelette. Repeated use of the product may be required to ensure that the surface remains visibly wet for [1 minute]. [3 minutes for [Adenovirus Type 5] and [*Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*)] at room temperature (68°F/20°C).]

[[This contact time][A 1 minute contact time] will not be sufficient for Adenovirus Type 5 and *Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*)(which requires a 3 minute contact time)]. at room temperature (68°F/20°C).]

Notes:

Critical and semi-critical devices must be followed by appropriate terminal sterilization/high-level disinfection process.

The following paragraph will be included in its entirety when the production label references medical devices:

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream, or normally sterile areas of the body, or (2) contacts intact mucous membranes, but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

The following HIV-1* / HBV* / HCV* instructions are to be included on production labels when claims against HIV-1* or HBV* or HCV* are listed.

CaviWipes 1 effectively kills [Human Immunodeficiency Virus Type 1* (HIV-1),] [Hepatitis B Virus* (HBV),] [[and] Hepatitis C Virus* (HCV)] on pre-cleaned environmental surfaces/objects previously soiled with blood/body fluids in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood/body fluids, and in which the surfaces/objects can be associated with the potential for transmission of [HIV-1*,] [HBV*,] [[and] [HCV*].

Special Instructions for Cleaning and Decontamination Against [HIV-1*,] [HBV*,] [[and] HCV*] of Surfaces or Objects Soiled With Blood or Body Fluids:

Personal Protection: When handling items soiled with blood or body fluids, wear appropriate barrier protection: disposable protective gloves, gowns, face masks and eye coverings.

Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfecting towelette. A CaviWipes 1 towelette can be used for this purpose.

Contact Time: Thoroughly wet surface with a new CaviWipes 1 towelette. Repeated use of the product may be required to ensure that the surface remains visibly wet for 1 minute.

Disposal of Infectious Materials: Dispose of used towelette in accordance with Federal, State and local regulations for infectious materials disposal.

[The following statement is to be included on production labels when food contact areas or surfaces are listed.]

[+A thorough rinse with potable water is required for surfaces in direct contact with food.]

CLEANING AND STAIN REMOVAL

DIRECTIONS FOR USE:

To clean -and/or- remove stains: Wipe surface with a wipe; [rinse and] let air dry. [If streaking is observed, wipe with a clean, damp [cloth] [or] [paper towel].] [[Works on] [effective against]][removes the following stains:]

[CLEANING] [and] [DEODORIZING]: [Wipe surface with towelette until visibly clean. Let air dry.]

Claims: <ul style="list-style-type: none">• Cleans• Removes stains• Removes (<i>insert stain(s)/soil(s) shown to the right:</i>)	Stains/Soils <ul style="list-style-type: none">• Blood• Body oils• Bodily fluids• Coffee [stains]• Common soils• Dead skin• Dirt• Fingerprints• Food+ residue[s] -or- soil[s]• Food stains• Fruit [stains]• Grease• Grime• Laboratory stains	Stains/Soils continued <ul style="list-style-type: none">• Mucus• [Other] common soils -and/or- stains• [Other] organic matter• Phlegm• Protein-based matter• Saliva• Spit• Spittle• Sputum• Throw-up• Tea [stains]• Urine• Vomit
---	--	--

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

ALLERGEN DESTRUCTION

To [clean and] destroy specified allergens: Wipe surface. Use enough wipes for treated surfaces to remain visibly wet for 1 minute.
[Rinse.] Let air dry. [If streaking is observed, wipe with a clean, damp [cloth or] paper towel after contact time has expired.]

Claims:

- Breaks Down Many Allergens[!]
- Destroys -or- deactivates dust mite matter
- [Penetrates and] breaks down dust mite matter -or- particles, cockroach matter -or- particles, and pet dander allergens [found in dust]

The following statement is to be included on production labels when animal housing facilities are listed. (Including Veterinary Clinics, Farms and Poultry Houses)]

For cleaning and disinfecting the following hard nonporous surfaces: feeding and watering equipment, utensils, instruments, cages, kennels, stables, catteries, etc.

1. Remove all animals and feeds from premises, animal transportation vehicles, crates, cages etc.
2. Remove all litter, droppings and manure from floors, walls and surfaces of facilities (including pens, chutes, and barns) occupied or traversed by animals.
3. Empty all troughs, racks, and other feeding and watering appliances.
4. Thoroughly clean all surfaces with soap or detergent and rinse with potable water. Saturate surfaces by wiping with product for a period of 1 minute. [3 minutes for [Adenovirus Type 5] and [*Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*).]]
5. Wipe all handling and restraining equipment such as leashes, muzzles, halters or ropes, as well as forks, shovels, and scrapers used for removing litter and manure.
6. Ventilate buildings and other closed spaces.
7. Do not house animals or employ equipment until treatment has been absorbed, set or dried.
8. Thoroughly scrub treated feed racks, troughs, mangers, automatic feeders, fountains, and waterers with soap or detergent, and rinse with potable water before reuse.

[DO NOT USE AS A BABY WIPE.] [Not For Use On Skin] [DO NOT FLUSH WIPE]

[Not for cleaning or sanitizing skin.] [Do not use as a diaper wipe or for personal cleansing.] [Refill only with this product.] [Always refer to manufacturer's care instructions before using on equipment -or- devices.]

[The following graphics are optional for canister containers only. They will be affixed to the label/canister so that it does not interfere with any label text. Due to size constraints, they will not appear on individually wrapped packets.]



THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

DISINFECTION

[Description] [CaviWipes 1 Disinfecting Towelettes are non-woven disposable towelettes pre-saturated with CaviCide 1]

[Description] [CaviWipes 1 Disinfecting Towelettes are multi-purpose [disinfectant] cleaning wipes for use on hard, non-porous inanimate surfaces.]

[CaviWipes 1 are effective against the following microorganisms on hard, non-porous surfaces when used as directed:]

[This product has demonstrated effectiveness against the following organisms with a [1 minute contact] [3 minutes contact] time:]

[Disinfects (*insert organisms*) on hard non-porous environmental surfaces listed on this label]

[When used as directed, CaviWipes 1 provides effective Tuberculocidal / Virucidal*^[^] / Bactericidal / Fungicidal^{^^} activity against the following organisms:]

[This product has demonstrated effectiveness against the following organisms with a [1 minute] [3 minutes] contact time:]

[EFFECTIVE AGAINST]

Pathogen:	Bacteria, Enveloped Viruses*, Bloodborne Pathogens **, TB, Pathogenic Fungi	[Fungicidal ^{^^} ,] [Non-enveloped Virus* [^]]
Contact Time:	1 min[ute]	3 min[utes]

ORGANISMS

	[Strain and/or ATCC number]
<i>Mycobacterium (1 Minute Contact)</i>	
[<i>Mycobacterium tuberculosis</i> var: <i>bovis</i> (BCG) (TB)]	
<i>Bacteria (1 Minute Contact)</i>	
[<i>Klebsiella pneumoniae</i>]	[ATCC 4352]
[<i>Pseudomonas aeruginosa</i>]	[ATCC 15442]
[<i>Salmonella enterica</i>]	[ATCC 10708]
[<i>Staphylococcus aureus</i>]	[ATCC 6538]
<i>Pathogenic Fungi (1 minute Contact)</i>	
[<i>Candida albicans</i>]	[ATCC 10231]
<i>Fungicide ^^ (3 minute Contact)</i>	
[<i>Trichophyton interdigitale</i> (formerly <i>Trichophyton mentagrophytes</i>)]	[ATCC 9533]
<i>Drug-Resistant Bacteria (1 Minute Contact)</i>	
[Carbapenem-Resistant <i>Klebsiella pneumoniae</i> [(CRKP)] [(CRE)]]	[ATCC BAA-2470]
[ESBL <i>Escherichia coli</i>] [ESBL <i>E. coli</i>]	[ATCC BAA-196]
[Methicillin Resistant <i>Staphylococcus aureus</i> (MRSA)]	[ATCC 33592]
[Methicillin Resistant <i>Staphylococcus epidermidis</i> (MRSE)]	[ATCC 51625]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

[Multi-drug resistant [MDR] <i>Acinetobacter baumannii</i>]	[ATCC BAA-1605]
[Vancomycin Intermediate <i>Staphylococcus aureus</i>] [VISA]	[ATCC 700699]
[Vancomycin Resistant <i>Enterococcus faecalis</i> (VRE)]	[ATCC 51299]
<i>Viruses* [Enveloped] , [Virucidal*[^]](1 Minute Contact)</i>	
[Hepatitis B Virus (HBV)*, **]	[Strain: Grimaud]
[Hepatitis C Virus (HCV)*, **]	[Strain: NADL]
[Herpes Simplex Virus Type 1*]	[ATCC VR-260]
[Herpes Simplex Virus Type 2*]	[ATCC VR-734]
[Human Immunodeficiency Virus (HIV-1)*, **]	
[Human Coronavirus* (not associated with Severe Acute Respiratory Syndrome or SARS)*]	[ATCC VR-740]
[Influenza A Virus, Strain A/ [Hong Kong 8/68] [H3N2 Virus],]*	[Charles River Labs]
<i>Viruses* [Non-Enveloped][Virucidal*[^]](3 Minute Contact)</i>	
[Adenovirus Type 5] [^]	[Strain: Adenoid 75, ATCC VR-5]

* Virus

**Bloodborne pathogens

[^]Virucidal – One minute contact time will not be sufficient for Adenovirus Type 5.

(^^Fungicidal claim will only be used when *Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*) is on the field/ product label. Effectiveness against *Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*)requires precleaning prior to disinfection)

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRODUCT CLAIMS

▲ to represent any use site(s) listed in below as applicable.

◆ to represent any surface(s) listed in below as applicable.

- [__# pre-saturated towelettes] [X" x Y"]
- [% bigger [textured] wipes] [vs. X" x Y" wipes]
- [1 minute] [[3 minutes for [Adenovirus Type 5] and [*Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*).]]
- [1 minute kill time] [product labeled claims] [[3 minutes for [Adenovirus Type 5] and [*Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*).]]
- [1 minute contact time] [[3 minutes for [Adenovirus Type 5] and [*Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*).]]
- [1 minute kill time] [[3 minutes for [Adenovirus Type 5] and [*Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*).]]
- [3 minute kill time]
- [1 minute contact time] [[3 minutes for [Adenovirus Type 5] and [*Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*).]]
- [3 minute contact time]
- [1 Step Germicidal Wipe][Surface Disinfectant Cleaner]
- [X" x Y" Wipes]
- [__# pre-saturated towelettes] [X" x Y"]
- [After use, just toss wipe(s) away]
- [A Metrex Research[®]™ brand]
- [A Metrex[®]™ brand]
- [A CaviWipes™ Brand]
- [Absorbs] [Absorbent] [wipe]
- [A disinfectant cloth for use in ▲ on ◆]
- [Cleaning Formula]
- [[Aids] [Helps] in reducing -or- in the reduction of cross-contamination between treated hard non-porous surfaces]
- [An effective method to clean against odor-causing organisms on hard, non-porous surfaces listed on this label].
- [[A] one-step cleaner/disinfectant]
- [A disinfectant towelette for use on ◆ and other ◆]
- [After use, just toss wipe(s) away]
- [Always ready-to-use]
- [An effective method to deodorize and disinfect against odor-causing organisms on hard non-porous surfaces]
- [An effective way to disinfect hard, non-porous environmental surfaces]
- [An effective way to disinfect hard, non-porous environmental surfaces listed on this label.]
- [A non-woven disposable towelettes for use in ▲ and other critical care areas where control between treated surfaces is required]
- [Antibacterial]
- [Antimicrobial]
- [Antibacterial][Antimicrobial] [Health club equipment wipes]
- [Antifungal]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRODUCT CLAIMS continued

- [A One Step Cleaner [and disinfectant] (does not include fungicidal^{^^} activity, precleaning is required for fungicidal^{^^} activity)
- [A disinfectant [cloth] [wipe] [towelette] for use on telephone surfaces and other telecommunications equipment surfaces]
- [A solution impregnated in a wiping cloth]
- [[Appropriate] [for] Cleaning Patient Discharge]
- [Appropriate] for [Occupied Room] [and] [Terminal Room] [Discharge] Cleaning]
- [Assures proper dilution strength, product effectiveness and standardizes technique]
- [Attacks and kills [insert organism] [on hard, nonporous surfaces]
- [Bacteria -and/or- virus* -and/or- germ fighting [formula]]
- [Bactericidal] [Virucidal*[^]] [Tuberculocidal] [in 1 minute]
- [Fungicidal^{^^}] [in 3 minutes]
- [Bactericide -or- Bactericidal][in 1 minute]]
- [Barber Disinfectant Cleaner]
- [Bathroom disinfectant wipes]
- [Bleach-free formula]
- [Does not contain bleach]
- [Broad spectrum disinfectant]
- [Built-in [tough] scrubbing fibers to help clean effectively]
- [Can help reduce cross contamination between treated hard, non-porous surfaces]
- [CaviWipes 1 have been tested for [Stainless Steel] [Porcelain][Glass][Plastic][Glazed Ceramic] surface compatibility]
- [CaviWipes 1 kills germs that cause odors]
- [CaviWipes 1 is effective against odor causing germs]
- [Can help reduce cross contamination between treated hard, non-porous surfaces]
- [Cleaner and disinfectant [towelette] [wipe]
- [Cleaner and Disinfectant in One]
- [Cleans]
- [Cleans] [♦][in ▲]
- [[Cleans and] disinfects ♦]
- [[Cleans] [and] [disinfects] [on the go]]
- [[Cleans and] disinfects [hard, nonporous surfaces]]
- [Cleans, deodorizes and disinfects]
- [Cleans, Deodorizes and Kills Common Household Germs]
- [Cleans] [disinfects] [Cleaner and disinfectant] [in one] [all in one [easy] step]
- [Clean Home]
- [Cleans the toughest soils]
- [Cleaning Wipe[s] [Towel][s] refill[s] [able]]
- [Cleaning Made Easy]
- [Clinical Clean]
- [Clipper Blade Disinfectant/Cleaner]
- [[Cleans [and /or Disinfects] [grocery cart handles or shopping cart handles or grocery cart child seats or shopping] cart child seats]
- [Cleans grocery carts and shopping carts]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRODUCT CLAIMS continued

- [Contains [number] canisters [plus][and][number refills]
- [[Cloth-like texture][Textured] [strong][for][with]scrubbing power]
- [Contains no abrasive harsh acids]
- [Convenient]
- [[Convenient] [Portable] way to [disinfect] [clean] hard non-porous surfaces [of ♦]]
- [Cuts Cleaning time]
- [Deep cleaning][formula]
- [Delivers a clean you can see and smell]
- [Dental Office Disinfectant/Cleaner]
- [Designed for Home Healthcare -and/or- patients -and/or caregivers]
- [Designed -or- developed for Home Caregivers]
- [Designed -or- developed for use in [▲] Home Healthcare -or- Home Care]
- [Delivers consistent [amount of solution] concentration every time [!]]
- [Deodorizes, and kills common household germs]
- [Deodorizes by killing the germs that cause odors]
- [Deodorize hard, non-porous surfaces] [♦][in ▲]]
- [Designed for the ▲ [[industry]]]
- [Designed] [for] [with] [the] Environmental Services Professional[s][in] [mind][!]
- [Destroys [Deactivates] [[Helps] Eliminate] (insert organisms) [on ▲]]
- [Disinfectant]
- [Disinfect]
- [Disinfectant/Cleaner]
- [Disinfect external surfaces of ♦]
- [Disinfect surfaces[♦]]
- [Disposable]
- [Disinfectant cleaner [that cleans, disinfects and deodorizes] in one step]
- [Disinfecting formula]
- [Disinfects] [♦][in ▲]]
- [[Disinfects][Disinfectant][Cleaner] on the go]
- [Disinfects [and deodorizes] ♦[in one step]]
- [Disinfecting towelettes]
- [Disinfects and deodorizes in 1 minute] [against product labeled claims] [3 minutes for [Adenovirus Type 5] and [Trichophyton interdigitale (formerly Trichophyton mentagrophytes).]]
- Disinfects hard non-porous environmental surfaces listed on this label {Use to}
- [Disinfects hard, non-porous surfaces in 1 minute][against product labeled claims] [3 minutes for [Adenovirus Type 5] and [Trichophyton interdigitale (formerly Trichophyton mentagrophytes).]]
- [[Disinfects] hard non-porous surfaces [that people touch]]
- [Disinfects ♦]
- [Disinfects [in [just] one [1] minute][against product labeled claims] [3 minutes for [Adenovirus Type 5] and [Trichophyton interdigitale (formerly Trichophyton mentagrophytes).]]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRODUCT CLAIMS continued

- [[Disinfects and deodorizes by killing germs and their odors]
- [[Disinfects hard, nonporous surfaces [throughout the ▲]
- [[Disinfects [as it cleans] [and scrubs in one easy step]]
- [Disinfect hard non-porous surfaces]
- [Dispenses one wipe at a time]
- [Does not contain dyes]
- [DO NOT FLUSH WIPE DOWN TOILET]
- [Double sided-scrubbing wipes]
- [Durable, lint-free wipe designed for sensitive equipment surfaces]
- [Easy]
- [Easy and convenient to use]
- [[Easy] [and convenient] way to clean [and] [disinfect]
- [Easy One Step Cleaning [and disinfecting] (except when precleaning is required for heavy soil and blood and body fluids)] [Precleaning is required for fungicidal[^] activity]
- [Easy] pop-up-[tub][canister][pail][lid][tab][dispenser][bucket]
- [Easy [and convenient] way to disinfect [bathrooms][office][nursing stations][high touch] hard, non-porous surfaces everyday]
- [Easy] [convenient] [dispensing] lid [closure][cap][package]
- [Easy drop-in refills]
- [Easy to use]
- [Easy to handle]
- [Easy to refill]
- [Easy to use package [canister][container][refill] keeps [your] wipes moist]]
- [Easy way to [clean] [and] [disinfect] [bathroom] [or exterior toilet surfaces] [hard, non-porous surfaces]]
- [Easy and convenient disinfecting [throughout the ▲]]
- [Easy to use package [canister][container][refill] keeps [your] wipes moist]]
- [Easy, one-step cleaning and disinfecting]
- [Easy [and convenient] way to [touch-up] [bathroom] [office] everyday]
- [Easy start [tab][lid] [(insert package type)]
- [Easy] [and convenient] [way to clean] [and disinfect]
- [Economical]
- [Economy]
- [Economical] Refill[s] [one] [1] [two] [2] [three] [3] [four] [4] [various numbers] pack]]
- [Economical] Refills [now] available]
- [Effective against flu virus (Influenza A/Hong Kong, H3N2 Virus)]
- [Effective against ***Insert any organism from list of organisms***]
- [Effective against Influenza A and H1N1 [Pandemic][Influenza A Virus]
- [Effective against bathroom bacteria and viruses* [use at least two bacteria and two viruses from this list:] [ESBL *E. coli*] [*Salmonella enterica*] [*Staphylococcus aureus*] [Herpes Simplex Virus Type 2] [Hepatitis B Virus (HBV)*] [Hepatitis C Virus (HCV)*] [Adenovirus 5*][Influenza A/Hong Kong H3N2 Virus*] [*Carbapenem-Resistant Klebsiella pneumoniae* (CRKP)(CRE)]
- [Effective against -or- Kills a range of bacteria including *Staphylococcus aureus*, *Salmonella enterica*, *Pseudomonas aeruginosa* [in the presence of organic soil [(5% blood serum)]]
- [Effectively disinfects against MRSA]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRODUCT CLAIMS continued

- [[Effective against -or- Kills (*insert any organism(s) from Organisms table*)] [in the presence of organic soil [(5% blood serum)]] [Precleaning is required for fungicidal^{^^} activity]
- [Effective in the presence of 5% blood serum contamination]
- [Effective against flu virus (Influenza A)]
- [[Effective -or- For daily use against (insert organisms)
- [[Effectively] disinfects hard, nonporous [environmental] surfaces]
- [Effectively removes everyday soils]
- [Efficacy tests have demonstrated that [Product Name] [this product] [are] [is] an effective bactericide, virucide*, and tuberculocide, in the presence of organic soil (5% blood serum)]
- [Efficacy tests have demonstrated that [Product Name] [this product] [are] [is] an effective fungicide^{^^}.]
- [Eliminates odors]
- [Eliminates odors from fecal matter and urine]
- [Eliminates airborne particles associated with sprays]
- [Eliminates [daily] mixing and measuring]
- [Eliminates [lingering] [food+] odors [on surfaces]]
- [[Eliminates odors at their source [caused by germs -or- bacteria-and/or- mildew]]
- [[Eliminates -or- Reduces odors [caused by germs -or- bacteria -and/or- mildew]]
- [EVS [Environmental Services] [use]]
- [Exhibits effective virucidal* activity]
- [Extra large [disinfecting] [Y" x Z" compared to our A" x B" size] wipes]
- [Extra large] [Disinfecting Towelettes]
- [Convenient] [portable] way to disinfect hard non-porous surfaces]
- [Cleaner and disinfectant]
- [[Fight[s] -and/or- kill[s] -and/or- Effective against [99.9% [of]] ESBL E. coli]
- [[Fight[s] -and/or- kill[s] -and/or- Effective against [99.9% [of]] *Salmonella enterica* [(Salmonella)]]
- [[Fight[s] -and/or- kill[s] -and/or- Effective against flu (Influenza A) [virus] [on environmental - and/or- hard, nonporous surfaces]]
- [[Fight[s] -and/or- kill[s] -and/or- Effective against -and/or- flu (Influenza A) [virus(es)]]
- [[Fight[s] -and/or- Kill[s] -and/or- Effective against (*insert any organism from list*)]
- [For daily use]
- [For] general use
- [For Household Use]
- [For use on bathroom surfaces]
- [For use on nursery surfaces]
- [For use on ♦ surfaces]
- [For use in critical care areas [where control of cross contamination on treated hard non-porous surfaces is of prime importance]
- [For] Surface cleaning
- [For Use by Home Caregivers]
- [For Use in Home Healthcare]
- [For use in ▲]
- [Formulated -or- Designed to] remove[s] biological odors including Urine - and/or- Vomit]
- [Fragrance Free]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRODUCT CLAIMS continued

- [Freshens]
- [Free of fragrance]
- [Fungicide^^]
- [Fungicidal^^]
- [Fungicidal^^ in 3 minutes]
- [Germicidal disposable wipes]
- [Germicide -or- Germicidal]
- [Gets rid of [or eliminates] odors]
- [Great for everyday use]
- [[Heavy] cloth-like towel]
- [Healthcare Disinfectant/Cleaner]
- [[Helps] [kill][s] [eliminate][s] odor-causing bacteria]
- [Helps reduce cross contamination on treated hard non-porous surfaces]
- [Hospital Disinfectant/Cleaner]
- [Hospital [disinfectant] [disinfection] [use only when hospital use sites on label]
- [Includes ___ count refill [pack] [contains] ___ wipes]
- [Institutional [size]]
- [Intermediate Level Disinfectant]
- [Just wipe to clean and disinfect]
- [Just] [Simply] wipe [exterior toilet surface] [bathroom germs] [away]
- [Just [pull] wipe and toss]
- [Kills Adenovirus Type 5 #]
- [Kills Pandemic 2009 H1N1 Influenza A Virus [(formerly called swine flu)]
- [Kill TB/HIV-1 in 1 minutes]
- [Kills][Effective against] [MRSA] [Methicillin Resistant *Staphylococcus aureus*]
- [Kills][Effective against] [Pathogenic Fungi] [*Candida albicans*] [1 minute]
- [Kills][Effective against] [[*Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*)]][in] [just] [3 minutes]
- [Kills] [Effective against] [TB] [Extended Spectrum β -lactamase *Escherichia coli*] [ESBL *E. coli*] [Multi-drug resistant [MDR] *Acinetobacter baumannii*] [*Klebsiella pneumoniae*] [*Salmonella enterica*] [*Staphylococcus aureus*] [*Pseudomonas aeruginosa*] [Methicillin Resistant *Staphylococcus aureus* (MRSA)] [Methicillin Resistant *Staphylococcus epidermidis* (MRSE)] [Vancomycin Resistant *Enterococcus faecalis* (VRE)] [Vancomycin Intermediate *Staphylococcus aureus*] [*Carbapenem-Resistant Klebsiella pneumoniae* (CRKP)(CRE)] [Herpes Simplex Virus Type 1] [Herpes Simplex Virus Type 2] [Hepatitis B Virus (HBV)*] [Hepatitis C Virus (HCV)] [Human Immunodeficiency Virus (HIV-1)] [Human Coronavirus] [(not associated with Severe Acute Respiratory Syndrome or SARS)] [Adenovirus Type 5(3 minute contact time for Adenovirus Type 5)] [[*Trichophyton interdigitale* (formerly Trichophyton Mentagrophytes) (3 minute contact time for *Trichophyton interdigitale* (formerly *Trichophyton mentagrophytes*)] [Influenza A, Hong Kong, H3N2 Virus*] [in] [just] [1 minute] [3 minutes]
- [Kills [or eliminates] 99.9% of bacteria]
- [Kills [or eliminates] 99.9% of viruses*]
- [Kills [or eliminates] 99.9% of fungus]
- [Kills bacteria in 1 minute]
- [Kills viruses* [^] in 1 minute] [3 minutes for [Adenovirus Type 5]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRODUCT CLAIMS continued

- [Kills viruses* in 3 minutes]
- [Kills TB in 1 minute]
- [Kills TB/HIV-1 in 1 minute]
- [Fungicidal^{^^} in 3 minutes]
- [Kills [bacteria] [viruses*] [[pathogenic]fungi]]
- [Kills bacteria, viruses*]
- [Kills [bacteria] [viruses*] [[pathogenic]fungi]]
- [Kill germs and bacteria instead of germy paper towels, sponges and rags [Use this product to]]
- [Kills odor causing bacteria]
- [Kills -or- Effective against bacteria]
- [Kills -or- Effective against germs]
- [Kills -or- Effective against viruses*]
- [Kills odor-causing bacteria -or- germs]
- [Kills -or- Effective against [bacteria][and][germs][and][viruses*]]
- [[Kills drug resistant organisms [(list organisms)]]
- [Kills drug resistant bacteria [(list organisms)] [in [1] [one] minute]]
- [Kills][Effective against] [***Insert any organism from list of organisms***] [in] [just][1 minute] [Kill bacteria in 1 minute] [Kill viruses* [^] in 1 minute] [Kill TB in 1 minute] [Kill TB/HIV-1 in 1 minute] [Kill [[*Trichophyton interdigitale* (formerly Trichopyton Mentagrophytes)], [Adenovirus Type 5] in 3 minutes]]
- [Kitchen+ and Bath Wipes] [for Hard, Non-porous Surfaces]]
- [Large[r] Surface[s]]
- [Large[r] [wipe][s]]
- [Larger wipe covers [more surface area] larger area [with less effort][!]]
- [Lead Free]
- [Leaves bathroom fresh and clean in [just [1 minute]]
- [Leaves bathroom fresh and clean in [just] minutes]
- [Leaves [bathroom] [office] clean [and disinfected] [and smelling fresh]]
- [Less] [Reduced] [packaging] [less][to throw away]
- [Lightly scented –or- quick-dissipating scent]
- [Lint free] [wipe]
- [Making home healthcare [more manageable] [easier] [simpler]]
- [Maximum cleaning, minimum effort]
- [Meets [the disinfection requirements of] OSHA['s] Bloodborne Pathogen Guidelines -or- Standards]
- [Mercury free]
- [Moist wipes]
- [Moisture lock lid]
- [Multi-pack]
- [Multi-purpose cleaner]
- [Multi-purpose disinfectant]
- [Multi [-refill]] [pack]
- [Multi-surface]
- [Next wipe pops up automatically [everytime] [!]]
- [Neutralizes odors [doesn't [just] mask them]]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRODUCT CLAIMS continued

- [Multi [-refill]] [pack]
- [Multi-surface]
- [Next wipe pops up automatically [everytime] [!]]
- [Neutralizes odors [doesn't [just] mask them]
- [No canister needed] (for flat packs, singles)
- [No [pinched fingers] [rips] [tears]]
- [No sponges, no rags, no buckets, no mess]
- [No threading [required]](for flat packs, singles)
- [No fragrance added]
- [No lint wipe]
- [No measuring] [required]
- [No mixing] [required]
- [No sponges, no rags, no buckets, no mess]
- [Non-scratching [wipe]]
- [Non-woven disposable cloth]
- [Non-abrasive formula]
- [[Occupied Room] [and] [Terminal Room] [Discharge] Cleaning Wipe[s] [Towel][s] Refill[s][able]]
- [Odor [elimination] [neutralization]
- [On the go disinfectant]
- [One Step Germicidal Wipe]
- [One-step cleaner and disinfectant](precleaning is required for Fungicidal^{^^} activity)
- [One step [cleaning] [disinfecting] [wipe]] [(precleaning is required for Fungicidal^{^^}activity)
- [One-step cleaning and disinfecting [saves time]] [(precleaning is required for Fungicidal^{^^} activity)
- [One step cleaning [and/or disinfecting] [and/or deodorizing] disinfect and/or touch-up clean]] (precleaning is required for Fungicidal^{^^} activity)
- [One step cleaning [and/or disinfecting] [and/or deodorizing] disinfect and/or touch-up clean] [cleans and/or disinfects and/or deodorizes] [anytime] [daily] [every day] [in one easy step] [on hard non-porous][on ▲] [bathroom and/or kitchen] surfaces [on household surfaces][on ♦ surfaces]] [(precleaning is required for Fungicidal^{^^} activity)
- [One step disinfectant cleaner that is effective against a broad spectrum of bacteria when used as directed] [(precleaning is required for Fungicidal^{^^} activity)
- [One step germicide]
- [One-step disinfectant cleaner designed for general cleaning and disinfecting hard, nonporous environmental surfaces in ▲]](precleaning is needed for Fungicidal^{^^} activity)
- [On-the-go disinfectant [Package] [insert type of package] made from [x%] recycled materials]
- [One [wipe] at a time dispensing]
- [Package] [Insert type of package] made from [x%] recycled materials]
- [Recommended for cleaning [and disinfecting] ♦.]
- [Recommended for cleaning [and disinfecting] around the toilet.]
- [Recommended-for disinfecting exterior toilet surfaces – or – around the exterior toilet surface]
- [Picks up [particles] [dust] [dirt] [hair] [crumbs]]
- [Pop-up [dispensing] [top] [tab] [lid]]
- [Portable]
- [Pouch[es]]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRODUCT CLAIMS continued

- [Powers through tough messes]
- [Pull n/ Go [pack]]
- [Pre-measured]
- [Pre-moistened [towelettes]
- [Premixed], [premeasured], [premoistened]
- [Premoistened [wet] household cleaning wipe]
- [Pre-measured to have the recommended amount of germicidal solution, wipe after wipe]
- [Pre-moistened] [Easy] [and convenient] way to disinfect]
- [Pre-moistened with specially formulated cleaners to handle daily clean-ups without bottles, sponges or buckets]
- [Ready to Use [RTU] wipe [prevents formulation errors]
- [[Product Name] are useful in a variety of [healthcare], [nursery/daycare] [and] [▲] applications listed on the [this] label.]
- [Easy] [and] [Convenient] [way to clean] [and disinfect]
- [Easy [Convenient] way to clean [and disinfect] [bathroom] [or exterior toilet surfaces] [hard, non-porous surfaces]
- [Ready-To-Use]
- [[Ready-to-use] disinfectant [for use in ▲]
- [Recyclable] [package]
- [Reduces] [cross contamination] [on treated hard non-porous surfaces]
- [Reduces] [packaging] waste [by X%] [when compared to regular canister or bucket or pail]
- [Reduce]
- [Reduces cross contamination to (insert organism) on treated hard non-porous surfaces]
- [Reduces] [cross contamination] [on treated hard non-porous surfaces]
- [Reduces] [packaging] waste [by X%] [when compared to regular canister or bucket or pail]
- [Reduces of cross-contamination [on treated hard non-porous surfaces] [from (*insert any organism from organisms list*) [in 1 minute)]]].
- [Refill package contains [one] [1] [two] [2] [three] [3] [four] [4] [various numbers] [wet] [pre-moistened] [wipe] [refill(s)] [roll(s)]
- [Refill [size] [pack] [value pack]
- [Refillable [with ___# wipes] [dispenser] refill [pack] [for dispenser]
- [Respiratory illnesses attributable to Pandemic 2009 H1N1 are caused by Influenza A virus. This product is a broad-spectrum hard surface disinfectant that has been shown to be effective against (Influenza A tested and listed on this label) and is expected to inactivate all Influenza A viruses including Pandemic 2009 H1N1 (formerly called swing flu).]
- [Requires no mixing or measuring and are ready-to-use]
- [Re-sealable [stay-moist] pouch [package] [container] inside]
- [Re-use your [empty] canisters [again [and again] [!]
- [Reuse[able] [Container][Dispenser][Canister][Pail][Bucket]]
- [Salon Disinfectant/Cleaner]
- [Sample]
- [School Disinfectant/Cleaner]
- [Scrubbing power of a sponge in a disinfecting [wipe] [towelettes]]
- [Shelf-stable formula]
- [So] [surfaces are more than [just] [clean] [they're] [disinfected]
- [Spa Disinfectant/Cleaner]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRODUCT CLAIMS continued

- [Specifically -or- Specially designed for the Medical/Dental/ Veterinary/Health Care [▲] industry]
- [Starts cleaning on contact]
- [Strong, durable low-lint wipe] wipe [designed for sensitive equipment surfaces]]
- [[Strong,] [Durable,] cloth-like wipes]
- [Strong, durable textured wipe [cleans] [and] [deodorizes]]
- [[Strong,] [Durable,] non-woven disposable cloth]
- [[Strong,] [Durable,] non-woven wipe does not leave lint on surfaces]
- [Suitable for [▲]]
- [Suitable for use on [◆]]
- [Suitable for use on [titanium-coated] [medical grade] stainless steel]
- [Suitable for use on hard, nonporous inanimate environmental surfaces]
- [Strong cleaning power]
- [Sturdy][Container][Dispenser][Canister][Pail][Bucket]
- [Tear at perforation]
- [Textured for better [cleaning] [scrubbing] power]
- [Textured design allows for] [scrubbing] [cleaning] [removing] [wiping] [tough] [stains] [soils] [messes]
- [Textured wipe for effective cleaning]
- [Textured] wipe is low-linting [designed] [for sensitive equipment surfaces]
- [Textured design [cleans] [deodorizes] hard, non-porous surfaces] [helps remove soil from surfaces]]
- [Textured] [strong] [towelettes] [for] [with] [scrubbing power]
- [The 1-minute [germicidal wipe] [disinfectant wipe] [[*Trichophyton interdigitale* (formerly *Trichopyton Mentagrophytes*), [Adenovirus Type 5] in 3 minutes]]
- [The answer to your [disinfecting] [cleaning] needs]
- [The] smell of clean]
- [[The] easy -and/or- convenient way to disinfect] [with CaviWipes 1™]
- [This product] controls the spread of germs on most treated hard, nonporous surfaces
- [This product has demonstrated effectiveness against Influenza A Virus and is expected to inactivate all Influenza A Viruses including Pandemic 2009 H1N1 Influenza A Virus]
- [This product meets AOAC efficacy testing requirements –or standards for hospital disinfection]
- [[Thick], [durable] [and] [heavy duty] wipes are bactericidal, tuberculocidal, and virucidal*[^] in 1 minute]
- [Thick, durable wipes are bactericidal, tuberculocidal, and virucidal* in [1] [3] minute[s]]
- [Thick, large wipe [cleans] [disinfects]]
- [This] [outer] [selling] [package] [container] [canister] contains X% post-consumer [material] [plastic] [resin]
- [Time to reorder]
- [Trial]
- [Tuberculocidal (1 minute)]
- [Tub refill]
- [Tub with pop-up action]
- [To break down and kill [inactivate]](insert organisms) on treated hard, nonporous surfaces.]
- [To clean and disinfect (insert organisms) on hard, nonporous laden surfaces.]
- [To kill [inactivate](insert organisms)[on treated hard, non-porous surfaces]
- [To fight (insert organisms) on hard, nonporous surfaces.]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRODUCT CLAIMS continued

- [To reduce (insert organisms) on surfaces in [▲]
- [To reduce cross contamination of (insert organisms) [on treated hard, nonporous surfaces [on ♦][in ▲]]
- [[Tub] [Pail] [Bucket] refill] [available]
- [Twin pack]
- [Two in one action [cleans and disinfects in one easy step]
- [Tuberculocide -or- Tuberculocidal]
- [Usable on variety of soils]
- [Use in [public] [common places] [▲]where [bacteria -and/or- viruses*-and/or- germs may be of concern] [on hard, nonporous surfaces]
- [Use to] [Deodorize hard, non-porous surfaces]
- [Use to] refill[s] [count] [wipe] [canisters] [tubs] [bucket] [tub] [pail]
- [Use to pre-clean critical or semi-critical medical devices prior to sterilization / high-level disinfection]
- [Use wipes to kill germs and bacteria instead of germ paper towels, sponges and rags]
- [Use this product to][Kill bacteria in 1 minute] [Kill viruses* [^] in 1 minute] [Kill TB in 1 minute]
- [Use to pre-clean critical or semi-critical medical devices prior to sterilization / high-level disinfection]
- [Use where control of cross-contamination between treated hard non-porous surfaces is of prime importance]
- [Use on hard, non-porous surfaces only: [stainless steel] [Formica®][Porcelain][Plastic][Glazed Ceramic] [glass] [glass tables] [glazed tile] [Test wipe on small inconspicuous area first.]
- [Value]
- [Versatile]
- [Veterinary Disinfectant/Cleaner]
- [Virucide*[^] -or- Virucidal*[^]][in 1 min]
- [Wet wipes]
- [When used according to label directions, [this product] reduces cross contamination (insert organisms) between treated hard non-porous surfaces]
- [Where control of cross contamination between treated hard non-porous surfaces is of prime importance]
- [Wipe away odors]
- [Wipes out] [Kills] [germs] [or bacteria] [while it cleans] [that cause odor]
- [Wipe] [material] made from [x %] [sustainable] [ingredients] [fibers] [may only be used when cellulose wipe material used]
- [Wipes out] [Kills] [common] [household] [bacteria] [viruses*[^]] [pathogenic fungi] [clinically relevant organisms] [germs] ***[(insert any organism from organism list)*** [in 1 minute] ***[[[Trichophyton interdigitale (formerly Trichopyton Mentagrophytes)], [Adenovirus Type 5] in 3 minutes]]***
- [Wipes out] [Kills] [Eliminates] [list organisms] in [just] [1 minute] ***[(insert any organism from organism list)*** [in 1 minute] [3 minutes for [Trichophyton interdigitale (formerly Trichophyton mentagrophytes) and Adenovirus Type 5]
- [Wipes out] [Kills] [the] flu virus (Influenza A/Hong Kong, H3N2 Virus)]
- [Wipes out] [Kills] [viruses* and bacteria [including the flu virus (Influenza A/Hong Kong, H3N2 Virus)]
- [X" x Y" Wipes]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

PRODUCT CLAIMS continued

- [X% [more] [wipes] [free] [vs. Z count] [canisters] [packs]
- [X% reduction in packaging [material]]
- [Wipes out] [Kills] [common] [household] [bacteria] [viruses*] [germs]
- [Wipes out] [Kills] [Eliminates] [list organisms] in [just] [1 minute] [3 minutes]
- [Wipes out] [Kills] [the] flu virus (Influenza A/Hong Kong, H3N2 Virus)
- [Wipes out] [Kills] [viruses* and bacteria [including the flu virus (Influenza A/Hong Kong, H3N2 Virus)]
- [With a bleach free formula]

[Note: The following claims can be used only for the first 6 months of the product on shelf]

- [New (Organisms added)] [Strong] [Thick] Wipe [!]
- [New dispensing] lid [closure] [cap] [package] [dispenses] [dispensing] [better [than before]
- [Big][Strong] [Thick] [wipe] [!]
- [Bigger] [Better] [Stronger] [Thicker] [wipe] [!] versus previous CaviWipes1]
- [Multi-purpose [cleaner] [disinfectant] [Multi [-refill] pack] New [larger] thicker [textured] wipe] versus previous CaviWipes1]
- [New (Organisms added)]
- [New] [Package] [packaging]
- [New thick and large embossed wipe cleans more surface area] versus previous CaviWipes1]
- [Reinforced] [With] [Quilted][strength]
- [Now with [%] more towel] versus previous CaviWipes1]
- [Now with [#] more wipes] versus previous CaviWipes1]
- [Package] [Packaging] uses [X%] less material [plastic] [than previous CaviWipes1 canister]
- [Scrubs] (&) [cleans]
- [Stronger] more durable [wipe] versus previous CaviWipes1]
- [Thick[er] [wipe] [with] cloth-like texture] versus previous CaviWipes1]
- [X% [Bigger] [larger] [quilted] wipes] versus previous CaviWipes1]
- [x%] thicker wipes] versus previous CaviWipes1]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

[Note – the chart shown below shall be considered optional – e.g. inclusive in a bracket]

[When used as directed, CaviWipes 1 provides effective Tuberculocidal / Virucidal*[^] / Bactericidal / Fungicidal^ activity against the following organisms:

<u>Tuberculocidal</u>	Associated with:
<i>Mycobacterium tuberculosis</i> var: <i>bovis</i> (BCG) (TB)	Tuberculosis (BCG)
<u>Viruses*</u>	
Human Immunodeficiency Virus (HIV-1)*	AIDS
Influenza A, Hong Kong, H3N2 Virus*	Common flu
<u>Fungus/Yeast</u>	
<i>Trichophyton interdigitale</i> (formerly <i>Trichophyton mentagrophytes</i>) (3 minute contact time)	Athletes Foot Fungus

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

Table 1 Medical Use Sites:

CaviWipes 1 are recommended for use on hard non-porous surfaces in[▲] [health care settings] [such as]

3D Imaging Services Centers	ENT Facility	Ophthalmic [Facilities] [Offices]
Acute Care Facilities	Eye Care [Facilities][Centers	Optometric [Facilities] [Offices]
Alternate Care Facilities	Eye Surgical Centers	Oral Health Care Facilities
Ambulances	Forensic Labs	Orthodontic Offices
Ambulatory Surgery Centers (ASC)	Full-Care Nursing Facility	Orthopedics
Animal Research Facilities	Gastrointestinal (GI) [Labs][Laboratories]	Outpatient Care Facilities
Assisted Living Facility	Gastrointestinal (GI) Centers	Outpatient [Surgery][Facilities] [Centers]
Autopsy Rooms	Gastrointestinal (GI) Suites	Out-Patient Surgical Centers (OPSC)
Bathroom	Home[health] Care Services	Pathology Rooms
Bioskill Laboratories	Hospice	Patient Care Areas
Blood Drawing Centers	Hospital	Patient Care rooms
Blood Apheresis Facilities	Hospital operating rooms	Patient Rooms
Blood Units	Hydrotherapy [Facilities][Rooms]	Pediatric Exam Rooms
Burn Center	In vitro Fertilizer Center (IVF)	Pediatric Intensive Care Units (PICU)
Cardiac Intensive Care Unit (CICU)	Instrument Reprocessing [Facilities][Departments][Center]	Pediatric Offices
CCU	Instrument Reprocessing [Rooms]	Personal Care Service Facilities
CAT Scan [Lab] [Laboratories]	Intensive Care Units (ICU)	Pharmacy Clean Rooms
Cath Labs	Isolation Areas	Phototherapy Rooms
Central Service Areas	Isolation Rooms	Physical Therapy Facilities
Central Services Department	Labor Units	Physicians' Offices
Central Supply Areas	[Lab] [Laboratories]	Plasma Apheresis [Centers][Facilities][Units]
Central Supply Rooms	Lasik Eye Centers	Post [Ops][Operatory] Rooms
Clean Rooms	Lasik Units	Pre[Ops] [Operatory] Areas
Clinical Areas	Laundry Rooms	Psychiatric Care Facility
Clinics	Life Science Laboratories	Radiology Department
Critical Care Units	Long-Term Acute Center (LTAC)	Recovery Rooms
Computer Science Labs	Long-Term Care [Facilities]	Rehabilitation Facilities
CS [Labs][Laboratories]	Medical Clinics	Respiratory Therapy Facilities
Dialysis Centers	Medical Facilities	Skilled Nursing [Facility] [Homes]
Dialysis Clinics	Medical Related Facilities	Sterile Processing

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

		Departments
Dissection [Labs][Laboratories]	Medivans	Sterile Storage Areas
Doctor's Offices	Medical Intensive Care Unit (MICUs)	Surgery Centers
Donation Centers [blood] [plasma] semen milk aphaeresis	Magnetic Resonance Imaging (MRI)[facilities][laboratories][labs]	Surgery Rooms
Drug Preparation [Centers] [Facilities] [Rooms]	Neonatal Intensive Care Units (NICU)	Surgical Centers
Drug Preparation Rooms	Neonatal Nursery Areas	Surgical Suites
Electrolysis Facilities	Newborn Nursery Center	Teaching Hospitals
Emergency Departments	Nuclear Medicine Facilities	Tissue Banks
Emergency Medical Services (EMS) Units	Nurses' Stations	Transplant Center
Emergency Medical Settings	Nursing Homes	Trauma Center
Emergency Rooms (ER)	Nursing Stations	University Research Lab
Emergency Vehicles	Occupational Therapy [Centers][Facilities]	Wound Center
Endo Suites	Oncology Areas	X-Ray Rooms
Endoscopy [Centers] [Facilities] [Departments]	Out-patient surgical centers (OPSC)	
Endoscopy Centers	Operating Rooms	

Table 2 Institutional Use Sites:

CaviWipes 1 are recommended for use on hard non-porous surfaces in [▲]

Adult School	Early Learning Centers	Middle School
Air Transport Facilities	Elementary Schools	Military Installations
Airplane Interiors	[Fire][Rescue Departments]	Nursery School
Airports	Fire Vehicles	Police Vehicles
Bathroom	Forestry Services	Pre-K School [Room]
Casino	Grammar School	Preparatory School
Child Care Facilities	High School	Preschool
Churches	Hospitality Facilities	Primary School
Colleges	Hotels	Prisoner Detention Facilities
Community Colleges	Industrial Facilities	Prisoner Lockups
Conference [Centers][Rooms]	Infant Care Centers	Prisons
Conference Rooms Fixtures	Intermediate School	School Buses
Convention Facilities	Jails	School Kitchens+
Correctional Facilities	Junior High School	Schools
Daycare Centers	Kindergarten	[University][Universities]
[Dormitories][Dorm]	Learning Institutions	

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

Table 3 Dental Use Sites:

CaviWipes 1 are recommended for use on hard non-porous surfaces in [▲]

Dental [Laboratories][Lab]	Dental Office	Dental [Operatory] [Operatories]
----------------------------	---------------	-------------------------------------

Table 4 Vet Use Sites:

CaviWipes 1 are recommended for use on hard non-porous surfaces in [▲]

Animal Care Facilities	Avian Units [Hatchery]	Pet Homes
Animal Control Vehicles	Bathroom	Pet Motels
Animal Dental Operatory	Doggy Day Care Centers	Pet Spas
Animal Detention Room	Feline Facilities	Pet Stores
Animal Research Facilities	Feline Units	Small Animal Facilities
Animal Shelters	Kennels	Veterinary Offices
Avian [Care] Facilities	Pet Areas	Veterinary Facilities

Table 5 Food Prep+ Use Sites:

CaviWipes 1 are recommended for use on hard non-porous surfaces in [▲]

Bakeries+	Food Processing Facilities+	Restaurants+
Banquet Hall Facilities+	Grocery Stores +	School Kitchens+
Butcher Shops+	Kitchen+	Supermarkets+
Food Preparation Areas+	Produce Areas+	

[+ A thorough rinse with potable water is required for surfaces in direct contact with food.]

Table 6 Miscellaneous Use Sites:

CaviWipes 1 are recommended for use on hard non-porous surfaces in [▲]

Airplanes	Factories	Money Handling Facilities
Arboretums	Foots Spas	Morgues
Arcades	Foreign Current Exchange Facilities	Mortuaries
[Athletic] [Training] Facilities	Funeral Homes	Motels
Auto Facilities	Gaming Facilities	Movie Theaters
Banks	Garden Centers	Outdoor Playground
Barber Shops	Golf Courses	Personal Care Service Facilities
[Bathroom(s) or public restrooms]	Grounds Keeping Services	Public Transportation Vehicles
Beauty Salons	Gymnasium	Recreational [center] [facilities]
Body Piercing Salons	Health Club Facilities	Recycling Centers
Body Service Facilities	Helicopter Interiors	Salon Settings
Body Storage Facilities	Household Bathrooms	Weather Shelters
Botanical Gardens	Housekeeping Rooms	Shower Rooms
Bowling Alleys	Indoor Playground	Skincare Salons
Buses	Laundromats	Storage [Areas] [Rooms]
Cabs	Laundry Rooms	Subway
Cadaver Processing areas	Lawn Services	Tanning Salons

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

Cars	Limousine	Tattoo Parlors
Centers	Living Rooms	Taxi
Condominiums	Locker Rooms	Theme Parks
Cruise Ships	Lockup Facilities	Transport Vehicles
Currency Exchange	Manicure Salons	
Day Spa	Manufacturing [Facilities] [Plant]	
Delivery Units	Massage Facility	

Table 7 Surfaces:

CaviWipes 1 are recommended for use on hard non-porous surfaces [♦]

Ambulances [Equipment Surfaces]	Bathroom Fixtures	Carts
Anesthesia Equipment	Bathtubs	Cash Register
Anesthesia Machines	Bed Pans	CAT Scan Machines
Animal Cages	Bed Railings [Bedside Rails]	Ceilings
Animal Equipment	Bedside Commode	Cell Phones
Aphaeresis Machines	Behind and Under Sinks	Center Venous Lines
Appliances+	Bidet	Chairs
Arm Rests	Biohazard Container	Changing Tables
ATM Surfaces	Bladder Scan	Chemotherapy Hoods
Autoclave Surfaces	Blood Glucose Meters	Child Seats+
Automatic Feeders	Blood Pressure Cuff	Chrome
Automatic Teller Machines (ATM)	Booster Chairs+	Clipper Blades
Baby Cribs	Booster Seats+	Closet Handles
Baby Warmers	BP Monitors	Coated Mattresses
Baked Enamel	Brushes	Coated Pillows
Bars	Cabinet Handles	Combs
Basins	Cabinets	Compression Devices
Baskets	Cafeteria Trays+	Computer [Keyboards] [Mouse]
Bassinet	Cages	Computers
Bathing Units	Cardiac Gym Equipment	Copy Machines
<ul style="list-style-type: none"> • [For use on Hard Non-Porous [Food Prep Surfaces+] [Surfaces] [Medical Surfaces][Veterinary Surfaces][Dental Surfaces] [Institutional Surfaces][▲ surfaces] • [and other critical care areas where control of cross contamination between treated hard non-porous surfaces is important. [Use on hard non-porous surfaces and equipment listed on this label made of stainless steel, plastic, Formica, and glass.] • [Surfaces which are made of plastic (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene), and use sites listed on this label consisting of non-porous vinyl, stainless steel, painted surfaces, Plexiglas®, Formica® and glass.] • + A thorough rinse with potable water is required for surfaces in direct contact with food. • +Any washable (food and non-food contact) surface where disinfection is required 		

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

Table 7 Surfaces continued:

Counters	Diaper Changing Stations	Examination Table
Countertops+	Diaper Pails	Exterior Surface of Digital X-Ray Sensors
Cribs+	Dictating Equipment	Exterior Surfaces of Amalgamators
Crutches	Digital Phosphor Plates	Exterior Surfaces of Anesthesia Machines
Defibrillators	Dining Tables+	Exterior Surfaces of Appliances such as Refrigerators+, Microwaves+, Food Processors+ and Stoves+
Dental Care Facilities Hand Pieces	Dish Racks+	Exterior surfaces of CICU Equipment
Dental Chairs	Display Cabinets	Exterior Surfaces of Dental Operatories
Dental Countertops	Diving Equipment	Exterior Surfaces of Dental Equipment
Dental Curing Lights	Diving Masks+	Exterior Surfaces of Diagnostic Equipment
Dental Equipment Lines	Diving Snorkels+	Exterior Surfaces of Dialysis Machines
Dental Equipment Tubing	Doorknobs	Exterior surfaces of EKG Machines
Dental Impression Trays	Dopplers	Exterior surfaces of Electronic Thermometers
Dental Operatory Surfaces	Drain Boards+	Exterior surfaces of Electrosurgical Generators
Dental Unit Instrument Trays	Drained Whirlpool Tank Surfaces	Exterior surfaces of Emergency Resuscitative Equipment
Dentist Chair	Emergency vehicles	Exterior Surfaces of Endodontic Equipment (Apex Locators, Pulp Testers, Motors)
Desk Tops	Enamel	Exterior surfaces of Endodontic equipment such as apex locators
Desks	Endoscope Cabinet	Exterior surfaces of ICU Equipment
Dialogistic Equipment	Exam Tables	Exterior surfaces of Incubation Equipment
<ul style="list-style-type: none"> • [For use on Hard Non-Porous [Food Prep Surfaces+] [Surfaces] [Medical Surfaces][Veterinary Surfaces][Dental Surfaces] [Institutional Surfaces][▲ surfaces] • [and other critical care areas where control of cross contamination between treated hard non-porous surfaces is important. [Use on hard non-porous surfaces and equipment listed on this label made of stainless steel, plastic, Formica, and glass.] • [Surfaces which are made of plastic (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene), and use sites listed on this label consisting of non-porous vinyl, stainless steel, painted surfaces, Plexiglas®, Formica® and glass.] • + A thorough rinse with potable water is required for surfaces in direct contact with food. • +Any washable (food and non-food contact) surface where disinfection is required 		

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

Table 7 Surfaces continued:

Exterior surfaces of Incubators	Exterior surfaces of Patient Care Monitor	Exterior surfaces of Ultrasound Monitors
Exterior surfaces of Infusion Pumps	Exterior surfaces of Pneumatic Tourniquets	Exterior Surfaces of Vanity Tops
Exterior surfaces of Isolettes	Exterior surfaces of Probes	Exterior Toilet Surfaces
Exterior surfaces of IV Pumps	Exterior surfaces of Pulp Tester Motors	External Surfaces of Glucometers
Exterior surfaces of Kendrick Extrication Device (KED)	Exterior surfaces of Pulse Oximeter	External Surfaces of Ultrasound Transducers
Exterior surfaces of Life Support Equipment	Exterior surfaces of Radiology Equipment	External Surfaces of Veterinary Equipment
Exterior surfaces of Medical Equipment	Exterior Surfaces of Refrigerator Units+	Eye Glasses (do not use on contact lenses)
Exterior surfaces of Medical Warmers	Exterior Surfaces of Respiratory Therapy Equipment	Faucets+
Exterior surfaces of Microwaves+	Exterior surfaces of Respiratory Ventilators	Fax Machines
Exterior surfaces of Monitors	Exterior surfaces of Resuscitators	Feed Racks+
Exterior surfaces of Monitors Electrodes	Exterior surfaces of Sequential Compression Devices	Filing Cabinets
Exterior surfaces of MRI Machines	Exterior or Exposed Interior Surfaces of Medical Equipment	Filing Doors
Exterior surfaces of OLED Monitors	Exterior surfaces of Sterilizers	Filing Handles
Exterior surfaces of Ophthalmoscopes	Exterior surfaces of Tympanic Thermometers	Fire [Vehicles] [Trucks]
Exterior surfaces of Oscopes	Exterior surfaces of Ultrasound Machines	Floors
<ul style="list-style-type: none"> • [For use on Hard Non-Porous [Food Prep Surfaces+] [Surfaces] [Medical Surfaces][Veterinary Surfaces][Dental Surfaces] [Institutional Surfaces][▲ surfaces] • [and other critical care areas where control of cross contamination between hard non-porous treated surfaces is important. [Use on hard non-porous surfaces and equipment listed on this label made of stainless steel, plastic, Formica, and glass.] • [Surfaces which are made of plastic (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene), and use sites listed on this label consisting of non-porous vinyl, stainless steel, painted surfaces, Plexiglas®, Formica® and glass.] • + A thorough rinse with potable water is required for surfaces in direct contact with food. • +Any washable (food and non-food contact) surface where disinfection is required 		

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

Table 7 Surfaces continued:

Food Cases+	Hair Cutting Implements	IV Stands
Food Service Countertops	Hair Dryers	Jacuzzi
Food Serving Trays+	Hair Rollers	Kennels
For use as a cleaner and disinfectant on cured, set and hardened, hard non-porous [dental impressions][dental impression materials]	Hampers	Keyboards
For use as a cleaner and disinfectant on hard nonporous [dental impressions] [dental impression materials] prior to sending to a dental lab.	Handles	Knee Walkers
Formica	Handrails	Knobs
Fountains+	Hard non-porous surfaces of vehicles including steering wheel	[Lab]oratory Equipment
Freezers+	Hard Plastic or Metal Flooring	[Lab]oratory Surfaces
Garbage Cans	Hard Surface Weight Lifting Surfaces	Laminated Surfaces
Glass	Hard, non-porous environmental surfaces	Lamps
Glazed Ceramic Tile	Headsets	Light Lens Covers
Glazed Porcelain	High Chairs+	Lights
Glazed Tiled Walls	High Touch Objects	Linoleum
Glazed Tiles	Hoods+	Loupes
Golf Cart	Hospital Bed (Railings, Frames)	Manicure Implements
Grab Bar	Infant Incubators (interior and exterior surfaces)	Manicure Tables
Grocery Carts	Infant/Child Care Equipment Surfaces	Marlite
Gurneys	Inside of Drawers	Mayo tray
Gym Equipment	Instrument Stands	Mayo tray table
Hair Clippers	IV Poles	Nail/Hair Care Implements
<ul style="list-style-type: none"> • [For use on Hard Non-Porous [Food Prep Surfaces+] [Surfaces] [Medical Surfaces][Veterinary Surfaces][Dental Surfaces] [Institutional Surfaces][▲ surfaces] • [and other critical care areas where control of cross contamination between hard non-porous treated surfaces is important. [Use on hard non-porous surfaces and equipment listed on this label made of stainless steel, plastic, Formica, and glass.] • [Surfaces which are made of plastic (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene), and use sites listed on this label consisting of non-porous vinyl, stainless steel, painted surfaces, Plexiglas®, Formica® and glass.] • + A thorough rinse with potable water is required for surfaces in direct contact with food. • +Any washable (food and non-food contact) surface where disinfection is required 		

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

Table 7 Surfaces continued:

Non-food contact surfaces+	Police [Vehicles][Cars]	Silk Peel™ with DermalInfusion™
Nurse Call [Button][Device][Cord]	Porcelain Enamel	Sinks+
Operating Room Furniture	Plastic Posey Vest	Slit Lamps
Operating Room Lights	Public Restrooms	Smart Phone Chargers
Operating Room Tables	Razors	Spine Back Boards
Operating Tables	Reception Areas	Stainless Steel
Operatory Light Switches	Reception Counters	Stalls
Outdoor Furniture+ (excluding wood frames and upholstery)	Refrigerator Carts	Stands
Ovens+	Refrigerators+	Stethoscopes
Overbed table	Rubber	Stools
Oxygen Hoods	Safety Straps	Storage unit
Patient Delivery Equipment	Salad Bar Sneeze Guards+	Stoves+
Patient Lift Equipment	Salon Surfaces	Stovetops+
Patient Monitoring Equipment	School Buses	Stretchers
Patient Scales	Scissors	Suction Pumps
Patient Support Equipment	Seats	Support Bars
Pens	Shampoo Bowls	Surgery Microscopes
Phone Chargers	Shears	Surgery Robots
Phone Cradle	Shelves	Surgical Headlights
Physical Therapy (PT) equipment surfaces	Shopping Carts	Surgical Light Sources
Pipes	Shower Chair	Synthetic Marble
Plastic Laminate	Shower Fixtures	Tables+
Plastic Mattress Covers	Shower Stalls	Tank
Playpens	Showers	Tanning Beds
<ul style="list-style-type: none"> • [For use on Hard Non-Porous [Food Prep Surfaces+] [Surfaces] [Medical Surfaces][Veterinary Surfaces][Dental Surfaces] [Institutional Surfaces][▲ surfaces] [and other critical care areas where control of cross contamination between hard non-porous treated surfaces is important. [Use on hard non-porous surfaces and equipment listed on this label made of stainless steel, plastic, Formica, and glass.] • [Surfaces which are made of plastic (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene), and use sites listed on this label consisting of non-porous vinyl, stainless steel, painted surfaces, Plexiglas®, Formica® and glass.] • + A thorough rinse with potable water is required for surfaces in direct contact with food. • +Any washable (food and non-food contact) surface where disinfection is required 		

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses, use sites and surfaces may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable.

Telemetry Equipment	Trash Cans	Walkers
Telephones	Tray Tables	Walls
Tiles+	Troughs	Warmers
Toilet (Rim, Seat, Tank, Handle,)	Tubing	Wash Basin
Toilet above the water line	Urinals	Washable Nail Files
Touch Screen Monitors	Vanity Tops+	Watering Appliances
Towel Dispensers	Vaporizer	Weight-lifting Surfaces
Toys (Plastic or hard non-porous)	Veterinary Care Surfaces	Wheelchairs
Transfer Bench	Vinyl	Whitening Lamps
Transport Vehicles	Vital Sign Monitors	Workstations
<ul style="list-style-type: none"> • [For use on Hard Non-Porous [Food Prep Surfaces+] [Surfaces] [Medical Surfaces][Veterinary Surfaces][Dental Surfaces] [Institutional Surfaces][▲ surfaces] [and other critical care areas where control of cross contamination between hard non-porous treated surfaces is important. [Use on hard non-porous surfaces and equipment listed on this label made of stainless steel, plastic, Formica, and glass.] • [Surfaces which are made of plastic (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene), and use sites listed on this label consisting of non-porous vinyl, stainless steel, painted surfaces, Plexiglas®, Formica® and glass.] • + A thorough rinse with potable water is required for surfaces in direct contact with food. • +Any washable (food and non-food contact) surface where disinfection is required 		