



U.S. ENVIRONMENTAL PROTECTION AGENCY
 Office of Pesticide Programs
 Antimicrobials Division (7510P)
 1200 Pennsylvania Ave., N.W.
 Washington, D.C. 20460

EPA Reg. Number:

46781-19

Date of Issuance:

6/13/22

NOTICE OF PESTICIDE:

Registration
 Reregistration
 (under FIFRA, as amended)

Term of Issuance:

Conditional

Name of Pesticide Product:

QHA-45-W

Name and Address of Registrant (include ZIP Code):

Rebecca Lei
 Senior Regulatory Affairs Specialist
 Electronic Transmittal: rebecca.lei@kavokerr.com

Note: Changes in labeling differing in substance from that accepted in connection with this registration must be submitted to and accepted by the Antimicrobials Division prior to use of the label in commerce. In any correspondence on this product always refer to the above EPA registration number.

On the basis of information furnished by the registrant, the above named pesticide is hereby registered under the Federal Insecticide, Fungicide and Rodenticide Act (FIFRA).

Registration is in no way to be construed as an endorsement or recommendation of this product by the Agency. In order to protect health and the environment, the Administrator, on his motion, may at any time suspend or cancel the registration of a pesticide in accordance with the Act. The acceptance of any name in connection with the registration of a product under this Act is not to be construed as giving the registrant a right to exclusive use of the name or to its use if it has been covered by others.

This product is conditionally registered in accordance with FIFRA section 3(c)(7)(A). You must comply with the following conditions:

1. Submit and/or cite all data required for registration/reregistration/registration review of your product under FIFRA when the Agency requires all registrants of similar products to submit such data.

Signature of Approving Official:

Steven Snyderman, Product Manager Team 33
 RMB II, Antimicrobials Division (7510M)

Date:

6/13/22

2. You are required to comply with the data requirements described in the DCI identified below:
 - a. Alkyl* dimethyl benzyl ammonium chloride *(50%C14, 40%C12, 10%C16) (ADBAC): GDCI-069105-1679
 - b. 1-Decanaminium, N-decyl-N, N-dimethyl-, chloride (DDAC): GDCI-069149-1681

You must comply with all of the data requirements within the established deadlines. If you have questions about the Generic DCI listed above, you may contact the Reevaluation Team Leader (Team 36): <http://www2.epa.gov/pesticide-contacts/contacts-office-pesticide-programs-antimicrobial-division>

3. Make the following label changes before you release the product for shipment:
 - Revise the EPA Registration Number to read, “EPA Reg. No. 46781-19.”
4. Submit one copy of the final printed label for the record before you release the product for shipment.

Should you wish to add/retain a reference to the company’s website on your label, then please be aware that the website becomes labeling under FIFRA and is subject to review by the Agency. See FIFRA section 2(p)(2). If the website is false or misleading, the product would be misbranded and unlawful to sell or distribute under FIFRA section 12(a)(1)(E). 40 CFR 156.10(a)(5) lists examples of statements EPA may consider false or misleading. In addition, regardless of whether a website is referenced on your product’s label, claims made on the website may not substantially differ from those claims approved through the registration process, FIFRA section 12(a)(1)(B). Therefore, should the Agency find or if it is brought to our attention that a website contains false or misleading statements or claims substantially differing from the EPA approved registration, the website will be referred to the EPA’s Office of Enforcement and Assurance.

If you fail to satisfy these data requirements, EPA will consider appropriate regulatory action including, among other things, cancellation under FIFRA section 6(e). Your release for shipment of the product constitutes acceptance of these conditions. A stamped copy of the label is enclosed for your records. Please also note that the record for this product currently contains the following CSFs:

- Basic CSF dated 2/8/2022
- Alternate CSF 1 dated 2/8/2022
- Alternate CSF 2 dated 2/8/2022

If you have any questions, please contact Aidan Fife by phone at 202-566-0822 or via email at fife.aidan@epa.gov.

Enclosure: Stamped Final Label

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

QHA-45-W

ACCEPTED
 06/13/2022
Under the Federal Insecticide, Fungicide and Rodenticide Act as amended, for the pesticide registered under
 EPA Reg. No. 46781-19

[TUBERCULOCIDAL[£]] [BACTERICIDAL] [VIRUCIDAL*] [FUNGICIDAL]

<p>Active Ingredient:</p> <p>Ethanol.....55.0% Didecyl dimethyl ammonium chloride0.3% Alkyl (C₁₄50%, C₁₂40%, C₁₆10%) dimethyl benzyl ammonium chloride0.2%</p> <p>Other Ingredients†.....44.5%</p> <p>Total.....100.00%</p>	<p>KEEP OUT OF REACH OF CHILDREN</p> <p>CAUTION [See other panel for additional Precautionary and First Aid statements.]</p>
<p>EPA Reg. No. 46781-RO</p>	<p>Reorder No:</p>
<p>EPA Est. No.</p>	<p>Net Contents: #[[lbs.] [oz]]</p>
<p>[insert EPA Est. No of Producer List on Box 2 of CSF]</p>	<p># Pre-Saturated Towelette # x # in.</p>
<p>8251-WI-003 [(A1)], 8251-WI-004 [(A2)]</p>	
<p>8251-WI-005 [(A3)], 8251-OH-001 [(A4)]</p>	<p>[Packed For:]</p>
<p>46781-MI-001 [(A5)], 56952-WI-001 [(A6)]</p>	<p>[Distributed By:]</p>
<p>2686-WI-008 [(A7)], 71979-SC-001 [(A8)]</p>	<p>[Sold By:]</p>
<p>75272-WI-001 [(A9)], 93725-TN-001 [(A10)]</p>	<p>[Manufactured By [or for]:]</p>
<p>0312-WI-3 [(A11)], 51435-WI-004 [(A12)]</p>	<p>[Manufacturer:]</p>
<p>56952-CHN-001 [(A13)], 90532-WI-002 [A14]</p>	
<p>90532-WI-003 [(A15)], 90532-WI-001 [(A16)]</p>	
<p>98377-IL-001 [(A17)]</p>	
<p>[Beginning of batch code indicates Est No.] [End of batch code indicates Est No.] [AXX code indicates Est No.]</p>	<p>Metrex Research 28210 Wick Road Romulus, MI 48174</p>
<p>Made in USA</p>	<p>[To reorder] [,] [For product or technical information] contact Metrex at 800-841-1428 or visit our web site at www.metrex.com</p>
	<p>[For SDS information, visit www.metrex.com]</p>

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

PRECAUTIONARY STATEMENTS
Hazards to Humans and Domestic Animals

CAUTION

Causes moderate eye irritation. Avoid contact with eyes or clothing. Wear protective eyewear. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

PHYSICAL AND CHEMICAL HAZARDS: Flammable. Keep away from heat and open flame.

FIRST AID

If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for treatment advice.

[For Chemical Emergency ONLY (spill, leak, fire, exposure or accident), call CHEMTREC at 1-800-424-9300]

For [ALL] [other] inquiries about this product, call Metrex Customer Care at X-XXX-XXX-XXXX, Monday through Friday, between X[insert any hour] a.m. and X[insert any hour] p.m. Pacific Time.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

Packaging Statements and Wipe Dispensing Instructions:
[Use A, B, or C as applicable depending on the wipe container/dispenser type.]

[A – Refillable, Non-refillable canister, buckets, tubs or pails]

[To dispense towelettes] [container] [canister] [pail] [bucket] [tubs] [Dispensing Directions]:

[To start feed, remove cover and discard seal. From center of towelette roll, pull up a towelette corner, twist it into a point and thread it through the hole in the container cover. Pull through about one inch. [Replace cover.] Pull out first towelette and tear off at [(any angle)] [45°-90° angle]. [Pull Wipe at [(any angle)] [45°-90° angle]]. Remaining towelettes feed automatically, ready for next use. [When not in use, keep center cap closed to prevent solution loss.]

[To refill] [canister] [bucket] [pail] [to be refilled by user]: Place refill [pouch] [bag] in bucket [canister]. Remove the lid from the [pail] [bucket] [canister]. [Use scissors] [to] [Cut] [Rip] [Tear] [Pull] [Open] refill pouch [bag]. Place [put] [insert] refill pouch into [bucket] [pail] [canister]. DO NOT remove wipe roll from refill pouch [bag]. From center of towelette roll, pull up a towelette corner, twist it into a point and thread it through the hole in the [bucket] [pail] [container] cover. Pull through about one inch. [Place [lid] [cover] on [canister] [bucket] [pail]]. [Replace cover.] Pull out first towelette and tear off at [(any angle)] [45° – 90° angle]. [Pull Wipe at [(any angle)] [45°-90° angle]]. Remaining towelettes feed automatically, ready for next use. When not in use, keep center cap closed to prevent solution loss.]

[B – Flat Pack]

[To dispense towelettes] [from flat pack:] [from Flat Pack wipes container:]

[Grasp the white tab that states "PULL HERE". Gently swing the lid open [peel back lid] and grasp a towelette. Remove towelette from package and then swing lid closed [close lid]. Gently tap the lid to ensure seal is made between it and the towelette package.]

[C – Single Use Packet]:

[Tear open packet. Unfold towelette and use as directed. Discard as directed.]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

EMERGING VIRAL PATHOGEN CLAIMS

This product qualifies for emerging viral pathogen claims per the EPA's 'Guidance to Registrants: Process for Making Claims Against Emerging Viral Pathogens not on EPA-Registered Disinfectant Labels' when used in accordance with the appropriate use directions indicated below. This product meets the criteria to make claims against certain emerging viral pathogens from the following viral category:

- Enveloped Viruses
- Large Non-enveloped
- Small Non-enveloped

<i>For an emerging viral pathogen that is a/an...</i>	<i>...follow the directions for use for the following organisms on the label:</i>
Enveloped virus	Feline Calicivirus (ATCC VR-782)
Large, non-enveloped virus	Feline Calicivirus (ATCC VR-782)
Small, non-enveloped virus	Feline Calicivirus (ATCC VR-782) Enterovirus 71, Strain: H (ATCC VR-1432)

[Product name] has demonstrated effectiveness against viruses similar to [name of emerging enveloped/small and large non-enveloped virus] on hard, non-porous surfaces. Therefore, [product name] can be used against [name of emerging enveloped/small and large non-enveloped virus] when used in accordance with the directions for use against (cited above) on hard, non-porous surfaces. Refer to the [CDC or OIE] website at [pathogen-specific website address] for additional information.

[Name of illness/outbreak] is caused by [name of emerging enveloped/small and large non-enveloped virus]. [Product name] kills similar viruses and therefore can be used against [name of emerging enveloped/small and large non-enveloped virus] when used in accordance with the directions for use against (cited above) on hard non-porous surfaces. Refer to the [CDC or OIE] website at [website address] for additional information.

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal.

[Note to Reviewer: Instructions for Residential and/or Non-Residential Use Sites will be used depending on Use Sites mentioned on the Production Labels. Non-Residential Use Sites Disposal Directions may be used when Product is offered for both Residential/ Household use sites and Non-Residential (Professional/ Institutional/ Industrial) use sites.]

[Use A, B or C below as applicable depending on the wipe container/ dispenser type.]

- **[A – Nonrefillable Containers, Canisters, Buckets or Pails]**
- **[B – Refillable Containers, Canisters, Buckets or Pails [to be refilled by the user]]**
- **[C – Flat Packs or Packets]**

[Note to Reviewer: the following Pesticide Storage directions apply to Residential and Non-Residential Use Sites for A, B or C container types above]

PESTICIDE STORAGE: [Store at room temperature] -or- [Store product in a cool [dry] area, [away from direct sunlight and heat] to avoid deterioration.] [When not in use, keep [center cap of lid] [flap] [tab] closed to prevent moisture loss.] [Close lid after use.]

Residential Use Sites:

[A – Nonrefillable Containers, Canisters, Buckets or Pails]

[B – Refillable Containers, Canisters, Buckets or Pails [to be refilled by the user]]

TOWELETTE DISPOSAL: Dispose of used towelette in trash after use. [Do not re-use towelette.] [Do not flush [in toilet].]

[Refillable] Container Handling: Refillable Container [with this brand name pesticide only]. Do not reuse or refill except as described in direction for use. Refill only with this product. Offer for recycling. If recycling is not available, discard in trash.

[C – Flat Packs or Packets]

TOWELETTE DISPOSAL: Dispose of used towelette in trash after use. [Do not re-use towelette.] [Do not flush [in toilet].]

Container Handling: Nonrefillable container. Do not reuse or refill this container. Wrap container and put in trash or offer for recycling if available.

Non-Residential Use Sites:

[A – Nonrefillable Containers, Canisters, Buckets or Pails]

[C – Flat Packs or Packets]

PESTICIDE DISPOSAL: To avoid waste, use all material in this container by application according to label directions. If wastes cannot be avoided, offer remaining product to a waste disposal facility or pesticide disposal program. [Do not re-use towelette.] [Do not flush [in toilet].]

Container Handling: Nonrefillable container. Do not reuse or refill this container. Offer for recycling if available or dispose of in a sanitary landfill or by other procedures approved by state and local authorities.

[B – Refillable Containers, Canisters, Buckets or Pails [to be refilled by the user]]

PESTICIDE DISPOSAL: To avoid waste, use all material in this container by application according to label directions. If wastes cannot be avoided, offer remaining product to a waste disposal facility or pesticide disposal program. [Do not re-use towelette.] [Do not flush [in toilet].]

Container Handling: Refillable Container [with this brand name pesticide only]. Do not reuse or refill except as described in direction for use. Refill only with this product. Offer for recycling if available or dispose of in a sanitary landfill or by other procedures approved by state and local authorities.

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

[FOR CANISTERS, FLAT PACKS, REFILLS AND SINGLE USE WIPE PACKETS]

[Optional Texts for SINGLE USE WIPE PACKETS: [Refer to Box Label for Full Use Directions.]]

To [Clean and] Disinfect [and Deodorize] Hard, Non-Porous Surfaces:

Remove visible filth and soil [loads] [prior to disinfecting]. To disinfect hard, non-porous surfaces. Use [enough] [Product Name] wipes to treat the surface, allow surface to remain visibly wet for 1 minute or 60 seconds [contact time] [dwell time]. [Rinse.] [Let air dry]. [If streaking is observed, wipe with a clean, damp [cloth or] paper towel after 1-minute or 60-second contact time [dwell time] has expired.] [Use 1-minute or 60-second contact time [dwell time] for *Mycobacterium bovis* [(BCG)].] [Use 1-minute or 60-second contact time to disinfect against SARS-CoV-2*^B on hard, nonporous surfaces.] [To kill SARS-CoV-2*^B on hard, nonporous surfaces, use 1-minute or 60-second contact time.]

[The following HIV-1 / HBV / HCV instructions are to be included on production labels when claims against HIV-1 or HBV or HCV are listed:]

(Y) This product effectively kills [Human Immunodeficiency Virus* Type 1 (HIV-1),] [Hepatitis B Virus* (HBV),] [[and] Hepatitis C Virus* (HCV)] on pre-cleaned hard, nonporous environmental surfaces/objects previously soiled with blood/body fluids in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood/body fluids, and in which the surfaces/objects likely to be soiled with blood/ body fluids can be associated with the potential for transmission of [HIV-1,] [HBV,] [[and] [HCV].

Special Instructions for Cleaning and Disinfection Against [HIV-1,] [HBV,] [[and] HCV] of Surfaces or Objects Soiled with Blood or Body Fluids:

Personal Protection: When handling items soiled with blood or body fluids, wear [appropriate] barrier protection: [such as] disposable protective gloves, gowns, face masks and eye coverings.

Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfecting towelette. A [Product Name] towelette can be used for this purpose.

Contact Time: Thoroughly wet surface with a new [Product Name] towelette. Ensure surface remains visibly wet for 1 minute or 60 seconds.

Disposal of Infectious Materials: Blood and other body fluids must be autoclaved and disposed of in addition to used wipe in accordance with Federal, State or local regulations for infectious materials disposal.

For use as a Disinfectant on Hard, Non-Porous Surfaces of Non-Critical Medical Devices, Instruments and Implements:

Instruments must be thoroughly cleaned to remove visible soils, rinsed and dried. Visibly clean and rinse lumens of hollow instruments. Then visibly treat [wet] the instruments with the [Product Name] product towelette. Ensure surface remains visibly wet for 1 minute or 60 seconds. [Use 1-minute or 60-second contact time [dwell time] for *Mycobacterium bovis* [(BCG)].] [Use 1-minute or 60-second contact time to disinfect against SARS-CoV-2*^B on hard, nonporous surfaces.] [To kill SARS-CoV-2*^B on hard, nonporous surfaces, use 1-minute or 60-second contact time.]

Wipe with a clean, damp cloth or paper towel and allow to air dry prior to use. Discard wipe after each use.

[Note to Reviewer: Contact time can be presented in 1 [one] min[ute] or 60 sec[onds] on Production Label.]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

DIRECTIONS FOR USE *continued*

[The following direction is to be included when hard, non-porous exterior surfaces of ultrasound transducers or probes are listed as a use site:]

To disinfect external surfaces of ultrasound transducers that do not go into the human body only: Pre-clean ultrasound gel from external transducer to be disinfected [by using a clean washcloth]. Then visibly treat -or- wet the probe with the product [Product Name] towelette. Ensure surface remains visibly wet for 1 minute -or- 60 seconds. [Use 1-minute -or- 60-second contact time [dwell time] for *Mycobacterium bovis* [(BCG)].] [Use 1-minute -or- 60-second contact time to disinfect against SARS-CoV-2*^β on hard, non-porous surfaces.] [To kill SARS-CoV-2*^β on hard, non-porous surfaces, use 1-minute -or- 60-second contact time.]

[The following paragraph will be included in its entirety when the production label references medical devices:]

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream, or normally sterile areas of the body, or (2) contacts intact mucous membranes, but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean critical or semi-critical medical devices prior to sterilization or high level disinfection.

This product may be used to disinfect non-critical medical devices, which come in contact only with intact skin.

[The following statement is to be included on production labels when animal housing facilities are listed: (Including Veterinary Clinics, Farms and Poultry Houses)]

For cleaning and disinfecting the following hard, non-porous surfaces: feeding and watering equipment, utensils, instruments, cages, kennels, stables, catteries, etc.

1. Remove all animals and feeds from premises, animal transportation vehicles, crates, cages etc.
2. Remove all litter, droppings and manure from floors, walls and surfaces of facilities (including pens, chutes, and barns) occupied or traversed by animals.
3. Empty all troughs, racks, and other feeding and watering appliances.
4. Thoroughly clean all surfaces with soap or detergent and rinse with potable water. Ensure surface remains visibly wet for 1 minute or 60 seconds.
5. Wipe all handling and restraining equipment such as forks, shovels, and scrapers used for removing litter and manure.
6. Ventilate buildings and other closed spaces.
7. Do not house animals or employ equipment until treatment has been absorbed, set or dried.
8. Thoroughly scrub treated feed racks, troughs, mangers, automatic feeders, fountains, and waterers with soap or detergent, and rinse with potable water before reuse.

+ For food-contact surfaces, a potable water rinse is required.

[DO NOT USE AS A BABY WIPE.] [Not for Use On Skin] [DO NOT FLUSH [WIPE]] [DO NOT RE-USE WIPE] [SINGLE USE ONLY]

[Not for cleaning skin.] [Do not use as a diaper wipe or for personal cleansing.] [APPLY CAREFULLY] [Refill only with this product.] [Always refer to manufacturer's care instructions before using on equipment -or- devices.]

[Note to Reviewer: Contact time can be presented in 1 [one] min[ute] or 60 sec[onds] on Production Label.]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

DISINFECTION [Organism List]

[[Product Name] [Disinfecting Towelettes] are non-woven disposable towelettes pre-saturated with the [Product Name] solution.]

[[Product Name] [Disinfecting Towelettes] are multi-purposed [disinfectant]/ [cleaning] wipes for use on hard, non-porous inanimate surfaces.]

[[Product Name] [This product] is effective against the following microorganisms on hard, non-porous surfaces when used as directed:]

[[Product Name] [This product] has demonstrated effectiveness against the following organisms with a 1-minute or 60-second contact time [dwell time]. [Use 1-minute -or- 60-second contact time [dwell time] for SARS-CoV-2*^β on hard, non-porous surfaces.]

[Disinfects (*insert organisms from Organism List*) on hard, non-porous environmental surfaces listed on this label]

[When used as directed, [this product] [Product Name] provides effective Bactericidal / Virucidal* / Fungicidal / Tuberculocidal^ε activity against the following organisms:]

Pathogen:	Bacteria	Viruses*	Bloodborne Pathogens**	ESKAPE Pathogens***	<i>Mycobacterium bovis</i> (BCG)	Fungi	SARS-CoV-2* ^β
Contact Time:	1 min[ute]	1 min[ute]	1 min[ute]	1 min[ute]	1 min[ute]	1 min[ute]	1 min[ute]

ORGANISMS	[Strain and/or ATCC number]
<i>Mycobacterium</i> [(1 Minute Contact)]	
<i>Mycobacterium tuberculosis</i> var: <i>bovis</i> (BCG) ^ε -or- <i>Mycobacterium bovis</i> (BCG) ^ε [(Causative Agent of Tuberculosis [(TB))]]	[ATCC 35743]
Bacteria [(1 Minute Contact)]	
<i>Pseudomonas aeruginosa</i> ***, ^	[ATCC 15442]
<i>Staphylococcus aureus</i> ***, ^^	[ATCC 6538]
<i>Salmonella enterica</i>	[ATCC 10708]
<i>Escherichia coli</i> -or- <i>E. Coli</i>	[ATCC 11229]
<i>Shigella dysenteriae</i>	[ATCC 11835]
<i>Serratia marcescens</i>	[ATCC 14756]
<i>Bordetella pertussis</i>	[ATCC 12743]
<i>Enterobacter cloacae</i> ***	[ATCC 13047]
<i>Acinetobacter baumannii</i> ***	[ATCC BAA-1709]
Fungicidal [Pathogenic Fungi] [(1 Minute Contact)]	
<i>Trichophyton interdigitale</i>	[ATCC 9533]
<i>Candida albicans</i> [(Resistant to Anidulafungin, Voriconazole, Itraconazole, Fluconazole)] (Note to Reviewer: select or all Antibiotics may be included)	[ATCC 10231]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

ORGANISMS <i>continued</i>	[Strain and/or ATCC number]
[Multi] Drug-Resistant Bacteria [(1 Minute Contact)]	
Vancomycin-Resistant <i>Enterococcus faecalis</i> [(VRE)] ^{***, €}	[ATCC 51575]
[Multidrug-Resistant] [MDR] <i>Enterococcus faecium</i> ^{***, €}	[ATCC 51559]
Methicillin-Resistant <i>Staphylococcus aureus</i> [(MRSA)] ^{***, ^^, €}	[ATCC 33592]
[Community Acquired] Methicillin-Resistant <i>Staphylococcus aureus</i> [(CA-MRSA)] ^{***, ^^, €}	[Genotype USA 400] [(NRS 123)]
Vancomycin Intermediate [Resistant] <i>Staphylococcus aureus</i> [(VISA)] ^{***, ^^, €}	[CDC HIP 5836]
<i>Staphylococcus aureus</i> [Vancomycin Resistant] [(VRSA)] ^{***, ^^, €}	[NARSA] [VRS1]
Linezolid-Resistant <i>Staphylococcus aureus</i> [(LRSA)] ^{***, ^^, €}	[NARSA] [NRS 119]
Carbapenem-Resistant <i>Klebsiella pneumoniae</i> [(CRKP)] ^{***, €}	[ATCC BAA-1705]
Multidrug-Resistant [MDR] <i>Klebsiella pneumoniae</i> ^{***, €} [(Resistant to Ampicillin, Cefepime, Ceftriaxone, Ceftazidime, Ciprofloxacin, Gentamicin, Trimethoprim/Sulfa, Tobramycin)] (Note to Reviewer: select or all Antibiotics may be included)	[ATCC 51503]
Multidrug-Resistant <i>Acinetobacter baumannii</i> [(MRAB)] ^{***, € -or-} Carbapenem-Resistant <i>Acinetobacter baumannii</i> [(CRAB)] ^{***, €}	[ATCC BAA-1605]
NDM-1 [(New Delhi metallo-beta-lactamase 1)] [-Positive] [-Producing] [Carbapenem-Resistant] [Multidrug-Resistant] <i>Enterobacter cloacae</i> ^{***, €}	[CDC 1000654]
ESBL [(Extended-Spectrum beta-lactamase)] [-Positive or -Producing] [Multidrug-Resistant] <i>Klebsiella pneumoniae</i> ^{***, €}	[ATCC 700603]
ESBL [(Extended-Spectrum beta-lactamase)] [-Positive or -Producing] [Multidrug-Resistant] <i>Escherichia coli</i> -or- ESBL <i>E. Coli</i> €	[ATCC BAA-196]
Metallo-beta lactamase -or- MBL [-Positive or -Producing] [Multidrug-Resistant] [Carbapenem-Resistant] <i>Pseudomonas aeruginosa</i> ^{***, ^, €}	[CDC 2012059]
Methicillin-Resistant <i>Staphylococcus epidermidis</i> [(MRSE)] €	[ATCC 51625]
Multidrug-Resistant [MDR] <i>Klebsiella aerogenes</i> [Carbapenem-Resistant] [CRKA] ^{***, €} [(Resistant to Imipenem, Ertapenem, Ceftazidime)] (Note to Reviewer: select or all Antibiotics may be included)	[ATCC BAA-2356]
Multidrug-Resistant [MDR] <i>Pseudomonas aeruginosa</i> ^{***, ^, €} [(Resistant to Ceftazidime, Ciprofloxacin, Gentamicin)] (Note to Reviewer: select or all Antibiotics may be included)	[ATCC BAA-2108]
Viruses [Non-enveloped][:] [(1 Minute Contact)]	
Feline Calicivirus surrogate for Human Norovirus -or- [Human] Norovirus – Feline Calicivirus as surrogate *	[ATCC VR-782]
Feline Calicivirus *	[ATCC VR-782]
[Human] Rotavirus *	[ATCC VR-2018] [Strain WA]
Rhinovirus [Type 1A] *	[ATCC VR-1559] [Strain 2060]
[Human] Adenovirus [Type 5] *	[ATCC VR-5] [Strain Adenoid 75]
Enterovirus [[Type] 71] *	[ATCC VR-1432] [Strain H]
Enterovirus [[Type] D68] *	[ATCC VR-1825] [Strain 14-18953]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

ORGANISMS <i>continued</i>	[Strain and/or ATCC number]
Viruses * [Enveloped][:] [(1 Minute Contact)]	
Herpes simplex virus [Type 1] *	[ATCC VR-733] [Strain F(1)]
Herpes simplex virus [Type 2] *	[ATCC VR-734] [Strain G]
Influenza A virus [(H3N2)] *	[ATCC VR-544] [Strain A/8/68]
Influenza B virus [(H1N1)] *	[ATCC VR-823] [Strain B/5/72]
Parainfluenza virus [Type 3] *	[ATCC VR-93] [Strain C243]
Duck Hepatitis B virus (DHBV) [surrogate for Human Hepatitis B virus (HBV)] *, ** -or- Hepatitis B virus (HBV) - Duck Hepatitis B virus as surrogate *, **	[Strain Grimaud] [HepadnaVirus Testing, Inc.]
Bovine Viral Diarrhea virus (BVDV) [surrogate for Human Hepatitis C virus (HCV)] *, ** -or- Human Hepatitis C virus (HCV) – Bovine Viral Diarrhea virus as surrogate *, **	[Strain NADL] [American BioResearch Lab] [ABL]
Human Immunodeficiency virus type 1 [(HIV-1)] *, **	[Strain IIIB] [ZeptoMetrix]
Severe Acute Respiratory Syndrome-related Coronavirus 2* ^β -or- SARS-CoV-2* ^β	[Strain: 1/2020] [BEI Resources, NR-52281]
SARS-Associated Coronavirus [(SARs-CoV)] *	[Strain CDC 200300592] [Source ZeptoMetrix/CDC]
Measles virus *	[ATCC VR-24] [Strain Edmonston]
Vaccinia virus *	[ATCC VR-119] [Strain WR]
Human orthopneumovirus * -or- [Human] Respiratory Syncytial virus [(RSV)] *	[ATCC VR-26] [Strain Long]
Varicella-Zoster virus [(VZV)] * -or- Human Herpes Virus 3 *	[ATCC VR-916] [Strain Webster]

[Note to Reviewer: Contact time can be presented in 1 [one] min[ute] or 60 sec[onds] on].

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

PRODUCT CLAIMS

[Note to Reviewer: Contact time can be presented in 1 [one] min[ute] or 60 sec[onds]]

- [# pre-saturated towelettes] [X" x Y"]
- [1-minute -or- 60-second contact time [dwell time] for bacteria [,] [or] viruses* [,] [or] [multi] drug-resistant bacteria [,] [or] bloodborne pathogens** [,] [or] ESKAPE pathogens*** [,] [or] Tuberculosis[£] -or- TB[£] [,] [or] SARS-CoV-2*^β on hard, non-porous surfaces]
- [1-minute -or- 60-second contact time [dwell time]]
- [1-minute -or- 60-second contact time [dwell time] against or for TB[£] -or- tuberculosis[£]]
- [1-minute -or- 60-second contact time [dwell time] against/for SARS-CoV-2*^β on hard, non-porous surfaces [in the presence of [5%] soil load]]
- [99.9% -or- 3 [three] log reduction against/for SARS-CoV-2*^β on hard, non-porous surfaces [in [just] 1 minute -or- 60 seconds] [in the presence of [5%] soil load] [in one-step π]]
- [Kills 99.9% -or- 3 [three] log of SARS-CoV-2*^β on hard, non-porous surfaces [in [just] 1 minute -or- 60 seconds] [in one step π] [in the presence of [5%] soil load]]
- [A Metrex Research[®] brand]
- [A Metrex[®] brand]
- [A CaviWipes[™] Brand]
- [A CaviWipes[™] [Brand] [by Metrex [Research]]]
- [A disinfectant cloth for use in ▲ on ◆]
- [[Aids] [Helps] in reducing -or- in the reduction of cross-contamination between treated hard non-porous surfaces]
- [An effective method to clean against odor-causing organisms on hard, non-porous surfaces listed on this label]
- [[A] one-step π cleaner/disinfectant]
- [[A] one-step π [cleaner and] disinfectant [towelette] against SARS-CoV-2*^β on hard, non-porous surfaces]
- [A disinfectant towelette for use on ▲ and ◆ surfaces]
- [After use, just toss wipe(s) away]
- [Always ready-to-use]
- [An effective method to deodorize and disinfect against odor-causing organisms on hard, non-porous surfaces]
- [An effective way to disinfect hard, non-porous environmental surfaces] [listed on this label]
- [A non-woven disposable towelettes for use in ▲ and other critical care areas where reduction of cross-contamination between treated hard non-porous surfaces is required]
- [Antimicrobial]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

PRODUCT CLAIMS continued

- [A solution impregnated in a wiping cloth]
- [[Appropriate] [for] Cleaning Patient Discharge]
- [[Appropriate] for Occupied Room -and/or Terminal Room Cleaning -and/or- Room Discharge Cleaning]
- [[Appropriate] [for] Cleaning and disinfecting dental operatories]
- [[Appropriate] [for] Cleaning and disinfecting medical sites]
- [[Appropriate] [for] Cleaning and disinfecting home]
- [Assures proper dilution strength, product effectiveness and standardizes technique]
- [Bacteria -and/or- virus* -and/or- germ killing [formula]]
- [Bactericide -or- Bactericidal] [in [just] 1 minute or 60 seconds]
- [Bathroom disinfectant wipes]
- [Big [textured] virucidal* [,] [or] bactericidal wipes [,] [or] tuberculocidal[£] [,] [or] [fungicidal] in 1-minute or 60-secondkill time]
- [Bonus]
- [Broad spectrum disinfectant]
- [Built-in [tough] scrubbing fibers to help clean effectively]
- [[Product Name] has been tested for surface safety [,] [or] compatibility on common hard, non-porous surfaces [and equipment] to ensure compatibility with surfaces commonly used in hospital/ medical setting including: [acrylic] [brass] [polycarbonate] [titanium-coated] [medical grade] [stainless steel] [Formica®] [Porcelain] [Plastic] [Glazed Ceramic] [glass] [glass tables] [glazed tile] [Vinyl]]
- [[Product Name] has great surface compatibility with [medical devices] [dental equipment] [♦] [in ▲] when used to clean -or- pre-clean]
- [[Product Name] has great surface compatibility with non-critical, non-semi-critical [medical devices] [dental equipment] [♦] [in ▲] when used to disinfect]
- [[Product Name] kills germs that cause odors] [is effective against odor causing germs]
- [[Product Name] will effectively clean and disinfect when used as directed]
- [Fungicidal -or- Fungicide] [in [just] 1 minute or 60 seconds]
- [Can help reduce cross contamination between treated hard non-porous surfaces]
- [Cleaner and disinfectant [towelette] [wipe]]
- [Cleaning Formula]
- [[Cleans] [and] [disinfects] [♦] [in ▲]]
- [[Cleans] [and] [disinfects] hard non-porous surfaces [on the go]]
- [[Cleans and] disinfects hard non-porous surfaces] [on [high-touch] [frequently used] areas]
- [Cleans, deodorizes and disinfects]
- [Cleans] [disinfects] [Cleaner and disinfectant] [in one] [all in one step]
- [Clean Home]
- [Cleans the toughest soils]
- [Cleaning Wipe[s] [Towel] [s] refill[s][able]]
- [Cleaning Made Easy]
- [[Cleans [and/ or Disinfects] [grocery cart handles or shopping cart handles or grocery cart child seats or shopping] cart child seats]
- [Cleans grocery carts and shopping carts]
- [[Cloth-like texture] [Textured] [strong] [for] [with] scrubbing power]
- [Compatible for use on hard, nonporous surfaces commonly found in [dental] [healthcare] [medical] [settings] such as [♦] [in ▲]]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

PRODUCT CLAIMS continued

- [Contains [number] canisters [plus] [and] [number refills]]
- [Contains (name of active ingredient), the [same] active ingredient used in [Brand name]. This product is [not] manufactured or distributed by ® /or M/]
- [Convenient]
- [[Convenient] [Portable] way to [disinfect] [clean] hard, non-porous surfaces [of ♦] [in ▲]]
- [Color safe]
- [Cuts Cleaning time]
- [Dependable] [and] [Trustworthy]
- [Deep cleaning [formula]]
- [Delivers a clean [▲] you can see and smell]
- [Demonstrates 99.9% -or- 3 [three] log] reduction against SARS-CoV-2*^B on hard, non-porous surfaces [in [just] 1 minute] [in one stepπ] [in the presence of [5%] soil load]]
- [Designed for Home Healthcare -and/or- patients -and/or caregivers]
- [Designed for professional dental settings]
- [Designed -or- Developed for use in Dental [Offices -or- Labs - or Laboratories -or- Operatory -or- Operatories]
- [Designed -or- developed for Home Caregivers]
- [Designed -or- developed for use in [▲] in Home Healthcare -or- Home Care]
- [Delivers consistent [amount of solution] concentration every time [!]]
- [Deodorizes, and kills household germs on hard non-porous surfaces (**Note to Reviewer:** any organism(s) of the following list will be included: Norovirus, Rotavirus, Rhinovirus, Influenza A virus, Influenza B virus, Parainfluenza virus, SARs-CoV-2*^B, *Staphylococcus aureus*, *Salmonella enterica*, *E. coli*)]
- [Deodorizes by killing the germs that cause odors]
- [Deodorize hard, non-porous surfaces] [of ♦] [in ▲]]
- [Designed for the ▲ [industry]]
- [Designed for killing (**insert any organism(s) from Organism List**) [on hard, non-porous surfaces [of ♦] [in ▲]]]
- [Designed for killing SARS-CoV-2*^B on hard, nonporous surfaces [of ♦] [in ▲] [in one stepπ] [in the presence of [5%] soil load] [in [just] 1 minute]
- [[Designed] [for] [with] [the] Environmental Services [in] [mind][!]]
- [Destroys -or- Deactivates 99.9%] [[Helps] Eliminate 99.9%] (**insert any organism(s) from Organism List**) [of ♦] [in ▲]]
- [Destroys -or- Deactivates -or- Disinfects -or- [Helps] Eliminate 99.9% SARS-CoV-2*^B on hard, non-porous surfaces [of ♦] [in ▲] [in one stepπ] [in the presence of [5%] soil load] [in [just] 1 minute]]
- [Disinfectant]
- [Disinfect]
- [Disinfect external surfaces [of ♦] [in ▲]]
- [Disinfect [♦] surfaces]
- [Dispense one wipe at a time]
- [Disposable]
- [Disinfectant cleaner [that cleans, disinfects and deodorizes] in one stepπ]
- [Disinfecting formula]
- [Disinfects] [♦] [in ▲]]
- [Disinfects [and deodorizes] ♦ [in one stepπ]]
- [Disinfecting towelettes]
- [Disinfects and deodorizes in [just] 1 minute] (**insert any organism(s) from Organism List**)
- [Disinfects hard, non-porous surfaces in [just] 1 minute] (**insert any organism(s) from Organism List**)
- [[Disinfects] hard, non-porous surfaces [that people [frequently] touch -or- use]]
- [Disinfects [(**insert any organism(s) from Organism List**)] [in [just] 1 minute]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

PRODUCT CLAIMS continued

- [Disinfects [bacteria] [viruses*] [[multi] drug-resistant bacteria] [bloodborne pathogens**] [ESKAPE pathogens***] [fungi] [in [just] 1 minute]
- [Disinfects [Tuberculosis[£] -or- TB[£]] [in [just] 1 minute]
- [[Disinfects and deodorizes by killing germs and their odors]
- [[Disinfects hard, non-porous surfaces [throughout the ▲]]
- [[Disinfects [as it cleans] [and scrubs] in one step]]
- [Disinfect [the] surface] [in one step]]
- [Does not contain dyes]
- [Does not contain carcinogens] [Is [not carcinogenic] [non-carcinogenic]]
- [DO NOT FLUSH [WIPE DOWN TOILET]]
- [Double sided-scrubbing wipes]
- [Durable, lint-free wipe designed for sensitive equipment surfaces]
- [Convenient to use]
- [Convenient way to clean [and] [disinfect]
- [One-step]] cleaning and] disinfecting against SARS-CoV-2*^β on hard, non-porous surfaces [in 1 minute]]
- [Easy] pop-up-[tub] [canister] [pail] [lid] [tab] [dispenser] [bucket]]
- [Convenient way to disinfect hard, non-porous surfaces [bathrooms] [office] [nursing stations] [high touch areas] [frequently used areas] everyday]
- [[Easy] [convenient] [dispensing] lid [closure] [cap] [package]]
- [Easy drop-in refills]
- [Easy to use]
- [Easy to handle]
- [Easy to refill]
- [Easy to use package [canister] [container] [refill] keeps [your] wipes moist]]
- [Convenient way to [clean] [and] [disinfect] [bathroom] [or exterior toilet surfaces] hard, non-porous surfaces]
- [Convenient disinfecting [throughout the ▲]]
- [Convenient] one-step]] cleaning and disinfecting]
- [Convenient] way to [touch-up] [bathroom] [office] everyday]
- [Convenient way to [clean] [disinfect] [use]]
- [Economical]
- [Economy]
- [Economical] Refill[s] [one] [1] [two] [2] [three] [3] [four] [4] [various numbers] pack]]
- [[Economical] Refills [now] available]]
- [Effective -or- For daily use against (***Insert any organism(s) from Organism List***)]
- [Effective -or- For daily use against Influenza A -and/or- B Viruses]
- [Effective -or- For daily use against SARS-CoV-2*^β on hard, non-porous surfaces [in 1 minute] [in one step]] [in the presence of [5%] soil load]]
- [Effective against bathroom bacteria and viruses* on hard nonporous surfaces (***Note to Reviewer:*** use at least two bacteria and two viruses* from this list: *E. coli*, *Salmonella enterica*, *Staphylococcus aureus*, Herpes Simplex Virus [Type 1 -or- 2], Hepatitis B Virus (HBV), Hepatitis C Virus (HCV), Adenovirus [Type 5], Influenza A -or- B Virus) [in [just] 1 minute] [in the presence of organic soil [(5% blood serum)]]
- [Effective against -or- Kills a range of bacteria including *Staphylococcus aureus*, *Salmonella enterica*, *Pseudomonas aeruginosa* (***Insert any organism(s) from Organism List***)] [in [just] 1 minute]
- [Effectively disinfects [Effective] against (***Insert any organism(s) from Organism List***)] [in [just] 1 minute]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

PRODUCT CLAIMS continued

- [[Effective against -or- Kills [in the presence of [organic soil load] [(5% blood serum)]]
- [[Effectively] disinfects hard, non-porous [environmental] surfaces]
- [Effectively removes everyday soils]
- [[Efficacy tests have demonstrated that] [Product Name] [this product] [are] [is] an effective bactericide, virucide*, tuberculocide² in the presence of organic soil load [5% blood serum]]
- [Eliminates odors [from fecal matter and urine]]
- [Eliminates airborne particles associated with sprays]
- [Eliminates [daily] mixing and measuring]
- [Eliminates [lingering] [food] odors on hard, non-porous surfaces]
- [[Eliminates -or- Reduces odors [at their source] [caused by germs -or- bacteria]]
- [EVS [Environmental Services] [use]]
- [Exhibits effective [bactericidal] [virucidal*] [tuberculocidal²] activity]
- [Extra large] [disinfecting] [Y" x Z" compared to our A" x B" size] [wipes] [Disinfecting Towelettes]
- [Kill[s] -and/or- Effective against [99.9%] (*insert any organism(s) from Organism List*) [on environmental -and/or- hard, non-porous surfaces]]
- [Kill[s] -and/or- Effective against [99.9%] Influenza A -or- B viruses [(which may cause cold -and/or- flu)] in [just] 1 minute]
- [Help[s] reduce[s] cross-contamination of [99.9% of] [clinically relevant] [bacteria] [pathogens] [organisms] -and/or- viruses* -and/or- germs between treated hard, non-porous surfaces [in your ▲] [in this cold -and/or- flu season]]
- [Kill[s] -and/or- Help[s] reduce[s] cross-contamination of -and/or- Effective against SARS-CoV-2*² on hard, non-porous surfaces] [in [just] 1 minute] [in one stepπ] [in the presence of [5%] soil load]]
- [For [daily] [general] use]
- [For [daily] [general] use on hard nonporous surfaces]
- [For more information visit us at www.metrex.com -or- www.kerrdental.com]
- [For use to disinfect hard nonporous bathroom -and/or- nursery surfaces]
- [For use to clean bathroom -and/or- nursery surfaces]
- [For use [in ▲] [on ♦ surfaces]]
- [For use in critical care areas where reduction of cross contamination between treated hard non-porous surfaces is of prime importance]]
- [For surface cleaning -and/or- deodorizing -and/or- disinfecting]
- [For Use by Home [Caregivers] [Healthcare]]
- [Formulated -or- Designed to remove[s] biological odors including Urine -and/or- Feces -and/or- Vomit]
- [Formulated to kill (*insert any organism(s) from Organism List*) [on hard, non-porous surfaces] [when used according to instructions]]
- [Fragrance Free] [Free of fragrance]
- [Freshens]
- [From CaviWipes, with more than XX [insert number] years of experience]
- [[Fungicidal disinfectant] [against *T. interdigitale* -and-or *Candida albicans*]]
- [Compatible with Furniture]
- [Germs are no match with [Product Name]]
- [[A] one-stepπ Germicidal [disposable] wipes]
- [Germicide -or- Germicidal]
- [Gets rid of [eliminates] odors]
- [Great for [everyday] [daily] [general] use]
- [[Heavy] cloth-like towel]
- [[Helps] [kill][s] [eliminate][s] odor-causing bacteria]
- [Helps prevent -or- reduce cross contamination between treated hard non-porous surfaces]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

PRODUCT CLAIMS continued

- [Improved canister and lid]
- [Improved dispensing system]
- [Includes ___ count refill [pack] [contains] ___ wipes]
- [Institutional [size]]
- [[Just] [Simply] wipe [exterior toilet surface] to kill -or- eliminates 99.9% bathroom germs on hard nonporous surfaces (**Note to Reviewer:** use at least two bacteria and two viruses* from this list: *E. coli*, *Salmonella enterica*, *Staphylococcus aureus*, Herpes Simplex Virus [Type 1 -or- 2], Hepatitis B Virus (HBV), Hepatitis C Virus (HCV), Adenovirus [Type 5], Influenza A -or- B Virus) [in [just] 1 minute] [in the presence of organic soil [(5% blood serum)]]
- [Just [pull] wipe and toss]
- [Kills -and/or- inactivates [(**Insert any organism(s) with from Organism List**)] [in [just] 1 minute] on hard, non-porous surfaces]
- [Kills [the] SARS-CoV-2*^β on hard, non-porous surfaces [in [just] [1 minute] [in one step]] [in the presence of [5%] soil load]]
- [Kills [bacteria] [viruses*] [germs] [fungi] [clinically relevant organisms [pathogens]] on hard, non-porous surfaces]
- [Kill germs and bacteria instead of paper towels, sponges and rags [Use this product to]]
- [Kills -or- Destroys [odor causing bacteria]]
- [Kills -or- Disinfects [Human] Norovirus [Feline Calicivirus]] [in [just] 1 minute]
- [Kills -or- eliminates 99.9% -or- Effective against [bacteria] [germs] [viruses*] [fungi] [clinically relevant organisms [pathogens]]
- [Kills -or- Disinfects -or- Effective against [bacteria] [and] [germs] [and] [viruses*] [and] [[multi] drug resistant bacteria] [and] [ESKAPE pathogens***] [and] [fungi] [and] [clinically relevant organisms] [in [just] 1 minute] [on hard, non-porous surfaces]]
- [Kills -or- Disinfects -or- Eliminates -or- [Is] Effective against [Feline Calicivirus] [[Human] Norovirus] [Tuberculosis[£] -or- TB[£]] [(**Insert any organism(s) from Organism List**)] [in [just] 1 minute] [on hard, non-porous surfaces]]
- [Kill [Effective against] bacteria in 1 minute] [Kill [Effective against] viruses* in 1 minute] [Kill [Effective against] HIV-1 in 1 minute] [Kill bloodborne pathogens** in 1 minute] [Kill ESKAPE pathogens*** in 1 minute] [Kill [Effective against] SARS-CoV-2*^β on hard, non-porous surfaces in 1 minute]
- [[Kills] [Effective against] Tuberculosis[£] -or- TB[£] in [just] 1 minute]
- [Kill [Effective against] [fungi] [pathogenic fungi] in [just] 1 minute]
- [Kills -or- Destroys -or- Disinfects -or- Eliminates 99.9% -or- Effective against SARS-CoV-2*^β on hard, non-porous surfaces [in [just] 1 minute] [in one step]] [in the presence of [5%] soil load]]
- [Kitchen+ and Bath Wipes]
- [Large[r] Surface[s]]
- [Large[r] [Wipe][s]]
- [Larger wipe covers [more surface area] larger area [with less effort] [!]]
- [Leaves [bathroom] [office] [surfaces] [▲] [◆] fresh and clean [and disinfected (on hard nonporous surfaces)] [and smelling fresh] in [just] 1 minute]
- [Leaves no [sticky] [active] residue, reduce environmental impact [sustainable]]
- [Less] [Reduced] [packaging] [less] [to throw away]
- [Lightly scented -or- quick-dissipating scent]
- [Low linting [Lint free] [No lint] [wipe]]
- [Longer lasting [canister] [flatpack] with X% more wipes than regular CaviWipes]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

PRODUCT CLAIMS continued

- [Registered to disinfect hard nonporous surfaces in healthcare facilities [intended for Healthcare Workers]]
- [Made to reduce the risk of cross contamination between treated hard non-porous surfaces]
- [Making home healthcare [manageable] [convenient]
- [Maximum cleaning, minimum effort]
- [Meets surface disinfection recommendations from OSA Bloodborne Pathogen Standard]
- [Moist wipes]
- [Moisture lock lid]
- [More wipes per canister provides more [ecofriendly] [environmentally friendly] packaging]
- [More wipes in the same canister]
- [Multi-pack]
- [Multi-purpose disinfectant]
- [Multi [-refill] [pack]]
- [Multi-surface]
- [Next wipe pops up automatically [everytime] [!]]
- [Neutralizes odors [doesn't [just] mask them]]
- [No canister needed]
- [No [pinched fingers] [rips] [tears]]
- [No sponges, no rags, no buckets, no mess]
- [No threading [required]]
- [No fragrance added]
- [No measuring] [No mixing] [required]
- [Non-scratching [wipe]]
- [Non-woven disposable cloth]
- [Now] [Reinforced] [With] [Quilted] [strength]
- [Now with [%] more towel]
- [Now with [#] more wipes [per canister]] [220 ct. canister provides X% less waste per canister]
- [[Occupied Room] [and] [Terminal Room] [Discharge] Cleaning -and/or- Disinfecting Wipe[s] [Towel][s] Refill[s][able]]
- [Odor [elimination] [neutralization]]
- [On the go disinfectant on hard nonporous surfaces]
- [[Package] [insert type of package] made from [x%] recycled materials]
- [One step cleaning] [disinfecting] [wipe]] [in one step]
- [One step cleaning and disinfecting [saves time]]
- [One step cleaning [and/or disinfecting] [and/or deodorizing] disinfect and/or touch-up clean] [cleans and/or disinfects and/or deodorizes] [anytime] [daily] [every day] [in one step] on hard, non-porous of [on ▲] [bathroom and/or kitchen+] surfaces [on household surfaces] [on ♦ surfaces]]
- [One step disinfectant cleaner [that is] effective against a broad spectrum of bacteria -and/or- viruses* when used as directed]
- [One step germicide] [bactericide] [virucide*] [fungi]
- [One step disinfectant cleaner designed for [general] cleaning and disinfecting on hard, non-porous environmental surfaces in ▲]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

PRODUCT CLAIMS continued

- [One [wipe] at a time dispensing]
- [Other -or- Additional CaviWipes products available for your facility's disinfectant needs]
- [Package] [Insert type of package] made from [x%] recycled materials]
- [Demonstrates effectiveness against SARS-CoV-2*^β on hard, nonporous surfaces [in the presence of [5%] soil load] [in 1 minute]]
- [Picks up [particles] [dust] [dirt] [hair] [crumbs]]
- [Pop-up [dispensing] [top] [tab] [lid]]
- [Portable]
- [Pouch[es]]
- [Powers through tough messes]
- [Pull n/ Go [pack]]
- [Pull, Clean, and Disinfect [in] [only] [just] [1 minute]] [as a [virucide] [bactericide] [tuberculocide^ε] [fungicide]]
- [Premixed], [premeasured], [premoistened] [towelettes]
- [Premoistened [wet] household cleaning wipe]
- [Pre-measured to have the recommended amount of germicidal solution, wipe after wipe]
- [Pre-moistened [convenient] way to disinfect]
- [Pre-moistened with specially formulated cleaners to handle daily clean-ups without bottles, sponges or buckets]
- [Prevents formulation errors]
- [[Product Name] are pre-moistened with a stabilized [Product Name] [disinfectant] solution and detergent solution and are ready- to-use.]
- [[Product Name] are useful in a variety of [healthcare], [medical], [dental], [nursery/daycare] [and] [▲] applications listed on the [this] label.]
- [Pseudomonacidal[^]] [in [just] 1 minute]
- [Ready-to-use [one-step^π] disinfectant [disinfection] for [killing] SARS-CoV-2*^β on hard, non-porous surfaces]
- [[Ready-to-use] disinfectant [for use in ▲]
- [Intended -or- Designed for cleaning [and disinfecting] ♦]
- [Intended -or- Designed for disinfecting exterior toilet surfaces -or- around the exterior toilet surface]
- [Recyclable] [package]
- [Reduces [packaging] waste [by X%]] [Get [X%] more wipes] [when compared to regular canister or bucket or pail or flat pack]
- [Reduces [risk of] cross-contamination [from (*insert any organism(s) from Organism List*)] [in [just] 1 minute] between treated hard non-porous surfaces]
- [Refill package contains [one] [1] [two] [2] [three] [3] [four] [4] [various numbers] [wet] [pre- moistened] [wipe] [refill(s)] [roll(s)]]
- [Refill [size] [pack] [value pack]]
- [Refillable [with ____# wipes] [dispenser] refill [pack] [for dispenser]]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

PRODUCT CLAIMS continued

- [Requires no mixing or measuring and are ready-to-use]
- [Re-sealable [stay-moist] pouch [package] [container] inside]
- [Re-use your [empty] canisters [again [and again] [!]]
- [Reuse[able]] [Container] [Dispenser] [Canister] [Pail] [Bucket]
- [Sample]
- [Scrubbing power of a sponge in a disinfecting [towelettes]]
- [Scrubs] (&) [cleans]
- [Shelf-stable] [formula]
- [So] [surfaces are more than [just] [clean] [they're] [disinfected]]
- [Specifically -or- Specially designed for the Medical/ Dental/ Veterinary/ Institutional/ Home Health Care [▲] industry]
- [Specifically -or- Specially designed [for] [with] [dentists] [hygienists] [patients] [in mind]]
- [Staphylocidal^^] [in [just] 1 minute]
- [Starts cleaning on contact]
- [Strong, durable, low-lint [lint free] [cloth-like] wipe [designed for sensitive equipment surfaces]]
- [Strong, durable textured wipe [cleans] [and] [deodorizes] [and] [disinfects]]
- [[Strong,] [Durable,] non-woven disposable cloth [wipe]] [does not leave lint on surfaces]
- [[Suitable] [Compatible] for [▲]]
- [[Suitable] [Compatible] for use on [♦]]
- [Suitable for use on hard, non-porous surfaces such as [acrylic] [brass] [polycarbonate] [titanium-coated] [medical grade] [stainless steel] [Formica®] [Glazed Porcelain] [Plastic] [Glazed Ceramic] [glass] [glass tables] [glazed tile] [Vinyl]]
- [[Suitable] [Compatible] for use on hard, non-porous inanimate environmental surfaces]
- [Strong cleaning power]
- [[Strong] [Sturdy] canister lid's hinge]
- [Sturdy] [Container] [Dispenser] [Canister] [Pail] [Bucket]
- [Tear at perforation]
- [Textured for better [cleaning power] [scrubbing power]]
- [Textured design allows for] [scrubbing] [cleaning] [removing] [wiping] [tough] [stains] [soils] [messes]
- [Textured wipe for effective cleaning]
- [[Textured] wipe is low-linting [lint-free] [designed] [for sensitive equipment surfaces]]
- [[Textured design [cleans] [deodorizes] hard, non-porous surfaces] [helps remove soil from surfaces]]
- [Textured] [strong] [towelettes] [for] [with] [scrubbing power]
- [The 1-minute [germicidal wipe] [disinfectant wipe] [bactericidal wipe] [virucidal* wipe]]
- [The 1-minute wipe against (*insert any organism(s) from Organism List*)]
- [The answer to your [disinfecting] [cleaning] needs]
- [[The] smell of clean -or- Clean smell]
- [[The] convenient way to disinfect] [with [Product Name]]
- [[This product] reduces cross-contamination [of germs] between hard, non-porous treated surfaces]
- [This product has demonstrated effectiveness against SARS-CoV-2*^β in 1 minute on hard, non-porous surfaces [in the presence of [5%] soil load] [in one-step]]
- [[Thick], [durable] [and] [heavy duty] wipes [that are] [bactericidal -and/or- virucidal* -and/or- in 1 minute] [tuberculocidal^ε in 1 minute]]
- [Thick, large wipe [that] [cleans] [disinfects] [deodorizes]]
- [[This] [outer] [selling] [package] [container] [canister] contains X% post-consumer [material] [plastic] [resin]]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

PRODUCT CLAIMS continued

- [Time to reorder]
- [Trial] [Pack] [Canister]
- [Tub with pop-up action]
- [To break down -and/or kill -and/or- inactivate (*insert any organism(s) from Organism List*)] [on surfaces] [on hard, non-porous surfaces].
- [To kill (*insert any organism(s) from Organism List*)], let stand [undisturbed] for 1 minute]
- [To kill SARS-CoV-2*^β on treated hard, non-porous surfaces [,] [let stand [undisturbed] for 1 minute]
- [To reduce [cross-contamination of] SARS-CoV-2*^β between treated, hard, non-porous surfaces [of ♦] [in ▲]]
- [To reduce cross-contamination between treated, hard non-porous surfaces of dental impression trays for both dentist office personnel and dental laboratory personnel.] (**Note to Reviewer: Dental impression trays do not go into the human mouth.**)
- [Try our other CaviWipes products for your disinfectanting needs]
- [[Tub] [Pail] [Bucket] refill] [available]]
- [Twin pack]
- [Two in one action] [cleans and disinfects in one step]
- [Tuberculocide^ε -or- Tuberculocidal^ε] [in [just] 1 minute]
- [Usable on variety of soils]
- [Unobstrusive smell -or- odor -or- low odor]
- [Use in [public] [hard nonporous surfaces of common places] [▲] where [bacteria -and/or- viruses*-and/or- germs may be of concern]]
- [Use to] [Clean -and/or Disinfect -and/or- Deodorize hard, non-porous surfaces]
- [[Use to] refill[s] [count] [wipe] [canisters] [tubs] [bucket] [tub] [pail]]
- [Use to pre-clean critical or semi-critical medical devices prior to sterilization / high-level disinfection]
- [Use wipes to kill germs -and/or- bacteria -and/or- and viruses* instead of paper towels, sponges and rags]
- [Use this product to] [Kill bacteria in 1 minute] [Kill viruses* in 1 minute] [Kill [pathogenic] fungi in 1 minute] [Kill TB^ε in 1 minute] [Kill HIV-1 in 1 minute] [Kill clinically relevant organisms [pathogens]] [Kill ESKAPE pathogens*** in 1 minute] [Kills bloodborne pathogens** in 1 minute] [Kill SARS-CoV-2*^β in 1 minute on hard, non-porous surfaces]
- [Use where reduction of cross-contamination between treated hard non-porous surfaces is of prime importance]
- [Use on hard, non-porous surfaces only: [acrylic] [brass] [polycarbonate] [titanium-coated] [medical grade] [stainless steel] [Formica®] [Glazed Porcelain] [Plastic] [Glazed Ceramic] [glass] [glass tables] [glazed tile] [Vinyl]] [Test wipe on small inconspicuous area first]
- [Value [Canister]]
- [Versatile]
- [Virtually [Lint Free] [residue free] [streak free] [clean] [shine]]
- [Virucide* -or- Virucidal*] [in [just] 1 minute]
- [Virucide* -or- Virucidal* against SARS-CoV-2*^β on hard, non-porous surfaces [in [just] 1 minute] [in the presence of [5%] soil load] [in one-step]]
- [Wet wipes]
- [When used according to label directions, [this product] reduces cross-contamination of (*insert any organism(s) from Organism List*) between treated hard non-porous surfaces]
- [Where control of cross contamination between treated hard non-porous surfaces is of prime importance]
- [Quick Cleaning action -or- activity]
- [Wipe away odors]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

PRODUCT CLAIMS continued

- [Wipes out 99.9%] [Kills] [germs] [bacteria] [viruses*] [while it cleans] [that cause odor]
- [[Wipe] [material] made from [x %] [sustainable] [ingredients] [fibers]] [may only be used when cellulose wipe material used]
- [Wipes away -or- out 99.9%] [Kills] [Eliminates 99.9%] [bacteria] [viruses*] [fungi] [clinically relevant organisms [pathogens]] [germs] [***(insert any organism(s) from Organism List)***] [in [just] 1 minute] on hard nonporous surfaces]
- [Wipes away -or- out 99.9%] [Kills] [Eliminates 99.9%] [household bacteria -or- viruses (**Note to Reviewer:** any organism(s) of the following list will be included: Norovirus, Rotavirus, Rhinovirus, Influenza A virus, Influenza B virus, Parainfluenza virus, SARs-CoV-2*^β, *Staphylococcus aureus*, *Salmonella enterica*, *E. coli*)] [***(insert any organism(s) from Organism List)***] [in [just] 1 minute on hard nonporous surfaces]
- [Wipes away -or- out 99.9%] [Kills] [Eliminates 99.9%] [viruses* and bacteria [including (***insert any organism(s) from Organism List)***]] [in [just] 1 minute]
- [[Wipes away -or- out 99.9%] [Kills] SARS-CoV-2*^β on hard, non-porous surfaces [in one step]] [in the presence of [5%] soil load] [in [just] 1 minute]]
- [X" x Y" Wipes]
- [X% [more] [wipes] [free] [vs. (*insert a Metrex disinfectant wipes*) count] [canisters][packs]]
- [For use on Hard, Non-Porous [Prep Surfaces+] [Surfaces] [Medical Surfaces] [Veterinary Surfaces] [Dental Surfaces] [Institutional Surfaces] [▲ surfaces]].
- [Any hard, non-porous surface where disinfection is required+].

[Note: The following claims can be used only for the first 6 months of the product on shelf]

- [New] [Strong] [Thick] Wipe [!]
- [New] [&] [dispensing] lid [closure] [cap] [package] [dispensing] [better [than before]]
- [New]
- [New] [Product Name] Same [trusted] [great] [reliable] brand
- [New] [Package] [Packaging]
- [New thick and larger embossed wipe cleans more surface area]
- [New Strong durable [wipe]
- [New Thick [wipe] [with] cloth-like texture]
- [New Large] [quilted] wipes]
- [New Thick wipes]
- [Multi-purposeπ [cleaner] [disinfectant] [Multi [-refill] pack] New [large] Thick [textured] wipe]]
- [New] [Big] [Strong] [Thick] [wipe] [!]
- [New] [dispensing] [lid] [closure] [cap] [package] [dispenses]

Alternate Brand Names

- CaviWipes Forte
- CaviWipes PRO
- CaviWipes Zeus
- CaviWipes Purple
- CaviWipes Indigo
- CaviWipes Violet
- CaviWipes Amethyst
- CaviWipes Diamond
- CaviWipes QHA

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

OPTIONAL GRAPHICS

[The following graphics are optional for all containers. They will be affixed to the label/ canister such that they do not interfere with any label text.]



Note to Reviewer: the above Graphic has been added as an Optional Graphic, all the other Graphics remain unchanged.

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

OPTIONAL GRAPHICS

[The following graphics are optional for all containers. They will be affixed to the label such that they do not interfere with any label text.]



On Hard Non-Porous surfaces



On Hard Non-Porous surfaces



On Hard Non-Porous surfaces



Packaging



Packaging



Packaging



Packaging



Packaging



Packaging



Packaging



Packaging



Packaging



Packaging

[Note to Reviewer:

- All ESKAPE graphics will be accompanied by the qualifier “***”
- All MDRO graphics will be accompanied by the qualifier “€”
- All SARs-CoV-2 graphics will be accompanied by the text “on hard nonporous surfaces”
- All ECO FRIENDLY logos will be accompanied by the text “packaging”]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

[Note to Reviewer: following section is optional.]

[CLEANING AND STAIN REMOVAL]

[To clean -and/or- remove stains: Wipe surface with a wipe; [rinse and] let air dry. [If streaking is observed, wipe with a clean, damp [cloth] [or] [paper towel].] **[[Works on] [effective against] [removes the following stains:]]]**

<i>Claims:</i>	<i>Stains/Soils</i>	<i>Stains/Soils continued</i>
<ul style="list-style-type: none">• Cleans• Removes stains• Removes (<i>insert stain(s) / soil(s) shown to the right:</i>)	<ul style="list-style-type: none">• Blood• Body oils• Bodily fluids• Coffee [stains]• Common soils• Dead skin• Defecation• Dirt• Fecal matter• Feces• Fingerprints• Food+ residue[s] -or- soil[s]• Food [stains]• Fruit [stains]• Grease• Grime• Laboratory stains	<ul style="list-style-type: none">• Mildew stains• Mold stains• Mucus• [Other] common soils -and / or- stains• [Other] organic matter• Phlegm• Protein-based matter• Saliva• Spit• Spittle• Sputum• Stool• Throw-up• Tea [stains]• Urine• Vomit

[Note to Reviewer: following section is optional.]

[ALLERGEN DESTRUCTION]

[To [clean and] destroy specified allergens: Wipe surface. Use enough wipes to treat surfaces for 1 minute or 60 seconds. [Use 1-minute or 60-second contact time [dwell time].] [Rinse.] Let air dry. [If streaking is observed, wipe with a clean, damp [cloth or] paper towel after contact time [dwell time] has expired.]

[Claims:]

- [Breaks Down Allergens[!]]
- [Destroys -or- deactivates dust mite matter]
- [[Penetrates and] breaks down dust mite matter -or- particles, cockroach matter -or- particles, and pet dander allergens [found in dust]]

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

Table 1 Medical Use Sites:
[Product Name] are recommended for use on hard, non-porous surfaces in [▲]

3D Imaging Service Centers	Endoscopy Centers	Ophthalmic Facilities -or- Offices
Acute Care Facilities	ENT [(Ear, Nose, Throat)] Facility	Optometric Facilities -or- Offices
Alternate Care Facilities	Eye Care [Facilities] [Centers]	Oral Health Care Facilities
Ambulances	Eye Surgical Centers	Orthodontic Offices
Ambulatory Surgery Centers (ASC)	Forensic Labs	Orthopedic Offices
Animal Research Facilities	Full-Care Nursing Facility	Outpatient Care Facilities
Assisted Living Facility	Gastrointestinal [(GI)] [Labs] [Laboratories]	Outpatient [Surgery] [Facilities] [Centers]
Autopsy Rooms	Gastrointestinal [(GI)] [Centers] [Suites]	Out-Patient Surgical Centers (OPSC)
Bathroom	Hallways	Pathology Rooms
Bioskill Laboratories	Home Care Services	Patient Care Areas
Blood Drawing Centers	Hospice	Patient Care Rooms
Blood Apheresis Facilities	Hospital [Operating Rooms] [Kitchens+]	Patient Rooms
Blood Units	Hydrotherapy [Facilities] [Rooms]	Pediatric Exam Rooms
Burn Center	<i>In vitro</i> Fertilizer Center (IVF)	Pediatric Intensive Care Units (PICU)
Cardiac Intensive Care Unit (CICU)	Instrument Reprocessing [Facilities] [Departments] [Center]	Pediatric Offices
CAT Scan [Lab] [Laboratories]	Intensive Care Units (ICU)	Personal Care Service Facilities
Cath Labs	Isolation Areas	Pharmacy Clean Rooms
Central Service Areas	Isolation Rooms	Phototherapy Rooms
Central Services Department	Labor Units	Physical Therapy Facilities
Central Supply Areas	[Lab] [Laboratories]	Physicians' Offices
Central Supply Rooms	Lasik Eye Centers	Plasma Apheresis Centers -or- Facilities -or- Units
Clean Rooms	Lasik Units	Post [Ops] [Operatory] Rooms
Clinical Areas	Laundry Rooms	Pre [Ops] [Operatory] Areas
Clinics	Life Science Laboratories	Psychiatric Care Facility
Critical Care Units	Long-Term Acute Center (LTAC)	Radiology Department
CS [(Blood Culture)] [Labs] [Laboratories]	Long-Term Care Facilities	Recovery Rooms
Dialysis Centers	Medical [Facilities] [Clinics] [Sites] [Areas]	Rehabilitation Facilities
Dialysis Clinics	Medical Related Facilities	Respiratory Therapy Facilities
Dissection [Labs] [Laboratories]	Medivans	Skilled Nursing Facilities -or- Homes
Doctor's Offices	Medical Intensive Care Unit (MICU)	Environmental Surfaces of Sterile Processing Departments
Donation Centers [blood] [plasma] [semen] [milk] [aphaeresis]	Magnetic Resonance Imaging [(MRI)] [facilities] [laboratories] [labs]	Environmental Surfaces of Sterile Storage Areas

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

Drug Preparation Centers -or- Facilities -or- Rooms	Neonatal Intensive Care Units (NICU)	Surgery Centers Surgery Rooms
Electrolysis Facilities	Neonatal Nursery Areas	Surgical Centers
Emergency Departments	Neonatal Units	Surgical Suites
Emergency Medical Services (EMS) Units	Newborn Nursery	Teaching Hospitals
Emergency Medical Settings	Nuclear Medicine Facilities	Tissue Banks
Emergency Rooms (ER)	Nurses' Stations	Transplant Center
Emergency Vehicles	Nursing [Homes] [Stations]	Trauma Center
Endo Suites	Occupational Therapy [Centers] [Facilities]	University Research Labs
Endoscopy [Centers] [Facilities] [Departments]	Oncology Areas	Waiting Rooms
X-Ray Rooms	Operating Rooms	Wound Center
Extended Care Facilities	Skilled Nursing Facilities	

Table 2 Institutional Use Sites:

[Product Name] are recommended for use on hard, non-porous surfaces in [▲]

Adult School	Early Learning Centers	Middle Schools
Air Transport Facilities	Elementary Schools	Military Installations
Airplane Interiors	[Fire] [Rescue Departments]	Nursery Schools
Airports	Fire Vehicles	Police Vehicles
Bathroom	Forestry Services	Pre-K School [Room]
Casinos	Grammar School	Preparatory School
Child Care Facilities	High School	Preschool
Churches	Hospitality Facilities	Primary School
Colleges	Hotels	Prisoner Detention Facilities
Community Colleges	Industrial Facilities	Prisoner Lockups
Conference [Centers] [Rooms]	Infant Care Centers	Prisons
Conference Rooms Fixtures	Intermediate School	School Buses
Convention Facilities	Jails	School Kitchens+
Correctional Facilities	Junior High School	Schools
Daycare Centers	Kindergarten	[University] [Universities]
[Dormitories] [Dorm]	Learning Institutions	

Table 3 Dental Use Sites:

[Product Name] are recommended for use on hard, non-porous surfaces in [▲]

Dental Laboratories -or- Labs	Dental Office	Dental Operatory -or- Operatories
-------------------------------	---------------	-----------------------------------

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

Table 4 Vet Use Sites:

[Product Name] are recommended for use on hard, non-porous surfaces in [▲]

Animal Care Facilities	Avian Units	Pet Homes
Animal Control Vehicles	Bathroom	Pet Motels
Animal Dental Operatory	Doggy Day Care Centers	Pet Spas
Animal Detention Room	Feline Facilities	Pet Stores
Animal Research Facilities	Feline Units	Small Animal Facilities
Animal Shelters	Kennel	Veterinary Offices
Avian [Care] Facilities	Pet Areas	Veterinary Facilities

Table 5 Food Prep+ Use Sites:

[Product Name] are recommended for use on hard, non-porous surfaces in [▲]

Bakeries+	Food Processing Facilities+	Restaurants+
Banquet Hall Facilities+	Grocery Stores +	School Kitchens+
Butcher Shops+	Kitchen+	Supermarkets+
Food Preparation Areas+	Produce Areas+	

Table 6 Miscellaneous Use Sites:

[Product Name] are recommended for use on hard, non-porous surfaces in [▲]

Airplanes	Facilities	Manufacturing [Facilities] [Plant]
Arboretums	Factories	Money Handling Facilities
Arcades	Foots Spas	Morgues
Athletic -or- Training Facilities	Foreign Current Exchange Facilities	Mortuaries
Auto Facilities	Funeral Homes	Motels
Banks	Gaming Facilities	Movie Theaters
Barber Shops	Garden Centers	Outdoor Playground
Bathroom	General Public Facilities	Personal Care Service Facilities
Beauty Salons	Golf Courses, not for use on grass	Public Areas
Body Piercing Salons	Grounds Keeping Services	Public Facilities
Body Service Facilities	Gymnasium [Gym]	Public Transportation Vehicles
Body Storage Facilities	Health Club Facilities	Public Restrooms
Botanical Gardens, not for use on plants/ grass	Helicopter Interiors	Recreational Centers -or- Facilities
Bowling Alleys	Household Bathrooms	Recycling Centers
Buses	Housekeeping Rooms	Salon Settings
Cabs	Indoor Playground	Shelters
Cadaver Processing areas	Laundromats	Shower Rooms
Cars	Laundry Rooms	Skincare Salons
Centers	Lawn Services	Storage Areas -or- Rooms
Condominiums	Limousine	Subway
Cruise Ships	Living Rooms	Tanning Salons
Currency Exchange	Locker Rooms	Tattoo Parlors
Day Spa	Lockup Facilities	Taxis
Delivery Units	Manicure Salons	Theme Parks
Dissection Work Tables	Massage Facility	Transport Vehicles
Universities	Trade Schools	

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

Table 7 Surfaces:

[Product Name] is recommended for use on hard, non-porous surfaces of [♦].

Ambulance -and/or- Ambulance Equipment Surfaces	Dental impressions [materials] [prior to sending to a dental lab]	
Animal Cages	Cages	Dental Operatory Surfaces
Animal Equipment	Cardiac Gym Equipment	Dental Unit Instrument Trays
External Surfaces of Appliances+	Carts	Dentist Chair
Arm Rests	Cash Register	Desk Tops
ATM Surfaces	Ceilings	Desks
Autoclave Surfaces	Cell Phones	Diaper Changing Stations
Automatic Feeders	Chairs	Diaper Pails
Automatic Teller Machines (ATM)	Changing Tables	Digital Sensors
Baby Warmers	Chemotherapy Hoods	Digital Phosphor Plates
Baked Enamel	Child Seats+	Tables+
Bars	Chrome	Drying Racks+
Basins	Clipper Blades	Display Cabinets
Baskets	Closet Handles	Diving Equipment
Bathing Units	Nonporous Coated Mattresses	Diving Masks
Bathroom Fixtures	Nonporous Coated Pillows	Diving Snorkels
Bathtubs	Combs	Door Knobs
Bed Pans	Computer	Dopplers
Bed Railings [Bedside Rails]	Computer Display	Drain Boards+
Bedside Commode	Computer [Keyboards] [Mouse]	Drained Whirlpool® Tank Surfaces
Behind and Under Sinks	Copy Machines	Enamel
Bidet	Counter Tops+	Endoscope Cabinet
Biohazard Container	Counters+	Equipment
Blood Pressure Cuff	[Baby] Cribs	Equipment Surfaces
Booster Chairs+	Crutches	Examination -or- Exam Table
Booster Seats+	Dental Chairs	External Surfaces of Kendrick Extrication Device (KED)
Brushes	Dental Countertops	Exterior Surfaces of Toilet
Cabinets Handles	Dental Curing Lights [‡]	Exterior or Exposed Interior Surfaces of Medical Equipment
Cabinets	Dental Impression Trays	Bassinets

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

Table 7 Surfaces continued:

Exterior Surfaces of Dental Equipment [‡]	Exterior Surfaces of Vanity Tops+	External Surfaces of Intubation Equipment
Exterior Surfaces of Amalgamators [‡]	External Surfaces of Emergency Resuscitative Equipment	External Surfaces of Isolettes
Exterior Surfaces of Anesthesia Equipment -or- Machines [‡]	External Surfaces of Bladder Scan[ners]	External Surfaces of IV Pumps
Exterior Surfaces of Aphaeresis [Machines]	External Surfaces of Blood Glucose Meters	External Surfaces of Life Support Equipment
Exterior Surfaces of Appliances such as Refrigerators+, Microwaves+, Food Processors+, Stoves+	External Surfaces of Blood Pressure (BP) Monitors	External Surfaces of Loupes
Exterior Surfaces of Center Venous Lines	External Surfaces of CAT Scan Machines	External Surfaces of Medical Equipment
Exterior Surfaces of CICU Equipment	External Surfaces of Compression Devices	External Surfaces of Medical Warmers
Exterior Surfaces of Defibrillators	External Surfaces of Dialogistic Equipment	External Surfaces of Monitors Electrodes
Exterior Surfaces of Dental Care Facilities Hand Pieces	External Surfaces of Dictating Equipment	External Surfaces of MRI Machines
Exterior Surfaces of Dental Equipment Lines	External Surfaces of Digital X-Ray Sensors	External Surfaces of OLED, LED Monitors
Exterior Surfaces of Dental Equipment Tubing	External Surfaces of EKG (Electrocardigram) Machines	External Surfaces of Ophthalmoscopes
Exterior Surfaces of Dental Operatories	External Surfaces of Electronic Thermometers	External Surfaces of Otopscopes
Exterior Surfaces of Diagnostic Equipment	External Surfaces of Electrosurgical Generators	External Surfaces of Patient Care Monitor [Unit]
Exterior Surfaces of Dialysis Machines	External Surfaces of Equipment _u	External Surfaces of Patient Monitoring Equipment
Exterior Surfaces of Endodontic Equipment (Apex Locators, Pulp Testers, Motors)	External Surfaces of Glucometers	External Surfaces of Probes
Exterior Surfaces of Radiology Equipment	External Surfaces of Headsets	External Surfaces of Pulp Tester Motors
Exterior Surfaces of Refrigerator Units	External Surfaces of ICU Equipment	External Surfaces of Pulse Oximeter
Exterior Surfaces of Respiratory Therapy Equipment	External Surfaces of Incubators	External Surfaces of Sequential Compression Devices
Exterior Surfaces of Urinals	External Surfaces of Infusion Pumps	External Surfaces of Respiratory Ventilators

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

Table 7 Surfaces continued:

External Surfaces of Non-Critical Medical Devices	Fire Trucks -or- Vehicles	Handles
External Surfaces of Resuscitators	Floors	Handrails
External Surfaces of Instrument Reprocessors (Note to Reviewer: Product can be used to disinfect external surfaces of Instrument Reprocessors that are not to be sterilized.)	External Surfaces of Food Cases+	Hard Plastic or Metal Flooring
External Surfaces of Suction Pumps	Food Serving Trays+, Countertops+	Hard, non-porous environmental surfaces
External Surfaces of Surgery Microscopes	Furniture	Hard, non-porous environmental surfaces of Vehicles including steering wheel
External Surfaces of Surgery Robots	Formica®	High Chairs+
External Surfaces of Surgical Headlights	Exterior Surfaces of Drinking/ Water Fountains+µ	High Touch/ Frequently Used Areas/ Objects
External Surfaces of Surgical Light Sourcesµ	Exterior Surfaces of Freezers	Hoods+µ
External Surfaces of Tympanic Thermometers	Garbage Cans	Hospital Bed (Railings, Frames)
External Surfaces of Ultrasound Machines	Glass	Hospital Patient Bed
External Surfaces of Ultrasound Monitors	Glazed Ceramic Tile	Infant Incubatorsµ (interior and exterior surfaces)
External Surfaces of Ultrasound Transducers	Glazed Porcelain	Surfaces of Infant/ Child Care Equipment+
External Surfaces of Vaporizer	Glazed Tile Walls	Inside, Outside of Drawers
External Surfaces of Veterinary Equipment	Glazed Tiles	Instrument Stands
External Surfaces of Vital Sign Monitors	Golf Cart	IV [Stands] [Poles]
External Surfaces of X-Ray Machines	Grab Bar	Jacuzziµ
Eye Glasses (do not use on contact lenses)	Grocery Cart	Kennels
Facilities [Surfaces]	Gurneys	Knee Walkers
Faucets	Gym Equipment	Knobs
Fax Machines	Hair Clippers	Laboratories
External Surfaces of Feed Racks+	Hair Cutting Implements	Laboratory Equipment
Filing Cabinets	Hair Dryers	Laboratory Surfaces
Filing Doors	Hair Rollers	Laminated Surfaces
Filing Handles	Hampers, nonporous surfaces	Lampsµ

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operatory(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

Table 7 Surfaces continued:

Light Lens Covers μ	Pipes	Stainless Steel
Lights μ	Plastic Laminate	Stalls
Linoleum	Plastic Mattress Covers	Stands
Manicure Implements	Playpens	Stethoscopes
Manicure Tables	Pneumatic Tourniquets	Stools
Marlite	Police Cars -or- Vehicles	Storage [Units]
Nonporous Mattress Covers	Porcelain Enamel	Exterior Surfaces of Stoves+, μ
Mayo Tray -or- Table	[Plastic] Posey Vest	Exterior Surfaces of Stovetops+, μ
Monitors	Public Restrooms	Stretchers
Nail/Hair Care Implements	Razors	Stretches
Nurse Call Button -or- Device -or- Cord	Reception [Areas] [Counters]	Support Bars
NaugaHyde®	Exterior Surfaces of Refrigerator Carts	Synthetic Marble
Operating Room [Furniture] [Light] [Light Switches] [Tables]	Exterior Surfaces of Refrigerators	Tables+
Operating Tables	Seats	Tanning Beds μ
Outdoor Furniture+ (excluding wood frames and upholstery)	Safety Straps	Telemetry Equipment
Exterior Surfaces of Ovens μ	Salad Bar Sneeze Guards+	Telephones
Overbed table	Salon Surfaces	Tiles+
Oxygen Hoods	School Buses	Toilet above the water line
Patient Monitoring Equipment	Scissors	Toilet (Rim, Seat, Tank, Handle)
Patient Chairs	Shampoo Bowls	Touch Screen Monitors
Patient Delivery Equipment	Shears	Towel Dispensers
Patient Lift Equipment	Shelves	Toys, not for infant/ toddler toys
Patient Scales	Shopping Carts	Transfer Bench
Patient Support Equipment	Shower [Fixtures] [Chair] [Stalls]	Transport Vehicles
Pens	Synthetic Marble	Trash Cans
Phone Chargers	Silk Peel™ with DermalInfusion™	Tray Tables
Phone Cradle	Sinks+	Troughs
Physical Therapy (PT) Equipment surfaces	Slit Lamps μ	Tubing
Wash Basin	Smart Phone Chargers	Vanity Tops+
Washable Nail Files	Spine Back Boards	Veterinary Care Surfaces
Workstations	Tanks	Vinyl
Rubber [material]	Walls	Walkers
Titanium-Coated [material]	Watering Appliances	Wheelchairs
Brass [material]	Weight-lifting Surfaces	Whitening Lamps
Medical Grade [material]	Acrylic [material]	Titanium-Coated [material]
Polycarbonate [material]		

THE INFORMATION IN THIS BOX IS NOT PART OF THE LABEL

Statements in brackets, [], are optional or instructional. Bold, italicized text and italicized words within parentheses are information for the reader and are not included on production labels. Sequence of statements is optional unless specified in 40 CFR 156.10. Variables such as capitalizing, pluralizing, singularizing, bolding, italicizing and abbreviating are variable unless specified in 40 CFR. Lists of uses and use sites may be used partially or in their entirety. The term 'towelette (s)' is interchangeable with wipe(s), towel(s), and cloth(s) and the term 'container' is interchangeable with dispenser(s), bucket(s), pail(s), tub(s) throughout the label. The term center(s), facility(ies), area(s), unit(s), clinic(s), room(s), lab(s), department(s), suites, office(s), operator(ies) in the use site tables are interchangeable as applicable. Section Headers are optional.

[CLAIMS QUALIFIERS]

[Note to Reviewer: the following definition will be included on Production Labels if the qualifiers are used. Qualifier symbols will be added when the specific organism(s), claim(s), use site(s), and/or surface(s) are listed on Product Labels.]

†	Does not include the weight of the towelette.
*	Virus
**	Bloodborne pathogens
***	ESKAPE pathogens
Υ	HIV-1 / HBV / HCV: Follow [the] Special Instructions prior to Disinfection [listed on label].
β	The virus that causes COVID-19.
€	Refer to Multidrug-Resistant Bacteria [listed on label].
π	when used according to disinfection directions.
^	Pseudomonacidal
^^	Staphylocidal
+	For food-contact surfaces, a potable water rinse is required.
¥	For surfaces that are not inserted into the human mouth and body.
μ	Allow surfaces to come to room temperature prior to treatment.
▲	Any use site(s) listed in Tables 1 – 6.
◆	Any surface(s) listed in Table 7.
£	Caused by <i>Mycobacterium bovis</i> (BCG)