



OMNI-II™

E.P.A. Reg. No. 46851-1
Est. No. 46851-MT-01

INSTRUMENT & EQUIPMENT DISINFECTANT

Non-Rusting

Pleasant Fragrance

Active Ingredients

o-phenylphenol
o-Benzyl-p-chlorophenol
Inert Ingredients

9.0%
1.0%
90.0%
100.0%

- Bacteriocidal
- Fungicidal — destroys pathogenic fungi
- Virucidal*
- Tuberculocidal

* Viruses including Herpes virus types 1 & 2, Polio virus types 1 & 2, Vaccinia virus, Influenza virus type A (New Jersey) and Influenza B (Maryland)

KEEP OUT OF REACH OF CHILDREN

DANGER:

See handling precautions and practical treatments on back label

Contents: 1/2 gallon (64 fluid oz.)

Manufactured & Distributed by:
ADM MEDICAL DIVISION
2800 Reserve, Missoula, MT 59801

46851-1

1982

DIRECTIONS FOR USE:

Omni II should be diluted 1:32 with water for disinfection of medical and dental instruments and devices. Rinse thoroughly after immersion. Discard the used solution daily or sooner if it becomes diluted or soiled. IT IS IN VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING.

Clean all excess organic debris from items to be disinfected and rough dry. Immerse in 1:32 concentration of Omni II for 10 minutes to destroy bacteria including *Escherichia coli* 0157, *Staphylococcus aureus*, *Salmonella typhosa*, *Pseudomonas aeruginosa*, *Mycobacterium tuberculosis* (if tuberculosis contamination is suspected, increase immersion time to 20 minutes for added safety), fungi including *Trichophyton interdigitale*, viruses including Herpes virus types 1 & 2, Polio virus types 1 & 2, Vaccinia virus, Influenza virus

type A1 (New Jersey) and Influenza B (Maryland). The ability of this product to destroy the etiologic agent for viral hepatitis has not been established or demonstrated.

For use on metals, plastics, rubber and glass. Glass may etch over prolonged or repeated exposure. Don't store lensed articles in Omni II.

When diluted as directed, this product is stable. All label claims of effectiveness are based on previously unused batches of Omni II diluted as directed. Organic material or water introduced into the diluted product in re-use may affect the product effectiveness.

Omni II should not be used for disinfection of hypodermic needles or any items with deep and/or narrow crevices.

DISPOSAL:

Do not reuse empty container. Rinse container and discard in trash collection.

PRECAUTIONARY STATEMENTS:

Hazard to Humans

DANGER: Corrosive. Causes eye and skin damage. Do not get in eyes, on skin or on clothing. Wear goggles or face shield and rubber gloves when handling. Harmful or fatal if swallowed.

FIRST AID:

In case of contact, immediately flood eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Remove and wash contaminated clothing before reuse. If swallowed, drink promptly a large quantity of milk, egg whites, gelatin solution, or if these are not available, drink large quantities of water. Avoid alcohol. Call a physician immediately.

NOTE TO PHYSICIAN: Probable mucosal damage may contraindicate the use of gastric lavage.

To prepare 1:32 use dilution

WATER	OMNI II
1 pint	1/2 oz (15 ml)
1 quart	1 oz (30 ml)
1/2 gallon	2 oz (60 ml)
1 gallon	4 oz (120 ml)