

46851-1

08/01/2000

1/2

COTTRELL

Omni II

Concentrated Surface Disinfectant and Holding Solution

Tuberculocidal, Virucidal*, Bactericidal, Fungicidal

Recommended for use in Hospitals,
Medical/Dental Clinics, Nursing Homes,
Healthcare Institutions and Veterinary Clinics.

Active Ingredients	
o-phenylphenol	9.0%
o-benzyl-p-chlorophenol	1.0%
Inert Ingredients	90.0%
Total	100.0%

DIN: 02231481

Lof Number is printed on container.

E.P.A. Reg No. 46851-1
E.P.A. Est. No. 37265-CA-01

Contents: 1 Quart
(32 oz. - 946 ml)

Reorder No. COMN32

Keep Out of Reach of Children

See Side Panel for Additional Precautionary Statements

DANGER:
EXTREMELY CORROSIVE



POISON:



ACCEPTED

AUG - 1 2000

Under the Federal Insecticide, Fungicide, and
Rodenticide Act as amended, for this
pesticide, registered under
EPA Reg. No. 46851-1

2/2

COTTRELL

DIRECTIONS FOR USE

IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING.

DANGER:
EXTREMELY CORROSIVE.
POISON. HARMFUL IF
ABSORBED THROUGH
SKIN. Contains O-phenylphenol and O-benzyl-p-chlorophenol. Avoid contact with eyes, skin and clothing. Causes eye and skin damage. Do not get in eyes, on skin or on clothing. Wear protective eyewear and rubber gloves when handling. Harmful if swallowed.

FIRST AID TREATMENT: If swallowed, give a glass of milk. If splashed in eyes or on skin, flush thoroughly with water. Call physician or poison control center immediately.

NOTE TO PHYSICIAN
 Probable mucosal damage may contraindicate the use of gastric lavage.

STORAGE & DISPOSAL
 Keep Omni II covered in storage or use. Do not contaminate water, food or feed by storage or disposal. Do not reuse empty container. Triple rinse container, puncture and dispose of in a sanitary landfill or incinerate. Used disinfectant solutions are toxic. Store, use and dispose of according to label instructions

Thoroughly clean surface of items to be disinfected and rough dry. Omni II is a concentrated solution and must be diluted 1:32 with water prior to instrument soak and/or decontamination of dental and medical instruments or disinfection of hard environmental surfaces. This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection.

Immerse items in diluted Omni II solution for instrument soak and/or decontamination of medical and dental instruments. For disinfection of hard environmental surfaces such as walls, floors and countertops, diluted Omni II should be mopped, sponged, sprayed or wiped in a manner to thoroughly wet the surface. A properly prepared solution of Omni II intended for use as a hard surface disinfectant, has a shelf life of 60 days (use life) when stored in a closed container such as a spray bottle.

A contact time of 10 minutes at 20°C (68°F) with a 1:32 dilution of Omni II has been shown to be effective on bacteria including: Escherichia Coli 0113, Staphylococcus aureus, Salmonella typhosa, Pseudomonas aeruginosa, Mycobacterium tuberculosis; fungi including: Trichophyton mentagrophytes; viruses* including Polio virus 1, Coxsacke B2, Influenza virus type A1 (New Jersey) and influenza B (Maryland), Herpes virus types 1 and 2, Vaccinia virus and HIV-1.

Rinse articles thoroughly following immersion. **Discard the used solution daily or sooner if it becomes diluted or soiled.**

When diluted as directed, this product is stable. All label claims of effectiveness are based on previously unused batches of Omni II diluted as directed. Organic material or water introduced into the diluted product in reuse may affect the product's effectiveness.

For use on metals, plastics, rubber and glass. Glass may etch over prolonged or repeated exposure to concentrated solutions. Do not store lensed articles in concentrated Omni II.

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV (Human Immunodeficiency Virus) ON SURFACES OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS.

PERSONAL PROTECTION: When handling items soiled with blood or body fluids or when conducting procedures which might lead to contact with blood or body fluids, always wear items such as disposable latex gloves, gowns, masks and eye coverings.

CLEANING PROCEDURES: Blood and body fluids must be thoroughly cleaned from surfaces and objects before application of the disinfectant.

DISPOSAL OF INFECTIOUS MATERIAL: Blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

PRECAUTIONARY STATEMENTS


Hazard to Humans. This product must not be used in hospital nurseries.

MADE IN U.S.A. BY:
 COTTRELL, LTD
 7399 SOUTH TULSON WAY
 ENGLEWOOD, CO 80112 USA
 1 800 THE EDGE

CANADIAN AGENT:
 TRANS CANADA DENTAL LTD.
 2891 PORTLAND DRIVE
 OAKVILLE, ONTARIO
 CANADA L6H 5S4
 PHONE: (905) 829-1466

COMNBL01

TO PREPARE 1:32 USE-DILUTION	
Water	ProSpray SC
To Make 1.2 Liter	
484.38 ml.	15.38 ml.
To Make 1 Liter	
968.75 ml.	31.25 ml.
To Make 1.2 Imperial Gallon	
1937.5 ml.	62.5 ml.
To Make 1 Imperial Gallon	
3875.0 ml.	125 ml.

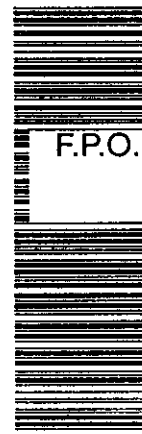

+H816C, 11/11/02/07

ACCEPTED
 AUG - 1 2000
 Under the Federal Insecticide, Fungicide, and Rodenticide Act as amended, for the pesticide, registered under EPA Reg. No. **H6851-1**