

Omni II™

Concentrated Surface Disinfectant
and Holding Solution

Tuberculocidal Virucidal Bactericidal
Fungicidal

Active Ingredients

4,4'-Dichloro-2,2'-dimethyl-2,2'-diphenyl
propane diol
Sodium hypochlorite
Sodium hydroxide

Net Content: 1 Quart
(32 fl. oz. / 946 ml)

Contents: 1 Quart
(32 oz. / 946 ml)
Register No. OMN232

DANGER - Keep Out of Reach of Children

**SPECIAL INSTRUCTIONS FOR
CLEANING AND DECONTAMINATION
AGAINST HIV ON
SURFACES OR OBJECTS
SOILED WITH BLOOD OR
BODY FLUIDS**

PERSONAL PROTECTION:

When handling items soiled with blood or body fluids or when conducting procedures which might lead to contact with blood or body fluids, always wear items such as disposable latex gloves, gowns, masks and eye coverings.

CLEANING PROCEDURES:

Blood and body fluids must be thoroughly cleaned from surfaces and objects before application of the disinfectant.

DISPOSAL OF INFECTIOUS

MATERIAL: Blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

PRECAUTIONARY STATEMENT

Hazards to Humans and Domestic Animals

DANGER: Corrosive. Causes irreversible eye damage and skin burns. Harmful if inhaled, swallowed or absorbed through skin. Do not get in eyes, on skin or on clothing. Avoid breathing spray mist. Wear goggles or face shields (safety glasses), protective clothing (long-sleeve shirt and long pants, sock, plus shoes and chemical resistant gloves such as water proof gloves). Wash hands before eating, drinking, chewing gum, using tobacco or using the toilet. Remove contaminated clothing and wash before reuse.

FIRST AID

IF IN EYES:

Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for treatment advice.

IF ON SKIN OR CLOTHING:

Take off contaminated clothing. Rinse skin immediately with plenty of water for 1-20 minutes. Call a poison control center or doctor for treatment advice.

IF SWALLOWED:

Call a Poison Control Center or doctor immediately for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told by a Poison Control Center or doctor.

IF INHALED:

Move person to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration preferably mouth-to-mouth if possible. Call a Poison Control Center or doctor for further treatment advice.

In case of an emergency for additional information call toll free 800-535-5053. Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

NOTE TO PHYSICIAN

Probable mucosal damage may contraindicate the use of gastric lavage.

DIRECTIONS FOR USE

IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING.

Omni II is a concentrated solution and must be diluted 1:32 with water prior to instrument soak and/or decontamination of dental and medical instruments or disinfection of hard environmental surfaces. This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection.

Soak/Decontamination - Immerse items in diluted Omni II solution for instrument soak and/or decontamination of medical and dental instruments. Rinse articles thoroughly following soak/decontamination. Organic materials or water introduced into the diluted product may affect products effectiveness. Discard the used solution daily or sooner if it becomes diluted or soiled.

Ultrasonic Cleaning - Place instruments into diluted Omni II and activate ultrasonic unit. When instruments are clean, remove and rinse thoroughly. Change solution daily or sooner if it becomes diluted or soiled.

Cleaning - Apply Omni II directly to surface using mop or spray bottle. Wipe surface to remove debris and solution using a clean paper or cloth towel and discard.

Hard Surface Disinfection - diluted Omni II solution should be mopped, sponged, sprayed or wiped onto hard environmental surfaces such as walls, floors and countertops in a manner to thoroughly wet the surface. A properly prepared solution of Omni II intended for use as a hard surface disinfectant, has a shelf life of 60 days (see life) when stored in a closed container such as a spray bottle.

A contact time of 10 minutes at 20°C (68°F) with a 1:32 dilution of Omni II has been shown to be effective on bacteria including: *Escherichia Coli* 0113, *Staphylococcus aureus*, *Salmonella typhosa*, *Pseudomonas aeruginosa*, *Mycobacterium tuberculosis*; fungi including: *Trichophyton mentagrophytes*; viruses* including Influenza virus type A1 (New Jersey) and Influenza B (Maryland), Herpes virus types 1 and 2, Vaccinia virus, and Polio virus 1.

Omni II when diluted 1:32 has been shown to be effective in destroying the HIV-1 (AIDS virus) in 60 seconds at 20°C (68°F). Although efficacy in 60 second contact time has been shown to be adequate against HIV-1, this time would not be sufficient for other organisms. Use 10 minute contact time for disinfection against all the organisms claimed. Effective in waters up to 400 ppm hardness calculated as calcium carbonate.

For use on metals, plastics, rubber and glass. Glass may etch over prolonged or repeated exposure to concentrated solutions. Do not store lensed articles in concentrated Omni II.

MANUFACTURED BY:
COTTRELL INTERNATIONAL, LLC a Division of
CERTOL
INTERNATIONAL, LLC
5450 LOUIET STREET
DENVER, CO 80239 USA
1 800 THE EDGE
(1.800.843.3343)



ACCEPTED
JUN 19 2003
Under the Federal Insecticide, Fungicide, and
Rodenticide Act as amended, for the
pesticide, registered under
EPA Reg. No. 46851-1

TO PREPARE 1:32 USE-DILUTION	
WATER	OMNI II
To Make: 1 Pint	
15 1/2 fl. oz.	1/2 fl. oz. (15 ml.)
To Make: 1 Gallon	
31 fl. oz.	1 fl. oz. (30 ml.)
To Make: 1 1/2 Gallon	
62 fl. oz.	2 fl. oz. (60 ml.)
To Make: 1 Gal. op	
124 fl. oz.	4 fl. oz. (120 ml.)

STORAGE AND DISPOSAL

Keep Omni II covered in storage or use. Do not contaminate water, food or feed by storage or disposal. Do not reuse empty container. Triple rinse container, puncture and dispose of in a sanitary landfill or incinerate. Used disinfectant solutions are toxic. Store, use and dispose of according to label instructions.

OMN232/BL07

46851-1

6-19-2003

11