

RECEIVED

OCT 24 1983

Registration Division (TS-757)

OMNIGEN

STERILIZING and DISINFECTING SOLUTION

For use in hospitals, dental offices, nursing homes, and health care institutions.

Cat No. U8996

28 Day Activated Dialdehyde

Qty. 1 Gallon
(3.79 liters)

- Sporicidal • Virucidal† • Fungicidal
- Tuberculocidal • Bactericidal • Pseudomonacidal

Non-Rusting

Complete Sterilant

Pleasant Fragrance

† Influenza virus types A1 (New Jersey) and B (Maryland), Herpes simplex virus types 1 and 2, Polio virus types 1 and 2 and Vaccinia virus

EPA Establishment No. 7368-CA-3
EPA Registration No. 45951-2

KEEP OUT OF REACH OF CHILDREN

DANGER:

BEST DOCUMENT AVAILABLE

ACCEPTED
with COMMENTS
in EPA Letter Dated:

DEC 20 1983

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, for sale or use
registered under EPA Reg. No.

46851-2

Recommended for Disinfection or Sterilization of: MEDICAL INSTRUMENTS, CARBON STEEL CUTTING INSTRUMENTS, BURS, ALUMINUM, GLASS, PLASTIC or RUBBER ARTICLES, THERMOMETERS, MIRRORS, RESPIRATORY EQUIPMENT.

STORAGE AND DISPOSAL

STORAGE. Keep Omnicide covered in storage or use. Store in a cool place. Do not contaminate water, food or feed by storage or disposal.

DISPOSAL. Triple rinse container (or equivalent) then puncture and dispose of in a sanitary landfill or appropriate used disinfectant/sterilant solutions are a very hazardous. Store, use, and dispose of according to label instructions.

PRECAUTIONARY STATEMENTS:

HAZARDS TO HUMANS AND DOMESTIC ANIMALS.

DANGER. Corrosive to tissue. Causes burns. Causes eye damage and skin irritation. Do not get in eyes, on skin or on clothing. May be absorbed through skin. Wear goggles or face shield and rubber gloves when handling.

HARMFUL IF SWALLOWED. Avoid contact with food. Use in ventilated area. Use containers which can be tightly closed during use or storage of solution.

STATEMENT OF PRACTICAL TREATMENT

Eyes — In case of eye contact, immediately flush thoroughly with water. **Skin** — In case of skin contact immediately flush thoroughly with water. **Ingestion:** If swallowed, drink milk, egg whites, gelatin solution or, if these are not available, large quantities of water. Avoid alcohol. Call a physician.

Note to Physician: Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.

Right Panel

ACCEPTED
with COMMENTS
in EPA Letter Dated:

DEC 20 1983

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, for the pesticide
registered under EPA Reg. No.
46851-2

BEST DOCUMENT AVAILABLE

RECEIVED

OCT 24 1983

Registration Division (TS-767)

20 DEC 1983

MSM Medical Division
P.O. Box 4546
2000 Reserve
Missoula, MT 59801

Attention: Wendell Guthrie
Vice President

Gentlemen:

Subject: Omnicide[®]
EPA Registration Number 49851-1
Your Application Dated October 20, 1983

The amendment referred to above, submitted in connection with registration under FIFRA, is acceptable provided that you:

1. Make the labeling changes listed below before you release the product for shipment bearing the amended labeling:

a. The active ingredient, and the name and address of the company must be clearly legible on the label.

2. Submit five (5) copies of your final printed labeling before you release the product for shipment. Refer to the A-79 Enclosure for a further description of final printed labeling.

Your release for shipment of the product bearing the amended labeling constitutes acceptance of these conditions.

A stamped copy of the label is enclosed for your records.

Sincerely yours,



John H. Lee
Product Manager (31)
Disinfectants Branch
Registration Division (TS-767C)

Enclosure

DCR-32691:R.Taylor:RD-02:CM#2:Rm243:557-1800:12/9/83:Del.12/25/83:efs

CONCURRENCES							
SYMBOL							
SURNAME							
DATE							