

FRONT PANEL

STERILIZING and DISINFECTING SOLUTION

For use in hospitals, dental offices, nursing homes, and health care institutions.

28 Day Activated Dialdehyde

Cat. No. TD/GL-BLK • Sporicidal • Virucidal† • Fungicidal
• Tuberculocidal • Bactericidal • Pseudomonacidal

Qty. 1 Gallon
(3.79 liters)

Non-Rusting

Complete Sterilant

Pleasant Fragrance

† Includes virus types A1 (New Jersey) and B (Maryland), Herpes, herpes virus types 1 & 2, Polio virus types 1 & 2, Vaccinia virus and HIV-1 (AIDS Virus) on impermeable hard surfaces.

E.P.A. Registration No. 4451-2
E.P.A. Establishment No. 2766-CA-01
(See Bottom of Container for E.P.A. Est. No.)

E.P.A. Establishment No. 7366-CA-3
E.P.A. Establishment No. 302-01-01
(See Bottom of Container for E.P.A. Est. No.)

TM of ADM Medical Division.
KEEP OUT OF REACH OF CHILDREN
DANGER: See inside panel for
additional precautionary statements

Active Ingredient
Glutaraldehyde 2.0%
Inert Ingredients 98.0%
Total 100.0%

Manufactured for & Distributed by
ADM MEDICAL DIVISION
25431 Cobble Rd., Suite 202
Laguna Hills, Ca. 92653

Recommended for Disinfection or Sterilization of: MEDICAL INSTRUMENTS, CARBON STEEL
CUTTING INSTRUMENTS, BURS, ALUMINUM, GLASS, PLASTIC or RUBBER ARTICLES,
THERMOMETERS, MIRRORS, RESPIRATORY EQUIPMENT.

STORAGE AND DISPOSAL

STORAGE. Keep Omnicide covered, in storage or use. Store in a cool place. Do not contaminate water, food or feed by storage or disposal.

DISPOSAL. Triple rinse container (or equivalent) then puncture and dispose of in a sanitary landfill or incinerate. Used disinfectant / sterilant solutions are acutely hazardous. Store, use, and dispose of according to label instructions.

PRECAUTIONARY STATEMENTS:

HAZARDS TO HUMANS AND DOMESTIC ANIMALS.

DANGER. Corrosive to tissue. Causes burns. Causes eye damage and skin irritation. Do not get in eyes, on skin or on clothing. May be absorbed through skin. Wear goggles or face shield and rubber gloves when handling.

HARMFUL IF SWALLOWED. Avoid contact with food. Use in ventilated area. Use containers which can be tightly closed during use or storage of solution.

STATEMENT OF PRACTICAL TREATMENT

Eyes — in case of eye contact, immediately flush thoroughly with water. Skin — in case of skin contact, immediately flush thoroughly with water. Ingestion: If swallowed, drink egg whites, gelatin solution or, if these are not available, large quantities of water. Avoid alcohol. Call a physician.

Note to Physician: Probable mucosal damage may contraindicate the use of gastric lavage. Measures against circulatory shock, respiratory depression and convulsion may be needed.



ACCEPTED

JUL 05 1989

Under the Federal Insecticide, Fungicide, and Rodenticide Act is amended, for the pesticide registered under EPA Reg. No. 46851-2

BEST AVAILABLE COPY

DIRECTIONS FOR USE:

IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING

To activate solution — add activator contents to this one gallon container — recap — shake. Activated solution changes color to blue. Do not dilute. Always clean away excess organic debris, rinse and rough dry instruments or equipment before immersion.

Omnicide™ may be USED AND REUSED as a disinfectant or sterilant in a manual (bucket or tray) system for up to 28 days after activation. Do not use activated solution beyond 28 days.

To destroy HIV-1 (AIDS Virus) immerse articles for 60 seconds at 20° in activated unused solution (see accompanying literature). Although 60 seconds contact time is adequate against HIV-1, this time would not be sufficient for other organisms.

FOR DISINFECTION:

Immerse articles for a minimum of 10 minutes at 20°C to destroy vegetative pathogens including — *Escherichia coli* 0157, *Staphylococcus aureus*, *Salmonella typhosa*, *Pseudomonas aeruginosa*, *Acinetobacter baumannii*, *Legionella pneumophila*, *Trichophyton interdigitale*, viruses including — Herpes simplex virus types 1 & 2, Polio virus types 1 & 2, Vaccinia virus, Influenza virus types A1 (New Jersey) and B (Maryland).

Rinse articles thoroughly with sterile water after disinfection.

For tuberculocidal activity during 28 day reuse, immerse articles for a minimum of 45 minutes at 20°C. For single use of activated unused solution at 20°C immerse completely for 10 minutes. Remove articles and rinse thoroughly.

For Complete Sterilization — to destroy resistant pathogenic spores — including *Clostridium botulinum* and *Clostridium Sporogenes* — immerse articles for a minimum of 10 hours at 20°C. Remove instruments using aseptic technique and rinse thoroughly in sterile water.

All instruments and devices must be thoroughly cleaned to remove excess organic debris, rinsed and rough dried before immersion in Omnicide. Hypodermic needles cannot be effectively disinfected or sterilized with Omnicide.

REUSABLE

Dating Record
OMNICIDE
HAS A 28-DAY SHELF
LIFE AFTER
ACTIVATION.

ACTIVATED

EXPIRATION



RIGHT PANEL

LEFT PANEL

ADM MEDICAL DIVISION, Inc.
dba Omnitec Medical Corp.
25431 Cabot Road, Suite 202
Laguna Hills, CA 92653
81-800-548-7215

Pre-Cleaning Requirements: See product label. Always clean away excess organic debris, rinse and rough dry instruments or equipment before immersion.

Active Ingredients: Glutaraldehyde 2%

Use-Solution: Omnicide is not intended to be diluted prior to use.

Effective Temperature of Use Solution: Omnicide has been shown to be effective at and above 68°F (20°C).

Minimum Contact Time: Omnicide™ is bactericidal, virucidal, tuberculocidal, and fungicidal in 10 minutes at 20°C. For tuberculocidal activity during 28-day reuse, increase immersion time to 45 minutes at 20°C (68°F). For complete sterilization, to destroy resistant pathogenic spores, immerse articles for a minimum of 10 hours at 20°C (68°F). To destroy HIV-1 (AIDS Virus) immerse in activated unused solution for 60 seconds at 20°C (see reverse side).

Non-Activated Shelf Life: Indefinite. Keep Omnicide™ covered in storage or use. Store in cool, dry location. Inadvertent freezing will not alter efficacy when thawed.

Shelf-Life After Activation: Omnicide is proven to be effective for use and reuse for 28 days after date of activation.

Re-use Life: Omnicide™ has a 28-day re-use life after date of activation.

Test Micro-Organisms Killed:


<u>Bactericidal</u> Escherichia coli 0113 Staphylococcus aureus Pseudomonas aeruginosa Salmonella typhosa	<u>Fungicidal</u> Trichophyton interdigitale <u>Tuberculocidal</u> Mycobacterium tuberculosis
---	--

Virucidal
Polio virus types 1 and 2
Herpes simplex virus types 1 and 2
Vaccinia virus
Influenza virus types A1 and B
HIV-1 (AIDS Virus)*
(on inanimate hard surfaces)
*See Reverse Side

Rinsing Instructions: See Product Label. Rinse articles thoroughly with sterile water after disinfection or sterilization.

Possible Adverse Reactions: Danger. Corrosive to tissue, causes burns, eye damage and skin irritation. Do not get in eyes, on skin or on clothing, wear goggles or face shield and rubber gloves when handling. In case of contact with eyes or skin, immediately flush with water.

Methods of Disposal: Triple rinse container, puncture and dispose of in a sanitary landfill or incinerate. Used disinfectant/sterilant solutions are acutely hazardous. Store, use and dispose of according to label instructions.

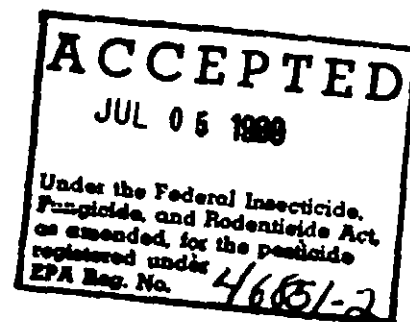

This compound is recommended only for items which cannot be sterilized by steam autoclave, unsaturated chemical vapor, dry heat or ethylene oxide due to physical limitations, incompatibility or residue.

Special instructions for cleaning and decontamination against HIV-1 (AIDS Virus) of surfaces or objects soiled with blood or body fluids.

Personal Protection:

When handling items soiled with blood or body fluids or when conducting procedures which might lead to contact with blood or body fluids always wear appropriate barrier protection items such as; disposable latex gloves, gowns, masks and eye coverings.

Cleaning Procedures:

Blood and body fluids must be thoroughly cleaned from surfaces and objects before application of the disinfectant/sterilant.

Contact Time:

See product label. Omnicide has been shown to be effective in destroying the HIV-1 (AIDS Virus) in 60 seconds at 20°C (68°F) when used undiluted (activated unused solution). Although efficacy at 60 seconds contact time has been shown to be adequate against HIV-1, this time would not be sufficient for other organisms. Use a 10 minute contact time for disinfection against all the organisms claimed.

Disposal of Infectious Materials:

Blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.