

911-22

46851-5

OMC - II  
Ready to Use

E.P.A. Reg. No. 46851-5  
Est. No. 37265-CA-01

## INSTRUMENT & EQUIPMENT DISINFECTANT

Non-Rusting

Pleasant Fragrance

### Active Ingredients

O-phenylphenol . . . . . 0.28%  
O-Benzyl-p-chlorophenol . . 0.03%  
Inert ingredients . . . . . 99.69%  
100.0%

- \* Bactericidal
- \* Fungicidal - destroys pathogenic fungi
- \* Virucidal\*
- \* Tuberculocidal
- \* Viruses including Herpes virus types 1 & 2, Polio virus types 1 & 2, Vaccinia virus, Influenza virus type A1 (New Jersey) and influenza type B (Maryland), on inanimate surfaces.

**KEEP OUT OF REACH OF CHILDREN  
CAUTION:**

See handling precautions and practical treatments on back panel.

Manufactured & Distributed by:  
**ADM MEDICAL DIVISION**

Contents: 1 quart (32 fluid oz.) 25431 Cabot Road Laguna Hills, CA 92653

### DIRECTIONS FOR USE:

**IT IS A VIOLATION OF FEDERAL LAW TO USE THIS  
PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING**

OMC II is ready to use for disinfection of medical and dental instruments and devices. Rinse thoroughly after immersion. Other hard surface areas such as walls, floors and countertops should be mopped, sponged, sprayed or wiped in a manner to thoroughly wet the surface with OMC II. Discard the used solution daily, or sooner if it becomes diluted or soiled.

Remove heavy soil and thoroughly clean surface prior to application of the product. Immerse in OMC II for 10 minutes at 121°C to destroy bacteria including *Escherichia coli* 0113, *Staphylococcus aureus*, *Salmonella typhosa*, *Pseudomonas aeruginosa*, *Mycobacterium tuberculosis*; fungi including *Trichophyton interdigital*; viruses including Herpes virus 1 & 2, Polio virus types 1 & 2, Vaccinia virus, Influenza virus type A1 (New Jersey) and Influenza B (Maryland).

For use on metals, plastics, rubber and glass. Glass may etch over prolonged or repeated exposure. Do not store lensed articles in OMC II.

**ACCEPTED  
SEP 28 1988**

Under the Federal Insecticide,  
Fungicide, and Rodenticide Act,  
as amended, for the pesticide  
registered under:  
EPA Reg. No. 46851-5

When used as directed, OMC II is stable. All label claims of effectiveness are based on previously unused batches of OMC II.

OMC II should not be used for disinfection of hypodermic needles or any items with deep and/or narrow crevices.

### DISPOSAL

Do not reuse empty container. Rinse container and discard in trash collection. Waste from the use of this product may be disposed of on site in an approved waste disposal facility.

### PRECAUTIONARY STATEMENTS:

**CAUTION:** May cause eye irritation. Avoid contact with eyes. Wash thoroughly with soap and water after handling.

### STATEMENT OF PRACTICAL TREATMENT:

If in eyes: Flush with plenty of water. Get medical attention if irritation persists.

### NOTE TO PHYSICIAN:

Probable mucosal damage may contraindicate the use of gastric lavage.