



## CHEMICAL WARNING LABEL

|                                                                                                              |                                                                                                                                            |
|--------------------------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------|
| <b>Health Hazard</b><br>■ Irritant<br>■ Carcinogen<br>■ Toxic<br>■ Sensitizer<br>■ Normal Material           | <b>Fire Hazard</b><br>■ Below 131<br>■ Below 100°F<br>■ Above 100°F, Not Exceeding 200°F<br>■ Above 200°F<br>■ Will Not Burn               |
| <b>Physical Hazard</b><br>■ Oxidizer<br>■ Acid<br>■ Alkali<br>■ Corrosive<br>■ Use No Water<br>■ Radioactive | <b>Reactivity</b><br>■ May Detonate<br>■ Shock and Heat<br>■ May Detonate<br>■ Violent Chemical Change<br>■ Unstable if Heated<br>■ Stable |

Product Name: OMC II Spray  
 Hazardous Chemicals:  
 o-phenylphenol  
 Route of Entry:  
 ■ Inhalation  
 ■ Ingestion  
 ■ Skin/Eye Absorption  
 Personal Protection: Nitrile  
 Gloves, Glasses

# OMC II<sup>TM</sup> Spray

## Surface Disinfectant/Cleaner

**Tuberculocidal, Virucidal,  
Bactericidal, Fungicidal**

|                         |         |
|-------------------------|---------|
| Active Ingredients      |         |
| o-phenylphenol          | 0.28%   |
| o-benzyl-p-chlorophenol | 0.03%   |
| Inert Ingredients       | 99.69%  |
| Total                   | 100.00% |

E. P.A. Reg No. 46851-5  
 E.P.A. Est. No. 46851-CO-3  
 E.P.A. Est. No. 39754-WI-1

(See bottom of container for E.P.A. Est. No.)

**Contents:**  
**24 fl. oz. (720ml)**  
**Reorder No. OMN240**

## PRECAUTIONARY STATEMENTS

**CAUTION:** Avoid contact with skin, eyes or clothing. Protective gloves should be worn.

**STATEMENT OF PRACTICAL TREATMENT:** If in eyes - flush with plenty of water. Get medical attention if irritation persists.

**NOTE TO PHYSICIAN:** Possible mucosal damage may contraindicate the use of gastric lavage.

## STORAGE &amp; DISPOSAL

Store at room temperature. Rinse and discard empty container in regular trash. Do not reuse empty container.

Manufactured by  
**PROCHEM COMPANY**

A wholly owned subsidiary of  
**COTTRELL, LTD.**  
 7399 SOUTH TUCSON WAY  
 ENGLISWOOD, CO 80112 USA  
 1 800 THE EDGE

## DIRECTIONS FOR USE

**IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING.**

OMC II Spray is a ready to use surface disinfectant/cleaner for use in the medical, dental, hospital, clinical, office and veterinary environments.

**FOR DISINFECTION:** Remove heavy soil and thoroughly clean surface prior to application of OMC II Spray. Non-porous surface areas such as walls, floors and countertops should be sprayed and mopped, sponged or wiped in a manner to thoroughly wet the surface with OMC II Spray. OMC II Spray may remain on the surface until dry or removed, after the recommended 10 minute contact time, by wiping with a clean, dry cloth. OMC II Spray is effective in 10 minutes at 25°C (68°F) to destroy the following bacteria: *Escherichia Coli* 0113, *Staphylococcus aureus*, *Salmonella typhosa*, *Pseudomonas aeruginosa*, *Mycobacterium tuberculosis*, fungi including *Trichophyton mentagrophytes*; the following viruses: Polio Virus 1, Herpes virus types 1 & 2, Vaccinia virus, Influenza virus type A1 (New Jersey) and Influenza B (Maryland) on inanimate surfaces. OMC II Spray has been shown to destroy HIV-1 (AIDS virus) in 60 seconds contact time at 20°C (68°F). Although efficacy at 60 seconds contact time has been shown to be adequate against HIV-1, this time would not be sufficient for other organisms. Use a 10-minute contact time for disinfection against all the organisms claimed.

This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise

enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection.

**\* KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, an in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-1) (associated with AIDS).

**SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 OF SURFACES OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS**

**PERSONAL PROTECTION:** When handling items soiled with blood or body fluids or when conducting procedures which might lead to contact with blood or body fluids always wear items such as heavy duty Nitrile gloves, gowns, masks, and eye coverings.

**CLEANING PROCEDURES:** Blood and other body fluids must be thoroughly cleaned from surfaces and objects prior to disinfection. Apply OMC II Spray directly to surface using mop or spray bottle. Wipe surface to remove debris and solution using a clean paper or cloth towel and discard.

**DISPOSAL OF INFECTIOUS MATERIAL:** Blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

**CAUTION:** Keep Out of Reach of Children  
 See Side Panel for Additional Precautionary Statements

OMN240/BL01