

46851-5

08/02/2000

1/2

Directions for Use - General Classification

IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING.

OMC II™ is a ready to use surface disinfectant for use in the medical, dental, hospital, clinical, office and veterinary environments.

FOR DISINFECTION: Remove excess organic debris and thoroughly clean surface prior to application of OMC II. Non-porous surface areas

such as walls, floors and countertops should be sprayed and mopped, sponged or wiped in a manner to thoroughly wet the surface with OMC

II. Allow surface to remain wet for 10 minutes at 20°C (68°F) to destroy bacteria including

Escherichia coli 0113, *Staphylococcus aureus*, *Salmonella typhosa*, *Pseudomonas aeruginosa*, *Mycobacterium tuberculosis*, fungi including *Trichophyton mentagrophytes*, viruses* including Herpes virus types 1 & 2, Vaccinia virus, Influenza virus types A1 (New Jersey), Influenza B (Maryland) and HIV-1 (AIDS virus).

STORAGE & DISPOSAL: Store at room

temperature. Rinse and discard empty container in regular trash. Do not reuse empty container.

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 OF SURFACES OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS.

PERSONAL PROTECTION: When handling items soiled with blood or body fluids or when conducting procedures which might lead to contact with blood or body fluids, always wear items such as disposable latex gloves, gowns, masks, and eye coverings.

OMC II

Surface Disinfectant Tuberculocidal, Virucidal, Bactericidal, Fungicidal

Recommended for use in hospitals, nursing homes, healthcare institutions

Active Ingredients	
O-phenylphenol	0.28%
O-benzyl-p-chlorophenol	0.03%
Inert Ingredients	99.69%
Total	100.00%


E.P.A. Reg. No. 46851-5

E.P.A. Est. No. 37265-CA-01

Keep Out of Reach of Children
Additional information on side panel

☒ CAUTION: Keep Out of Reach of Children

☒ CAUTION: In Use

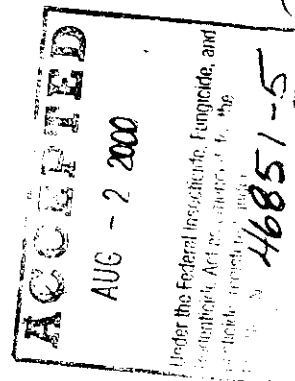

COTTRELL

CII

Disinfectant

**cidal, Virucidal,
al, Fungicidal**

for use in hospitals, medical/dental clinics,
healthcare institutions and veterinary clinics.

phenol
0.28%
0.03%
99.69%
100.00%

Contents:
1 Gallon (3.785 Liters)

Reorder No. CPS128

2001, Manufactured by Cottrell Ltd., London, Ontario

DNW62231483

CAUTION: Irritant

CLEANING PROCEDURES: Blood and body fluids must be thoroughly cleaned from surfaces and objects before application of the disinfectant.

DISPOSAL OF INFECTIOUS MATERIAL: Blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

PRECAUTIONARY STATEMENTS

CAUTION: May cause eye irritation. Avoid contact with eyes. Protective gloves should be worn. This product must not be used in hospital nurseries.

STATEMENT OF PRACTICAL TREATMENT: If in eyes - flush with plenty of water. Seek medical attention if irritation persists.

NOTE TO PHYSICIAN: Possible mucosal damage may contraindicate the use of gastric lavage.

REGISTERED
COTTRELL, LTD.
7399 SOUTH JESSIE WAY
ENGLEWOOD, CO 80112 USA
1 800 THE FDGE

CANADIAN AGENT
TRANS CANADA DENTAL LTD.
2891 PORTLAND DR
OAKVILLE, ONTARIO,
CANADA L6H 1S4
(905) 829-1466

CPS128/UL01

ACCEPTED

AUG - 2 2000

Under the Federal Insecticide, Fungicide, and
Rodenticide Act as amended, for the
pesticide, registered under
IT's Reg. No. 46851-5

2/2