

A BISYNTHETIC PHENOL SOLUTION IMPREGNATED IN A UNIFORM RELEASE TOWELETTE

ProSpray wipes, Disinfectant Towelettes are intended for use in hospitals, medical, dental and veterinary offices, laboratories, industrial clean rooms, nursing homes, ambulances, police, hotels, schools, homes, health spas, boats, autos, and toilet seats. Use on inanimate surfaces and equipment such as stainless steel, formica, glass tables, carts, baskets, counters, cabinets, telephones and other hard non-porous surfaces.

Preparation of Dispenser: Lift up tab and pull back to expose the entire opening. Do Not Remove Peel & Seal Tab. Grasp exposed edge of towelette and pull out. Close Peel & Seal tab to prevent evaporation, by returning it to its original position and press down firmly.

IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING.


DIRECTIONS FOR USE: DO NOT USE AS A BABY WIPE. Remove one ProSpray wipes, Disinfectant Towelette, unfold, wipe surface to remove gross filth. Use a second towelette to thoroughly wet surface. Ensure surface remains wet for 10 minutes to destroy or inactivate bacteria including Escherichia Coli 0113, Staphylococcus aureus, Salmonella typhosa, Pseudomonas aeruginosa, Mycobacterium tuberculosis, fungi including Trichophyton mentagrophytes, and the following viruses* Polio virus 1, Herpes simplex virus types 1 & 2, Vaccinia virus, Influenza virus type A 1 (New Jersey) and Influenza B (Maryland).

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the blood stream or normally sterile areas of the body, (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

ProSpray™ wipes

Disinfectant Towelettes
Tuberculocidal, Virucidal,
Bactericidal, Fungicidal

Active Ingredients	
o-phenylphenol	0.28%
o-benzyl-p-chlorophenol	0.03%
Inert Ingredients	99.69%
Total	100.00%

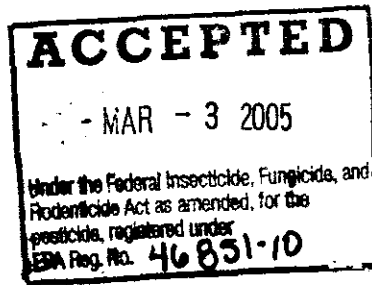

with
Lemon Scent

Reorder No: PSW

72 Pre-moisten Towelettes

9 x 10 in. (22.86 x 25.4 cm)

CAUTION: Keep Out of Reach of Children

46851-10

3/3/2005

Page 182

MANUFACTURED FOR:
CERTOL
INTERNATIONAL, LLC
6120 E. 35TH AVE.
COMMERCIAL CITY, CO 80022 USA
1 800 THE EDGE
(1-800-843-3343)

E.P.A. Reg. No. 46851-10
E.P.A. Est. No. 70806-OH-001



PSW/SP72/JUL01

***KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-1) (associated with AIDS).


SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 OF SURFACES OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS:

- **PERSONAL PROTECTION:** When handling items soiled with blood or body fluids or when conducting procedures which might lead to contact with blood or body fluids always wear items such as heavy-duty Nitrile gloves, gowns, masks, and eye coverings.
- **CLEANING PROCEDURES:** Blood and body fluids must be thoroughly cleaned from surfaces and objects prior to disinfection. Apply ProSpray wipes, Disinfectant Towelettes directly to the surface to be cleaned and then discard.

- **DISPOSAL OF INFECTIOUS MATERIAL:** Blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

PRECAUTIONARY STATEMENTS

- **CAUTION:** Avoid contact with eyes and clothing. Protective gloves should be worn.
- **STATEMENT OF PRACTICAL TREATMENT:** If in eyes - flush with plenty of water. Seek medical attention if irritation persists.
- **STORAGE AND DISPOSAL:** Store at room temperature. Do not reuse towelettes. Dispose of used towelettes and empty packets in regular trash.

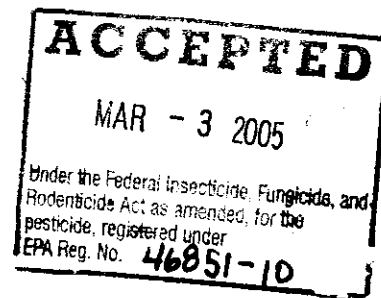

082