

# Omni II™ [Alternate Brand Names: "Vital Defense-D" and "Prophene Plus"]

## Concentrated Surface Disinfectant and Holding Solution

### Tuberculocidal, Virucidal\* Bactericidal, Fungicidal

#### Active Ingredients

o-phenylphenol	9.0%
o-benzyl-p-chlorophenol	1.0%
Inert Ingredients	90.0%
Total	100.0%

Contents: 1 Quart  
(32 oz. - 946 ml)  
Reorder No. OMN232

E.P.A. Reg. No. 46851-11  
E.P.A. Est. No. 46851-CO-3

OMN232FL06

**DANGER, Keep Out of Reach of Children**  
See Side Panel for Additional Precautionary Statements

#### SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV ON SURFACES OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS

**PERSONAL PROTECTION:** When handling items soiled with blood or body fluids or when conducting procedures which might lead to contact with blood or body fluids, always wear items such as heavy duty Nitrile gloves, gowns, masks and eye coverings.

**CLEANING PROCEDURES:** Blood and body fluids must be thoroughly cleaned from surface and objects before application of the disinfectant.

**DISPOSAL OF INFECTIOUS MATERIAL:** Blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

#### STORAGE AND DISPOSAL

Do not contaminate water, food or feed by storage or disposal.

**PESTICIDE STORAGE:** Omni II covered in storage or use.

**PESTICIDE DISPOSAL:** Pesticide wastes are toxic. Improper disposal of excess pesticide, spray mixture, or residue is a violation of Federal Law. If these wastes cannot be disposed of by use according to label instructions, contact your State Pesticide or Environmental Control Agency, or the Hazardous Waste representative at the nearest EPA Regional Office for guidance.

**PESTICIDE CONTAINER:** Triple rinse (or equivalent). Then offer for recycling or reconditioning, or puncture and dispose of in sanitary landfill, or incineration, or if allowed by state and local authorities, by burning. If burned, stay out of smoke.

#### FIRST AID

##### IF IN EYES:

- Hold eye open and rinse slowly and gently with water for 15-20 minutes.
- Remove contact lenses, if present, after the first 5 minutes, then continue rinsing.
- Call a poison control center or doctor for treatment advice.

##### IF SWALLOWED:

- Call a Poison Control Center or doctor immediately for treatment advice.
- Have person sip a glass of water if able to swallow.
- Do not induce vomiting unless told by a Poison Control Center or doctor.
- Do not give anything by mouth to an unconscious person.

##### IF ON SKIN OR CLOTHING:

- Take off contaminated clothing.
- Rinse skin immediately with plenty of water for 15-20 minutes.
- Call a poison control center or doctor for treatment advice.

In case of an emergency for additional information call toll free 800-535-5053. Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

TO PREPARE 1:32 USE-DILUTION				
WATER	15 1/2 fl. oz.	31 fl. oz.	62 fl. oz.	124 fl. oz.
OMNI II	1/2 fl. oz.	1 fl. oz.	2 fl. oz.	4 fl. oz.
	(15 ml.)	(30 ml.)	(60 ml.)	(120 ml.)

#### DIRECTIONS FOR USE

IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING.

Omni II is a concentrated solution and must be diluted 1:32 with water prior to instrument soak and/or decontamination of dental and medical instruments or disinfection of hard environmental surfaces. This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the blood-stream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high-level disinfection.

**Soak/Decontamination** - immerse items in diluted Omni II solution for instrument soak and/or decontamination of medical and dental instruments. Rinse articles thoroughly following soak/decontamination. Organic materials or water introduced into the diluted product may effect products effectiveness. Discard the used solution daily or sooner if it becomes diluted or soiled.

**Ultrasonic Cleaning** - Place instruments into diluted Omni II and activate ultrasonic unit. When instruments are clean, remove and rinse thoroughly. Change solution daily or sooner if it becomes diluted or soiled.

**Cleaning** - Apply Omni II directly to surface using mop or spray bottle. Wipe surface to remove debris and solution using a clean paper or cloth towel and discard.



46851C03E2K

**Hard Surface Disinfection** - diluted Omni II solution should be mopped, sponged, sprayed or wiped onto hard environmental surfaces such as walls, floors and countertops in a manner to thoroughly wet the surface. A properly prepared solution of Omni II intended for use as a hard surface disinfectant, has a shelf life of 90 days (use life) when stored in a closed container such as a spray bottle.

A contact time of 10 minutes at 20°C (68°F) with a 1:32 dilution of Omni II has been shown to be effective on bacteria including: *Escherichia Coli* 0113, *Staphylococcus aureus*, *Salmonella typhosa*, *Pseudomonas aeruginosa*, *Mycobacterium tuberculosis*; fungi including: *Trichophyton mentagrophytes*; viruses\* including *Influenza virus* type A1 (New Jersey) and *Influenza B* (Maryland), *Herpes virus* types 1 and 2, *Vaccinia virus*, and *Polo virus* 1.

Omni II when diluted 1:32 has been shown to be effective in destroying the HIV-1 (AIDS virus) in 60 seconds at 20°C (68°F). Although efficacy in 60 second contact time has been shown to be adequate against HIV-1, this time would not be sufficient for other organisms. Use 10 minute contact time for disinfection against all the organisms claimed. Effective in waters up to 400 ppm hardness calculated as calcium carbonate.

For use on metals, plastics, rubber and glass. Glass may etch over prolonged or repeated exposure to concentrated solutions. Do not store lensed articles in concentrated Omni II.

MANUFACTURED BY:  
**CERTOL**  
INTERNATIONAL, LLC  
5450 JOULET STREET  
DENVER, CO 80229 USA  
1 800 TYZE EDGE  
(1 800 843 3343)

OMN232FL06

**ACCEPTED**  
OCT 11 2002  
Under the Federal Insecticide, Fungicide, and  
Rodenticide Act as amended, for the  
pesticide, registered under  
EPA Reg. No. 46851-11

46851-11

10/11/2002

Page 1 of 1