

1/2 49199-2 Panel

Front Panel

Isotex G
Disinfecting Towelettes

ACTIVE INGREDIENTS

Glutaraldehyde (1,5 Pentanedial)-----0.25%

Triethylene Glycol-----1.50%

INERT INGREDIENTS*-----98.25%

*Does not include weight of dry towelettes
Contains sodium nitrite

Meets efficacy data requirements for use on surfaces in
hospital and medical environments.

WARNING: KEEP OUT OF REACH OF CHILDREN.

GERMICIDAL

EPA REG NO 49199-2
EPA EST NO 49199-MA-01

for tub containers
160/5.5x9 in. lemon Scented Towelettes.

Net contents: 10 fluid oz.
Net Weight: 15 oz.

for foil packets with folded towelettes
1/5.5x9 in. lemon scented towelette

Net Contents: .063 fluid oz.
Net Weight: .094 oz.

for foil packets with towelette attached to inside of foil
1/4x4 in. lemon scented towelette


Net Contents: .063 fluid oz.
Net Weight: .094 oz.

Manufactured by: Isotex Inc., 4 Nancy La., Framingham, MA
01701

ACCEPTED

DEC 1 0 1986

Under the Federal Insecticide, Fungicide, and Rodenticide Act, as amended, for the pesticide registered under EPA Reg. No. 49199-2

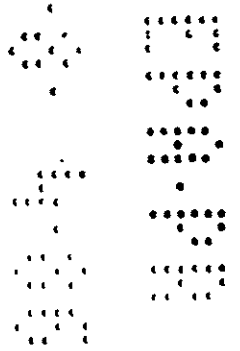

2/2

SIDE PANEL
Isotex G

ACCEPTED
DEC 10 1986
U.S. Environmental Protection Agency
Pesticide and Herbicide Act
as amended, for the pesticide
registered under
EPA Reg. No. 49199-2

PRECAUTIONARY STATEMENT

Hazards to humans and domestic animals.
Physical or chemical hazards.
Use rubber gloves when handling towelette.
May cause eye and skin irritation and dermal sensitization.
If irritation occurs, discontinue use.

Directions for use.

It is a violation of federal law to use this product in a manner inconsistent with its labeling.

for tubs

To start feed: Flip open cap. Remove and discard foil seal. Remove large cover from container. From center of towelette roll, pull up towelette corner, twist it into a point and thread it through the hole in the container cover. Pull through about one inch. Replace cover. Pull out first towelette and snap off at a 90 angle. Remaining towelettes feed automatically, ready for next use. When through using, keep small center cap closed to prevent moisture loss..

for foil packets with folded towelette
Remove towelette from package and unfold.

for foil packets with towelette attached to inside of foil
Peel foil package apart. Towelette is bonded to inside of package.

To disinfect clean, hard, non-porous surfaces such as chairs, counters, trays, drawer pulls: Thoroughly wipe contaminated surface with wet towelette and allow it to dry. Discard towelette after use.

KILLS: Salmonella cholerasuis, Staphylococcus aureus, Psuedomonas aeruginosa.

EFFECTIVE AGAINST: Gram negative and gram positive bacteria..

for tubs

DISPOSAL: Do not reuse empty container. Wrap container and put in trash collection.

for foil packets

DISPOSAL: Place towelette and wrapper in trash collection.

