



U.S. ENVIRONMENTAL PROTECTION AGENCY

Office of Pesticide Programs
Antimicrobials Division (7510P)
1200 Pennsylvania Avenue NW
Washington, D.C. 20460

EPA Reg. Number:
52252-11

Date of Issuance:
JAN 6 2012

Term of Issuance:

Conditional

Name of Pesticide Product:
REVOX PA Sterilant

NOTICE OF PESTICIDE:

- Registration
- Reregistration

(under FIFRA, as amended)

Name and Address of Registrant (include ZIP Code):

Richard Ormsbee
Minnotech Corporation
14605 28th Avenue North
Minneapolis, MN 55447

Note: Changes in labeling differing in substance from that accepted in connection with this registration must be submitted to and accepted by the Registration Division prior to use of the label in commerce. In any correspondence on this product always refer to the above EPA registration number.

On the basis of information furnished by the registrant, the above named pesticide is hereby registered/reregistered under the Federal Insecticide, Fungicide and Rodenticide Act.

Registration is in no way to be construed as an endorsement or recommendation of this product by the Agency. In order to protect health and the environment, the Administrator, on his motion, may at any time suspend or cancel the registration of a pesticide in accordance with the Act. The acceptance of any name in connection with the registration of a product under this Act is not to be construed as giving the registrant a right to exclusive use of the name or to its use if it has been covered by others.

This product is conditionally registered in accordance with FIFRA sec 3(c)(7)(a) provided that you:

1. Submit and/or cite all data required for registration of your product under FIFRA sec. 3(c)(5) when the Agency requires all registrants of similar products to submit such data; and submit acceptable responses required for re-registration of your product under FIFRA section 4.
2. Make the labeling changes listed below before you release the product for shipment:
 - a. Revise the "EPA Registration Number to read, EPA Reg. No. 52252-11"

Signature of Approving Official:

Marshall Swindell
Product Manager Team-33
Regulatory Management Branch I
Antimicrobials Division (7510P)

Date:


JAN 6 2012

Page 2
EPA Reg. No. 52252-11

If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA sec. 6(e). Your release for shipment of the product constitutes acceptance of these conditions.

A stamped copy of the label is enclosed for your records. Submit one (3) copy of your final printed labeling prior to release of this product for shipment. If you have any questions concerning this letter, please contact Demson Fuller at (703) 308-8062.

Sincerely,

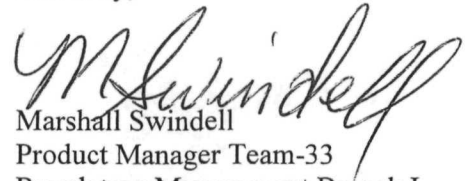

Marshall Swindell
Product Manager Team-33
Regulatory Management Branch I
Antimicrobials Division (7510P)

Enclosure: (Stamped Label)

REVOX PA Sterilant Master Label

For the sterilization and disinfection of hard surfaces and non-porous food contact surfaces.

For use in the following locations;

- | | |
|-------------------------------------|-------------------------------|
| Industrial Laboratories | Hospitals (as a disinfectant) |
| Pharmaceutical Manufacturers | University Laboratories |
| Medical Products Manufacturers | Semi Conductor Manufacturers |
| Electrical Utility Companies | Cosmetic Manufacturers |
| Dairies and Pasteurizing Facilities | Bio Tech Companies |
| Beverage and Food Processing Plants | Animal Hospitals |
| Breweries/Wineries | Veterinary Clinics |

Read the REVOX PA label and application notes before using this product.

Failure to follow directions for use per package insert may result in user injury.

This product must be diluted with Purified Water prior to use.

Active Ingredients	
Hydrogen Peroxide	22.0%
Peroxyacetic Acid	4.5%
Inert Ingredients	<u>73.5%</u>
Total	100.0%

ACCEPTED
with COMMENTS
EPA Letter Dated:

JAN - 6 2012

Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended for the pesticide,
registered under EPA Reg. No. 52252-11

Net Contents: _____

KEEP OUT OF REACH OF CHILDREN

DANGER - PELIGRO

See side panel for additional precautionary statements.

EPA Reg. NO.: 52252-_____
EPA Est. NO.: 52252-MN-01

(Minntech logo)

Minntech Corporation
14605 28th Avenue North
Minneapolis, Minnesota 55447 U.S.A.
Telephone: (763) 553-3300
Toll Free: (800) 328-3345
Fax: (763) 553-3387
E-mail: info@minntech.com

4/10

ACCEPTED
with COMMENTS
EPA Letter Dated:

JAN - 6 2012

Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended, for use as a pesticide,
registered under EPA Reg. No. 52252-11

PRECAUTIONARY STATEMENTS

Hazard to Humans and Domestic Animals.

DANGER - PELIGRO

Corrosive. Causes irreversible eye damage. Harmful if absorbed through skin. Do not get in eyes, on skin, or on clothing. Avoid contact with skin. Prolonged or frequent repeated skin contact may cause an allergic reaction in some individuals. Wash hands before eating, drinking, chewing gum, using tobacco or using the toilet. Remove contaminated clothing and wash hands before reuse. Caution should be used when applying indoors because pets may be at risk.

FIRST AID

IF INHALED:	<ul style="list-style-type: none"> • Move person to fresh air. • If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably mouth-to-mouth, if possible. • Call a poison control center or doctor for further treatment advice.
IF ON SKIN OR ON CLOTHING:	<ul style="list-style-type: none"> • Take off contaminated clothing. • Rinse skin immediately with plenty of water for 15-20 minutes. • Call a poison control center or doctor for treatment.
IF IN EYES:	<ul style="list-style-type: none"> • Hold eye open and rinse slowly and gently with water for 15-20 minutes. • Remove contact lenses, if present, after first 5 minutes, then continue rinsing. • Call a poison control center or doctor for treatment advice.
IF SWALLOWED:	<ul style="list-style-type: none"> • Call a poison control center or doctor immediately for treatment advice. • Have person sip a glass of water if able to swallow. • Do not induce vomiting unless told to by a poison control center or doctor. • Do not give anything by mouth to an unconscious person.

HOT LINE NUMBER

For chemical emergency, spill, leak, fire, exposure and accident, call Chemtrec, day or night
(800) 424-9300, (703) 527-3387.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact Chemtrec at 1-800-424-9300 for emergency medical treatment information.

Note to Physician:

Probable mucosal damage may contraindicate the use of gastric lavage.

PRECAUTIONARY STATEMENTS Cont.

Personal Protective Equipment (PPE)

Handlers must wear: ANSI approved protective eyewear, and protective gloves.

Environmental Hazards

This pesticide is toxic to birds, fish and aquatic invertebrates

Do not discharge effluent containing this product into lakes, streams, ponds, estuaries, oceans, or public waters unless this product is specifically identified and addressed in a NPDES permit. Do not discharge effluent containing this product to sewer systems without previously notifying the sewage plant authority. For guidance contact your State Water Board or Regional Office of the U.S. Environmental Protection Agency.

ACCEPTED
with COMMENTS
EPA Letter Dated:
JAN -6 2012

Physical and Chemical Hazards

This product contains an oxidizing agent.

Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended, for the pesticide,
registered under EPA Reg. No. 52252-1

PRODUCT EFFICACY

- Sterilant 100X dilution, 11 hours 20°C
- Sporicidal 100X dilution, 11 hours 20°C
- Bactericidal 100X dilution, 11 hours 20°C
- Virucidal 100X dilution, 11 hours 20°C
- Hepatitis B 100X dilution, 11 hours 20°C
- HIV-1* 100X dilution, 11 hours 20°C
- Fungicidal 100X dilution, 11 hours 20°C
- Effective against non-tuberculous mycobacterium (chelonae abscessus) 100X dilution, 10 min., 20°C
- Cleaner/Sanitizer (Non-food contact surfaces) 32x dilution, 10 min., 20°C
- (food contact surfaces) 320x dilution, 1 min., 25°C
- Staphylococcus aureus and Escherichia coli)
- Hospital Disinfectant 100X dilution, 10 min., 20°C
- Pseudomonacidal
- Broad Spectrum Disinfectant 100X dilution, 10 min., 20°C
- Germicidal Spray Disinfectant 100X dilution, 10 min., 20°C

* KILLS HIV-1 ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/ objects likely to be soiled with blood or body fluids which can be associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-1) (associated with AIDS).

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS:

PERSONAL PROTECTION: Specific barrier protection items to be used when handling items soiled with blood or body fluids are disposable latex gloves, gowns, masks, or eye coverings.

CLEANING PROCEDURE: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of the sterilant.

DISPOSAL OF INFECTIOUS MATERIALS: Blood and other body fluids should be autoclaved and disposed of according to federal, state and local regulations for infectious waste disposal.

CONTACT TIME: Leave surfaces wet for 11 hours.

DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling. DO NOT USE AFTER EXPIRATION DATE.

Do not allow REVOX PA Sterilant to mix with alkaline substances such as bleach (Sodium Hypochlorite).

If spilled, flush away with large quantities of water. Use purified water when diluting REVOX PA Sterilant.

REVOX PA is a single-use product not intended for reuse. Fresh sanitizing solution should be prepared daily or more often if the solution becomes diluted or soiled.

ACCEPTED
with COMMENTS
EPA Letter Dated:

JAN -6 2012

DIRECTIONS FOR STERILIZATION

REVOX PA Sterilant may be used as a sterilant in any application other than a health care setting/hospital. For use on hard, non-porous surfaces such as plastics, stainless steel objects or containers, glass used by laboratories, manufacturers of pharmaceuticals, medical products, and cosmetics. Examples include glassware, laboratory instruments, containers, etc. Remove any obvious debris or organic material from the surface to be sterilized. This can often be accomplished by rinsing with purified water, mechanical action, or by detergent cleaning followed by a water rinse. Dilute REVOX PA Sterilant 100X (1 part REVOX PA plus 99 parts purified water). Do not store diluted REVOX PA Sterilant for use as a sterilant; once diluted, use immediately. Immerse the item to be sterilized in a sufficient volume of diluted REVOX PA Sterilant to cover the item and fill all passages requiring sterilization. Hold in the sterilizing solution for a minimum of 11 hours at 20°C (68°F). Remove items after 11 hours and rinse with sterile water until effluent testing shows acceptable levels when tested with Residual Test Strips.

DIRECTIONS FOR HOSPITAL DISINFECTION

REVOX PA Sterilant is to be used on hard, non-porous surfaces such as plastic, glass, or stainless steel in settings such as hospitals, clinics, or physician's offices. Examples include counters, bed rails, bathroom fixtures, bedpans, non-invasive medical instruments (scissors, clamps) etc. This product is **not** to be used as a terminal high level disinfectant on any critical/semi-critical medical device. Remove any obvious debris or organic material from the surface to be disinfected. This can often be accomplished by rinsing with purified water, mechanical action or by detergent cleaning followed by a water rinse. Dilute REVOX PA Sterilant 100X (1 part REVOX PA Sterilant plus 99 parts purified water). Once diluted, the solution must be used within seven (7) days. Immerse the item to be disinfected in sufficient volume of diluted REVOX PA Sterilant to cover the item and fill all passages requiring disinfecting. Hold in the disinfecting solution for a minimum of 10 minutes. Remove items after 10 minutes and rinse with sterile water until effluent shows acceptable levels when tested with Residual Test Strips. This product may be used to pre-clean or decontaminate critical/semi-critical medical devices **prior** to sterilization or high level disinfection.

ACCEPTED
with COMMENTS
EPA Letter Dated:

JAN -6 2012

Under the Federal Insecticide,
Fungicide and Rodenticide Act as
required for the pesticide,
registered under EPA Reg. No. 52252-11

FOGGING AS AN ADJUNCT TO REGULAR CLEANING AND DISINFECTING:

This product may be used in fogging as an adjunct following regular cleaning and disinfecting procedures for hard room surfaces.

Prior to fogging, remove or carefully protect all food products and packaging materials. Ensure room is properly ventilated. Vacate all personnel from the room during fogging and do not allow personnel into the room until the hydrogen peroxide air concentration has been verified to be less than or equal to 0.5 ppm. Fog areas using 1 to 10 ml of concentrated REVOX PA solution per cubic meter of room volume using a 0.3 to 10% aqueous diluted solution using any fogging equipment that is capable of achieving the desired droplet size of 0.5 to 150 microns and is constructed of compatible materials. Review the operations manual provided by the fogging equipment manufacturer for specific directions on the use of this equipment. Dilution directions: To obtain an aqueous diluted solution of 0.3%, add 0.3 ml of concentrated REVOX PA solution to 99.7 ml of high purity water (i.e. RO or DI). To obtain an aqueous diluted solution of 10%, add 10 ml of concentrated REVOX PA solution to 90 ml of high purity water (i.e. RO or DI). Allow surfaces to dry thoroughly before operations are resumed.

Note: Once diluted, the solution must be used within 7 days. Discard any unused diluted solution

DIRECTIONS FOR USE AS A GERMICIDAL DISINFECTANT SPRAY

For use as a germicidal spray on hard surfaces such as floors, walls, bathroom fixtures, vinyl or Formica surfaces. Dilute 100X with purified water. Note: Once diluted, the solution must be used within seven (7) days. Discard any unused diluted solution. Spray onto surfaces using a plastic spray bottle. Allow product to remain on surface for 10 minutes. Let air dry or rinse with purified water. Drain off excess water, if possible, and allow to dry.

DIRECTIONS FOR USE AS A CLEANER/SANITIZER
(NONFOOD CONTACT SURFACES)

For use on hard, non-porous surfaces such as plastics, stainless steel objects or Formica. Examples include counter tops, walls, floors, bedrails, bathroom fixtures etc. Using water or mechanical action, remove heavy soil and gross filth from hard, non-porous surfaces such as Formica, stainless steel, or plastic. Dilute 32X with purified water. Note: Once diluted, the solution must be used within seven (7) days. Discard any unused diluted solution. Fresh solution should be prepared daily or more often if the solution becomes diluted or soiled. Apply solution to surface or immerse items to be sanitized in the solution. REVOX PA Sterilant may be applied by cloth, mop or sponge. Allow 10 minutes of contact time. Rinse with purified water, drain excess, if possible, and allow to air dry.

ACCEPTED
with COMMENTS
EPA Letter Dated:

JAN -6 2012

DIRECTIONS FOR USE AS A CLEANER/SANITIZER (FOOD CONTACT SURFACES)

REVOX PA Sterilant may be used to sanitize hard, non-porous surfaces such as glass or stainless steel used in fillers, pipelines, silos, processing vats, bulk tanks and utensils used in dairies, food processing plants, wineries and breweries. Using water or mechanical action, remove heavy soil, gross food particles or gross filth from hard surfaces such as Formica, stainless steel, glass, or plastic. Using an appropriate detergent or cleaner, wash surfaces or objects. Rinse with potable water. Remove any food products or packaging materials from the room or cover completely. Dilute REVOX PA Sterilant at 2 ounces to 5 gallons of water. Use fresh solution with each use. Fresh sanitizing solution should be prepared daily or more often if the solution becomes diluted or soiled. Dip, spray, brush or fill equipment with solution and allow to sit at least 1 minute at 75°F(25°C). Allow to drain well. Do not rinse. Blow or air dry.

USES

Primarily intended use with surfaces such as:

- Plastic Items
- Stainless Steel Objects or containers e.g. bedpans, tanks
- Hard Surfaces e.g. counters, sinks, bulk tanks etc.
- Floors
- Walls
- Bathroom Fixtures
- Vinyl surfaces
- Formica surfaces
- Glass

ACCEPTED
with COMMENTS
EPA Letter Dated:

JAN -6 2012

Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended, for the pesticide,
registered under EPA Reg. No. 52252-11

10/10

ACCEPTED
with COMMENTS
EPA Letter Dated:

JAN 6 6 2012

Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended, for the pesticide, 52252-1,
registered under EPA Reg. No.

STORAGE AND DISPOSAL

Do not contaminate water, food or feed by storage or disposal.

Pesticide Storage

Store in shipping carton. Do not expose to direct sunlight. Maintain temperature below 75°F (25°C). Avoid contact with combustible materials. Avoid contamination from any source, including metals, dust, etc. Such contamination may cause rapid decomposition, generation of large quantities of oxygen gas and high pressures. Store in original closed container. NEVER TAMPER WITH VENT.

Pesticide Disposal

Wastes resulting from the use of this product may be disposed of on-site by dilution in a sanitary sewer or at an approved waste disposal facility.

Pesticide wastes are acutely hazardous. Improper disposal of excess pesticide, spray mixture, or rinsate is a violation of Federal Law. If these wastes cannot be disposed of by use according to label instructions, contact your State Pesticide or Environmental Control Agency, or the Hazardous Waste Representative at the nearest EPA Regional Office for Guidance.

Container Disposal

Nonrefillable container. Do not reuse or refill this container. Offer for recycling, if available.

{The following is applicable only to volumes of 5 gallons or less}

Triple rinse as follows: Empty the remaining contents into application equipment or a mix tank and drain for 10 seconds after the flow begins to drip. Fill the container ¼ full with water and recap. Shake for 10 seconds. Pour rinsate into application equipment or a mix tank or store rinsate for later use or disposal. Drain for 10 seconds after flow begins to drip. Repeat this procedure two more times.

{The following is applicable only to volumes of greater than 5 gallons}

Triple rinse as follows: Empty the remaining contents into application equipment or a mix tank. Fill the container ¼ full with water. Replace and tighten closures. Tip container on its side and roll it back and forth, ensuring at least one complete revolution, for 30 seconds. Stand the container on its end and tip back and forth several times. Turn the container over onto its other end and tip it back and forth several times. Empty rinsate into application equipment or a mix tank or store rinsate for later use or disposal. Repeat this procedure two more times.