

105 53128-1 11. 32

U.S. ENVIRONMENTAL PROTECTION AGENCY OFFICE OF PESTICIDES PROGRAMS REGISTRATION DIVISION (WH-567) WASHINGTON, D.C. 20460	EPA REGISTRATION NO.	DATE OF ISSUANCE
	53128-1	MAY 3 1985
	TERM OF ISSUANCE	
NOTICE OF PESTICIDE: <input checked="" type="checkbox"/> REGISTRATION <input type="checkbox"/> REREGERISTRATION (Under the Federal Insecticide, Fungicide, and Rodenticide Act, as amended)		NAME OF PESTICIDE PRODUCT Payerax Water With Stabilizer

NAME AND ADDRESS OF REGISTRANT (Include ZIP code)

American Dade
 American Hospital Supply Corporation
 1851 Delaware Parkway
 Miami, FL 33125

NOTE: Changes in labeling formula differing in substance from that accepted in connection with this registration must be submitted to and accepted by the Registration Division prior to use of the label in commerce. In any correspondence on this product always refer to the above U.S. EPA registration number.

On the basis of information furnished by the registrant, the above named pesticide is hereby Registered/Reregistered under the Federal Insecticide, Fungicide, and Rodenticide Act.

A copy of the labeling accepted in connection with this Registration/Reregistration is returned herewith.


Registration is in no way to be construed as an indorsement or approval of this product by this Agency. In order to protect health and the environment, the Administrator, on his motion, may at any time suspend or cancel the registration of a pesticide in accordance with the Act. The acceptance of any name in connection with the registration of a product under this Act is not to be construed as giving the registrant a right to exclusive use of the name or to its use if it has been covered by others.

This product is conditionally registered in accordance with FIFRA sec. 3(c)(7)(A) provided that you:

1. Submit and/or cite all data required for registration/reregistration of your product under FIFRA sec. 3(c)(5) when the Agency requires all registrants of similar products to submit such data.
2. Add the phrase "EPA Registration No. 53128-1" to your label before you release the product for shipment.
3. Submit five (5) copies of your final printed labeling before you release the product for shipment. Refer to the A-79 Enclosure for a further description of final printed labeling.

If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA sec. 6(c). Your release for shipment of the product constitutes acceptance of these conditions.

A stamped copy of the label is enclosed for your records.

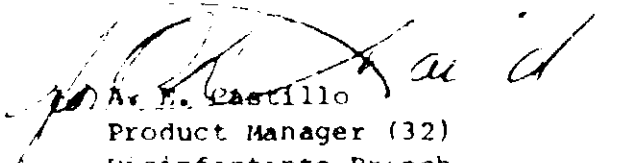
 A. E. Castillo
 Product Manager (32)
 Disinfectants Branch
 Registration Division (TS-767C)

Enclosure

ATTACHMENT IS APPLICABLE

SIGNATURE OF APPROVING OFFICIAL	DATE
---------------------------------	------

LI _____

PARAMAX®

Water Bath Stabilizer

For control of slime-forming and fungal populations in recirculating water systems. For use in the Paramax Analyzer recirculating water bath.

Material Provided: a solution containing the following Active Ingredients:

- 2,2-Dibromo-3-nitrilopropionamide.....5%
- Inert Ingredients.....95%
- E.P.A. Registration No. _____
- E.P.A. Est. _____

ACCEPTED
with COMMENTS
in EPA Letter Dated:

MAY 3 1985

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
as amended, for the pesticide
registered under EPA Reg. No.
53128-1

KEEP OUT OF REACH OF CHILDREN.

DANGER

Precautionary Statements
Hazards to Humans and Domestic Animals.

CAUSES SEVERE BURNS OF EYES. CAUSES SKIN IRRITATION. HARMFUL IF SWALLOWED.

Do not get in Eyes, on Skin, or on Clothing. Wear Safety Goggles When Handling. Wash Thoroughly After Handling.

FIRST AID: In case of eye contact, immediately flush eyes with plenty of water for at least 15 minutes. Call a physician. In case of skin contact, immediately wash skin with soap and plenty of water. Wash contaminated clothing before reuse. Get medical attention if irritation persists. If swallowed, immediately induce vomiting by giving two glasses of water and sticking finger down throat. Repeat until vomit is clear. Call a physician. Never give anything by mouth to an unconscious person.

ENVIRONMENTAL HAZARDS: This product is toxic to fish. Apply this product only as specified in this insert. Do not contaminate water by cleaning of equipment, or disposal of wastes. NOTE: Do not discharge into lakes, streams, ponds, or public waters unless in accordance with a NPDES permit. For guidance, contact your regional office of the EPA.

STORAGE AND DISPOSAL: Do not contaminate water, food, or feed by storage or disposal.

312 #23127-7 1113-

STORAGE: To maintain product quality, store at temperatures below 60C. Keep container tightly closed when not in use.

PESTICIDE DISPOSAL: Pesticide wastes are acutely hazardous. Improper disposal of excess pesticide, spray mixture, or rinsate is a violation of Federal Law. If these wastes cannot be disposed of by use according to label instructions, contact your State Pesticide or Environmental Control Agency, or the Hazardous Waste representative at the nearest EPA Regional Office for guidance.

CONTAINER DISPOSAL: Do not reuse empty container. Triple rinse (or equivalent). Replace cap and dispose of in Paramax Biohazard Disposal Bin or wrap container and throw in the trash, or by other procedures approved by state and local authorities.

DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling. For use in a hospital or clinical laboratory setting.

Add one vial of Water Bath Stabilizer to the 30 L water bath in the Paramax Analyzer every 7 days. Do not mix with other additives. In case of fouling, drain and flush the water bath and replace the ion exchange cartridge.

This product is warranted to perform as described in the labeling and in American Dade Chemistry Systems' literature, and American Dade Chemistry Systems disclaims any implied warranty of merchantability or fitness for any other purpose, and in no event shall American Dade Chemistry Systems be liable for any consequential damages arising out of the aforesaid express warranty.


Printed in USA

Manufactured for
American Dade Chemistry Systems
American Hospital Supply Corporation
Costa Mesa, CA USA 92626

195 # 55121-1 111 32

VIAL LABEL

86110-2		12 mL
Paramax [®]	DANGER	
Water Bath Stabilizer	See insert for pre-cautionary statements and directions for use.	
Store at temperatures below 60 C.		
Active Ingredients		
2,2-Dibromo-3-nitrilo-propionamide...5%		
Inert Ingredients.....95%		
E.P.A. Registration No. _____		
E.P.A. Est. _____		
Manufactured for American Dade Chemistry Systems American Hospital Supply Corporation Costa Mesa, CA USA 92626		
Lot No.	Exp. Date	VL

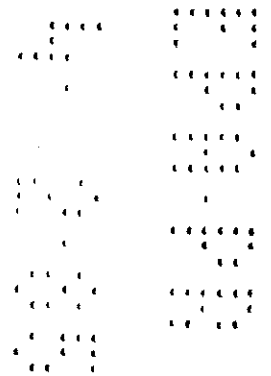

BOX LABEL

B 6110-2

B6110-2

B 6110-2

Paramax

Paramax

Para

WATER BATH
STABILIZER

WATER BATH STABILIZER

Manufactured
for

LOT NO:
EXP. DATE:

FOR USE IN THE PARAMAX ANALYZER
RECIRCULATING WATER BATH.

American DADE
Chemistry Systems
American Hospital
Supply Corporation
Costa Mesa CA 92626

Store at temperatures
below 60 C

CONTENTS: 6VIALS, 12 mL EA.

Active Ingredients:
2,2-Dibromo-3-nitrilo propionamide... 5%
Inert Ingredients.....95%
E.P.A. Registration No. _____
E.P.A. Est. _____

BL

WATER B

DANGER:
of eyes
irritat
swallow

See ins
precaut
(first
for use

BEST AVAILABLE COPY

**COMPANY
NUMBER** 53886-

**PRODUCT
SERIAL NO.** 2

**LABEL
APPROVAL
DATE** 5-6-85