

Sterilizing and Disinfecting Concentrate Solution Glutaraldehyde/Synergistic Dual Phenolics

ColdCide-10

Can Use and Reuse
For Up to 30 Days
In a Manual Bucket System.

- * Dilute for disinfection
- * Corrosion inhibitor
- * Stable, low pH formula
- * Dilute for sterilization
- * No activator required

Sporicidal Virucidal*	Tuberculocidal Fungicidal	Bactericidal Pseudomonacidal
--------------------------	------------------------------	---------------------------------

Active ingredients
Glutaraldehyde 10.0%
o-Phenyphenol 0.5%
m-Cresol Amylphenol 0.1%
Inactive ingredients 89.4%
Total 100.00%

Product No.
MX-5332

EXPIRATION DATE: 2 YEARS AFTER
MANUFACTURING DATE

**KEEP OUT OF REACH OF CHILDREN
DANGER**

See right panel for additional precautionary statements.

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 (HUMAN IMMUNODEFICIENCY VIRUS OR AIDS VIRUS) OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS: **Personal Protection:** Wear appropriate barrier protection such as latex gloves, gowns, masks or eye coverings. **Cleaning Procedure:** Blood and other body fluids must be thoroughly cleaned from surfaces and objects before any disinfection or sterilization. **Contact Time:** While the HIV-1 virus is inactivated in 30 seconds, use the recommended contact time for disinfection of other organisms on this label. **Infectious Materials Disposal:** Cleaning materials that may contain blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious disease.

Patented E.P.A. Reg. No. 55195-2

E.P.A. Est. No. 46781-CO-1

DIRECTIONS FOR USE:

It is a violation of U.S. Federal law to use this product in a manner inconsistent with its labeling.

DISINFECTION: All articles and surfaces to be disinfected must be thoroughly cleaned and rough dried before placing in disinfecting solution. Prepare a 1:20 dilution (4 gallons) by pouring entire contents of this container (25.6 fl. oz.) into an empty five-gallon container and filling with tap water to 4-gallon marker. Completely immerse instruments in disinfecting solution for 10 minutes at room temperature (20°C/68°F or higher). At a 1:20 dilution, ColdCide-10 kills *Salmonella choleraesuis*, *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Trichophyton mentagrophytes*, *Cryptosporidium parvum*, and *Candida albicans*. Mycobacterial tuberculosis is completely inactivated at room temperature (20°C/68°F or higher) in 10 minutes. Completely inactivate the following viruses: HIV-1, Herpes simplex virus 1 & 2 and Influenza A, HK, and inactivated polio vaccine. ColdCide-10 kills Adenovirus 1, Poliovirus 1&2, Rotavirus SA 11, Herpes virus 1, and Influenza virus A & B vaccine virus.

Use ColdCide-10 to inactivate the HIV-1 AIDS virus on hard, non-porous surfaces. Immerse in a 1:20 dilution of ColdCide-10 in a bucket or bin at a contact time of 30 seconds at room temperature (20°C/68°F or higher). After reuse of a 1:20 diluted solution for 30 days in manual bucket or tray systems. Although efficacy at a 30-second contact time has been shown to be adequate against HIV-1, this time would not be sufficient for other organisms. Use a 10-minute contact time for disinfection against all of the organisms claimed.

Remove instruments using aseptic technique, rinse with potable water, and dry. This solution may be used and reused for up to 30 days.

STERILIZATION: All articles and surfaces to be sterilized must be thoroughly cleaned and rough dried before placing in sterilizing solution. Prepare a 1:5 dilution by pouring entire contents of this container (25.6 fl. oz.) into an empty gallon container and filling with tap water. Completely immerse instruments in sterilizing solution for 6 hours at room temperature (20°C/68°F or higher). Remove instruments using aseptic technique, rinse with sterile water and dry. This solution may be used and reused for up to 30 days. A 1:20 dilution may be used for sterilization with 12 hours exposure at 20°C.

PRECAUTIONARY STATEMENTS:

Hazard to Humans and Domestic Animals

DANGER: Corrosive. Causes eye damage. Do not get in eyes, on skin or on clothing. Wear safety glasses and rubber gloves when handling. Harmful if swallowed. Avoid contamination of food. Avoid reuse of unwashed disinfectant-soaked gloves. Avoid prolonged inhalation.

STATEMENT OF PRACTICAL TREATMENT: If in eyes, immediately flush with water for at least 10 minutes. Get medical attention. If on skin, wash thoroughly with water. Wash contaminated clothing before reuse.

WASH THOROUGHLY AFTER HANDLING.

STORAGE AND DISPOSAL:

Store in cool, well-ventilated area. Unused product may be discarded by pouring down drain and flushing with large quantity of water. Rinse empty container thoroughly and discard in trash.

For use in hospitals, medical offices, dental offices, veterinary laboratories, convalescent care centers and other health care facilities.

Specifically recommended for sterilization and disinfection of Medical Instruments used in: Anesthesiology, Endoscopy, Respiratory therapy, Surgery and Urology. Dental Instruments such as: X-ray films, X-ray cassettes, Sharp instruments, Forceps, pliers, Cages, Plastic objects, Rubber objects, Thermometers, and Endoscopes.

Contents 25.6 fluid oz.

Made in U.S.A.

Production lot # is stamped on bottom of container.

Manufactured by: ColdCide, Inc., 909 Pershing Dr., Silver Spring, Md. 20910

BEST AVAILABLE COPY