

55364-7

UNITED STATES ENVIRONMENTAL PROTECTION AGENCY

9-8-2003

1/2

SEP - 8 2003

Bob MacDonald  
Agent for Maril Products  
Maril Products  
320 W 6<sup>th</sup> Street  
Tustin , CA 92780

Subject: Control III Elite  
EPA Registration No. 55364-7  
Amendment Dated: June 9, 2003  
EPA Received Date: June 16, 2003

Dear Mr. Bob MacDonald,

The following amendment submitted in connection with registration under section 3(c)(7)(a) of the Federal Insecticide, Fungicide, and Rodenticide Act (FIFRA) is acceptable with the following condition.

**Proposed Amendment**

- Update First Aid Statements in compliance with PR Notice 2001-1

**Condition**


Revise the Storage and Disposal section as follows

1. Store in original containers in a locked storage area in accessible to children. Do not reuse empty container. Wrap and discard in the trash.

**General Comments**

A stamped label is enclosed for your records. Please submit three (3) copies of your final printed label before distributing or selling the product bearing the revised labeling. If the above condition is not compiled with, the registration will be subject to cancellation in accordance with FIFRA section (e). Your release for shipment of the product bearing the amended labeling constitutes acceptance of this condition.

Sincerely,

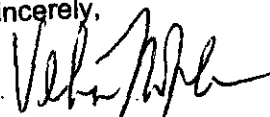

Velma Noble  
Product Manager (31)

CONCURRENCES						
SYMBOL						
SURNAME						
DATE						

**KILLS HIV-1 ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS**

in the health care settings (Hospitals, Nursing Homes) or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS).

**SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.**

**PERSONAL PROTECTION:** When handling items soiled with blood or body fluids use disposable latex gloves, gowns, masks, or eye coverings.

**CLEANING PROCEDURES:** Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this product.

**CONTACT TIME:** Allow surface to remain wet for 10 minutes.

**DISPOSAL OF INFECTIOUS MATERIALS:** Blood and other body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

**PRECAUTIONARY STATEMENTS**

**HAZARDS TO HUMANS AND DOMESTIC ANIMALS**

**WARNING:** Causes eye and skin irritation. Do not get in eyes, on skin, or on clothing. Harmful if swallowed. Avoid contamination of food. Remove contaminated clothing and wash before reuse. Wash thoroughly with soap and water after handling.

**STORAGE AND DISPOSAL**

Do not contaminate water, food, or feed by storage or disposal.

**CONTAINER DISPOSAL:** Do not reuse container. Wrap container and put in trash.

**STORAGE:** Store in a dry place no lower in temperature than 50°F or higher than 120°F.

**FIRST AID**

**If in eyes:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first five minutes, then continue rinsing eye. Call a poison control center doctor for treatment advice.

**If on skin or clothing:** Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment advice.

**If swallowed:** Call a poison control center or doctor immediately for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to do so by the poison control center or doctor. Do not give anything by mouth to an unconscious person.

**If inhaled:** Move person to fresh air. If person is not breathing, call 911 or an ambulance then give artificial respiration, preferably by mouth-to-mouth, if possible. Call a poison control center or doctor for further treatment advice.

**Note to Physicians:** Probable mucosal damage may contraindicate the use of gastric lavage.

**HOT LINE NUMBER**

For emergency medical advice, call your local poison control center (1-800-222-1222) or doctor. Have the product container or label with you when seeking medical advice or treatment.

# CONTROL III Elite

## DISINFECTANT \*VIRUCIDE TUBERCULOCIDAL CLEANER

<b>ACTIVE INGREDIENTS</b>	
n-Allyl (60% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chlorides	0.105%
n-Allyl (68% C12, 32% C14) dimethyl ethylbenzyl ammonium chlorides	0.105%
<b>INERT INGREDIENTS</b> .....	91.790%
<b>TOTAL</b> .....	100.000%

### KEEP OUT OF REACH OF CHILDREN WARNING

SEE LEFT PANEL FOR ADDITIONAL  
PRECAUTIONARY STATEMENTS

EPA REG. NO. 55364-7  
EPA EST. NO. 40873-CA-01, GA-01  
Net Contents 32 fluid ounces

**MARIL PRODUCTS, INC.**  
320 West 6th Street  
Tustin, CA 92780 USA  
714-544-7711 • 800-546-7711

### DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi critical medical devices prior to sterilization or high level disinfection.

Control III Elite is specifically designed for the cleaning and disinfection of non-critical medical devices such as bed frames, wheel chairs, physical therapy equipment and other hard non-porous environmental surfaces in hospitals, nursing homes, durable medical equipment suppliers, doctor offices, schools, gymnasiums, homes and animal clinics. Control III Elite may also be used to pre-clean or decontaminate instruments in hospitals or other medical facilities prior to sterilization or high level disinfection.

**DISINFECTION AND CLEANING:** Remove gross filth or heavy soil prior to application of Control III ELITE. Apply solution as is - DO NOT DILUTE - to equipment with a cloth, sponge or by spraying so as to thoroughly wet all surfaces. For disinfection surfaces must remain wet for ten minutes. Allow to air dry or wipe off with a clean cloth or sponge.

**PRECLEANING AND SOAKING:** Control III ELITE can be used for pre-cleaning and soaking small medical devices prior to sterilization or high level disinfection. All items to be soaked must be pre-cleaned to remove gross filth or heavy soil. Items should then be fully immersed in Control III ELITE solution for ten minutes; then thoroughly rinsed and allowed to air dry.

**TUBERCULOCIDAL ACTIVITY:** This product exhibits disinfectant efficacy against Mycobacterium tuberculosis (BCG) in 10 minutes at 20 degrees Centigrade when used as directed on previously cleaned hard non-porous inanimate surfaces.

**BACTERICIDAL ACTIVITY:** When used as directed, this product exhibits effective disinfectant activity against the organisms: Staphylococcus aureus, Salmonella choleraesuis, Pseudomonas aeruginosa, Escherichia coli 0157:H7, and meets the requirements for hospital use.

**\*VIRUCIDAL ACTIVITY:** This product, when used on environmental, inanimate, non-porous surfaces exhibits effective virucidal activity against HIV-1 (associated with AIDS) and Canine Parvo virus.

P/N 10007 REV. B

ACCEPTED  
with COMMENTS  
in EPA Letter Dated:

SEP 8 - 2003

Under the Federal Insecticide,  
Fungicide, and Rodenticide Act as  
amended, for the pesticide,  
registered under EPA Reg. No. 55364-7

2/6