

← PEEL HERE

C-100A

CIDE-SWIPES™

• DISINFECTANT
• VIRUCIDE*
• CLEANER

Contents: 1 Towellette (Net wgt. 0.58 oz.) (16.5 gms.)

Cide Swipes are pre-saturated wipes containing broad-spectrum antibacterial and antifungal agents. For use in medical, dental, home, office, and institutional applications.

TO USE:

- This is a variation of Federal Law. Use this product in a manner inconsistent with its labeling.
- Tear open protective wrap. Allow towel to unfold by holding and gently shaking. Wipe surface and allow to remain wet for 60 seconds. Each 1 wipe treats and disinfects a surface area 40 times its size.

SEE PRODUCT INFORMATION SHEET FOR ADDITIONAL INSTRUCTIONS

CAUTION:

- May cause eye and skin irritation upon contact. In case of contact with eye or skin, flush thoroughly with water. If irritation persists, contact a physician.
- Avoid contact with food.
- Keep (un)opened towels in dry area. Do not open until ready to use. Do not re-use towel. Discard after use.

KEEP OUT OF REACH OF CHILDREN

*Cide Swipes are effective as a virucide against Herpes Type 1 and Herpes Simplex Type 1 and 2, for a 15 minute contact time on pre-cleaned hard nonporous surfaces.

EPA Registration No. 52631-111-1 • EPA Registration No. 56095-2

Made by IAR PRODUCTS, INC. Maywood, NJ 07607

ACCEPTED

SEP 19 1989

Under the Federal Insecticide
Fungicide and Rodenticide Act
as amended for the pesticide
registration for
EPA Reg. No. 526095-2