

ACCEPTED
 SEP 19 1989
 Under the Federal Insecticide, Fungicide, and Rodenticide Act, as amended, for the pesticide registered under EPA Reg. No. **56095-4**

← PEEL HERE

P-100B CIDE-SWIPES

- DISINFECTANT
- VIRUCIDE*
- CLEANER

Contents: 1 Towellette
(Net wgt. 0.367 oz. / 10.4 gms)

Cide-Swipes are pre-saturated wipes containing known antibacterial and antiviral agents. For use in medical, dental, home, school, and institutional applications.

Active Ingredient	
GLUTARAL DEHYDE	0.24%
Inert Ingredients**	99.76%

**Does not include weight of dry towellette

TO USE:

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

To disinfect heavily soiled surfaces, clean first with one Cide-Swipe and follow with the use of a second fresh Cide-Swipe. Simply tear open seal and pouch, thoroughly wipe surface for 60 seconds. Allow the treated surface to remain wet for 3 minutes or let air dry. Each pad cleans a surface area 40 times its size.

SEE PRODUCT INFORMATION SHEET FOR ADDITIONAL INSTRUCTIONS

Caution:

KEEP OUT OF REACH OF CHILDREN

- May cause eye and skin irritation upon contact. In case of contact with eye or skin, flush thoroughly with water. If irritation persists contact a physician.
- Dispose of container and used towellette in trash.

*Cide-Swipes are effective as a virucide against Poliovirus Type 1 and Herpes Simplex Type 1 and 2, for a 15 minute contact time on pre-cleaned, hard, nonporous surfaces.

PA Establishment No. 52631 NJ001 • EPA Registration No. 56095-4

Made by LAB PRODUCTS, INC. Maywood, NJ 07607

