

PM: 31 SF 095-4

Page 142

P-100B Packet Label 9/22/94 23113 (Page 1)

PEEL HERE

CIDE-SWIPES

DISINFECTANT

VIRUCIDE*

CLEANER

P-100B

Contents: 1 Pad
(Net wt) 0.367 oz. (10.4 gms.)

Cide-Swipes are pre-saturated wipes containing known anti-bacterial and antiviral agents. For use in medical, dental, home, school and institutional applications.

Active Ingredient

Glutaraldehyde	0.24%
Inert Ingredients**	89.76%

**Does not include weight of dry pad.

DIRECTIONS FOR USE:

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

To disinfect heavily soiled surfaces, clean first with one Cide-Swipes and follow with the use of a second fresh Cide-Swipes.

Peel open and remove front panel. Pad is attached to back panel. Use back panel as an applicator. Thoroughly wipe surface for 60 seconds. Allow treated surface to remain wet for 3 minutes or let air dry. Each pad cleans and disinfects a surface area 40 times its size.

KEEP OUT OF REACH OF CHILDREN

CAUTION:

- May cause eye and skin irritation on contact. In case of contact with eye or skin, flush thoroughly with water. If irritation persists, contact a physician.
- Avoid contact with food.

CONTAINER DISPOSAL:

- Place used pad in the trash container. Do not re-use pad.

* Cide-Swipes are effective as a virucide against HIV-1 (AIDS Virus) for a 3 minute contact time and Poliovirus Type 1 for a 15 minute contact time on pre-cleaned, hard, nonporous surfaces.

See back panel for Special Instructions for cleaning and Decontamination Against HIV Contaminated surfaces.

EPA Establishment No. 52631-NJ001 • EPA Registration No. 66095-4
Made by: LAB PRODUCTS, INC., Maywood, NJ 07807

BEST AVAILABLE COPY

ACCEPTED
 JAN 17 1995
 Under the Federal Insecticide, Fungicide, and Rodenticide Act, as amended, for the pesticide registered under EPA Reg. No. 56095-4

P-100B (Rear Label)

Corrected Draft Label

Ref: EPA Letter Dated November 22, 1994

**Special Instructions for Cleaning and
Decontamination Against HIV of
Surfaces or Objects Soiled
With Blood or Body Fluids.**

Personal Protection: When handling items soiled with blood or other body fluids, disposable latex gloves, gowns, surgical mask and eye protection are recommended.

Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of the disinfectant.

Contact Time: Cide-Swipes™ are effective as a virucide against HIV-1 (AIDS Virus) for a three minute contact time.

Disposal of Infectious Material: Blood and other body fluids should be autoclaved and disposed according to local regulations for infectious waste disposal.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

ACCEPTED

JAN 17 1995

Under the Federal Insecticide,
Fungicide and Rodenticide Act,
as amended, for the pesticide
registered under
EPA Reg. No. 56095-4

BEST AVAILABLE COPY