

PM 82 # 56392 - 1

TRAP  
WHEN  
NE

SHAKE OFTEN



SHAKE OFTEN

**PRECISE**

**Hospital Foam  
Cleaner Disinfectant**

**STREPTOCOIDAL ■ STAPHYLOCIDAL ■ PSEUDOMONACIDAL ■ FUNGICIDAL\*\*  
SALMONELLACIDAL ■ TUBERCULOCIDAL ■ VIRUCIDAL\***

\*Human Immunodeficiency Virus, Type 1 (HIV-1) ■ Human Simplex Virus, Type 2 (HSV-2) ■ Influenza A-2, strain Hong Kong (H3N2) ■ Kilo Carina Parvo Virus ■ Poliovirus, Type 1, Mahoney ■ Vaccinia Virus, ATCC VR-117  
\*\*Control pathogenic fungi such as athlete's foot fungi on hard non-porous surfaces.

For Hospital Hard Surfaces. Quick spray and wipe for cleaning. Always properly mixed and ready to use. Put this product to work throughout the hospital.

World - patient room, emergency room, recovery room, operating room, laboratory, clinic, bathroom, physician office, physical therapy, nursing station, radiology, morgue, dental operator, veterinary hospital.

Cleans and Shines all hospital washable surfaces—tile, tubs, bowls, basins, sinks, showers, urinals, empty bed pans, tables, cabinets, doors and door handles, empty enamel basins, floors, walls, woodwork, work station tables and counter tops, housekeeping carts, vanity tops, vending machines, bed springs and frames, bedside furniture, dressing and surgical carts, wheelchairs, walkers, stainless steel, chrome, plastic, metal, vinyl, fiberglass, baked enamel, porcelain, examination and x-ray tables, film cassettes, chart holders, shower chairs, sitz baths, commodes and commode seats, instrument cabinets and trays, catheter trays, electrodes, bacteriology laboratory benches, counter tops, whirlpool baths, operating tables, emergency vehicles, telephones, reception counters, dental hand pieces, dental chairs, medical instruments.

Disinfects—Kills germs commonly found on hard non-porous hospital surfaces. See directions for disinfecting. Deodorizes—Kills odor causing germs on hard surfaces. Leaves surfaces smelling clean and fresh.

Labor Saving—Just spray on—wipe off. No rubbing, rinsing or scouring.

Won't Scratch—Contains no abrasive to scratch or leave gritty deposit.

Removes Hospital Sells—Grime, greasy soil, soap scum, water marks, scuff marks, blood, drainage, stains, smears, fingermarks.

**DIRECTIONS FOR USE**

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

Directions: Shake well. Hold can six inches from surface.

To Clean: Quick spray and wipe clean with a damp sponge, cloth or paper towel. No rinsing needed.

To Clean and Disinfect: Spray surface thoroughly until completely covered with foam, allow ten minutes to wet surface and disinfect. Wipe clean with damp sponge, cloth or paper towel. No rinsing needed. If gross soil is present, remove prior to disinfecting.

To Clean and Disinfect for TB: Spray surface thoroughly until completely covered with foam, allow ten minutes to wet surface and disinfect. Wipe clean with a damp sponge, cloth or paper towel. No rinsing needed. Tuberculecidal activity requires ten minutes contact at room temperature.

To Clean and Disinfect for HIV-1 (AIDS Virus): Spray pre-cleaned surface thoroughly until completely covered with foam, allow one minute to wet surface and disinfect. Wipe clean with damp sponge, cloth or paper towel. No rinsing needed. Kills HIV-1 (AIDS Virus) in one minute at room temperature (68° - 77°F, 20° - 25°C). Contact time required for other organisms is ten minutes.

**KILLS HIV ON PRECLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS).

Special Instructions for Cleaning and Decontamination Against HIV-1 (AIDS Virus) on Surfaces/Objects Soiled with Blood/Body Fluids:

Personal Protection: When handling items soiled with blood or body fluids, appropriate personal protective equipment such as disposable plastic gloves, gowns, masks, or eye coverings should be considered.

Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects prior to disinfecting.

Contact Time: PRECISE Kills HIV-1 (AIDS Virus) in one minute. Contact time required to kill other organisms is ten minutes.

Disposal of Infectious Materials: Blood and other body fluids and cleaning materials should be autoclaved and disposed of according to federal, state and local regulations for infectious waste disposal.

**PRECAUTIONARY STATEMENTS**

Irritant to Humans and Domestic Animals.

WARNING: Contents under pressure. May cause eye irritation. Avoid contact with eyes. In case of contact, flush eyes with plenty of water. Get medical attention.

Storage: Do not use or store near heat or open flame. Store out of reach of children.

Disposal: Return cap and discard canister in trash. Do not puncture or incinerate.


© 1982-1985, Caltech Industries, Inc.

© 1985, Caltech Industries, Inc.

P.O. Box 1138, Midland, MI 48641-1138

1-800-234-7700


**REORDER NO. 40513**

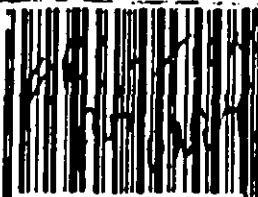

7 29969 40513 7



**PRECISE**  
**Hospital Foam  
Cleaner Disinfectant**

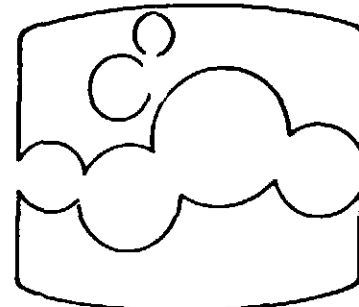

**For Hard Surfaces  
Cleans, Shines  
Disinfects  
Deodorizes  
Won't Scratch**

EPA REG. NO. 56392-1

Active ingredients:

o-Phenylphenol ..... 0.37%


Inert ingredients ..... 99.63%

**HOSPITAL SIZE**

**NET WT. 20 OZ. (1 LB. 4 OZ.)**

**KEEP OUT OF REACH OF CHILDREN**

**WARNING  
READ COMPLETE PRECAUTIONS AND  
DIRECTIONS ON REAR PANEL**

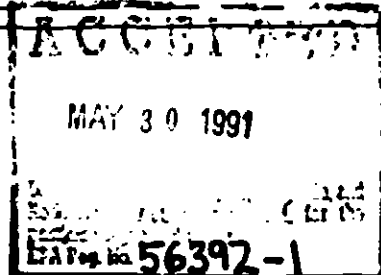

L.P.A. Est. No. 18746-11-1; 11525-81-1; 11525-11-1; 11525-CA-1; 37015-01-1; 43716-01-1. Superscript used is second letter of Lat. No.

Doc 03747/1/02-25-88

**BEST AVAILABLE COPY**

Copy 1 of 5