

PH 5053:1:WP5.0:CR-31:Farmer:C.Disk:KEVRIC:05/20/91:CL:WO:CL:5Fm

US ENVIRONMENTAL PROTECTION AGENCY
OFFICE OF PESTICIDES PROGRAMS
REGISTRATION DIVISION (TS-767)
WASHINGTON, DC 20460

NOTICE OF PESTICIDE: ☒ REGISTRATION
☐ RENEWAL
(Under the Federal Insecticide, Fungicide,
and Rodenticide Act, as amended)

EPA REGISTRATION NO.
50532-7

DATE OF ISSUANCE
MAY 29 1991

TERM OF ISSUANCE

NAME OF PESTICIDE PRODUCT
Dispatch Hospital
Stainless & Plastic
Cleaner Disinfectant

NAME AND ADDRESS OF REGISTRANT (Include ZIP code)

Caltech Industries, Inc.
217 East Main Street
Midland, MI 48640

NOTE: Changes in labeling formula differing in substance from that accepted in connection with this registration must be submitted to and accepted by the Registration Division prior to use of the label in commerce. In any correspondence on this product always refer to the above U.S. EPA registration number.

On the basis of information furnished by the registrant, the above named pesticide is hereby Registered/Reregistered under the Federal Insecticide, Fungicide, and Rodenticide Act.

A copy of the labeling accepted in connection with this Registration/Reregistration is returned herewith.

Registration is in no way to be construed as an indorsement or approval of this product by this Agency. In order to protect health and the environment, the Administrator, on his motion, may at any time suspend or cancel the registration of a pesticide in accordance with the Act. The acceptance of any name in connection with the registration of a product under this Act is not to be construed as giving the registrant a right to exclusive use of the name or to its use if it has been covered by others.

This product is conditionally registered in accordance with FIFRA section 3(c)(7)(A) provided that:

1. Submit/enclose all data or other material required for registration/reregistration of your product under FIFRA section 3(c)(5) or FIFRA section 4 when the Agency requires all registrants of similar products to submit such data.

2. Make the labeling changes listed below before you release the product for shipment:

a. Add the phrase "EPA Registration No. 50532-7."

b. Delete "General Classification" from the back panel.

3. Submit five (5) copies of your final printed labeling, before you release the product for shipment. Refer to the 7-7 enclosure for a further description of final printed labeling.

☐ ATTACHMENT IS APPLICABLE

SIGNATURE OF APPROVING OFFICIAL

DATE

If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA section 6(e). Your release for shipment of the product constitutes acceptance of these conditions.

A stamped copy of the label is enclosed for your records.

W8

Walter C. Francis
Product Manager (32)
Antimicrobial Program Branch
Registration Division (H7505C)

Enclosures



DISPATCH® Hospital

Stainless & Plastic Cleaner Disinfectant

with Bleach (5500 ppm NaOCl)

Delete



ACCEPTED
with COMMENTS
in EPA Letter Dated:

MAR 06 1988

Under the Federal Insecticide,
Fungicide, and Rodenticide Act
Registration No. EPA Reg. No.

56392-7

Disinfects • Cleans • Lifts Stains • Cuts Grease

EPA Reg. No. 56392-7

EPA Est. No. 62801-MI-001

NET 16 FL. OZ. (1 PINT)

Active Ingredient

Sodium Hypochlorite....0.55%

Inert Ingredients.....99.45%

KEEP OUT OF REACH OF CHILDREN

CAUTION

READ COMPLETE PRECAUTIONS ON REAR PANEL

DRAFT FRONT LABEL
Copy 1 of 5

DISPATCH®

Hospital Stainless & Plastic Cleaner Disinfectant

Delete

~~WITH BLEACH~~ (5500 ppm NaOCl)
STAPHYLOCIDAL-SALMONELLACIDAL-PSEUDOMONACIDAL
TUBERCULOCIDAL-VIRUCIDAL+

+Human Immunodeficiency Virus, Type 1(HIV-1)(AIDS Virus)
+Herpes Simplex Virus, Type 2(HSV-2) +Poliovirus, Type 1, Mahoney

•For hospital non-porous surfaces such as stainless steel, plastic, ceramic tile, laminated plastic countertops, vinyl, tile, linoleum, porcelain enamel, fiberglass, chrome, porcelain and grout which must be cleaned and disinfected. A rapid acting cleaner disinfectant that can't scratch. Effective on organic stains such as fruit, coffee, tea, food residues, grease, finger prints and laboratory stains.

•Useful in outpatient clinic, central service area, kitchen and food service area, laboratory, clinic, emergency room, radiology, operating room, physical therapy, nursing station reception counter, restroom, physician office, veterinary hospital, nursing home, dental operatory.

DIRECTIONS FOR USE

GENERAL CLASSIFICATION

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

•**To Clean:** Hold bottle 6-8" from surface. Direct gentle spray to soiled area or apply with damp applicator. Wipe clean with a damp applicator such as sponge/cloth/paper towel. Stubborn stains-allow 5 minute contact. No rinsing needed. If streaking is observed rinse with water. Polish with dry cloth.

•**To Clean and Disinfect in one step:** Spray surface thoroughly until completely wet. Allow 1 minute at room temperature (68-77°F) to wet surface and disinfect. Wipe clean with a damp sponge, cloth or paper towel. No rinsing needed. To kill TB allow 2 minute contact at room temperature. Gross soil should be removed prior to cleaning and disinfecting.

•**KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-1)(associated with AIDS).

**Special Instructions for Cleaning and Decontamination Against
HIV-1(AIDS Virus) on Surfaces/Objects Soiled with Blood/Body Fluids**

Personal Protection: When handling items soiled with blood or body fluids, personal protective equipment such as disposable plastic gloves, gowns, masks, or eye coverings should be used.

Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before disinfecting.

Contact Time: Leave surfaces wet for 1 minute.

Disposal of Infectious Materials: Blood, body fluids, and cleaning materials should be autoclaved and disposed according to federal, state and local regulations for infectious waste disposal.

STORAGE AND DISPOSAL

Store this product in a cool dry area, away from direct sunlight and heat to avoid deterioration. In case of spill, flood areas with large quantities of water. Product or rinsates that cannot be used should be diluted with water before disposal in a sanitary sewer. Reuse empty container only with DISPATCH Refill. When discarding container, place in trash collection. Do not contaminate food or feed by storage, disposal or cleaning of equipment.

PRECAUTIONARY STATEMENTS

Hazards to humans and domestic animals.

•**STATEMENT OF PRACTICAL TREATMENT:** If In Eyes: Flush with plenty of water for 15 minutes. Get medical attention.

•**PHYSICAL AND CHEMICAL HAZARD:** Do not use this product with ammonia or acids such as vinegar, rust removers or toilet bowl cleaners. This product not intended for use on fabrics.

•Trademark of Caltech Industries, Inc.

Manufactured for
©1984 Caltech Industries, Inc.
P.O. Box 1139, Midland MI 48641
1-800-234-7700

REORDER NO. 68965

EXP. DATE: _____

DRAFT BACK LABEL

Copy 1 of 5