

PM 32

56392-7


10/1



DISPATCH[®]
Hospital
Cleaner Disinfectant
with Bleach (5500 ppm NaOCl)



Disinfects • Cleans
Lifts Stains • Cuts Grease

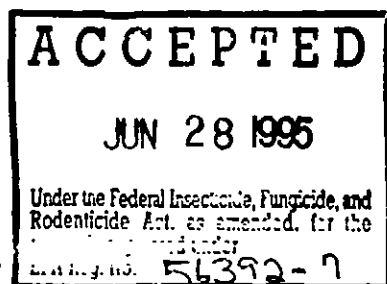

EPA Reg. No. 56392-7

EPA Est. No. 62801-MI-001

Active Ingredient: Sodium Hypochlorite 0.55%
 Inert Ingredients 99.45%

KEEP OUT OF REACH OF CHILDREN

CAUTION

READ COMPLETE PRECAUTIONS ON REAR PANEL

NET 16 FL. OZ. (1 Pint)

DISPATCH[®] Hospital Cleaner Disinfectant with Bleach (5500 ppm NaOCl)
 STAPHYLOCCIDAL • SALMONELLICIDAL • PSEUDOMONACIDAL • TUBERCULOCIDAL • VIRUCIDAL +
 + Human Immunodeficiency Virus, Type 1 (HIV-1) (AIDS Virus)
 + Herpes Simplex Virus, Type 2 (HSV-2) + Poliovirus, Type 1, Mahoney

• For hospital non-porous surfaces such as stainless steel, plastic, ceramic tile, laminated plastic countertops, vinyl, tile, linoleum, porcelain enamel, fiberglass, chrome, porcelain and grout which must be cleaned and disinfected. A rapid acting cleaner disinfectant that can't scratch. Effective on organic stains such as fruit, coffee, tea, food residues, grease, finger prints and laboratory stains.
 • Useful in outpatient clinic, central service area, kitchen and food service area, laboratory, clinic, emergency room, radiology, operating room, physical therapy, nursing station reception counter, restroom, physician office, veterinary hospital, nursing home, dental operator.

DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

- **To Clean:** Hold bottle 6-8" from surface. Direct gentle spray to soiled area or apply with damp applicator. Wipe clean with a damp applicator such as sponge/cloth/paper towel. Stubborn stains - allow 5 minute contact. No rinsing needed. If streaking is observed, rinse with water. Polish with dry cloth.
- **To Clean and Disinfect in one step:** Spray surface thoroughly until completely wet. Allow 1 minute at room temperature (68-77°F) to wet surface and disinfect. Wipe clean with a damp sponge, cloth or paper towel. No rinsing needed. To kill TB allow 2 minute contact at room temperature. Gross soil should be removed prior to cleaning and disinfecting.
- **KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-1) (associated with AIDS).

Special Instructions for Cleaning and Decontamination Against HIV-1 (AIDS Virus) on Surfaces/Objects Soiled with Blood/Body Fluids

Personal Protection: When handling items soiled with blood/body fluids, personal protective equipment such as disposable plastic gloves, gowns, masks, or eye coverings should be used.
Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before disinfecting. **Contact Time:** Leave surfaces wet for 1 minute.

Disposal of Infectious Materials: Blood, body fluids, and cleaning materials should be autoclaved and disposed according to federal, state and local regulations for infectious waste disposal. This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

STORAGE AND DISPOSAL

Store this product in a cool, dry area, away from direct sunlight and heat to avoid deterioration. In case of spill, flood areas with large quantities of water. Product or rinsates that cannot be used should be diluted with water before disposal in a sanitary sewer. Refill empty container only with DISPATCH Refill. When discarding container, place in trash collection. Do not contaminate food or feed by storage, disposal, or cleaning of equipment.

PRECAUTIONARY STATEMENTS • Hazards to Humans and Domestic Animals

- **STATEMENT OF PRACTICAL TREATMENT: If In Eyes** - Flush with plenty of water for 15 minutes. Get medical attention.
- **PHYSICAL OR CHEMICAL HAZARDS:** Do not use this product with ammonia or acids such as vinegar, rust removers, or toilet bowl cleaners. This product not intended for use on fabrics.

EXP. DATE 3/97

Manufactured for: ©1984 Caltech Industries, Inc.
 P.O. Box 1139, Midland, MI 48641
 1-800-234-7700

**Trademark of Caltech Industries, Inc.

REORDER
 NO. 68965

**BEST COPY AVAILABLE**