

### DISPATCH®

Hospital Cleaner Disinfectant Towels with Bleach

\* Ready-to-use towels pre-moistened with stabilized bleach stains such as crystal violet, safranin and iodine.

(5200 ppm) and detergent solution

May be used to clean & disinfect in one step

#### Kills bacteria & viruses in 1 Minute

Staphylococcus aureus, Salmonella choleraesuis, Pseudomonas HIV-1(AIDS virus), Herpes Simplex (HSV-2), Poliovirus (Type 1,-Mahoney)

### Kills TB\* in 2 Minutes

\* Mycobacterium tuberculosis

### EPA Reg.No.56392-8

EPA Est. Nos. 65288-WI-2, 50757-WI-1, 41925-WI-1

Active Ingredient: Sodium Hypochlorite....0.52%

inert ingredients......99.48%

# KEEP OUT OF REACH OF CHILDREN **CAUTION NET CONTENTS**

### **DISPATCH®** Hospital

### Cleaner Disinfectant Towels with Bleach

\*Rapid acting cleaner disinfectant towels that won't scratch "Stable to expiration date on package \*May be safely used on hospital non-porous surfaces such as stainless steel, plastic laminates, tile and porcelain \*For medical facilities such as laboratory, out-patient clinic, central service. radiology, operating room, nursing station, emergency room, home healthcare settings \*Effective on organic stains such as fingerprints \*Laboratory applications include histology stains and hematology stains as well as gram

#### **DIRECTIONS FOR USE**

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling. To Clean: Wipe surface with towel until clean. If streaking is observed, wipe with a damp cloth or paper towel

To Clean and Disinfect in One Step: Wipe surface with towel until completely wet. Allow 1 minute at room temperature (68-77°F) to disinfect. To kill TB allow 2 minute contact at room temperature. Gross soil should be removed prior to disinfecting. Gloves should be worn.

KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in health care settings or other settings in which there aeruginosa, Streptococcus pyogenes, Trichophyton mentagrophytes, is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids. and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficency virus Type 1 (HIVassociated with AIDS) Special Instructions for Cleaning and Decontamination
Against HIV-1(AIDS virus) on Surfaces/Objects Soiled with Blood/Body Fluids;
personal Protection: When handling items soiled with blood/body fluids, personal protective
equipment such as disposable plastic gloves, gowns, masks, or eye coverings should be used. Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before disinfecting. Contact time: Leave surfaces wet for 1 minute. Disposal of Infectious Materials: Blood, body fluids, and cleaning materials should be autoclaved and disposed according to federal, state and local regulators for infectious waste disposal.

> THIS PRODUCT IS NOT TO BE USED AS A TERMINAL STERILANT/HIGH LEVEL DISINFECTANT ON ANY SURFACE OR INSTRUMENT THAT (1) IS INTRODUCED DIRECTLY INTO THE HUMAN BODY, EITHER INTO OR IN CONTACT WITH THE BLOODSTREAM OR NORMALLY STERILE AREAS OF THE BODY, OR (2) CONTACTS INTACT MUCOUS MEMBRANES BUT WHICH DOES NOT ORDINARILY PENETRATE THE BLOOD BARRIER OR OTHERWISE ENTER NORMALLY STERILE AREAS OF THE BODY. THIS PRODUCT MAY BE USED TO PRECLEAN OR DECONTAMINATE CRITICAL OR SEMI-CRITICAL MEDICAL DEVICES PRIOR TO STERILIZATION OR HIGH LEVEL DISINFECTION.

#### STORAGE AND DISPOSAL

Store this product in a cool, dry area, away from direct sunlight and heat to avoid deterioration. In case of spill, flood areas with large quantities of water. Product or rinsates that cannot be used should be diluted with water before disposal in a sanitary sewer. When discarding container, place in trash collection. Do not contaminate food or feed by storage, disposal, or cleaning of equipment.

## PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

CAUTION: Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling.
STATEMENT OF PRACTICAL TREATMENT: If in Eyes: Flush eyes with plenty of water

Call a physician if imitation persists

PHYSICAL AND CHEMICAL HAZARDS: Do not use this product with ammonia or acids such as vinegar, rust removers, or toilet bowlicleaners. This product not intended for use on fabrics

READ COMPLETE DIRECTIONS AND PRECAUTIONS @Trademarks of Caltech Industries, Inc. ON REAR PANELS

e e e

EXP. DATE

©1999 Caltech Industries, Inc. Midland, MI 48642 1-800-234-7700

ACCEPTED APR 30 2001

Maler The Federal Insecticide, Function and fundatisida Act, on a moded, for the

56392-8

REORDER NO:

Not to be used as baby wipes.

Keep lid tightly closed.