



UNITED STATES ENVIRONMENTAL PROTECTION AGENCY
WASHINGTON, DC 20460

OFFICE OF CHEMICAL SAFETY
AND POLLUTION PREVENTION

February 20, 2020

Tony Herber
Agent
Clorox Professional Products Company
c/o PS&RC; P.O. Box 493
Pleasanton, CA 94566

Subject: Label Amendment – Add Emerging Viral Pathogen Statement to Label
Product Name: Dispatch Hospital Cleaner Disinfectant Towels with Bleach
EPA Registration Number: 56392-8
Application Date: November 28, 2018
Decision Number: 548733

Dear Mr. Herber:

The amended label referred to above, submitted in connection with registration under the Federal Insecticide, Fungicide and Rodenticide Act, as amended, is acceptable. This approval does not affect any conditions that were previously imposed on this registration. You continue to be subject to existing conditions on your registration and any deadlines connected with them.

Because you have opted to add statements pertaining to emerging viral pathogens to your label as described in the [August 19, 2016, Guidance To Registrants: Process For Making Claims Against Emerging Viral Pathogens Not On EPA-Registered Disinfectant Labels \("Guidance"\)](https://www.epa.gov/sites/production/files/2016-09/documents/emerging_viral_pathogen_program_guidance_final_8_19_16_001_0.pdf), https://www.epa.gov/sites/production/files/2016-09/documents/emerging_viral_pathogen_program_guidance_final_8_19_16_001_0.pdf, you are subject to the following additional terms of registration:

1. You may make statements pertaining to emerging viral pathogens only through the following communications outlets: technical literature distributed exclusively to health care facilities, physicians, nurses and public health officials, "1-800" consumer information services, social media sites and company websites (non-label related). These statements shall not appear on marketed (final print) product labels.
2. Your statements pertaining to emerging viral pathogens must adhere to the format approved on the Agency-accepted master label.
3. You may make statements pertaining to emerging viral pathogens only upon a disease outbreak that meets all the following criteria:
 - a. The causative organism must be a virus that causes an infectious disease that has appeared in a human or animal population in the U.S. for the first time, or that may have existed previously but is rapidly increasing in incidence or geographic range.
 - i. For human disease, the outbreak is listed in one of the following Centers for Disease Control (CDC) publications:

- A. CDC Current Outbreak List for “U.S. Based Outbreaks” (www.cdc.gov/outbreaks),
 - B. CDC Current Outbreak List for “Outbreaks Affecting International Travelers” with an “Alert” or “Advisory” classification (www.cdc.gov/outbreaks) (also released through the CDC’s Health Alert Network (HAN) notification process)
 - C. Healthcare-Associated Infections (HAIs) Outbreaks and Patient Notifications page (www.cdc.gov/hai/outbreaks)
- ii. For animal disease, the outbreak is identified as an infectious disease outbreak in animals within the U.S. on the World Organization for Animal Health (OIE) Weekly Disease Information page (www.oie.int/wahis_2/public/wahid.php/Diseaseinformation/WI).
- b. The CDC or OIE has identified the taxonomy, including the viral family and/or species, of the pathogen and provides notice to the public of the identity of the emerging virus that is responsible for an infectious disease outbreak. Based on the taxonomy of the outbreak pathogen identified by the CDC or OEI, the pathogen's viral subgroup is [small non-enveloped, large non-enveloped, enveloped].
 - c. The virus can be transmitted via environmental surfaces (non-vector transmission), and environmental surface disinfection has been recommended by the CDC, OIE or EPA to control the spread of the pathogen.
4. You may begin communicating statements pertaining to emerging viral pathogens only upon CDC or OIE’s publication per term B.3.a. of an outbreak of an emerging viral pathogen meeting all of the criteria of term B.3. You must cease and remove all such non-label communications intended for consumers no later than 24 months after the original publication of the outbreak per term B.3.a., unless the Agency issue written guidance to the contrary due to continued public health concerns. The emerging pathogen claim language may remain on the master label.
5. Terms B.1 through B.4 above shall become immediately void and ineffective if registration for use against Adenovirus is suspended or cancelled or no longer meets the criteria for a disinfectant claim (see EPA Product Performance Test Guideline 810.2200). In addition, terms B.1 through B.4 above shall become immediately void and ineffective upon your receipt of evidence of ineffectiveness against any pathogen in a less-resistant Spaulding category.

A stamped copy of your labeling is enclosed for your records. This labeling supersedes all previously accepted labeling. You must submit one copy of the final printed labeling before you release the product for shipment with the new labeling. In accordance with 40 CFR 152.130(c), you may distribute or sell this product under the previously approved labeling for 18 months from the date of this letter. After 18 months, you may only distribute or sell this product if it bears this new revised labeling or subsequently approved labeling. “To distribute or sell” is defined under FIFRA section 2(gg) and its implementing regulation at 40 CFR 152.3.

Should you wish to add/retain a reference to the company's website on your label, then please be aware that the website becomes labeling under the Federal Insecticide Fungicide and Rodenticide Act and is subject to review by the Agency. If the website is false or misleading, the product would be misbranded and unlawful to sell or distribute under FIFRA section 12(a)(1)(E). 40 CFR 156.10(a)(5) list examples of statements EPA may consider false or misleading. In addition, regardless of whether a website is referenced on your product's label, claims made on the website may not substantially differ from those claims approved through the registration process. Therefore, should the Agency find or if it is brought to our attention that a website contains false or misleading statements or claims substantially differing from the EPA approved registration, the website will be referred to the EPA's Office of Enforcement and Compliance.

Your release for shipment of the product constitutes acceptance of these conditions. If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA section 6. If you have any questions, please contact Wanda Henson by phone at (703) 308-6345, or via email at henson.wanda@epa.gov

Sincerely,

A handwritten signature in blue ink that reads "Wanda G. Fuller, for". The signature is cursive and fluid.

Demson Fuller, Product Manager 32
Regulatory Management Branch I
Antimicrobials Division (7510P)
Office of Pesticide Programs

Enclosure

Label identification

Dispatch® Hospital Cleaner Disinfectant Towels with Bleach, EPA Reg. No. 56392-8

R0887088

8:45 2/20/20

Formatting notes (applicable for all claims)

Now[!] -and/or- **New[ly][!]** -and/or- **Improved[!]** may be added anywhere to a claim and will only be used for the first 6 months of product on shelf.

Optional text may be placed anywhere on the label -and/or- container.

Bold, italicized text is information for the reader and is not part of the label.

Bracketed information is optional text.

All footnotes are on the last page, unless they are part of EPA's mandated text.

The word "and" may be substituted with "&". Plural words may be used in their singular form or singular words may be used in their plural form unless otherwise specified in 40 CFR.

All directions may be written in numbered form or in paragraph form.

All use surfaces and/or use sites on the label may be listed in conjunction with an image of the use surface and/or use site.

Highlighted text is new. Strike-through text means removed.

Dispatch®

Hospital Cleaner Disinfectant Towels with Bleach

KEEP OUT OF REACH OF CHILDREN

CAUTION:

See back label for additional precautionary statements

-or-

Read Complete Directions and Precautions on Back

-or- side -or- rear Panels -or- Dispenser Carton

SELECT A SIZE *For canister only*

Additional statement for single count wipe pouch:

See carton for additional directions for use. -or- Read carton before use.

ACTIVE INGREDIENT:

Sodium Hypochlorite0.65%[††]

OTHER INGREDIENTS: 99.35%

Total:100.00%

[(Yields Available Chlorine...0.62%)]

[6,200 PPM [Parts Per Million] Available Chlorine]

[Net] [Contents] _____ **# of towels and size and weight**

Each package type and size will bear the actual net contents in pounds and ounces

Premoistened Wipes -or- Towels

[Individually Packaged]

ACCEPTED

02/20/2020

Under the Federal Insecticide, Fungicide
and Rodenticide Act as amended, for the
pesticide registered under
EPA Reg. No. 56392-8

PRECAUTIONARY STATEMENTS: Hazards to humans and domestic animals.

CAUTION:

Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco, or using the toilet.

FIRST AID:

If in eyes, hold eye open and rinse slowly and gently with water for 15-20 minutes. If present, remove contact lenses after the first 5 minutes of rinsing, then continue rinsing eye. Call a poison control center or doctor for further treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment. [Emergency Information: 1-800-446-1014. -or- Poison Control Center: (800) 446-1014 -or- Clorox Information Line: 1-888-797-7225 -or- Questions? Call 1-888-797-7225.]

PHYSICAL OR CHEMICAL HAZARDS: STRONG OXIDIZING AGENT:

Do not mix with other chemicals. Mixing this product with chemicals (e.g., ammonia, acids, detergents) or organic matter (e.g., urine or feces) in a manner inconsistent with labeling instructions may release hazardous gases irritating to eyes, lungs, and mucous membranes.

For more [product -and/or- ingredient] information, visit IngredientsInside.com -or- www.smartlabel.org.

For [a] Spanish translation of product directions -or- instructions -or- information, visit www.cloroxprofessional.com.

Mfd. for & © YYYY Clorox Professional Products Company,
1221 Broadway, Oakland, CA 94612

Made in [the] U.S.A. [of global components -and/or- ingredients.]

EPA Reg. No. 56392-8

EPA Est. No. 8251-OH-1 (Q6), WI-3 (KV); 5813-CA-6 (TPP); GA-2 (VG); IL-2 (VF); 62401-NJ-1 (JW), NC-2 (PK) Beginning of batch code indicates Est. No.

additional optional text

Clorox Customer Assistance 1-800-234-7700. Any Questions?

The experts at Clorox are standing by to answer any of your questions. -or- [Questions or Comments?] Call [toll-free] 1-800-234-7700.

[Visit us at -or- For SDS information, [please] go to] www.cloroxprofessional.com

To re-order [call **insert company name** at **insert phone number**] -and/or- go to www.cloroxprofessional.com.

Clorox is a [registered] trademark of The Clorox Company.

Dispatch -or- **insert trademark** is a [registered] trademark of The Clorox Company -or- Clorox Professional Products Company

Patents: www.thecloroxcompany.com/patents/

Optional Graphics/Logos/Symbols
May be placed anywhere



[Do not flush]



[[Use your phone to] go here
-and/or- scan [[the] tag -or- [this]
code] to learn -or- get -or- see [more]
uses -or- information [on how-to-use
[This Product]] -or- [how-to] tips -or-
[how-to [-use]] videos]

Packaging Related Statements

For fold-pack labels

Peel [here] for [additional] directions [for use][.]

Dispenser

Canister

To open: Remove lid. Discard inner seal from canister.

To thread: Find center of wipe roll, twist corner of first wipe into a point and thread through the hole in the canister lid. Pull through about one inch. Replace lid. Dispense remaining wipes as necessary by pulling out and tearing off. When not in use, keep center cap of lid closed to prevent moisture loss.

Select-A-Size: For a 7" x 8" towel - pull one towel gently through the hole, angle sharply and detach at the first perforation. For a 7" x 16" towel - pull two towels gently straight up through the hole. Hold second towel, angle sharply and detach at the second perforation.
NOTE: Select-A-Size feature for the larger size towel works best after several smaller towels have been used.

Soft pack

Open package. Pull out wipes as needed and reseal.
When not in use keep sealed to prevent moisture loss.

Individually wrapped Pack

Tear open packet. Unfold.

- KEEP PACKAGE -OR- CONTAINER TIGHTLY SEALED WHEN NOT IN USE
- PEEL -and/or- RESEAL HERE
- PULL TAB
- Remove flaps -or- lid to create dispenser.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

Not for cleaning or sanitizing skin. Do not use as a diaper wipe or for personal cleansing. For surfaces that may come in contact with food, a potable water rinse is required. [Avoid contact with clothes, fabric, wood, [natural] rubber, and painted and paper surfaces. Do not use on silver, copper, or chipped enamel. APPLY CAREFULLY – This product contains bleach.] [Refill only with this product.] [Always refer to manufacturer's care instructions before using on equipment -or- devices.]

[Look for] Product expiration date on back of canister or pack. [Discard wipe after use.] To prevent cross contamination do not reuse wipe.

A potable water rinse is required for food contact surfaces. Do not use on glassware, utensils, or dishes.

To [clean and] disinfect [and deodorize] hard, nonporous surfaces: Wipe [hard, nonporous] surface to be disinfected. Use enough wipes for treated surface to remain visibly wet for the contact time listed below -or- on label. [Rinse.] Let air dry. Visible filth must be removed prior to disinfecting. [If streaking is observed, wipe with a clean, damp [cloth or] paper towel after appropriate contact time has expired.] [For] Killing *Clostridium difficile* [spores]¥: Pre-clean hard, nonporous surfaces by removing visible filth [loose dirt, debris, blood/bodily fluids, etc.]. Wipe [hard, nonporous] surface to be disinfected. Use enough wipes for treated surface to remain visibly wet for three -or- 3 min[utes]. [Rinse.] Let air dry.

Special Instructions for Cleaning Prior to Disinfection against *Clostridium difficile* spores. Personal Protection: Wear appropriate barrier protection such as gloves, gowns, masks and eye covering. **Cleaning Procedure:** Remove all visible fecal matter/waste from surfaces/objects before disinfection by application with **product name** -or- this product. Cleaning is to include vigorous wiping and/or scrubbing, until all visible soil is removed. Special attention is needed for high-touch surfaces. Surfaces in patient rooms are to be cleaned in an appropriate manner, such as from right to left or left to right, on horizontal surfaces, and top to bottom, on vertical surfaces, to minimize spreading of the spores. Restrooms are to be cleaned last. Do not reuse soiled cloths. **Infectious Materials Disposal:** Materials used in the cleaning process that may contain feces/wastes are to be disposed of immediately in accordance with local regulations for infectious materials disposal. **Disinfection procedure:** Wipe surface with towel until completely wet. Let stand for three -or- 3 min[utes]. Wipe dry -or- allow to air dry. Visible soil must be removed prior to disinfecting. Gloves must be worn.

Ultrasound transducers and probes

For use on external surfaces of ultrasound transducers and probes only.

To disinfect: Pre-clean gel from external surface of ultrasound transducer or probe to be disinfected with a paper towel or clean cloth. Then thoroughly wet the probe with a **product name** towel. Let stand for 1 -or- one min[ute]. To kill TB, let stand for 2 -or- two min[utes] at room temperature [20° C -or- 68°].

The following is only for products with medical use sites medical devices listed on the label

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream, or normally sterile areas of the body, or (2) contacts intact mucous membranes, but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

Special Instructions for Use Against HIV-1, HBV and HCV

This product kills HIV-1, HBV and HCV on precleaned hard, nonporous surfaces/objects previously soiled with blood/body fluids in health care settings or other settings in which there is an expected likelihood of soiling of hard, nonporous surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS), Human Hepatitis B Virus (HBV) and Human Hepatitis C Virus (HCV).

Special instructions for cleaning and decontamination against HIV-1 (AIDS Virus), HBV and HCV on surfaces/objects soiled with blood/body fluids

Personal Protection: When handling items soiled with blood or body fluids, personal protective equipment such as disposable gloves, gowns, masks and eye coverings must be used.

Cleaning Procedure: Remove all visible blood/body fluids from surfaces and objects before disinfecting.

Contact Time: Allow surfaces to remain wet for one -or- 1 min[ute], [rinse and] let air dry. For all other organisms, see directions for contact times.

Disposal of Infectious Materials: Blood, body fluids, and cleaning materials must be autoclaved and disposed according to federal, state, and local regulations for infectious waste disposal.

Farm Premise And Poultry House Disinfectant [Instructions -or- Directions For Use]

1. Remove all poultry or animals and feeds from premises, trucks, vehicles, coops, crates, and enclosures.
 2. Remove all litter and manure or droppings from floors, walls and surfaces of barns, pens, stalls, chutes, and other facilities and fixtures occupied or traversed by animals or poultry.
 3. Empty all troughs, racks, and other feeding and watering appliances.
 4. Thoroughly clean all surfaces with soap or detergent and rinse with water.
 5. Wipe all surfaces using enough wipes to keep surfaces visibly wet for 3 min[utes].
 6. Wipe all halters, ropes and other types of equipment used in handling and restraining animals, as well as forks, shovels, and scrapers used for removing litter and manure.
 7. Ventilate buildings, cars, boats, coops, and other closed spaces. Do not house livestock or poultry or employ equipment until treatment has been absorbed, set or dried.
 8. Thoroughly scrub all treated feed racks, mangers, troughs, automatic feeders, fountains, and waterers with soap or detergent, and rinse with potable water before reuse.
-

Cleaning and Stain Removal

To clean -and/or- remove stains: Follow directions above, but no standing time is required. - or- Wipe surface with a wipe; [rinse and] let air dry. [If streaking is observed, wipe with a clean, damp [cloth or] paper towel.]

STORAGE AND DISPOSAL

For canisters

Store this product in a cool, dry area away from direct sunlight and heat. When not in use keep center cap of lid closed to prevent moisture loss. Nonrefillable container. Do not reuse or refill this container. Discard used towels and container in trash or take container to recycling center. [Do not flush towels down toilet.]

For soft packs and individually wrapped packs

Store this product in a cool, dry area away from direct sunlight and heat. Nonrefillable container. Do not reuse or refill this container. Discard used towel and empty package in trash. [Do not flush towels down toilet.]

Organism Table

<i>Bacteria</i>	<i>Strain and/or ATCC</i>	<i>Contact Time</i>
Acinetobacter baumannii	[ATCC 19606]	1 min[ute]
Klebsiella aerogenes <i>tested as Enterobacter aerogenes</i>	[ATCC 13048]	1 min[ute]
Enterococcus faecalis, vancomycin resistant -or- Vancomycin resistant Enterococcus faecalis [(VRE)]	[ATCC 51299]	1 min[ute]
Enterococcus hirae	[ATCC 10541]	1 min[ute]
ESBL (Extended Spectrum Beta Lactamase) producing Escherichia coli (ESBL producing E. coli)	[ATCC BAA-196]	1 min[ute]
Klebsiella pneumoniae [(Kleb)]	[ATCC 4352]	1 min[ute]
Methicillin resistant Staphylococcus aureus [(MRSA)]	[ATCC 33592]	1 min[ute]
Multi-drug resistant Enterococcus faecium [(MDR E. faecium)] ^[1]	[ATCC 51559]	1 min[ute]
Pseudomonas aeruginosa [(Pseudomonas)]	[ATCC 15442]	1 min[ute]
Salmonella enterica <i>tested as Salmonella choleraesuis</i>	[ATCC 10708]	1 min[ute]
Serratia marcescens	[ATCC 14756]	1 min[ute]
Staphylococcus aureus [(Staph)]	[ATCC 6538]	1 min[ute]
Streptococcus pyogenes [(Strep)]	[ATCC 19615]	1 min[ute]
<i>Spore-forming bacterium</i>	<i>Strain and/or ATCC</i>	<i>Contact Time</i>
∂Clostridium difficile -or- C. difficile -or- C. diff [(C. diff)] spores [¥]	[ATCC 43598]	3 min[utes]
<i>Mycobacterium</i>	<i>Strain and/or ATCC</i>	<i>Contact Time</i>
Mycobacterium bovis (Tuberculosis -or- TB) [20°C -or- 68°F]	[(BCG)]	2 min[utes]
<i>Fungus</i>	<i>Strain and/or ATCC</i>	<i>Contact Time</i>
Trichophyton interdigitale <i>tested as Trichophyton mentagrophytes</i> [(Athlete's Foot Fungus)] [(Trichophyton)]	[ATCC 9533]	1 min[ute]

<i>Viruses Enveloped</i>	<i>Strain and/or ATCC</i>	<i>Contact Time</i>
‡Avian Influenza A Virus	[ATCC VR-2072, Strain A/ Washington/897/80 X A/Mallard/New York/6750/78]	1 min[ute]
‡, ‡‡HIV [Human Immunodeficiency Virus] [Type 1]	[Strain HTLV-3]	1 min[ute]
‡, ‡‡[Human] Hepatitis B Virus [(HBV)] (as duck HBV)		1 min[ute]
‡, ‡‡[Human] Hepatitis C Virus [(HCV)] (as bovine diarrhea virus)		1 min[ute]
‡[Human] Herpes simplex Virus [Type 2]	[MS Strain]	1 min[ute]
‡Influenza A Virus [(representative of the common flu virus)]	[ATCC VR-544] [Strain Hong Kong]	1 min[ute]
‡Influenza B Virus	[ATCC VR-823] [Strain B/Hong Kong/5/72]	1 min[ute]
<i>Viruses Large Non-enveloped</i>	<i>Strain and/or ATCC</i>	<i>Contact Time</i>
‡Adenovirus [Type 2] [(a cause of the common cold)]	[ATCC VR-846] [Strain Adenoid 6]	1 min[ute]
‡Rotavirus	[ATCC VR-899] [Strain SA-11]	1 min[ute]
<i>Viruses Small Non-enveloped</i>	<i>Strain and/or ATCC</i>	<i>Contact Time</i>
‡, § Canine Parvovirus	[ATCC VR-2017] [Strain Cornell]	1 min[ute]
‡, §Feline Parvovirus [(Feline Panleukopenia Virus)]	[ATCC VR-648] [Strain Philips-Roxane]	1 min[ute]
‡[Human] Hepatitis A Virus [(HAV)]	[Strain HM-175]	1 min[ute]
‡Norovirus (as Feline Calicivirus)	[ATCC VR-782] [Strain F-9]	1 min[ute]
‡Poliovirus [Type 1]	[Mahoney Strain, Thomas Francis 941]	1 min[ute]
‡Rhinovirus [Type 37] [(causes colds)]	[ATCC VR-1147] [Strain 151-1]	1 min[ute]

Antimicrobial Claims

- [3 min[ute]] broad spectrum disinfectant for non-food contact surfaces
- A disinfectant cloth for use on **insert surface(s) from Table 1**
- [A] one-step² cleaner -and/or- disinfectant
- Aids in reducing -or- in the reduction of cross-contamination between treated surfaces
- An effective method to clean², deodorize and disinfect against odor-causing organisms on hard, nonporous surfaces listed on this label
- An effective way to disinfect hard, nonporous surfaces listed on this label
- Antibacterial formula
- Appropriate -or- Perfect -or- Ideal [for] Terminal Cleaning [Patient Discharge[s][ing]]
- Assures proper dilution strength, product effectiveness and standardizes technique
- Attacks and kills Clostridium difficile^Y -or- C. difficile^Y -or- C. diff^Y [(C. diff)] [spores] [on precleaned, hard, nonporous surfaces]
- Bacteria -and/or- virus[‡] -and/or- germ fighting [formula]
- Bactericidal -and/or- Virucidal[‡] -and/or- Fungicidal -and/or- Sporicidal^Δ -and/or- Tuberculocidal -and/or- Disinfectant
- Bactericidal
- Bactericide
- Bathroom disinfectant wipes
- Broad spectrum disinfectant [with Clorox[®] bleach]
- Broad spectrum [Proven] disinfectant [and bactericide -and/or- virucide[‡] -and/or- fungicide] with efficacy against Clostridium difficile^Y -or- C. difficile^Y -or- C. diff^Y spores
- [Cleans² [and], Disinfects, [& Deodorizes] in One Step]
- [Cleans² and] disinfects **insert use site(s) from Table 2**
- [Cleans² and] disinfects [hard, nonporous surfaces]
- Cleans², deodorizes and disinfects
- Daily use product with Clostridium difficile^Y -or- C. diff^Y -or- C. difficile^Y sporicidal efficacy which allows for product standardization [reduces -or- reducing cleaning complexity]
- Daily use product with sporicidal^Δ efficacy
- Demonstrated Effectiveness against: **insert organism(s) from Organism Table** [on hard, nonporous surfaces -or- on **insert use surface(s) from Table 1**]
- Deodorizes by killing the germs that cause odors
- Designed for killing Clostridium difficile^Y -or- C. difficile^Y -or- C. diff^Y [(C. diff)] [spores] on surfaces in hospitals
- Designed for killing -or- controlling Clostridium difficile^Y -or- C. difficile^Y -or- C. diff^Y spores [on hard, nonporous surfaces [and patient care equipment]] in hospitals [and other healthcare facilities]
- Destroys -or- Deactivates [tough-to-kill] Clostridium difficile^Y -or- C. difficile^Y -or- C. diff^Y [(C. diff)] [spores] [on hospital surfaces]
- Disinfectant
- Disinfecting formula
- Disinfects [and deodorizes]
- Disinfects and deodorizes by killing germs and their odors
- Disinfects hard, nonporous surfaces [throughout the **insert surface(s) from Table 1**]
- Easy and convenient disinfecting on hard, nonporous surfaces [throughout the **insert use site(s) from Table 2**]
- Easy, one-step² cleaning and disinfecting
- Effective against Influenza A Virus [and] **insert organism(s) from Organism Table** [and] [C. difficile^Y spores] [on precleaned, hard, nonporous surfaces]
- Effective against **insert organism(s) from Organism Table** [and] Clostridium difficile^Y -or- C. difficile^Y -or- C. diff^Y [(C. diff)] [spores]
- Effective against -or- Kills Staphylococcus aureus, Salmonella enterica, Pseudomonas aeruginosa [in the presence of organic soil [(5% blood serum)]]
- Effective against -or- Kills **insert organism(s) Organism Table except C. diff** [in the presence of organic soil [(5% blood serum)]]
- Effective disinfectant cleaner² [and Clostridium difficile^Y sporicide] on hard, nonporous surfaces [for use in hospitals, ambulatory care centers, long term care facilities, and other healthcare facilities]
- Effective -or- For daily use against Clostridium difficile^Y -or- C. difficile^Y -or- C. diff^Y [(C. diff)] [spores]
- Effective against -or- kills **insert number up to 32** pathogens -or- organisms
- [Effectively] disinfects [hard, nonporous] surfaces
- [Efficacy tests have demonstrated that **product name** -or- this product [are -or- is] an effective bactericide, virucide[‡], tuberculocide, and fungicide in the presence of organic soil (5% blood serum)]
- Eliminates odors at their source [caused by germs -or- bacteria]
- Eliminates -or- Reduces odors [caused by germs -or- bacteria]
- Exhibits effective virucidal[‡] activity
- Fight[s] -and/or- kill[s] -and/or- Effective against [99.9% [of]] Salmonella enterica [(Salmonella)]
- Fight[s] -and/or- kill[s] -and/or- Effective against Avian Influenza A Virus [on hard, nonporous surfaces]
- Fight[s] -and/or- kill[s] -and/or- Effective against cold -and/or- flu Virus[es]⁸
- Fight[s] -and/or- kill[s] -and/or- Effective against [Human] Hepatitis A [(HAV)]
- Fight[s] -and/or- Kill[s] -and/or- Effective against **insert organism(s) from Organism Table** [in **insert appropriate contact time for respective organism**]
- [For] Disinfecting
- [For] Germ Kill[ing]
- Formulated to kill Clostridium difficile^Y -or- C. difficile^Y -or- C. diff^Y [(C. diff)] [spores]
- Fungicidal
- Fungicide
- Germicidal disposable wipes
- Germicidal
- Germicide
- Helps reduce the spread of germs -and/or- bacteria -and/or- viruses[‡] -or- pathogens -or- **insert organism(s) from Organism Table** [on hard, nonporous treated surfaces]
- [In circumstances of room^P contamination,] clean [room] from cleanest to dirtiest surfaces, [and] [Clean] from highest to lowest [surfaces] [using **insert ABN** -or- this product to clean and disinfect high-touch areas^P]

- [Kills [99.9% of] bacteria, viruses[‡] & a fungus in 1 Min[ute] on Hard, Nonporous Surfaces][⁶] **qualifier will only be used if *Mycobacterium bovis* listed on label**
- Kills [99.9% of] germs [on hard surfaces]
- Kill[s] -and/or- Fights -and/or- Effective against germs that can cause infections -or- illnesses: **insert organism(s) from Organism Table**
- Kill[s] -and/or- Fights -and/or- Effective against[: **insert organism(s) from Organism Table**] on hard, nonporous surfaces -or- on **insert use site(s) from Table 2**
- Kills bacteria -and/or- viruses[‡] -and/or- a fungus -and/or- Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] [spores] [on precleaned, hard, nonporous surfaces]
- Kills bacteria, viruses[‡]
- Kills both Influenza A and B viruses [on hard, nonporous surfaces]
- Kills Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] [spores] [on precleaned, hard, nonporous surfaces]
- Kills Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] [spores] in 3 min[utes] [on hard, nonporous surfaces]
- Kills [effective against] both enveloped and nonenveloped viruses[‡] [on hard, nonporous surfaces]
- Kills flu viruses⁷ [in US]
- Kills **insert organism(s) from Organism Table** [in **insert contact time**]
- Kills micro-organisms **insert organism(s) from Organism Table** [on hard, nonporous surfaces]
- Kills the spore form of Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] [spores]
- Kills multiple strains of Staphylococcus aureus^[9] -and/or- Influenza viruses^[7]
- Kills Norovirus⁴, a pathogen -or- organism commonly spread through surfaces
- Kills odor-causing bacteria -and/or- germs
- Kills -or- Fights -or- is effective against **insert organism(s) from Organism Table** [on hard, nonporous surfaces -or- on **insert use surface(s) from Table 1**
- Kills -or- Is effective against [99.9999%] Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] [spores] [in 3 min[utes] [on precleaned, hard, nonporous surfaces]
- Kills -or- Effective against bacteria -and/or- germs -and/or- viruses[‡]
- Kills -or- Is effective against [99.9999%] Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] [spores] [in 3 min[utes]], [in the presence of [three (3) part] soil [load]] [on hard, nonporous surfaces]
- May be used as part of a comprehensive approach to Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] spore control
- Meets [the disinfection requirements of] OSHA['s] Bloodborne Pathogens Guideline -or- Standard
- [Multi-purpose] disinfectant
- One-step² cleaner and disinfectant
- One-step² cleaning and disinfecting [saves time]
- One-step² disinfectant cleaner designed for general cleaning² and disinfecting hard, nonporous surfaces in health care facilities -or- **insert use site(s) from Table 2**
- Parvovirucidal³
- Parvovirucide³
- Pay close attention to hot spots or frequently touched surfaces **insert surface(s) from Table 1** and disinfect regularly, up to 3 times daily
- Pre-moistened disposable towels for use in healthcare and other areas where control of cross contamination is required
- Pre-moistened towels with 1:10 dilution of bleach [(6%)] and detergent
- **Product name** are intended for cleaning² and disinfecting hard, nonporous surfaces where control of cross-contamination is required.
- **Product name** may be used to clean² and disinfect in one step; see directions for use on this label for cleaning and disinfecting in one step
- Pseudomonacidal¹⁰
- Ready-to-use disinfectant [disinfection] for [killing] Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] [spores]
- [Ready-to-use] hospital disinfectant
- Reduces cross-contamination of **insert organism(s) from Organism Table** on treated, hard nonporous surfaces
- Reduces exposure to Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] from treated surfaces
- Reduces exposure to Methicillin-Resistant Staphylococcus aureus [(MRSA)] from treated surfaces
- Reduces cross-contamination on treated surfaces
- Reduce[s] the spread of [common] germs -or- pathogens on hard, nonporous treated surfaces
- Renders Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] [spores] inactive
- Staphylocidal⁹
- The answer to your disinfecting needs
- [The] easy -and/or- convenient way to disinfect
- Tested Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] spores [with soil load]
- [This product] reduces the spread of germs on treated, hard, nonporous treated surfaces
- To break down and kill -or- inactivate -or- [help prevent the spread of] Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] [spores] [on treated surfaces] [on precleaned, hard, nonporous treated surfaces]
- To kill Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] [spores] on precleaned, hard, nonporous surfaces, [let stand for three -or- 3 min[utes]]
- To kill -or- inactivate Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] [spores] [on surfaces]
- To reduce Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] [spores] on [precleaned, hard, nonporous] surfaces in **insert use site(s) from Table 2**
- Tuberculocidal
- Tuberculocide
- Use in public -or- common places where [bacteria -and/or- viruses[‡] -and/or- germs may be of concern] [you are concerned about bacteria -and/or- viruses[‡] -and/or- germs] [on hard, nonporous surfaces]
- Used the -or- Tested against the clinically relevant 43598 strain of Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] spores
- Used under the **insert current test method** as of **include date of test method**
- Use where control of the hazards of cross-contamination between treated surfaces is of prime importance
- Virucidal[‡]
- Virucide[‡]
- When used according to label directions, [this product] reduces exposure to Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)] from contact with -or- treated surfaces

Emerging Viral Pathogen Claims

This product qualifies for emerging viral pathogen claims per the EPA's 'Guidance to Registrants: Process for Making Claims Against Emerging Viral Pathogens not on EPA-Registered Disinfectant Labels' when used in accordance with the appropriate use directions for use indicated below.

This product meets the criteria to make claims against certain emerging viral pathogens from the following viral category:

- Enveloped viruses

<i>For an emerging viral pathogen that is an:</i>	<i>... follow the directions for use for the following organisms on the label:</i>
Enveloped virus	Adenovirus

Product name -or- this product has demonstrated effectiveness against viruses similar to **insert name of emerging virus** on hard, nonporous surfaces. Therefore, **product name** -or- this product can be used against **insert name of emerging virus** when used in accordance with the directions for use against **insert name of supporting virus(es)** on hard, nonporous surfaces. Refer to the CDC -or- OIE website at **insert pathogen-specific website address** for additional information.

Insert name of illness/outbreak is caused by **insert name of emerging virus**. **Product name** -or- this product kills similar viruses and therefore can be used against **insert name of emerging virus** when used in accordance with the directions for use against **insert name of supporting virus(es)** on hard, nonporous surfaces. Refer to the CDC -or- OIE website at **insert pathogen-specific website address** for additional information.

General Claims

- A solution impregnated in a wiping cloth
- Advanced cleaning formula
- Be ready
- Bleach-based formula means no quat-binding concerns [with wiping]
- Cleaning
- Cleaning made easy
- Cleans
- Cleans the toughest soils
- Cleans tough messes
- Clorox® clean
- Compatible with a wide variety of surfaces. Check cloroxhealthcare.com/compatible
- Contains 6,500 ppm Sodium Hypochlorite
- Contains [Clorox®] bleach
- Contains no abrasives, harsh acids
- Convenient
- Deep clean[ing] [formula]
- Delivers a clean you can see and smell
- Delivers consistent [amount of solution] concentration every time[!]
- Delivers consistent [sodium hypochlorite] concentration every time[!]
- Deodorizes
- [Designed] For daily use -or- Use for daily [or] [discharge] cleaning [in Healthcare facilities]
- [Designed] [for] [with] [the] Environmental Services Professionals [in] [mind][!]
- Designed to be used around -or- for care of -or- in the care of Immune-compromised -or- chemotherapy patients -or- individuals
- Does not contain nonylphenol ethoxylates [(NPEs)] -and/or- alkylphenol ethoxylates [(APEs)]
- Does not contain VOCs
- Does not require [the hassle of] dispensing systems
- Does not produce mist or overspray
- Dries quickly
- Easy [and convenient] to use
- Easy to handle
- Effectively removes everyday soils
- Eliminates airborne particles associated with sprays
- Eliminates [daily] mixing and measuring
- Eliminates odors from fecal matter and urine
- EVS [Environmental Services] [use]
- Expiration Date **insert date -or- length of time**
- [For] general use
- For more advice -or- tips -or- education -or- support -or- information visit us at www.cloroxprofessional.com
- For more advice -or- tips -or- education -or- support -or- information) call us [toll free] at **insert phone number**
- [For] Surface [care]
- [For] Surface cleaning
- For use on bathroom surfaces
- For use on nursery surfaces
- [Formulated -or- Designed to] remove[s] biological odors including Urine -and/or- Feces -and/or- Vomit
- Fresh [lingering] bleach scent
- Freshens
- Great for everyday use
- [Heavy] cloth-like towel
- Institutional [size]
- Large[r] Surfaces
- Leaves a fresh bleach scent
- Leaves nothing behind
- Lightly scented -or- quick-dissipating scent
- Maximum cleaning, minimum effort
- Moist wipes
- Multi-purpose
- Multi-surface
- Neutralizes odors [doesn't [just] mask]
- No harsh chemical odors or fumes
- No measuring [required]
- No mixing
- Non-abrasive formula
- Nonylphenol ethoxylate [(NPE)] -and/or- alkylphenol ethoxylate [(APE)] free
- Not for personal use
- NOT TO BE USED AS BABY WIPES
- [Note:] This product is compatible with the listed materials
- Powers through tough messes
- Pre-measured
- Pre-moistened [towelettes]
- **Product name** are pre-moistened with a unique, stabilized bleach and detergent solution and are ready-to-use.
- **Product name** are useful in a variety of healthcare and nursery/daycare applications listed on the [this] label
- Prevents formulation errors
- Ready to use
- Recyclable [bottle -or- product container]
- [Refer to] see [Clorox®] [Terminal] [Cleaning] protocol for additional [cleaning] information
Reuse[able] container -or- bucket
Pouch[es]
[Sturdy] Buckets
System
Protocols
- Removes **insert stain(s)/soil(s) from Table 4**
- Removes stains
- REORDER NO.
- Safe to -or- for use on **insert surface material(s) from Table 3**
- See expiration date [on package]
- Shelf-stable formula
- [Specifically -or- Specially] designed for the Medical/Dental/Veterinary/Health Care industry
- Starts cleaning on contact
- Strong cleaning power
- [Strong,] [Durable,] cloth-like wipes
- [Strong,] [Durable,] non-woven disposable cloth
- [Strong,] [Durable,] non-woven wipe does not leave lint on surfaces [following use]
- Suitable for hospital use
- Suitable for use on **insert surface material(s) from Table 3**

- | | |
|--|---|
| <ul style="list-style-type: none">• Suitable for use on titanium-coated -or- medical grade stainless steel• Suitable for use on hard, nonporous surfaces• Surfaces: For use on hard, nonporous surfaces such as <i>insert surface material(s) from Table 3</i>• Terminal Cleaning Wipes -or- Towel[s] [Refill[able]]• [The] smell of clean | <ul style="list-style-type: none">• This wipe is saturated with a 1:10 sodium hypochlorite solution• Usable on a variety of soils• Use of this product does not produce mist in the air.• VOC free• We're there for you• Works on -or- effective against -or- removes the following stains -and/or- soils: <i>insert stain(s)/soil(s) from Table 4</i> |
|--|---|

Table 1 Surfaces^P

Animal equipment	Exterior -or- external toilet surfaces	-and/or- Cords
Any washable (food ¹² -and/or- non-food contact surface where disinfection is required)	Exterior -or- external urinal surfaces	Other telecommunications equipment [surfaces]
Appliance exteriors	Exterior surfaces of air vents -or- air vent exteriors	Outdoor furniture (excluding wood frames and upholstery)
Appliances	External surfaces of [medical] equipment -or- [medical] equipment surfaces	Oven exteriors
Autoclaves	External [surfaces of] ultrasound transducers	[Patient] chairs
Automatic feeders	External surfaces of [veterinary] equipment	Patient monitoring equipment
[Bathroom] fixtures	Faucets	Pens
[Bath]tubs	Feed racks	Phlebotomy trays
[Bed] head -and/or- foot boards	Floors	Phone cradles
Bed pans	Food cases ^{[5], 12}	Pipes
Bed rails -or- railing	Food trays ^{[5], 12}	Plastic mattress covers
[Bedside] commodes -or- tables	Fountains	Playpens
Bedframes	Freezer exteriors	PVC tubing
Behind and under counters ^P	Garbage -or- trash cans	Reception counters -or- desks
Behind and under sinks ^P	Glazed ceramic tiles	Refrigerator exteriors
Booster chairs	Glucometer exteriors	Salad bar sneeze guards
BP Monitors	Glucometers -or- Blood Glucose Monitors	Shelves
Cabinet handles	[Grocery store -or- supermarket] carts	Shower fixtures
Cabinets	Gurneys	Showers -or- shower stalls
Cages	Hampers	Shower walls
Carts	Hand railings	Similar hard, nonporous surfaces except for those excluded by the label
Ceilings	Hard, nonporous dental -and/or- hospital -and/or- medical -and/or- veterinary surfaces	Sinks
Cell[ular] -or- wireless -or- mobile -or- digital phones	Headsets	Stall doors
Chemotherapy Hoods	Highchairs ^{[5], 12}	Stretchers
Closet handles	High touch surfaces	Stove[top]s
Coated mattresses ^P	Hoods	Support bars
Coated pillows ^P	[Hospital -or- patient] bed railings -and/or- linings -and/or- frames	Table[top]s
[Common] hard, nonporous surfaces	Infant incubators and care cribs	[Tele]phones
[Computer] keyboards	[Inner -or- inside of] drawers	[Tiled] walls
[Computer] monitors	IV poles	Toilet rims
Cribs	IV stands	Toilet seats
Dental counter[top]s	[Laminated plastic] counter[top]s	Towel dispensers
Dental operatory surfaces	Laminated surfaces	Toys
[Dentist -or- dental] chairs	Lamps	Troughs
Desks [tops]	Light lens covers	[TV] Remote [Control]s
[Diaper -and/or- infant] changing tables	Mammography equipment	Ultrasound Transducers [and Probes]
Diaper pails	Mattress covers	Vanities
Diagnostic equipment	Mayo -and/or- Instrument Stands	Vanity tops
Dictating equipment [surfaces]	Med[ication] carts	Veterinary care surfaces
Dish racks	Microwave exteriors	Wash basins
Doorknobs	Nurse-Call Device -and/or- Buttons	Watering appliances
Drain boards		Wheelchairs
Exam[ination] tables		X-Ray machine surfaces

Table 2 Use Sites

Acute Care Facilities	Fast Food Chains -and/or- Restaurants	Outpatient Clinics
Airplanes	Food Preparation and Processing Areas ¹²	Outpatient Surgical Centers [(OPSC)]
Ambulances	Food [Service -and/or- Processing] Establishments ¹²	Patient Care Areas
Animal Housing Facilities	Food Serving Areas ¹²	Patient Restrooms
Animal Life Science Lab[oratorie]s	Funeral Homes	Patient Rooms
[Animal -and/or- Pet] Grooming Facilities	Grocery Stores	Pet Areas
Anesthesia Rooms -and/or- Areas	Gym[nasium]s	Pet Shops -or- Stores
[Assisted Living -and/or- Full Care] Nursing Homes	Health Care Settings -or- Facilities	[Pediatric] Exam[ination] Rooms -and/or- Areas
Bars	Health Club Facilities	P[ediatric] I[ntensive] C[are] U[nit]s
Bathrooms	Home Health Care [Settings]	Pharmacies
Bedrooms	Hospices	Physicians Offices
Blood Banks	[Hospital] central service areas	Physical Therapy [Rooms -and/or- Areas]
[Blood -and/or- plasma -and/or- semen -and/or- milk -and/or- apheresis] Donation centers	[Hospital] Kitchens	Produce Areas
Bowling Alleys	Hospital [Premises]	Public Areas
Butcher Shops	Hotels	Radiology -or- X-Ray [Rooms -or- Areas]
Cafeterias	Industrial Facilities	Reception areas
CAT Lab[oratorie]s	I[ntensive] C[are] U[nit]s	Recovery Rooms
Central Supply Rooms -or- Areas	Isolation Areas	Recreational Centers -or- Facilities
Churches	Kennels	Rehabilitation Areas
Clinics	Lab[oratorie]s	Respiratory Therapy Rooms -or- Areas
Colleges	Laundromats	Restaurants
Commercial -or- Institutional Kitchens	Laundry Rooms	Restrooms -or- Restroom Areas
Correctional Facilities	Livestock -and/or- Swine -and/or- Poultry Facilities	School Buses
C[ritical] C[are] U[nit]s	Locker Rooms	School Kitchens
Day Care Centers	Long Term Care Facilities	Schools
Deli[catessen]s	Manufacturing Plants -and/or- Facilities	Shelters
Dental Operatories	[Medical] Clinics	Shower Rooms
Dental -or- Dentist's Offices	Medical Facilities	Sills Ledges
Dining Halls -and/or- Rooms	Medical -and/or- Physician's -and/or- Doctor's Offices	Small Animal Facilities
Dormitories	Military Installations	Stalls
Dialysis Clinics	Motels	Storage Rooms -and/or- Areas
Dialysis Machines	N[eonatal] I[ntensive] C[are] U[nits]	Supermarkets
[Emergency Medical] Transport Vehicles	[Newborn -or- Neonatal] Nurseries	Surgery Rooms -or- O[perating] R[oom]s
E[mergency] R[oom]s	Nursing Homes	Trains
Equine Farms	Nursing -or- Nurses' Stations	Transport Vehicles
[Eye] Surgical Centers	Ophthalmic Offices	Universities
Factories	Orthopedics	Veterinary -and/or- Animal Hospitals
Farms	Other Food Service Establishments	Veterinary Clinics -and/or- Facilities
		Veterinary Offices

Table 3 Surface Materials

[Baked] enamel
Chrome
Fiberglass [sealed] surfaces
Formica^{λλ} [® -or-TM]
Glass
Glazed porcelain
Linoleum

Marlite^{λλλ} [® -or-TM]
[Nonporous] vinyl
Plastic [lamine] ^{pp}
Plexiglas^{□□□} [® -or-TM]
Porcelain enamel
Stainless steel
Synthetic marble

Not Recommended For Use On -or- Avoid contact with:

Aluminum
Brass
Chipped enamel
Clothes
Copper
Corian^λ TM -or- ®
Fabric
Finished wood
Galvanized steel

Natural marble
Painted surfaces
Paper surfaces
Rubber
Sealed granite
Sealed marble
Silver
[Unfinished] wood

Table 4 Stains/Soils

- Blood
- Bodily fluids
- Body oils
- Coffee [stains]
- Common soils
- Dead skin
- Defecation
- Dirt
- Fecal matter

- Feces
- Fingerprints
- Food residues -and/or- soils
- Food stains
- Fruit [stains]
- Grease
- Grime
- Lab[oratory] stains
- Mildew stains

- Mold stains
- Mucus
- [Other] common soils -and/or- stains
- [Other] organic matter
- Phlegm
- Protein-based matter
- Saliva
- Spit

- Spittle
- Sputum
- Stool
- Tea [stains]
- Throw-Up
- Urine
- Vomit

Footnotes

¹ This strain of Enterococcus faecium has shown resistance to the following antibiotics: Ampicillin, Gentamicin, Penicillin]

² when used according to the disinfection directions^{5]}

³ Canine Parvovirus, Feline Parvovirus

⁴ Tested as Feline calicivirus

^{5]} Non-food contact surfaces

⁶ Kills TB in 2 min[utes]

⁷ Influenza A Virus -and/or- Influenza B Virus

⁸ Influenza A Virus -and/or- Influenza B Virus, Adenovirus, Rhinovirus

⁹ Staphylococcus aureus, Methicillin-resistant Staphylococcus aureus

¹⁰ Pseudomonas aeruginosa

¹² [The] surface must be rinsed after treatment -and/or- prior to use

^{††} Does not include weight of dry wipe

[‡ Viruses]

[‡‡ Bloodborne pathogens]

[¥] Follow [the] Special Instructions for Cleaning Prior to Disinfection [listed on label]

[§ Parvoviruses]

[‡] Clostridium difficile

^p [on] hard, nonporous surfaces [of]

^{pp} For unfamiliar surfaces, test in a small, inconspicuous area first. Consult medical equipment warranty information and equipment care and cleaning guides prior to selecting and using any surface disinfectant on plastic surfaces

^λ Corian is a [registered] trademark of [E. I. du Pont de Nemours and Company] [a subsidiary of] [E. I. du Pont de Nemours and Company] [or one or more of its affiliates]

^{λλ} Formica is a [registered] trademark of [The Diller Corporation] [a subsidiary of] [Formica] [Formica Corporation] [or one or more of its affiliates]

^{λλλ} Marlite is a [registered] trademark of [Marlite, Inc.] [a subsidiary of] [Marlite, Inc.] [or one or more of its affiliates]

^{□□□} Plexiglas is a [registered] trademark of [Arkema, Inc.] [a subsidiary of] [Arkema, Inc.] [or one or more of its affiliates]