

VIRAHOL®

HOSPITAL DISINFECTANT/CLEANER & INSTRUMENT PRESOAK*

NO MIXING REQUIRED

SEE RIGHT PANEL FOR ADDITIONAL INSTRUCTIONS

U.S. PATENT 5,145,663
KEEP OUT OF REACH OF CHILDREN
WARNING

ACCEPTED
APR 28 1995
Under the Federal Insecticide, Fungicide, and Rodenticide Act, as amended, for the pesticide registered under EPA Reg. No. 60142-1

ACTIVE INGREDIENT - Isopropanol ..	70%
INERT INGREDIENTS	30%
TOTAL	100%

SEE RIGHT PANEL FOR ADDITIONAL PRECAUTIONARY STATEMENTS

NET CONTENTS - 1 U.S. GALLON - (128 Fluid Ounces) - 3.785 Liters

BEST COPY AVAILABLE

RECAUTIONARY STATEMENTS

HAZARDS TO HUMANS & DOMESTIC ANIMALS

WARNING

Causes eye irritation. Do not get in eyes. Harmful if swallowed.

STATEMENT OF PRACTICAL TREATMENT: In case of eye contact, immediately flush eyes with plenty of water. Get medical attention if irritation persists.

PHYSICAL OR CHEMICAL HAZARDS: Flammable, keep away from heat & open flame.

STORAGE AND DISPOSAL

ORAGE: Store in a cool, dry place.

CONTAINER DISPOSAL: Triple rinse (or equivalent). Then refer to recycling or reconditioning, or puncture and dispose in a sanitary landfill, or incineration, or if allowed by state or local authorities, by burning. If burned, stay out of fire.

WASTE DISPOSAL: Do not contaminate water, food, or feed by storage or disposal. Wastes resulting from use of this product may be disposed of on site or at an appropriate waste disposal facility.

VERDIEN CORPORATION
11800 28th St. N.
St. Petersburg, FL 33716
(813) 572-6636

EPA Reg. No. 60142-1
EPA Est. No. 60142-FL-1

EFFECTIVE AGAINST A BROAD RANGE OF PATHOGENS

HIV-1 (Associated with AIDS)
Staphylococcus aureus
Poliovirus Type 1

Mycobacterium tuberculosis
Salmonella choleraesuis
Trichophyton mentagrophytes

Pseudomonas aeruginosa
Herpes simplex Type 2

COMPATIBLE !!! CONTACT WITH THESE MATERIALS

Aluminum
Epoxy
Nylon
Silicone
Titanium

APR 28 1995

Under the Federal Insecticide, Fungicide, and Rodenticide Act, as amended, for the pesticide registered under EPA Reg. No. 60142-1

Buna N
Natural Rubber
Polypropylene
Stainless Steel (304, 316, 440)
Tygon**

Cast Bronze
Neoprene
Rigid PVC
Teflon*
Viton*

* a registered trademark of E.I. duPont de Nemours ** a registered trademark of Norton

DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling. When using strong cleaners and disinfecting agents, it is always advisable to first test in an inconspicuous area. Virahol® Hospital Disinfectant/Cleaner & Instrument Presoak is designed for effective cleaning, disinfection, and deodorizing of hard inanimate surfaces such as walls, sink tops, tables, chairs, telephones and bed frames, and for the decontamination of instruments prior to sterilization.

TO DISINFECT CONTAMINATED SURFACES AND OBJECTS

For heavily soiled areas, or surfaces of objects soiled with blood or body fluids, a precleaning step is required. Use a clean sponge or wet cloth.

On precleaned surfaces, apply Virahol® Hospital Disinfectant/Cleaner & Instrument Presoak with a mop, cloth, sponge or hand sprayer so as to thoroughly wet precleaned surfaces. **DO NOT DILUTE.** Allow surfaces to remain wet for 10 minutes.

TO DECONTAMINATE INSTRUMENTS PRIOR TO STERILIZATION (Instrument Presoak)

Completely immerse instruments for 10 minutes prior to preparation for sterilization. This product is not to be used as a terminal sterilization level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either on a non-sterile site with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical medical devices prior to sterilization or high-level disinfection.

Kills HIV-1 on precleaned environmental surfaces & objects previously soiled with blood or body fluids in high traffic areas such as restrooms, which there is an expected likelihood of soiling of inanimate surfaces, objects with blood or body fluids, and in work areas. All objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus (HIV-1) (Associated with AIDS).

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON HARD INANIMATE SURFACES AND ON OBJECTS SOILED WITH BLOOD OR BODY FLUIDS

Personal Protection: When handling items soiled with blood or body fluids use disposable latex gloves, gown and shoe covers. Follow standard cleaning procedures. Blood and body fluids must be thoroughly cleaned from surfaces and objects before application of Virahol® Hospital Disinfectant/Cleaner & Instrument Presoak.

Disposal of Infectious Materials: Blood and other body fluids should be properly covered and disposed of according to applicable state and federal regulations for infectious waste disposal.

Contact Time: Apply Virahol® Hospital Disinfectant/Cleaner & Instrument Presoak with a mop, cloth, sponge or hand sprayer so as to thoroughly wet precleaned surfaces. **DO NOT DILUTE.** Allow surfaces to remain wet for 1 minute to assure that contact time is maintained. This contact time will not control other common types of viruses and bacteria. For example, 10 minutes contact time is required to kill Mycobacterium tuberculosis at 20°C.

BEST COPY AVAILABLE