

## **PLEASE NOTE**

**This image contains more than one label approved for this product on this date.**

60142-3

12/8/2004

Page 1 of 5

Veridien Corporation  
2875 MCI Drive, Suite B  
Pinellas Park, FL 33782

DEC 8 2004

Attention: Kresti A. Lyddon,

Subject: **Virahol Hospital Disinfectant Towlette**  
EPA Registration No. 60142-3  
Amendment Application Date: October 6, 2004  
EPA Received Date: November 29, 2004

This will acknowledge receipt of your notification, "for alternate brand names" submitted under the provisions of FIFRA section 3 (c) 9. Based on a review of the material, the following comment apply.


Additional Brand Name:

"Viranguard Medical Instrument Wipes "

This application is acceptable and has been made a part of the records for this file.

If you have any question concerning this letter, please contact me at 703-308-6341 or Portia Jenkins at (703) 308-6230.

Sincerely yours,

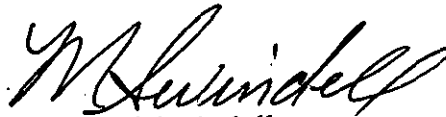

Marshall Swindell  
Product Manager 33  
Regulatory Management Branch I  
Antimicrobials Division (7510C)

CONCURRENCES

SYMBOL								
SURNAME								
DATE								

285



Please read instructions on reverse before completing form.

Form Approved. OMB No. 2070-0060



United States  
Environmental Protection Agency  
Washington, DC 20460

<input type="checkbox"/>	Registration
<input type="checkbox"/>	Amendment
<input checked="" type="checkbox"/>	Other

OPP Identifier Number

### Application for Pesticide - Section I

1. Company/Product Number 60142-3	2. EPA Product Manager Marshall Swindell	3. Proposed Classification <input checked="" type="checkbox"/> None <input type="checkbox"/> Restricted
4. Company/Product (Name) Virahol Hospital Surface Disinfectant Towelette	PM# 33	
5. Name and Address of Applicant (Include ZIP Code) Veridien Corporation 2875 MCI Drive, Suite B Pinellas Park, FL 33782  <input type="checkbox"/> Check if this is a new address	6. Expedited Review. In accordance with FIFRA Section 3(c)(3) (b)(ii), my product is similar or identical in composition and labeling to: EPA Reg. No. _____ Product Name _____	

### Section - II

<input type="checkbox"/> Amendment - Explain below.	<input type="checkbox"/> Final printed labels in response to Agency letter dated _____
<input type="checkbox"/> Resubmission in response to Agency letter dated _____	<input type="checkbox"/> "Me Too" Application.
<input checked="" type="checkbox"/> Notification - Explain below.	<input type="checkbox"/> Other - Explain below.

Explanation: Use additional page(s) if necessary. (For section I and Section II.)

Notification of Additional Brand Name -- "Viraguard Medical Instrument Wipes"

### Section - III

1. Material This Product Will Be Packaged In:				2. Type of Container	
Child-Resistant Packaging <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No	Unit Packaging <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No	Water Soluble Packaging <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No		<input type="checkbox"/> Metal	<input type="checkbox"/> Plastic
* Certification must be submitted		If "Yes" Unit Packaging wgt. .001oz	No. per container 100	If "Yes" Package wgt	No. per container
3. Location of Net Contents Information <input checked="" type="checkbox"/> Label <input type="checkbox"/> Container		4. Size(s) Retail Container 100 individual towelettes		5. Location of Label Directions <input type="checkbox"/> top <input type="checkbox"/> side	
6. Manner in Which Label is Affixed to Product <input checked="" type="checkbox"/> Lithograph <input type="checkbox"/> Paper glued <input type="checkbox"/> Stenciled <input type="checkbox"/> Other _____					

### Section - IV

1. Contact Point (Complete items directly below for identification of individual to be contacted, if necessary, to process this application.)		
Name Kresti A. Lyddon	Title Agent	Telephone No. (Include Area Code) 651-433-3328
Certification I certify that the statements I have made on this form and all attachments thereto are true, accurate and complete. I acknowledge that any knowingly false or misleading statement may be punishable by fine or imprisonment or both under applicable law.		6. Date Application Received (Stamped)
2. Signature 	3. Title Agent	
Typed Name Kresti A. Lyddon	5. Date 6 October 2004	

375

15 7/16"

100 WIPES  
SEE INSERT FOR USE INSTRUCTIONS  
AND PRECAUTIONS

**MEDICAL  
INSTRUMENT  
WIPES**



CONTAINS PATENTED VIRAGUARD® FORMULA

**VIRAGUARD**  
**KILLS  
99.9% OF  
VIRUSES &  
BACTERIA**

**KILLS  
99.9% OF  
VIRUSES &  
BACTERIA**

**VIRAGUARD**

**PRECAUTIONARY STATEMENTS**

**HAZARD TO HUMANS & DOMESTIC ANIMALS. KEEP OUT OF REACH OF CHILDREN.**

**Caution.** Causes moderate eye irritation. Harmful if inhaled or absorbed through the skin. Avoid contact with eyes, skin, or clothing. Wash thoroughly with soap and water after handling. **FIRST AID** Have the product container or label with you when calling a poison control center or doctor or going for treatment. **If in eyes:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing the eye. Call a poison control center or doctor for further treatment and advice. **If on skin or clothing:** Take off contaminated clothing. Rinse skin with plenty of water for 15-20 minutes. Call a poison control center or doctor for further treatment and advice. **If inhaled:** Move person to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably mouth-to-mouth, if possible. Call a poison control center or doctor for further treatment and advice. **PHYSICAL OR CHEMICAL HAZARDS:** Flammable. Keep away from heat, sparks and open flames. **STORAGE AND DISPOSAL:** Do not contaminate water, food, or feed by storage and disposal. **PESTICIDE STORAGE:** Store in a cool, dry place, at room temperature. **CONTAINER DISPOSAL:** Discard empty container in trash. **PESTICIDE DISPOSAL:** Discard used towlettes and empty package in regular trash.

See Carton Insert For Product Use Instructions and Other Important Information

**MEDICAL  
INSTRUMENT WIPES**

*Effective Against* BACTERIA, VIRUSES, FUNGI including: STREPTOCOCCUS, MRSA, VRE, E. coli, HERPES, HIV/AIDS, TB and more.

**CAUTION  
KEEP OUT OF REACH  
OF CHILDREN**

**ACTIVE INGREDIENT:**  
ISOPROPANOL.....70%  
OTHER INGREDIENTS\*...30%  
TOTAL.....100%

\*Does not include the weight of towlette

EPA Reg.No. 60142-3  
EPA Est.No. 81133-CA-001

**NOT FOR INDIVIDUAL SALE**



MADE IN CANADA for  
Veridien Corporation, Pinellas Park, FL 33782 1-800-345-5444 www.veridien.com

8 3/8"



POSITION ONLY



485




CMYK

AUGUST 18 2004

**KILLS  
99.9% OF  
VIRUSES &  
BACTERIA**

**VIRAGUARD**

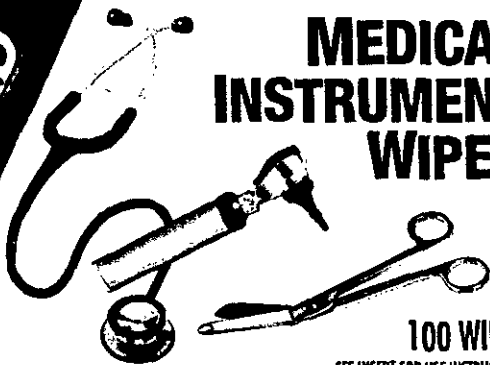

# MEDICAL INSTRUMENT WIPES

**100 WIPES**

SEE INSERT FOR USE INSTRUCTIONS  
AND PRECAUTIONS

CONTAINS PATENTED VIRAGUARD® FORMULA

# MEDICAL INSTRUMENT WIPES

Effective Against BACTERIA, VIRUSES,  
FUNGI including: STREPTOCOCCUS,  
MRSA, VRE, E. coli, HERPES,  
HIV/AIDS, TB and more.

**NOT FOR INDIVIDUAL SALE**



MADE IN CANADA for  
Veridien Corporation, Pinellas Park, FL 33782 1-800-345-5444 [www.veridien.com](http://www.veridien.com)

**CAUTION**  
KEEP OUT OF REACH  
OF CHILDREN

ACTIVE INGREDIENT:  
ISOPROPANOL.....70%  
OTHER INGREDIENTS\*...30%  
TOTAL.....100%

\*Does not include the weight  
of towlette

EPA Reg.No. 60142-3  
EPA Est No. 81133-CAN-001

**VIRAGUARD®**  
**MEDICAL INSTRUMENT WIPES**  
**FOR DISINFECTING, CLEANING AND**  
**DEODORIZING HARD, NON-POROUS**  
**ENVIRONMENTAL SURFACES**  
**NON-STAINING**

**KEEP OUT OF REACH OF CHILDREN**  
**CAUTION**

<b>ACTIVE INGREDIENT:</b>	
ISOPROPANOL.....	70%
OTHER INGREDIENTS*.....	30%
TOTAL.....	100%

\*Does not include the weight of towelette

EPA Reg. No. 60142-3 EPA Est. No. 81133-CAN-001  
NET CONTENTS: 100 Individual Saturated Swabs.

**PRECAUTIONARY STATEMENTS**

**HAZARD TO HUMANS & DOMESTIC ANIMALS**  
**KEEP OUT OF REACH OF CHILDREN**

**Caution.** Causes moderate eye irritation. Harmful if inhaled or absorbed through the skin. Avoid contact with eyes, skin, or clothing. Wash thoroughly with soap and water after handling.

**FIRST AID**

Have the product container or label with you when calling a poison control center or doctor or going for treatment. **If in eyes:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing the eye. **Call a poison control center or doctor for further treatment and advice. If on skin or clothing:** Take off contaminated clothing. Rinse skin with plenty of water for 15-20 minutes. Call a poison control center or doctor for further treatment and advice. **If inhaled:** Move person to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably mouth-to-mouth, if possible. Call a poison control center or doctor for further treatment and advice.

**PHYSICAL OR CHEMICAL HAZARDS:**

Flammable. Keep away from heat, sparks and open flame.

**STORAGE AND DISPOSAL:** Do not contaminate water, food, or feed by storage and disposal.

**PESTICIDE STORAGE:** Store in a cool, dry place, at room temperature.

**CONTAINER DISPOSAL:** Discard empty container in trash.

**PESTICIDE DISPOSAL:** Discard used towelettes and empty package in regular trash.

Verdien Corporation  
2875 MCI Drive, Suite B  
Pinellas Park, FL 33762-6105  
1-800-345-5444

**EFFECTIVE AGAINST**  
**A BROAD RANGE OF PATHOGENS**

**TUBERCULOCIDAL:** *Mycobacterium bovis* (BCG)  
**BACTERICIDAL:** *Staphylococcus aureus* (MRSA), ATCC#33591; Vancomycin-resistant *Enterococcus faecium* (VRE), ATCC#51559; *Streptococcus pyogenes*, ATCC#19815; *Pseudomonas aeruginosa*, ATCC#15442; *Staphylococcus aureus*, ATCC#6538; *Salmonella choleraesuis*, ATCC#10708; *Escherichia coli*, ATCC#11229.  
**FUNGICIDAL:** *Trichophyton mentagrophytes*, ATCC#9533; **VIRUCIDAL:** Herpes simplex type 1,

ATCC#VR260; HIV-1 (AIDS Virus) inactivates HIV-1 on pre-cleaned environmental surfaces and objects previously soiled with blood or body fluids in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces or objects with blood or body fluids, and in which the surfaces and objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus (HIV-1) (AIDS virus).

**DIRECTIONS FOR USE:**

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

**FOR DISINFECTING, CLEANING AND DEODORIZING HARD, NON-POROUS SURFACES:**

**GENERAL INFORMATION:**

An effective disinfectant and deodorant towelette for hospitals, clinics, medical and dental offices, laboratories, industrial clean rooms, nursing homes, ambulances, veterinary offices, hotels, restaurants, schools, professional salons and health spas. This product is designed for effective cleaning, disinfecting and deodorizing of hard non-porous, inanimate surfaces such as walls, sink tops, stainless steel surfaces, equipment, tables, chairs, telephones, toilet seats and bed frames. When using strong cleaners and disinfecting agents, it is always advisable to first test in an inconspicuous area. This product may be used in kitchen preparation areas on counters, sinks, tables, chairs, thermometers and appliances. For surfaces which may come into contact with food a potable water rinse is required. This product may also be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body.

**APPLICATION INSTRUCTIONS:**

Pull out towelette from dispenser and wipe surface to be disinfected with towelette. Make certain to wet surface thoroughly. Surface must remain visibly wet for a full 10 minutes. Use additional towelettes if needed to assure a 10 minute wet contact time. For surface which may come into contact with food, a potable water rinse is required. Dispose of towelette in regular trash.

**SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON HARD INANIMATE SURFACES AND/OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS:**

**Personal Protection:** When handling items soiled with blood or body fluids, use disposable latex gloves, gown, mask and/or eye coverings.

**Cleaning Procedures:** Blood and body fluids must be thoroughly cleaned from surfaces and objects before application of this product.

**Contact Time:** Surface must remain visibly wet for a full 10 minutes. Use additional towelettes if needed to assure a 10 minutes wet contact time.

**Disposal of Infectious Materials:** Blood and other body fluids should be autoclaved and disposed of according to federal, state, and local regulations for infectious waste disposal.

No warranty, either expressed or implied, is given to use contrary to instructions given herein.

**NEXT**

**LABEL**

60142-3

12/8/2004

Page 175  
DEC 08 2004

**Veridien Corporation**  
2875 MCI Drive, Suite B  
Pinellas Park, FL 33782

Attention: Kresti A. Lyddon

Subject: **Virahol Hospital Disinfectant Towelette**  
EPA Registration No. 60142-3  
Amendment Application Date: November 29, 2004  
EPA Received Date: November 29, 2004

This will acknowledge receipt of your notification, "for alternate brand names" submitted under the provisions of FIFRA section 3 (c) 9. Based on a review of the material, the following comment apply.


Additional Brand Name

Viraguard Telephone & Keyboard Wipes

This application is acceptable and has been made a part of the records for this file.

If you have any question concerning this letter, please contact me at 703-308-6341 or Portia Jenkins at (703) 308-6230.

Sincerely yours,

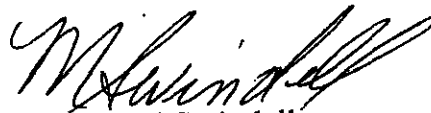

Marshall Swindell  
Product Manager 33  
Regulatory Management Branch I  
Antimicrobials Division (7510C)

CONCURRENCES

SYMBOL	SURNAME	DATE						



285



Please read instructions on reverse before completing form.

Form Approved. OMB No. 2070-0060

 <b>United States</b> <b>Environmental Protection Agency</b> Washington, DC 20460	<input type="checkbox"/> Registration <input type="checkbox"/> Amendment <input checked="" type="checkbox"/> Other	OPP Identifier Number
--------------------------------------------------------------------------------------------	--------------------------------------------------------------------------------------------------------------------------	-----------------------

### Application for Pesticide - Section I

<b>1. Company/Product Number</b> 60142-3	<b>2. EPA Product Manager</b> Marshall Swindell	<b>3. Proposed Classification</b> <input checked="" type="checkbox"/> None <input type="checkbox"/> Restricted
<b>4. Company/Product (Name)</b> Virahol Hospital Surface Disinfectant Towelette	<b>PM#</b> 33	
<b>5. Name and Address of Applicant (Include ZIP Code)</b> Veridien Corporation 2875 MCI Drive, Suite B Pinellas Park, FL 33782  <input type="checkbox"/> Check if this is a new address		<b>6. Expedited Review.</b> In accordance with FIFRA Section 3(c)(3) (b)(II), my product is similar or identical in composition and labeling to: EPA Reg. No. _____  Product Name _____

### Section - II

<input type="checkbox"/> Amendment - Explain below.	<input type="checkbox"/> Final printed labels in response to Agency letter dated _____
<input type="checkbox"/> Resubmission in response to Agency letter dated _____	<input type="checkbox"/> "Me Too" Application.
<input checked="" type="checkbox"/> Notification - Explain below.	<input type="checkbox"/> Other - Explain below.

**Explanation:** Use additional page(s) if necessary. (For section I and Section II.)

Notification of Additional Brand Name -- "Viraguard Telephone & Keyboard Wipes"

### Section - III

<b>1. Material This Product Will Be Packaged In:</b>				<b>2. Type of Container</b>	
<b>Child-Resistant Packaging</b> <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No	<b>Unit Packaging</b> <input checked="" type="checkbox"/> Yes <input type="checkbox"/> No	<b>Water Soluble Packaging</b> <input type="checkbox"/> Yes <input checked="" type="checkbox"/> No	<input type="checkbox"/> Metal <input type="checkbox"/> Plastic <input type="checkbox"/> Glass <input checked="" type="checkbox"/> Paper <input type="checkbox"/> Other (Specify) _____	* Certification must be submitted	
	If "Yes" Unit Packaging wgt. .001oz	No. per container 100	If "Yes" Package wgt	No. per container	
<b>3. Location of Net Contents Information</b>		<b>4. Size(s) Retail Container</b>		<b>5. Location of Label Directions</b>	
<input checked="" type="checkbox"/> Label <input type="checkbox"/> Container		100 individual towelettes		<input type="checkbox"/> Top <input type="checkbox"/> Bottom	
<b>6. Manner in Which Label is Affixed to Product</b>					
<input checked="" type="checkbox"/> Lithograph Paper glued <input type="checkbox"/> Other _____ <input type="checkbox"/> Stenciled					

### Section - IV

<b>1. Contact Point (Complete items directly below for identification of individual to be contacted, if necessary, to process this application.)</b>		
<b>Name</b> Kresti A. Lyddon	<b>Title</b> Agent	<b>Telephone No. (Include Area Code)</b> 651-433-3328
<b>Certification</b> I certify that the statements I have made on this form and all attachments thereto are true, accurate and complete. I acknowledge that any knowingly false or misleading statement may be punishable by fine or imprisonment or both under applicable law.		<b>6. Date Application Received (Stamped)</b>
<b>2. Signature</b> 	<b>3. Title</b> Agent	
<b>Typed Name</b> Kresti A. Lyddon	<b>5. Date</b> 6 October 2004	




CMYK

AUGUST 18 2004

**KILLS  
99.9% OF  
VIRUSES &  
BACTERIA**

**VIRAGUARD**

# TELEPHONE & KEYBOARD WIPES

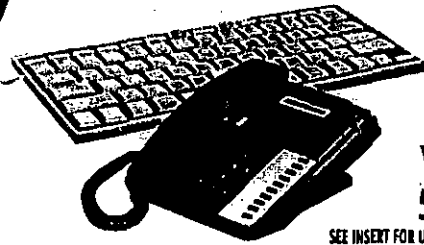

**VERIDIEN®  
50 WIPES**

SEE INSERT FOR USE INSTRUCTIONS  
AND PRECAUTIONS

CONTAINS PATENTED VIRAGUARD® FORMULA

### TELEPHONE & CELL PHONE WIPES

Effective Against  
BACTERIA, VIRUSES, FUNGI including:  
STREPTOCOCCUS, MRSA, VRE,  
E. coli, HERPES, HIV / AIDS, TB  
and more.

#### CAUTION

KEEP OUT OF REACH OF CHILDREN

#### ACTIVE INGREDIENT:

ISOPROPANOL.....	7.0%
OTHER INGREDIENTS*.....	30%
TOTAL.....	100%

\*Does not include the weight of towelette  
EPA Reg.No. 60142-3  
EPA Est.No. 01133-CAN-031

MADE IN CANADA for  
Veridien Corporation, Pinellas Park, FL 33782  
1-800-345-6444 www.veridien.com

485

12 1/2"

6 5/16"

50 WIPES  
CONTAINS ALCOHOL  
FOR EXTERNAL USE ONLY  
NOT FOR INDIVIDUAL SALE

# TELEPHONE & KEYBOARD WIPES

**VIRAGUARD**

**KILLS  
99.9% OF  
VIRUSES &  
BACTERIA**

**VIRAGUARD**

### PRECAUTIONARY STATEMENTS

**HAZARD TO HUMANS & DOMESTIC ANIMALS. KEEP OUT OF REACH OF CHILDREN.**

**Caution.** Causes moderate eye irritation. Harmful if inhaled or absorbed through the skin. Avoid contact with eyes, skin, or clothing. Wash thoroughly with soap and water after handling. **FIRST AID** Have the product container or label with you when calling a poison control center or doctor or going for treatment. **If in eyes:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing the eye. Call a poison control center or doctor for further treatment and advice. **If on skin or clothing:** Take off contaminated clothing. Rinse skin with plenty of water for 15-20 minutes. Call a poison control center or doctor for further treatment and advice. **If inhaled:** Allow person to rest. If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably mouth-to-mouth, if possible. Call a poison control center or doctor for further treatment and advice. **PHYSICAL OR CHEMICAL HAZARDS:** Flammable. Keep away from heat, sparks and open flame. **STORAGE AND DISPOSAL:** Do not contaminate water, food, or feed by storage and disposal. **PESTICIDE STORAGE:** Store in a cool, dry place, at room temperature. **CONTAINER DISPOSAL:** Discard empty container in trash. **PESTICIDE DISPOSAL:** Discard used towels and empty package in regular trash.

See Carton Insert For Product Use Instructions and Other Important Information

### TELEPHONE & CELL PHONE WIPES

*Effective Against*  
**BACTERIA, VIRUSES, FUNGI** including:  
**STREPTOCOCCUS, MRSA, VRE,**  
**E. coli, HERPES, HIV / AIDS, TB**  
and more.

**CAUTION**  
**KEEP OUT OF REACH OF CHILDREN**

**ACTIVE INGREDIENT:**  
ISOPROPANOL.....70%  
OTHER INGREDIENTS\*...30%  
TOTAL.....100%

\*Does not include the weight of towellette  
EPA Reg.No. 601423  
EPA Est No. 01133-CAN-001

MADE IN CANADA for  
Veridian Corporation, Pinellas Park, FL 33782  
1-833-345-5444 www.veridian.com



**VIRAGUARD®**  
**TELEPHONE & KEYBOARD WIPES**  
**FOR DISINFECTING, CLEANING AND**  
**DEODORIZING HARD, NON-POROUS**  
**ENVIRONMENTAL SURFACES**  
**NON-STAINING**  
**KEEP OUT OF REACH OF CHILDREN**  
**CAUTION**

**ACTIVE INGREDIENT:**  
 ISOPROPANOL.....70%  
 OTHER INGREDIENTS\*.....30%  
 TOTAL.....100%

\*Does not include the weight of towlettes

EPA Reg. No. 60142-3 EPA Est. No. 81133-CAN-001  
 NET CONTENTS: 100 Individual Saturated Swabs.

**PRECAUTIONARY STATEMENTS**

**HAZARD TO HUMANS & DOMESTIC ANIMALS**  
**KEEP OUT OF REACH OF CHILDREN**

Caution. Causes moderate eye irritation.  
 Harmful if inhaled or absorbed through the skin.  
 Avoid contact with eyes, skin, or clothing. Wash thoroughly with soap and water after handling.

**FIRST AID**

Have the product container or label with you when calling a poison control center or doctor or going for treatment. **If in eyes:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing the eye. Call a poison control center or doctor for further treatment and advice. **If on skin or clothing:** Take off contaminated clothing. Rinse skin with plenty of water for 15-20 minutes. Call a poison control center or doctor for further treatment and advice. **If inhaled:** Move person to fresh air. If person is not breathing, call 911 or an ambulance, then give artificial respiration, preferably mouth-to-mouth, if possible. Call a poison control center or doctor for further treatment and advice.

**PHYSICAL OR CHEMICAL HAZARDS:**

Flammable. Keep away from heat, sparks and open flame.

**STORAGE AND DISPOSAL:** Do not contaminate water, food, or feed by storage and disposal.

**PESTICIDE STORAGE:** Store in a cool, dry place, at room temperature.

**CONTAINER DISPOSAL:** Discard empty container in trash.

**PESTICIDE DISPOSAL:** Discard used towlettes and empty package in regular trash.

Veridien Corporation  
 2875 MCI Drive, Suite B  
 Pinellas Park, FL 33782-6105  
 1-800-345-5444

**EFFECTIVE AGAINST**  
**A BROAD RANGE OF PATHOGENS**

**TUBERCULOCIDAL:** Mycobacterium bovis (BCG)  
**BACTERICIDAL:** Staphylococcus aureus (MRSA), ATCC#33591; Vancomycin-resistant Enterococcus faecium (VRE), ATCC#51559; Streptococcus pyogenes, ATCC#19815; Pseudomonas aeruginosa, ATCC#15442; Staphylococcus aureus, ATCC#6538; Salmonella choleraesuis, ATCC#10708; Escherichia coli, ATCC#11229.  
**FUNGICIDAL:** Trichophyton mentagrophytes, ATCC#9533; **VIRUCIDAL:** Herpes simplex type 1,

ATCC#VR260; HIV-1 (AIDS Virus) inactivates HIV-1 on precleaned environmental surfaces and objects previously soiled with blood or body fluids in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces or objects with blood or body fluids, and in which the surfaces and objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus (HIV-1) (AIDS virus).

**DIRECTIONS FOR USE:**

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

**FOR DISINFECTING, CLEANING AND DEODORIZING HARD, NON-POROUS SURFACES:**

**GENERAL INFORMATION:**

An effective disinfectant and deodorant towlette for hospitals, clinics, medical and dental offices, laboratories, industrial clean rooms, nursing homes, ambulances, veterinary offices, hotels, restaurants, schools, professional salons and health spas. This product is designed for effective cleaning, disinfecting and deodorizing of hard non-porous, inanimate surfaces such as walls, sink tops, stainless steel surfaces, equipment, tables, chairs, telephones, toilet seats and bed frames. When using strong cleaners and disinfecting agents, it is always advisable to first test in an inconspicuous area. This product may be used in kitchen preparation areas on counters, sinks, tables, chairs, thermometers and appliances. For surfaces which may come into contact with food a potable water rinse is required. This product may also be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body.

**APPLICATION INSTRUCTIONS:**

Pull out towlette from dispenser and wipe surface to be disinfected with towlette. Make certain to wet surface thoroughly. Surface must remain visibly wet for a full 10 minutes. Use additional towlettes if needed to assure a 10 minute wet contact time. For surface which may come into contact with food, a potable water rinse is required. Dispose of towlette in regular trash.

**SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON HARD INANIMATE SURFACES AND/OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS:**

**Personal Protection:** When handling items soiled with blood or body fluids, use disposable latex gloves, gown, mask and/or eye coverings.  
**Cleaning Procedures:** Blood and body fluids must be thoroughly cleaned from surfaces and objects before application of this product.  
**Contact Time:** Surface must remain visibly wet for a full 10 minutes. Use additional towlettes if needed to assure a 10 minutes wet contact time.  
**Disposal of Infectious Materials:** Blood and other body fluids should be autoclaved and disposed of according to federal, state, and local regulations for infectious waste disposal.  
 No warranty, either expressed or implied, is given to use contrary to instructions given herein.