

...recommended that disinfection be carried out immediately after and just prior to use of the hemodialysis delivery system.

Flush equipment thoroughly with water prior to filling the delivery system with DIACIDE HD Solution. After filling the delivery system, hold DIACIDE HD Solution in the system for a minimum of 15 minutes. Within 15 minutes contact time at 20°C, DIACIDE HD Solution will destroy vegetative pathogens including *Pseudomonas aeruginosa* and hydrophilic and lipophilic viruses (Poliovirus Type 1; Coxsackievirus B-1; Influenza, Type A; Dengue, Herpes simplex Type 1) on inanimate surfaces. Prior to machine use, drain DIACIDE HD Solution and thoroughly rinse with water. To assure complete rinsing, test final rinse water using a glutaraldehyde residual test ampule.

This is a single use product.

NOTE: DIACIDE HD Concentrate should remain in hemodialysis delivery systems for more than 12 hours.

MANUFACTURED BY:

GULFSTREAM MEDICAL INC.

550 Pleasant St.

Winthrop, MA 02152

DIACIDE HD

disinfecting solution

Active ingredient
Glutaraldehyde 20%
Inert ingredients 76%
Total 100.00%

NOTE: This product is a concentrate and must be activated and diluted immediately prior to use. Activator must be added to fluid before this product is effective. See Directions for Activation and Dilution.

Keep out of reach of children.

DANGER: See right side panel for precautions.

GULFSTREAM

GULFSTREAM

ACCEPTED
MAR 27 1991
...the Food and Drug Administration.
...the Federal Insecticide, Fungicide, and Rodenticide Act.
...the Pesticide
EPA Reg. No. 63720-1

BEST AVAILABLE COPY

...removal of pooling.

Eyes — Corrosive, causes eye damage. Do not get in eyes. In case of contact, flush with water immediately and get medical attention.

Ingestion — Harmful if swallowed. Avoid contamination of food.

Inhalation — Avoid splashes. Use in a well ventilated area.

Skin — Avoid skin contact as possibility of sensitization exists. Can cause skin irritation. For skin contact, flush thoroughly with water. Get medical attention if irritation occurs.

STORAGE AND DISPOSAL:

Keep in a cool place. Triple rinse empty container with water and dispose in an incinerator or landfill approved for pesticide containers.

CONTENTS 32 FL. OZ. (0.95 LITERS)

MADE IN U.S.A.

CONTROL NUMBER ON BOTTOM

NOTE DATED HERE

MIXING DATE _____

EXPIRATION DATE _____