

65787-2

5/11/2011

1/3



UNITED STATES ENVIRONMENTAL PROTECTION AGENCY  
WASHINGTON, D.C. 20460

**FILE COPY**

May 11, 2011

OFFICE OF CHEMICAL SAFETY  
AND POLLUTION PREVENTION

Jeff Jones  
Agent, Alcavis HDC, LLC  
c/o Delta Analytical Corporation  
12510 Prosperity Drive, Suite 160  
Silver Spring, MD 20904

Subject: BLEACH-WIPE  
EPA Reg.#: 65787-2  
Amendment Date: February 28, 2011  
Receipt Date: March 3, 2011

Dear Mr. Jones:

This acknowledges the receipt of your letter of February 28, 2011 in response to the Agency initiated Letter dated February 9, 2011, regarding the label amendment for Bleach-Wipe, EPA Reg# 65787-2.

**Proposed Amendment:**

Label corrections made for "FIRST AID", "PRECAUTIONARY STATEMENTS", "DIRECTIONS FOR USE", and "STORAGE DISPOSAL" statements, are acceptable.

**General Comments:**

A stamped copy of the accepted label is enclosed. Submit one copy of your final printed labeling before distributing or selling the product bearing the revised labeling.

If you have any questions on this letter, please contact David Liem by email at [liem.david@epa.gov](mailto:liem.david@epa.gov) or call at 703-305-1284.

Sincerely

A handwritten signature in black ink that reads "Wanda Y. Henson".

Wanda Y. Henson  
Acting Product Manager (32)  
Regulatory Management Branch II  
Antimicrobial Division (7510P)


ORIG

[LOGO]

[CAT # [09041][09053]]

TM

# BLEACH-WIPE

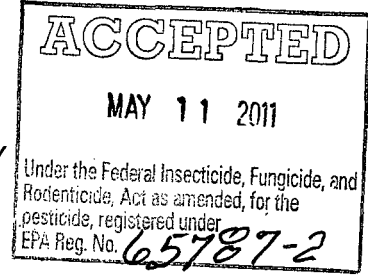

FOR INSTITUTIONAL AND COMMERCIAL USE ONLY

Bleach Disinfectant Wipe

***For wiping hard non-porous environmental surfaces and patient care equipment***

Active ingredient:

Sodium hypochlorite: ..... 0.525%

Other ingredients:..... 99.475%

Total:..... 100.000%

5,000 ppm Available Chlorine

## KEEP OUT OF REACH OF CHILDREN CAUTION


[See [back][reverse] [label][side] for additional precautionary statements]

FIRST AID	
If in eyes:	<ul style="list-style-type: none"> <li>• Hold eye open and rinse slowly and gently with water for 15 - 20 minutes.</li> <li>• Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.</li> </ul>
If on skin or clothing:	<ul style="list-style-type: none"> <li>• Take off contaminated clothing.</li> <li>• Rinse skin immediately with plenty of water for 15 - 20 minutes.</li> </ul>
Have the product container or label with you when calling a poison control center or doctor, or going for treatment. In an emergency, call a doctor or Poison Control Center at 1-800-222-1222.	

Manufactured by:  
Alcavis HDC, LLC  
8322 Helgerman Ct.  
Gaithersburg, MD 20877  
800-726-2308

EPA Reg. No. 65787-2  
EPA Est. No. 65787-MD-001

PREMOISTENED [8" x 10"] [5" x 8"] TOWELETTE  
Net Contents: [0.7] [0.4] oz.

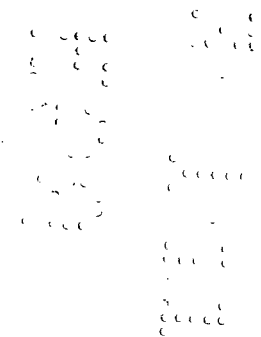

**PRECAUTIONARY STATEMENTS**  
**Hazards to Humans and Domestic Animals**

**CAUTION:** Harmful if absorbed through skin. Liquid may cause moderate eye irritation. Avoid contact with skin, eyes or clothing. Wear gloves for sensitive skin or prolonged use since it may cause skin irritation. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

**DIRECTIONS FOR USE:**

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Suggested areas of usage:

Medical, Dental and Laboratory Counters, Exam Tables, Carts, Point of Care Equipment, Telephones, Sink Tops.

- Always use personal protective equipment (gloves).
- Open wipe packet.
- Remove premoistened towelette.
- Apply towelette to hard non-porous surface to be disinfected.
- Thoroughly wet surface making sure treated surface remains visibly wet for [one] [1] full minute.
- Use additional wipe if needed to assure continuous [one] [1] minute wet contact time.
- Allow surface area to air dry and discard used towelette.
- Do not re-use used towelette.

Do not use this product as a terminal sterilant/high level disinfectant on any surface or instrument that is introduced directly into human body, either into or in contact with the bloodstream or normally sterile areas of the body, or contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

**STORAGE AND DISPOSAL**

Do not contaminate water, food or feed by storage or disposal.

**Storage:** Store this product in a cool, dry, and well-ventilated area away from direct sunlight and heat. **Towelette Disposal:** Do not reuse towelette. Dispose of used towelette and empty pack in trash. Do not flush in toilet.

*[Optional claims:]*

[Bleach-Wipe is a 1:10 dilution of 5.25% sodium hypochlorite solution with filtered water to comply with recommended infection control protocol.]

[Tuberculocidal] [and] [Bactericidal]

[One [1] minute minimum contact time]

[Maximum surface area that can be disinfected per towelette is [3' x 3'] [3' x 1.5']]