



UNITED STATES ENVIRONMENTAL PROTECTION AGENCY
WASHINGTON, DC 20460

OFFICE OF CHEMICAL SAFETY
AND POLLUTION PREVENTION

June 23, 2016

J. Evelyn Lawson
Federal Registration Specialist
The Clorox Company
C/O PS&RC
P.O Box 493
Pleasanton, CA 94566-0803

Subject: PRIA Label Amendment Add Additional Disinfection Microorganisms
Product Name: CPPC Tsunami
EPA Registration Number: 67619-12
Application Date: November 20, 2015
Decision Number: 519594

Dear Ms. Lawson:

The amended label referred to above, submitted in connection with registration under the Federal Insecticide, Fungicide and Rodenticide Act, as amended, is acceptable. This approval does not affect any conditions that were previously imposed on this registration. You continue to be subject to existing conditions on your registration and any deadlines connected with them.

A stamped copy of your labeling is enclosed for your records. This labeling supersedes all previously accepted labeling. You must submit one copy of the final printed labeling before you release the product for shipment with the new labeling. In accordance with 40 CFR 152.130(c), you may distribute or sell this product under the previously approved labeling for 18 months from the date of this letter. After 18 months, you may only distribute or sell this product if it bears this new revised labeling or subsequently approved labeling. "To distribute or sell" is defined under FIFRA section 2(gg) and its implementing regulation at 40 CFR 152.3.

Should you wish to add/retain a reference to the company's website on your label, then please be aware that the website becomes labeling under the Federal Insecticide Fungicide and Rodenticide Act and is subject to review by the Agency. If the website is false or misleading, the product would be misbranded and unlawful to sell or distribute under FIFRA section 12(a)(1)(E). 40 CFR 156.10(a)(5) list examples of statements EPA may consider false or misleading. In addition, regardless of whether a website is referenced on your product's label, claims made on the website may not substantially differ from those claims approved through the registration process. Therefore, should the Agency find or if it is brought to our attention that a website contains false or misleading statements or claims substantially differing from the EPA approved registration, the website will be referred to the EPA's Office of Enforcement and Compliance.

Your release for shipment of the product constitutes acceptance of these conditions. If these conditions are not complied with, the registration will be subject to cancellation in accordance

Page 2 of 2
EPA Reg. No. 67619-12
Decision No. 519594

with FIFRA section 6. If you have any questions, please contact Ben Chambliss by phone at (703) 308-8174, or via email at chambliss.ben@epa.gov.

Sincerely,


A handwritten signature in black ink, appearing to read "DF", with a long horizontal stroke extending to the right.

Demson Fuller, Product Manager 32
Regulatory Management Branch II
Antimicrobials Division (7510P)
Office of Pesticide Programs

Enclosure

Note: Bold, italicized text and (italicized text within parentheses) is information for the reader and is not part of the label. [Bracketed] information and phrase sequence are optional unless specified in 40 CFR 156.10. Variables such as CAPITALIZATION, plurals or singular, bold and italics are optional unless specified in 40 CFR. The term "wipe(s)", is interchangeable with towel(s), towelette(s) and cloth(s) throughout except for the product brand name which can only be changed by notification or amendment. Listed uses and use sites can be used partially or in their entirety. R0618196

CPPC Tsunami

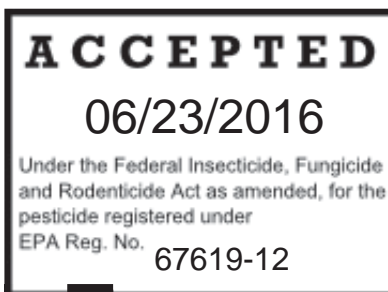

ACTIVE INGREDIENT:

Sodium Hypochlorite 0.55%[††]
Other Ingredients: 99.45%
Total: 100.00%
[Available Chlorine...0.52%]
[5200 PPM [Parts Per Million] Available Chlorine]

[††Does not include weight of dry wipe.]

**KEEP OUT OF REACH OF CHILDREN
CAUTION:**

Liquid causes moderate eye irritation.
See back label for additional precautionary statements
**Basic formula and alternates A02, A04, A06, A08 and A10
through 16:** Contains no phosphorus

**Additional statement for single count wipe pouch:
See carton for additional directions for use.
-or- Read carton before use.**

[Net] [Contents]_____
**Each package type and size will bear the
actual net contents in pounds and ounces**
Premoistened Wipes
[Individually Packaged]

Note: Bold, italicized text and (italicized text within parentheses) is information for the reader and is not part of the label. [Bracketed] information and phrase sequence are optional unless specified in 40 CFR 156.10. Variables such as CAPITALIZATION, plurals or singular, bold and italics are optional unless specified in 40 CFR. The term "wipe(s)", is interchangeable with towel(s), towelette(s) and cloth(s) throughout except for the product brand name which can only be changed by notification or amendment. Listed uses and use sites can be used partially or in their entirety. R0618196

A list of this product's ingredients is available at www.IngredientsInside.com
-or- A list of [this product's] [**product name**] ingredients is available at [**insert company website**].

Mfd. for & © YYYY Clorox Professional Products Company,
1221 Broadway, Oakland, CA 94612

Made in the USA

EPA Reg. No. 67619-12

EPA Est. No. 5813-CA-3 (A8 or H8), CA-6 (TPP), GA-1 (A4),
GA-2 (VG) IL-2 (VF);

2686-WI-8 (07); 8251-OH-1 (Q6), WI-2 (KW),

WI-3 (KV), WI-5 (26), 41925-WI-1 (TA), 56952-AR-1 (RL), WI-1 (RK),

63650-WI-1 (JZ).

Beginning of batch code indicates Est. No.



[[Use your phone to] go here -and/or- scan [[the] tag -or- [this] code] to learn -or- get -or- see [more] uses -or- information [on how-to-use [This Product]] -or- [how-to] tips -or- [how-to [-use]] videos]

additional optional text

Clorox Customer Assistance 1-800-234-7700. Any Questions?

The experts at Clorox are standing by to answer any of your questions. -or- Questions or Comments? Call [toll-free] 1-800-234-7700.

Satisfaction Guaranteed[!] At Clorox we have prided ourselves in making consistent quality products [for over 95 -or- **insert number**

years], and we are dedicated to ensuring your total satisfaction with our product. If you are not completely satisfied, please call the

number below -or- 1-800-234-7700. For MSDS information please go to www.cloroxprofessional.com

Visit us at www.cloroxprofessional.com

To re-order [call (**insert company name**) at (**insert phone number**)] -and/or- [go to (**insert website**)].

Commercial Solutions is a [registered] trademark of The Clorox Company -or- Clorox Professional Products Company.

Clorox is a [registered] trademark of The Clorox Company.

insert trademark is a [registered] trademark of The Clorox Company -or- Clorox Professional Products Company

PRECAUTIONARY STATEMENTS: HAZARDS TO HUMANS & DOMESTIC ANIMALS

CAUTION: Liquid causes moderate eye irritation. Do not get in eyes or on clothing. Avoid contact with clothing. Wear protective eyewear.

Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. [[For sensitive skin or prolonged use,] wear gloves.]

FIRST AID: If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor or going for treatment. For further information, call 1-800-234-7700. -or- Questions? Call 1-800-234-7700.

PHYSICAL AND CHEMICAL HAZARDS: This product contains bleach. Do not use this product with other chemicals such as ammonia, toilet bowl cleaners, rust removers, or acid, as this releases hazardous gases.

For canisters or buckets not intended to be refilled by the consumer

STORAGE AND DISPOSAL:

Store this product in a cool [dry] area, [away from direct sunlight and heat] to avoid deterioration. Dispose of wipe in trash after use. Do not flush. Nonrefillable container. Do not reuse or refill empty container. [Triple rinse promptly, then] Recycle empty container or discard in trash.

For canisters or buckets intended to be refilled by the consumer

STORAGE AND DISPOSAL:

Store this product in a cool [dry] area, [away from direct sunlight and heat] to avoid deterioration. Dispose of wipe in trash after use. Do not flush. Do not reuse or refill except as described in the directions for use. Refill only with this product. [Triple rinse promptly, then] Recycle empty container or discard in trash.

For single count or multi-count pouch

STORAGE AND DISPOSAL:

Dispose of wipe in trash after use. Do not flush. Recycle empty pouch -or- wrapper -or- bag or discard in trash.

PACKAGING RELATED STATEMENTS

DISPENSER DIRECTIONS: Remove lid and discard seal. Insert corner of center sheet through flap opening on lid. DO NOT push finger through opening. Replace lid and pull wipe at an angle. The next sheet pops up automatically. When finished, close lid flap to retain moisture.

TO OPEN PACKAGE: [Open lid and] pull out wipe from center of roll. [The] next sheet pops up automatically. [When finished] close lid to retain moisture.

DISPENSING DIRECTIONS: Starter Kit - [Pull down ribbed tab and lift up to] Remove lid from bucket [canister]. [Use scissors] [to] [Cut] [Rip] [Tear] [Pull] [Open] refill pouch. Insert corner of center sheet through flap opening on lid. DO NOT push finger through opening. Replace lid and pull wipe [towel] at an angle. The next sheet [wipe] [towel] pops up automatically. When finished, close lid flap to retain moisture.

REFILL [POUCH]: Place refill [pouch] in bucket [canister]. [Use scissors] [to] [Cut] [Rip] [Tear] [Pull] [Open] refill pouch. Place [put] [insert] refill pouch into bucket [canister]. DO NOT remove wipe roll from refill pouch. Insert corner of center sheet through flap opening on lid. DO NOT push finger through opening. Replace lid and pull wipe [towel] at an angle. The next sheet [wipe] [towel] pops up automatically. When finished, close lid flap to retain moisture.

For Pouch -or- Flat Pack:

Peel -and/or- reseal [here]

Peel -and/or- Lift [here]

To Open Package: [Open pouch by] peel[ing] back seal. When finished, close to retain moisture.

-or-

To Open Package:

[1.] Open [click-lock] [closure] and peel back -or- remove seal.

[2.] Pull out wipe [, use as directed].

[3.] [When finished,] [firmly] close [click-lock] [closure] to retain moisture. [A “snap” should be heard, indicating that it is properly closed.]

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

Not for cleaning or sanitizing skin. Do not use as a diaper wipe or for personal cleansing. For surfaces that may come in contact with food, a potable water rinse is required. [Avoid contact with clothes, fabric, wood, natural rubber, and painted and paper surfaces. Do not use on silver or chipped enamel. APPLY CAREFULLY – This product contains bleach.] [Refill only with this product.] [Always refer to manufacturer’s care instructions before using on equipment -or- devices.]

[Look for] Product expiration date on back of refill pouch. [Discard wipe after use.] To prevent cross contamination do not reuse wipe. If bucket becomes contaminated [or] [damaged] replace immediately. Replace bucket immediately if tabs below cap become damaged.

Special Instructions for Cleaning Prior to Disinfection against Clostridium difficile spores. Personal Protection: Wear appropriate barrier protection such as gloves, gowns, masks and eye covering. **Cleaning Procedure:** Fecal matter/waste must be thoroughly cleaned from surfaces/objects before disinfection by application with [**Product Name**] [**this product**]. Cleaning is to include vigorous wiping and/or scrubbing, until all visible soil is removed. Special attention is needed for high-touch surfaces. Surfaces in patient rooms are to be cleaned in an appropriate manner, such as from right to left or left to right, on horizontal surfaces, and top to bottom, on vertical surfaces, to minimize spreading of the spores. Restrooms are to be cleaned last. Do not reuse soiled cloths. **Infectious Materials Disposal:** Materials used in the cleaning process that may contain feces/wastes are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

SUPPLEMENTAL GRAPHICS/LOGOS/SYMBOLS

The following graphics/symbols/logos are optional and may be placed anywhere on the product label, container, package, etc.



DISINFECTION

To [clean and] disinfect [and deodorize] hard, nonporous surfaces: Wipe [hard, nonporous] surface to be disinfected. Use enough wipes for treated surface to remain visibly wet for the contact time listed below -or- on label. [Rinse.] Let air dry. Gross filth must be removed prior to disinfecting. [If streaking is observed, wipe with a clean, damp [cloth or] paper towel after appropriate contact time has expired.] [For] Killing *Clostridium difficile* [spores][¥]: Pre-clean hard, nonporous surfaces by removing gross filth [loose dirt, debris, blood/bodily fluids, etc.]. Wipe [hard, nonporous] surface to be disinfected. Use enough wipes for treated surface to remain visibly wet for 3 minutes. [Rinse.] Let air dry.

Pathogen	Bacteria	‡ Viruses	‡‡ Bloodborne Pathogens	TB (22.5°C -or- 72.5°F)	Parvoviruses	<i>Clostridium difficile</i> [¥]	Mold/Mildew/Fungi
Contact Time:	30 sec[onds]	1 min[ute]	1 min[ute]	3 min[utes]	3 min[utes]	3 min[utes]	5 min[utes]

-and/or-

ORGANISMS:

Bacteria:	<i>[Strain and/or ATCC number]</i>
<i>Acinetobacter baumannii</i> *	[ATCC 15308]
<i>Bordetella pertussis</i> *	[ATCC 12743]
<i>Campylobacter jejuni</i> *	[ATCC 29428]
Carbapenem resistant <i>Klebsiella pneumoniae</i> [(CRKP)]*	[ATCC BAA-1705]
[∅] <i>Clostridium difficile</i> -or- <i>C. difficile</i> -or- <i>C. diff</i> [(<i>C. diff</i>)] spores ^{¥, * **}	[ATCC 43598]
Community Acquired Methicillin resistant <i>Staphylococcus aureus</i> [CA-MRSA]*	[NARSA NRS123] [Genotype USA400]
Community Acquired Methicillin resistant <i>Staphylococcus aureus</i> [CA-MRSA]*	[NARSA NRS384] [Genotype USA300]
<i>Enterobacter aerogenes</i> *	[ATCC 13048]
<i>Enterobacter cloacae</i> *	[ATCC 13047]
<i>Enterococcus faecalis</i> , vancomycin resistant -or- Vancomycin resistant <i>Enterococcus faecalis</i> [(VRE)]*	[ATCC 51575]
<i>Enterococcus hirae</i> *	[ATCC 10541]
ESBL (Extended Spectrum Beta Lactamase) producing <i>Escherichia coli</i> (ESBL producing <i>E. coli</i>)*	[ATCC BAA-196]
ESBL (Extended Spectrum Beta Lactamase) producing <i>Klebsiella pneumoniae</i> (ESBL producing <i>K. pneumoniae</i>)*	[CDC 700603]
<i>Escherichia coli</i> O157:H7*	(ATCC 35150)
<i>Escherichia coli</i> [(<i>E. coli</i>)]*	[ATCC 11229]
<i>Escherichia coli</i> New Dehli Metallo-Beta Lactamase-1 [(NDM-1)] [NDM-1 <i>E. coli</i>]*	[CDC 1001728]
<i>Klebsiella pneumoniae</i> *	[ATCC 4352]
<i>Klebsiella pneumoniae</i> New Dehli Metallo-Beta Lactamase-1 [(NDM-1)] [NDM-1 <i>K. pneumoniae</i>]*	[CDC 1000527]
<i>Legionella pneumophila</i> [(<i>Legionella</i>)]*	[ATCC 33153]
Linezolid resistant <i>Staphylococcus aureus</i> [(LRSA)]*	[NRS 119]

*30 sec[ond] contact time

**1 min[ute] contact time

***3 min[ute] contact time

¥ Follow [the] Special Instructions for Cleaning Prior to Disinfection [listed on label]

Note: Bold, italicized text and (italicized text within parentheses) is information for the reader and is not part of the label. [Bracketed] information and phrase sequence are optional unless specified in 40 CFR 156.10. Variables such as CAPITALIZATION, plurals or singular, bold and italics are optional unless specified in 40 CFR. The term “wipe(s)”, is interchangeable with towel(s), towelette(s) and cloth(s) throughout except for the product brand name which can only be changed by notification or amendment. Listed uses and use sites can be used partially or in their entirety. R0618196

ORGANISMS continued:	
Bacteria:	[Strain and/or ATCC number]
Listeria monocytogenes [(Listeria)]*	[ATCC 19117]
Methicillin resistant Staphylococcus aureus [(MRSA)]*	[ATCC 33592]
[¥ ¥] Multi-drug resistant Enterococcus faecium [(MDR E. faecium)]*	[ATCC 51559]
Proteus mirabilis*	[ATCC 9240]
Pseudomonas aeruginosa [(Pseudomonas)]*	[ATCC 15442]
Salmonella enterica tested as Salmonella choleraesuis*	[ATCC 10708]
Serratia marcescens*	[ATCC 14756]
Shigella dysenteriae*	[ATCC 11835]
Staphylococcus aureus*	[ATCC 6538]
Staphylococcus epidermidis* [(Coagulase-negative Staphylococci)]	[ATCC 12228]
Streptococcus pneumoniae [(Strep)]*	[ATCC 6305]
Streptococcus pyogenes [(Strep)]*	[ATCC 12344]
Vancomycin intermediate resistant Staphylococcus aureus [(VISA)]*	[CDC HIP 5836]
Vancomycin resistant Staphylococcus aureus [(VRSA)]*	[NARSA VRS1]
Mycobacterium:	
Mycobacterium bovis [(BCG)] (Tuberculosis -or- TB)***	
Mold, Mildew, Fungi:	
Aspergillus brasiliensis tested as Aspergillus niger [(mildew)]*****	[ATCC 16404]
Candida albicans***	[ATCC 10231]
Candida glabrata***	[ATCC 32312]
Trichophyton mentagrophytes [(Athlete's Foot Fungus)]***	[ATCC 9533]

*30 sec[ond] contact time

**1 min[ute] contact time

*** 3 min[ute] contact time

****5 min[ute] contact time

¥Follow [the] Special Instructions for Cleaning Prior to Disinfection [listed on label]

[¥ ¥] **This strain of Enterococcus faecium has shown resistance to the following antibiotics:**

Amikacin

Ampicillin

Gentamicin

Penicillin

Tobramycin

Vancomycin]

ORGANISMS Continued:	
Viruses [Non-enveloped][:]	[Strain and/or ATCC number]
‡Adenovirus [type 2] [(causes colds)]**	[ATCC VR-846] [(Strain Adenoid 6)]
‡Enterovirus type D68**	[NR-49132] [(Strain US/KY/14-18953)]
‡[Human] Hepatitis A Virus [(HAV)] **	[(Strain HM-175)]
‡Norovirus -or- Norwalk virus (as Feline Calicivirus)**	[ATCC VR-782]
‡Poliovirus [type 1]**	[ATCC VR-1000] [(Strain Brunhilde)]
‡Rhinovirus [type 37] [(a leading] causative agent of the common cold)]**	[ATCC VR-1147] [(Strain 151-1)]
‡Rotavirus [(causative agent of viral diarrhea)]**	[(Strain WA)]
Viruses [Enveloped][:]	
‡Avian Influenza A Virus**, §	[ATCC VR-2072]
‡, ‡‡HIV [Human Immunodeficiency Virus] [type 1]*	[(Strain HTLV-IIIB)]
‡[Human] Coronavirus [(additional] causative agent of the common cold)]**	[ATCC VR-740] [(Strain 229E)]
‡, ‡‡[Human] Hepatitis B Virus [(HBV)] (as duck HBV)**	
‡, ‡‡[Human] Hepatitis C Virus [(HCV)] (as bovine diarrhea virus)**	[ATCC VR-1422]
‡[Human] Herpes virus [type 2] -or- Herpes simplex virus [type 2] [(causative agent of genital herpes)]**	[ATCC VR-734] [(Strain G)]
‡Influenza A Virus [(representative of the common flu virus)]**	[ATCC VR-544] [(Strain Hong Kong)]
‡Influenza B Virus**	[ATCC VR-823] [Strain B/Hong Kong/5/72]
‡Measles Virus**	[ATCC VR-24]
‡MERS - Coronavirus** [(Middle East Respiratory Syndrome Coronavirus)] [(MERS-CoV)]	
‡Respiratory syncytial virus [(RSV)] [(causes colds)]**	[ATCC VR-26] [(Strain Long)]
‡SARS - Associated Coronavirus** [(SARS - CoV)]	[ZeptoMetrix/CDC CDC#200300592]
Bloodborne Pathogens:	
‡, ‡‡HIV type 1 [Human Immunodeficiency Virus]*	[(Strain HTLV-IIIB)]
‡, ‡‡[Human] Hepatitis B Virus [(HBV)] (as duck HBV)**	
‡, ‡‡[Human] Hepatitis C Virus [(HCV)] (as bovine diarrhea virus)**	[ATCC VR-1422]
Parvoviruses Non-enveloped:	
‡Canine parvovirus***	[ATCC VR-2017] [(Strain Cornell)]
‡Feline parvovirus [(Feline panleukopenia virus)]***	[ATCC VR-648] [(Strain Philips-Roxane)]

*30 sec[ond] contact time

**1 min[ute] contact time

***3 min[ute] contact time

§Kills Avian Influenza A virus on precleaned hard, nonporous surfaces

‡ Viruses

‡‡ Bloodborne Pathogens

DISINFECTION continued

Now -and/or- New[!] -and/&/or- Improved[!] **to be used as a claim descriptor only for the first 6 months of product on shelf.**

Claims:

- [5 minute] broad spectrum no-rinse disinfectant for non-food contact surfaces
- A disinfectant cloth for use on (*insert use site(s) from table 1-5*)
- [A] one-step cleaner/disinfectant
- Aids in reducing -or- in the reduction of cross-contamination on -or- between treated surfaces
- Assures proper dilution strength, product effectiveness and standardizes technique
- Bacteria -and/or- virus[‡] -and/or- germ fighting [formula]
- Bactericidal -and/or- Virucidal[‡] -and/or- Fungicidal -and/or- Sporicidal^Δ -and/or- Tuberculocidal -and/or- Disinfectant
- Bactericide -or- Bactericidal
- Bathroom disinfectant wipes
- Broad spectrum disinfectant [with Clorox® Bleach]
- Broad spectrum [Proven] disinfectant [and bactericide -and/ or- virucide[‡] -and/or- fungicide] with efficacy against Clostridium difficile[¥] -or- C. diff[¥] -or- C. difficile[¥] spores.
- [Cleans and] disinfects (*insert use site(s) from table 1-5*)
- [Cleans and] disinfects [hard, nonporous surfaces]
- Cleans, deodorizes and disinfects
- Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] spores tested in the presence of [three (3) part] soil [load]
- Daily use product with Clostridium difficile[¥] -or- C. diff[¥] -or- C. difficile[¥] sporicidal^Δ efficacy which allows for product standardization [reduces -or- reducing cleaning complexity] [reduces the number of products needed] [eliminates the need for a separate sporicide].
- Daily use product with sporicidal^Δ efficacy
- Deodorizes by killing the germs that cause odors
- Designed for killing Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores] on surfaces in hospitals
- Designed for killing -and/or-controlling -and/or-eliminating Clostridium difficile[¥] -or- C. diff[¥] -or- C. difficile[¥] spores [on hard, nonporous surfaces [and patient care equipment]] in hospitals [and other healthcare facilities].
- Destroys [Deactivates] [[Helps] Eliminate] [tough-to-kill] Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores] [on hospital surfaces]
- Disinfecting formula
- Disinfects [and deodorizes]
- Disinfects and deodorizes by killing germs and their odors
- Disinfects hard, nonporous surfaces [throughout the (*insert use site(s) from table 1-5*)]
- Easy and convenient disinfecting [throughout the (*insert use site(s) from table 1-5*)]
- Easy, one-step cleaning and disinfecting
- Effective against [*Insert any organism from list of organisms*] [and] Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores]
- Effective against -or- Kills a wide range of bacteria including Staphylococcus aureus, Salmonella enterica, Pseudomonas aeruginosa [in the presence of organic soil [(5% blood serum)]]
- Effective against -or- Kills (*insert any organism(s) from table above*) [in the presence of organic soil [(5% blood serum)]]
- Effective disinfectant cleaner [and Clostridium difficile[¥] sporicide] [for use in hospitals, ambulatory care centers, long term care facilities, and other healthcare facilities.]
- Effective -or- For daily use against Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores]
- Effective against -or- kills **X** number of pathogens -or- organisms
- [Effectively] disinfects [hard, nonporous] surfaces
- Eliminates odors at their source [caused by germs -or- bacteria -and/or- mildew]
- Eliminates -or- Reduces odors [caused by germs -or- bacteria -and/or- mildew]
- Exhibits effective virucidal[‡] activity
- Fight[s] -and/or- kill[s] -and/or- Effective against [99.9% [of]] Escherichia coli [(E. coli)]
- Fight[s] -and/or- kill[s] -and/or- Effective against [99.9% [of]] Salmonella enterica [(Salmonella)]
- Fight[s] -and/or- kill[s] -and/or- Effective against Avian flu virus [on hard, nonporous surfaces]
- Fight[s] -and/or- kill[s] -and/or- Effective against cold -and/or- flu virus[es]^{ΔΔΔ}
- Fight[s] -and/or- kill[s] -and/or- Effective against [Human] Hepatitis A virus [(HAV)]
- Fight[s] -and/or- Kill[s] -and/or- Effective against (*insert any organism from EPA approved list*) [in (*insert appropriate contact time for respective organism*)].
- Fight[s] -and/or- kills[s] -and/or- Effective against measles virus on hard, nonporous surfaces
- Fight[s] -and/or- stop[s] -and/or- help[s] prevent[s] -and/or- reduce[s] the spread of [99.9% of] [illness-causing] bacteria -and/or- viruses[‡] -and/or- germs on hard, nonporous treated surfaces [in your (*list any use site*)] [this cold -and/or- flu season]
- Formulated to kill Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores]
- Fungicide -or- Fungicidal
- Germicidal disposable wipes
- Germicide -or- Germicidal
- Just wipe to clean and disinfect
- Kills bacteria -and/or- viruses[‡] -and/or- fungi Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores]

Use Sites: See Tables 1-5.

^{ΔΔΔ}Influenza A Virus, Influenza A virus H1N1, Influenza B Virus, Human Coronavirus, Rhinovirus

[¥] Follow [the] Special Instructions for Cleaning Prior to Disinfection [listed on label]

DISINFECTION continued

Now -and/or- New[!] -and/&/or- Improved[!] **to be used as a claim descriptor only for the first 6 months of product on shelf.**

- Kills bacteria, viruses[‡], mold, mildew
 - Kills both Influenza A and B viruses
 - Kills Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores] in 3 min[utes] [on hard, nonporous surfaces]
 - Kills common fungi most prevalent in healthcare settings
 - Kills [effective against] both enveloped and nonenveloped viruses[‡] [on hard, nonporous surfaces]
 - Kills [effective against] CRE^{◇◇} [(Carbapenem-resistant Enterobacteriaceae)]
 - Kills [effective] against Enterovirus D68
 - Kills **insert organism** [in **insert contact time**]
 - Kills major forms of microbes, including the spore form of Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores]
 - Kills most common causes of flu viruses^Δ [in US]
 - Kills multiple strains [of **insert species**]
 - Kills Norovirus, a highly contagious pathogen -or- organism commonly spread through surfaces
 - Kills odor-causing bacteria -or- germs
 - Kills -or- Is effective against [99.9999%] Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores] [in 3 min[utes]] [on hard, nonporous surfaces]
 - Kills -or- Destroys mold and mildew
 - Kills -or- Effective against bacteria
 - Kills -or- Effective against germs
 - Kills -or- Effective against viruses[‡]
 - Kills -or- Is effective against [99.9999%] Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores] [in 3 minutes], [in the presence of [three (3) part] soil [load]] [on hard, nonporous surfaces]
 - May be used as part of a comprehensive approach to Clostridium difficile[¥] -or- C. diff[¥] -or- C. difficile[¥] spore control.
 - Meets [the disinfection requirements of] OSHA[‘s] Bloodborne Pathogens Guideline -or- Standard
 - Mildewcide -or- Mildewcidal
 - Multi-purpose disinfectant
 - One-step cleaner and disinfectant
 - One-step cleaning and disinfecting [saves time]
 - One-step disinfectant cleaner designed for general cleaning and disinfecting hard, nonporous surfaces in health care facilities -or- (**insert use site(s) from table 1**)
 - Parvovirucide[‡] -or- Parvovirucidal[‡]
 - Prevents -or- controls mold and mildew growth
 - Pseudomonocidal⁴
 - Ready-to-use disinfectant [disinfection] for [killing] Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores]
 - [Ready-to-use] hospital [grade] disinfectant
 - Reduces exposure to Clostridium difficile[¥] -or- Clostridium difficile[¥] (C. diff) -or- C. difficile[¥] -or- C. diff[¥] from treated surfaces
 - Reduces exposure to Methicillin resistant Staphylococcus aureus [(MRSA)] from treated surfaces.
 - Reduces the risk of cross-contamination on treated surfaces
 - Renders Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores] inactive
 - Staphylocidal⁵
 - The answer to your disinfecting needs
 - The easy, convenient way to clean and disinfect with Clorox® Bleach
 - [The] easy -and/or- convenient way to disinfect
 - This wipe is saturated with a 1:10 sodium hypochlorite solution similar to that recommended by the CDC for effective disinfecting against Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores]
 - [This product] controls the spread of germs on most treated hard, nonporous surfaces
 - This product meets AOAC efficacy testing requirements -or- standards for hospital disinfection
 - To kill [inactivate] [[help] prevent the spread of] Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores] [on treated surfaces]
 - To decontaminate Clostridium difficile[¥] -or- C. difficile[¥] -or- C. diff[¥] [(C. diff)][¥] [spores] laden surfaces
- ⁴ Pseudomonas aeruginosa
⁵ Staphylococcus aureus
^{◇◇} Carbapenem resistant Klebsiella pneumoniae [ATCC BAA-1705]
^Δ Influenza A virus, Influenza B virus
[¥] Follow [the] Special Instructions for Cleaning Prior to Disinfection [listed on label]

DISINFECTION continued

Now -and/or- New[!] -and/&/or- Improved[!] **to be used as a claim descriptor only for the first 6 months of product on shelf.**

- To kill *Clostridium difficile*[¥] -or- *C. difficile*[¥] -or- *C. diff*[¥] [(*C. diff*)[¥] [spores], let stand for five -or- 3 min[utes]
 - To reduce *Clostridium difficile*[¥] -or- *C. difficile*[¥] -or- *C. diff*[¥] [(*C. diff*)[¥] [spores] on surfaces in **insert use site from tables 1-5**
 - Tuberculocide -or- Tuberculocidal
 - Use in public -or- common places where [bacteria -and/or- viruses[‡] -and/or- germs may be of concern] [you are concerned about bacteria -and/or- viruses[‡] -and/or- germs] [on hard, nonporous surfaces]
 - Used the -or- Tested against the clinically relevant 43598 strain of *Clostridium difficile*[¥] -or- *C. difficile*[¥] -or- *C. diff*[¥] spores
 - Used under the **insert current test method** as of **include date of test method**
 - Use where control of the hazards of cross-contamination between -or- on treated surfaces is of prime importance
 - Virucide[‡] -or- Virucidal[‡]
 - When used according to label directions, [this product] reduces exposure to *Clostridium difficile*[¥] -or- *Clostridium difficile*[¥] (*C. diff*)[¥] -or- *C. difficile*[¥] -or- *C. diff*[¥] from contact
- ¥ Follow [the] Special Instructions for Cleaning Prior to Disinfection [listed on label]

DISINFECTION continued

The following is only for products with medical devices listed on the label:

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream, or normally sterile areas of the body, or (2) contacts intact mucous membranes, but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

Special Instructions for Use Against HIV-1, HBV and HCV

This product kills HIV-1, HBV and HCV on precleaned environmental surfaces/objects previously soiled with blood/body fluids in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus (HIV-1) (associated with AIDS), Human Hepatitis B Virus (HBV) and Human Hepatitis C Virus (HCV).

Special instructions for using this product to clean and decontaminate against HIV-1, HBV and HCV on surfaces/objects soiled with blood/body fluids:

Personal Protection: When handling items soiled with blood or body fluids, use disposable impervious gloves, gowns, masks and eye coverings.

Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and other objects before applying this product.

Contact Time: Allow surfaces to remain wet for 1 min[ute], [rinse and] let air dry. For all other organisms, see directions for contact times.

Disposal of Infectious Materials: Use disposable impervious gloves, gowns, masks and eye coverings. Blood and other body fluids must be autoclaved and disposed of according to local regulations for infectious waste disposal.

Pandemic 2009 H1N1 Influenza A virus###

Claims:

- Respiratory illnesses attributable to Pandemic 2009 H1N1 are caused by influenza A virus. This product (**Product Name**) is a broad-spectrum hard surface disinfectant that has been shown to be effective against influenza A virus and is expected to inactivate all influenza A viruses including Pandemic 2009 H1N1 (formerly called swine flu).
- This product has demonstrated effectiveness against influenza A virus and is expected to inactivate all influenza A viruses including Pandemic 2009 H1N1 influenza A virus.
- This product has demonstrated effectiveness against influenza A virus and is expected to inactivate all influenza A viruses including Pandemic 2009 H1N1 (formerly called swine flu).
- Kills Pandemic 2009 H1N1 influenza A virus (formerly called swine flu).
- Kills Pandemic 2009 H1N1 influenza A virus.

per Agency guidance dated October 22, 2009 “Guidance for Testing and Labeling Claims against Pandemic 2009 H1N1 Influenza A Virus (formerly called Swine Flu)” www.epa.gov/oppad001/h1n1-guide.html

FARM PREMISE AND POULTRY HOUSE DISINFECTANT [INSTRUCTIONS]:

[Directions For Use:]

1. Remove all poultry or animals and feeds from premises, trucks, vehicles, coops, crates, and enclosures.
2. Remove all litter and manure or droppings from floors, walls and surfaces of barns, pens, stalls, chutes, and other facilities and fixtures occupied or traversed by animals or poultry.
3. Empty all troughs, racks, and other feeding and watering appliances.
4. Thoroughly clean all surfaces with soap or detergent and rinse with water.
5. Wipe all surfaces using enough wipes to keep surfaces visibly wet for 5 min[utes].
6. Wipe all halters, ropes and other types of equipment used in handling and restraining animals, as well as forks, shovels, and scrapers used for removing litter and manure.
7. Ventilate buildings, cars, boats, coops, and other closed spaces. Do not house livestock or poultry or employ equipment until treatment has been absorbed, set or dried.
8. Thoroughly scrub all treated feed racks, mangers, troughs, automatic feeders, fountains, and waterers with soap or detergent, and rinse with potable water before reuse.

CLEANING AND STAIN REMOVAL

To clean -and/or- remove stains: Wipe surface with a wipe; [rinse and] let air dry. [If streaking is observed, wipe with a clean, damp [cloth or] paper towel.]

Claims:

- Cleans
- Removes (*insert stain(s)/soil(s) from table below*)
- Removes stains

Stains/Soils:

- | | | | |
|-------------------|--------------------------------|---------------------|--------------------------|
| • Blood | • Fecal matter | • Grime | • [Other] common soils |
| • Body oils | • Fingerprints | • Laboratory stains | -and/or- stains |
| • Coffee [stains] | • Food residue[s] -or- soil[s] | • Mildew stains | • [Other] organic matter |
| • Dead skin | • Fruit [stains] | • Mold stains | • Tea [stains] |
| • Dirt | • Grease | | • Urine |

ALLERGEN DESTRUCTION

To [clean and] destroy specified allergens: Wipe surface. Use enough wipes for treated surfaces to remain visibly wet for 1 minute. [Rinse.] Let air dry. [If streaking is observed, wipe with a clean, damp [cloth or] paper towel after contact time has expired.]

Allergens:

- Cockroach matter -or- particles
- Dust mite matter -or- particles
- Pet dander -or- dog and cat dander

Claims:

- Breaks Down Many Allergens[!]
- Destroys Many Allergens[!]
- Destroys -or- deactivates dust mite matter -or- particles, cockroach matter -or- particles, and pet dander allergens [found in dust]
- [Penetrates and] breaks down dust mite matter -or- particles, cockroach matter -or- particles, and pet dander allergens [found in dust]
- Reduces dust mite matter -or- particles, cockroach matter -or- particles, and pet dander allergens [found in dust]
- Reduces Many Allergens[!]
- This product kills germs and destroys many allergens while cleaning hard, nonporous surfaces.

GENERAL CLAIMS

Now -and/or- New[!] -and/&/or- Improved[!]**to be used as a claim descriptor only for the first 6 months of product on shelf.**

- A solution impregnated in a wiping cloth
- Appropriate -or- Perfect -or- Ideal for Terminal Cleaning
- Bleach-based formula means no quat-binding concerns [with wiping]
- Compatible with a wide variety of surfaces. Check cloroxhealthcare.com/compatible
- Convenient
- Delivers consistent [amount of solution] concentration every time[!]
- Delivers consistent [sodium hypochlorite] concentration every time[!]
- [Designed] For daily use -or- Use for daily [or] [discharge] cleaning [in Healthcare facilities]
- [Designed] [for] [with] [the] Environmental Services Professional[s] [in] [mind][!]
- Disposable
- Does not contain mercury
- Does not contain nonylphenol ethoxylates (NPEs) -and/or- alkylphenol ethoxylates (APEs)
- Does not contain VOCs
- Does not require [the hassle of] dispensing systems
- [Contains] the power of Clorox® Bleach
- Easy and convenient to use
- Easy to handle
- Eliminates airborne particles associated with sprays
- Eliminates [daily] mixing and measuring
- EVS [Environmental Services] [use]
- Expiration Date [***insert date -or- length of time***]
- [For] general use
- For use on bathroom surfaces
- For use on nursery surfaces
- Fresh [lingering] bleach scent
- [Heavy] cloth-like towel
- Large[r] Surface[s]
- Larger wipe covers [more surface area] larger area [with less effort][!]
- Leaves a fresh bleach scent
- Mercury free
- Moist wipes
- No measuring [required]
- No mixing
- Nonylphenol ethoxylate (NPE) -and/or- alkylphenol ethoxylate (APE) free
- [Note:] This product is compatible with the listed materials
- Pre-measured
- Pre-moistened [towelettes]
- Prevents formulation errors
- Ready to use
- [Refer to] see [Clorox] [Terminal] [Cleaning] protocol for additional [cleaning] information
Reuse[able] container -or- bucket
Pouch[es]
[Sturdy] Bucket[s]
System
Protocol[s]
12x12 in[ch]
110 count [ct.]
Cloth[s]
Towel[s]
Large[r] [wipe][s]
Big[ger]
- Recyclable [bottle] [product container]
- Shelf-stable formula
- Specifically -or- Specially designed for the Medical/Dental/Veterinary/Health Care industry
- [Strong,] [Durable,] cloth-like wipes
- [Strong,] [Durable,] non-woven disposable cloth
- [Strong,] [Durable,] non-woven wipe does not leave lint on surfaces [following use]
- Suitable for hospital use
- Suitable for use on ***insert surface material(s) from Table 5***
- Suitable for use on [titanium-coated] [medical grade] stainless steel
- Suitable for use on hard, nonporous surfaces
- Terminal Cleaning Wipe[s] [Towel][s] Refill[s][able]
- [The] smell of clean
- VOC free
- VOC's below laboratory detection limits

TABLE 1 Medical:

USE SITES

Ambulances -or- [Emergency Medical] Transport Vehicles	Headboards Health Care Settings -or- Facilities	Outpatient Clinics Outpatient Surgical Centers [(OPSC)]
Anesthesia Rooms -or- Areas [Assisted Living -or- Full Care] Nursing Homes	Home Health Care [Settings] Hospices [Hospital] Kitchens	Patient Care Areas Patient Restrooms Patient Rooms
[Bed] Head [Foot] Board (blood) (plasma) (semen) (milk) (apheresis) Donation centers	Hospitals Intensive Care Units -or- ICUs Isolation Areas	[Pediatric] Examination Rooms -or- Areas Pediatric Intensive Care Units [(PICU)] Pharmacies
BP Monitors Carts	IV Stands Laboratories	Physicians Offices Physical Therapy Rooms -or- Areas
CAT Lab[oratories]	[Mayo] [Instrument] Stands	Radiology -or- X-Ray Rooms -or- Areas
Central Service Areas	[Medical] Clinics	Recovery Rooms
Central Supply Rooms -or- Areas	Medical Facilities	Rehabilitation
Chemotherapy Hoods	Medical -or- Physician's -or- Doctor's Offices	Respiratory Therapy Rooms -or- Areas
Critical Care Units -or- CCUs	Neonatal Intensive Care Units [(NICU)]	Sill[s] Ledge[s]
Dialysis Clinics	Newborn -or- Neonatal Nurseries	Surgery Rooms -or- Operating Rooms -or- ORs
Dialysis Machines	Nurse Call Device [and Cord]	Transport Vehicles
Emergency Rooms -or- ERs	Nurse-Call [Device] [Button] [and Cord]	[TV] Remote[s] [Control][s]
[Eye] Surgical Centers	Nursing -or- Nurses' Stations	X-Ray machine surfaces
Footboards	Ophthalmic Offices	Ultrasound Transducers [and Probes]
Glucometers -or- Blood Glucose Monitors	Orthopedics	

SURFACES

bed pans	external surfaces of [medical] equipment -or- [medical] equipment surfaces	phone cradle
behind and under sinks ^P	external [surfaces of] ultrasound transducers	pipes
cabinet handles	gurneys	plastic mattress covers
cabinets	hard, nonporous hospital -or- medical surfaces	reception counters -or- desks -or- areas
closet handles	high touch surfaces	shower fixtures
coated mattresses	[hospital -or- patient] bed railings -or- linings -or- frames	showers
coated pillows	[inner] [inside of] drawers	sinks
computer keyboards	IV poles	stretchers
counters	mattress covers	support bars
desk tops	patient chairs	tables
doorknobs		telephones
[exam -or- examination] tables		wash basins
exterior surfaces of air vents -or- air vent exteriors		wheelchairs

^Phard, nonporous surfaces of

TABLE 2 Dental:

USE SITES

Dental Operatories
Dental -or- Dentist's Offices

SURFACES

dental countertops
dental operatory surfaces
dentist -or- dental chairs
hard, nonporous dental surfaces
light lens covers
reception counters -or- desks -or- areas

TABLE 3 Veterinary:

USE SITES

Animal Housing Facilities
Animal Life Science Laboratories
[Animal -or- Pet] Grooming Facilities
Equine Farms
Farms
Kennels
Livestock -and/or- Swine -and/or- Poultry Facilities
Pet Areas
Pet Shops -or- Stores
Small Animal Facilities
Veterinary Clinics -or- Facilities
Veterinary Offices
Veterinary -or- Animal Hospitals

SURFACES

animal equipment
automatic feeders
cages
external surfaces of [veterinary] equipment
feed racks
fountains
hard, nonporous veterinary surfaces
pens
reception counters -or- desks -or- areas
stalls
troughs
veterinary care surfaces
watering appliances

TABLE 4 Food Service:

USE SITES

Bars
Cafeterias
Commercial -or- Institutional Kitchens
Delis
Fast Food Chains -or- Restaurants
Food Preparation and Processing Areas
Food [Service -or- Processing] Establishments
Food Serving Areas
Other Food Service Establishments
Restaurants
School Kitchens

SURFACES

any washable (food and non-food contact) surface where disinfection is required
appliances
dish racks
drain boards
food cases
food trays
freezers
hoods
microwaves
outdoor furniture (excluding wood frames and upholstery)
ovens
refrigerators
salad bar sneeze guards
stoves -or- stovetops

A potable water rinse is required for food contact surfaces.
Do not use on glassware, utensils, or dishes.

TABLE 5 Miscellaneous/General:

USE SITES

Acute Care Facilities
Airplanes
Blood Banks
Bowling Alleys
Butcher Shops
Churches
Colleges
Correctional Facilities
Day Care Centers
Dormitories
Factories
Funeral Homes
Grocery Stores
Gymnasiums -or- Gyms
Health Club Facilities
Hotels
Industrial Facilities
Laundromats
Laundry Rooms
Locker Rooms
Long Term Care Facilities
Manufacturing Plants -or- Facilities
Military Installations
Motels
Nursing Homes
Produce Areas
Public Areas
Recreational Centers -or- Facilities
Restrooms -or- Restroom Areas
School Buses
Schools
Shelters
Shower Rooms
Storage Rooms -or- Areas
Supermarkets
Trains
Universities

SURFACES

[bathroom] fixtures
[bath]tubs
behind and under counters^P
behind and under sinks^P
booster chairs
cabinets
ceilings
cell[ular] -or- wireless -or- mobile -or- digital phones
chairs
[computer] keyboards
[computer] monitors
counters -or- countertops
cribs
desks
[diaper -or- infant] changing tables
diaper pails
dictating equipment [surfaces]
doorknobs
exterior -or- external toilet surfaces
exterior -or- external urinal surfaces
faucets
floors
garbage -or- trash cans
[grocery store -or- supermarket] carts
hampers
hand railings
headsets
highchairs
lamps
linoleum
other telecommunications equipment [surfaces]
playpens
shelves
showers -or- shower stalls
sinks
stall doors
tables
telephones
[tiled] walls
toilet rims
toilet seats
towel dispensers
toys
vanity tops -or- vanities

SURFACE MATERIALS

[baked] enamel
chrome
[common] hard, nonporous surfaces
FormicaTM[[®]]¹
glass
glazed ceramic [tile]
glazed porcelain
laminated surfaces
MarliteTM[[®]]²
plastic [laminate]^{PP}
porcelain enamel
stainless steel
synthetic marble
vinyl [tile]
similar hard, nonporous surfaces
except for those excluded by the label

Not Recommended For Use On
-or- Avoid contact with:

aluminum	natural marble
brass	painted surfaces
chipped enamel	paper surfaces
clothes	rubber
copper	sealed granite
Corian TM [[®]] ³	sealed marble
fabric	silver
finished wood	unfinished wood
galvanized steel	wood

^Phard, nonporous surfaces of
^{PP}For unfamiliar surfaces, test in a small, inconspicuous area first. Consult medical equipment warranty information and equipment care and cleaning guides prior to selecting and using any surface disinfectant on plastic surfaces.

¹Formica is a [registered] trademark of [The Diller Corporation] [a subsidiary of] [Formica] [Formica Corporation] [or one or more of its affiliates].

²Marlite is a [registered] trademark of [Marlite, Inc.] [a subsidiary of] [Marlite, Inc.] [or one or more of its affiliates].

³Corian is a [registered] trademark of [E. I. du Pont de Nemours and Company] [a subsidiary of] [E. I. du Pont de Nemours and Company] [or one or more of its affiliates].