AN 1006

ONE KIT

CONTENTS: 25 Cartridges, 25 Liner Bags, 25 Dosimeters*, 25 Humidichips*. Each AN1006 cartridge contains 0.39 av. oz. (11.0 g) EOGas sterilant.

ANDERSEN STERILIZERS, INC.

Manufactured and Distributed by:

ANDERSEN STERILIZERS, INC.

3154 Caroline Drive • Haw River, NC 27258 USA
336-376-8622

Distributed by:

ANDERSEN PRODUCTS, INC.

3202 Caroline Drive • Haw River, NC 27258 USA
1-800-523-1276 • 336-376-3000
www.anpro.com • mailbox@anpro.com

European Distributor:

H. W. ANDERSEN PRODUCTS, LTD.

Clacton-On-Sea • Essex CO15 4XA UK
44-1255-428328

EPA Registration No. 69340-6

EPA Establishment No. 69340-NC-001

KEEP OUT OF REACH OF CHILDREN

DANGER

IMPORTANT: FOR PROFESSIONAL USE ONLY

Users must follow the requirements of the OSHA occupational exposure standard for Ethylene oxide (29 CFR 1910.1047).

Active ingredient: Ethylene oxide 96.0%
Inert ingredient: 4.0%
Total: 100.0%

PRECAUTIONARY STATEMENTS Hazard to Humans and Domestic Animals.

DANGER

Causes Irreversible eye damage and skin burns. Harmful if inhaled. Do not breath vapor. Do not get on eyes, skin, or clothing. Do not swallow. Cancer Hazard and Reproductive Hazard. May cause nervous system damage. Store and use with adequate ventilation in accordance with 29 CFR 1910.1047

See user's manual for additional precautionary statements.

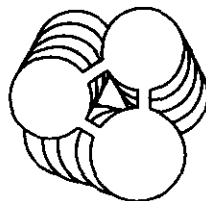
Emergency Contact: 1-800-255-3924

FIRST AID STATEMENTS

In all cases of exposure, get medical attention immediately. Take person to a doctor or emergency treatment facility. Have the product container or label with you when going for treatment.

IF IN EYES: Hold eyes open and rinse slowly and gently with water for 15 to 20 minutes. Remove contact lenses, if present, after the first 5 minutes then continue rinsing eye. Get medical treatment.

IF INHALED: Get exposed person to fresh air. Keep warm. *If not breathing call 911 or an ambulance, then give artificial respiration, preferably by mouth-to-mouth, if possible.* If breathing is difficult, give oxygen. Call a physician even if no symptoms are present. Keep under medical observation. Symptoms may be delayed.

EOGas™

Dosimeter*, EOGas*, and Humidichip* are trademarks of Andersen Products, Inc.
EOGas U.S. Pat. no. 4,937,046 and 5,160,700. Humidichip U.S. Pat. no. 5,062,636

IF ON SKIN: Immediately wash skin with plenty of water while removing contaminated clothing and shoes. Call a physician. Aerate, wash, or clean contaminated clothing and discard leather goods.

IF SWALLOWED: Drink at least two glasses of water. Do not induce vomiting. Do not give anything by mouth to an unconscious victim. Call a physician.

NOTE TO PHYSICIAN

Ethylene oxide is a gas. Skin exposure from contact with fabric, rubber or plastic containing residual ethylene oxide will commonly result in skin irritation with extensive blister formation. At high concentrations severe conjunctivitis can occur. Irritation of the respiratory tract may occur, but without acute lung edema. Symptoms of systemic intoxication are headache, nausea, vomiting, incoordination, and cardiac irregularities. Treatment is symptomatic.

ENVIRONMENTAL HAZARDS

Do not discharge effluent containing this product into lakes, streams, ponds, estuaries, oceans, or other waters unless in accordance with the requirements of a National Discharge Eliminations System (NDPES) permit and the permitting authority has been notified in writing prior to discharge. Do not discharge effluent containing this product to sewer systems without previously notifying the local sewage treatment plant authority. For guidance contact your State Water Board or Regional Office of the EPA.

PHYSICAL OR CHEMICAL HAZARDS

DANGER. FLAMMABLE LIQUID AND GAS UNDER PRESSURE

Ethylene oxide gas is extremely flammable. Do not use near flame, electrical sparks, or hot surfaces. Ground all equipment to prevent static sparks.

DIRECTIONS FOR USE

To be used only in EOGas sterilizers, and only for hospital, medical and veterinary sterilization.

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. It is the responsibility of the employer of any person engaged in the handling or application of this product to follow the requirements of 29 CFR 1910.1047. Read package insert for complete sterilization instructions and additional precautions.

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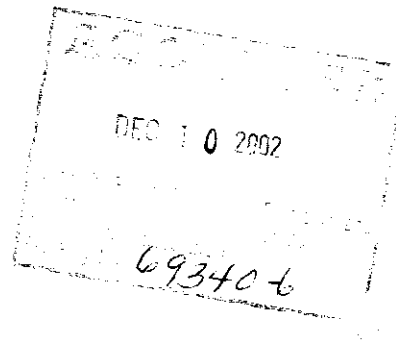
STORAGE: Store in a cool place and out of direct sunlight.

PESTICIDE DISPOSAL: Unwanted or expired EOGas cartridges should be returned to the manufacturer for disposal. Contact Andersen Products, Inc. (tel. no. 336-376-3000) for instructions. If unwanted or expired EOGas cartridges cannot be disposed of according to label instructions, contact your State Pesticide or Environmental Control Agency, or the Hazardous Waste Representative at the nearest EPA Regional Office for guidance.

CONTAINER DISPOSAL: Do not reuse empty box, used cartridge, or used liner bags. Wrap box, cartridge, and liner bags in paper and discard in the trash. P/N 2852 092002

69340-6

12-10-2002

11

REMOVE TAPE AND TRIGGER GUARD, PRESS BUTTON TO ACTIVATE CARTRIDGE

6 **EOGas**

Users must follow requirements of the OSHA Occupational Exposure Standard for Ethylene oxide (29 CFR 1910.1047). Use only in an Andersen EO Gas Sterilizer with a 16 Diffusion Bag and according to manufacturer's instructions. See outer box and package insert for precautions and sterilization instructions.

Active Ingredient: Ethylene Oxide 96%
Other Ingredient 4%
Net Contents: 0.39 av. oz. (11.0 g)

DANGER
KEEP OUT OF REACH OF CHILDREN
CONTENTS FLAMMABLE
DO NOT OPEN THIS PACKAGE
DISPOSE OF SPENT EO Gas CARTRIDGE IN TRASH

CE 0413

LOT NO.

USE BEFORE

Manufactured by: Andersen Sterilizers, Inc. • Health Science Park • Haw River, NC 27258 U.S.A.
Distributed by: Andersen Products, Inc. • Health Science Park • Haw River, NC 27258 U.S.A.
Distribué en Europe par: H. W. Andersen Products Ltd. • Clacton-On-Sea • Essex CO15 4XA U.K.
EPA Registration No. 69340-6 • EPA Est. No. 69340-NC-001 • U.S. Patent No. 4,937,046 & 5,160,700
PN 2966 092002

REMOVE TAPE AND TRIGGER GUARD, PRESS BUTTON TO ACTIVATE CARTRIDGE

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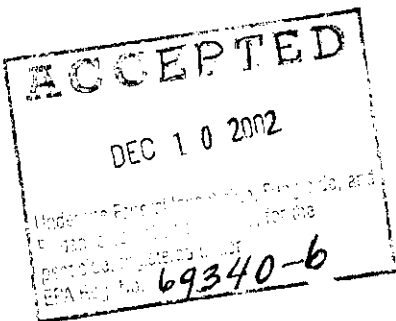
790

*Gaseous Sterilant for
50°C Sterilization*

Directions for Use: EOGas Cartridges

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1. Disassemble and scrub all instruments in detergent and water to the most critical standard of cleanliness possible.
2. Be sure that items to be sterilized are physically dry before wrapping and processing. Towel drying or draining dry is sufficient. Water on instruments at the time of exposure to EOGas may react with the gas and reduce its effectiveness.
3. Although EOGas is a highly diffusible gaseous sterilant, nevertheless occlusive caps, plugs or stylets must be removed from instruments so that the gas can penetrate freely. Hollow bore needles and plastic or rubber tubing must be open and free from plugs. Syringes must be packaged disassembled with their plungers out of the barrels.

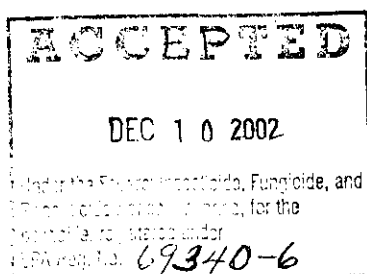
If the nature of the material to be sterilized is such that the water treatments specified above are harmful, then pretreatment in a sterilizer bag having a saturated humidity will be necessary for an overall pretreatment routine of 8 hours. Use the following procedure:

To humidify the contents of the sterilizer bag, wrap the items individually in paper or cloth in the usual manner. Place them in a sterilizer bag along with a Humidichip®, a Dosimeter® and card and a cartridge of EOGas from which the trigger guard has been removed. Heat seal the sterilizer bag and place it in the EOGas sterilizer WITHOUT pressing the cartridge trigger. After 4 hours, remove the bag from the EOGas sterilizer and leave it at room temperature for another 4 hours. Then, without opening the sterilizer bag, grasp the cartridge of EOGas through the wall of the sterilizer bag and press the trigger button firmly. Put the sterilizer bag with the activated cartridge back in the sterilizer cabinet for a normal length 16 hour sterilization cycle.

You must wrap all items individually in cloth or paper in the manner conventional for steam sterilization or in ANPRO Seal and Peel® Packaging.

ANPRO Seal and Peel Packaging offers a see-through, peel open, extended shelf life package, proven and controlled to be compatible with the EOGas Sterilizing System. No other plastic film packaging material may be used with EOGas unless the user first carefully tests the material to be sure that EOGas can penetrate it in adequate concentration. Polyamide (Nylon) and polyester (Mylar) films are known to be inappropriate to use with EOGas. In as much as Seal and Peel is waterproof, you must include an AN1071 Humidichip RH Stabilizer in each Seal and Peel package.

Do not pack the sterilizer bag so tightly with cloth or gauze that gas diffusion may be slowed.

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Technical description of the system

The EOGas Sterilizer cabinet is thermostatically controlled to assure an internal temperature of 50°C. It is provided with a dual-volume exhaust system: low-volume during the sterilization cycle for economy and high-volume for purging just before opening the door. The purge cycle will lower the cabinet temperature for a few minutes, but it will return to 50°C shortly after the door locks.

The sterilizer bag is a gas diffusion membrane retaining EOGas long enough to sterilize its contents and then allowing the gas to diffuse into the cabinet atmosphere from which it is vented harmlessly to the outside atmosphere.

The AN1071 Humidichip will help maintain adequate relative humidity in the sterilization bag during the cycle.

The EOGas cartridge will deliver a dose of at least 500 milligrams per liter hours to the contents of the sterilizer bag. Tests in our laboratory confirm that this dose will kill the most resistant spores known at 50°C within the 16 hour cycle providing that the spores have been rehydrated according to our instructions.

The Dosimeter will indicate the actual dose of ethylene oxide delivered by the EOGas cartridge. A blue color change reaching the triangle on the Dosimeter scale indicates that a dose of ethylene oxide, usually adequate for sterilization, has been delivered.

Storage and Disposal

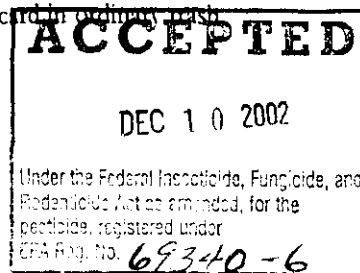
Do not contaminate water, food, or feed by storage or disposal.

Storage: Store in a cool place out of direct sunlight.

Shelf Life: Each cartridge is marked with an expiration date.

Pesticide Disposal: Unwanted or expired EOGas cartridges should be returned to the manufacturer for disposal. Contact Andersen Products, Inc. (tel. no. 336-376-8622) for instructions. If unwanted or expired EOGas cartridges cannot be disposed of according to label instructions, contact your State Pesticide or Environmental Control Agency, or the Hazardous Waste Representative at the nearest EPA Regional Office for guidance.

Container Disposal: Do not reuse empty box, used cartridge, or used liner bags. Wrap box, cartridge, and liner bags in paper and discard in ordinary trash.

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near the sterilizer while loading and unloading it. Sterilize batteries wrapped individually and separately from their electrical instruments.

Avoid breathing EOGas vapor. Breathing EOGas vapor is harmful. If you can smell EOGas you are breathing toxic amounts. In concentrated amounts EOGas sterilizing gas is as irritating to the lungs and mucous membranes as is ammonia gas.

As with other chemical vapors, there is a chance of an occasional allergic response to EOGas in a sensitive individual. Such individuals should not handle EOGas, and should neither breathe its vapors nor allow materials sterilized in it to come in contact with their skin or mucous membranes.

All users must avoid contact with skin, eyes and clothing. If contact with liquid EOGas occurs, users must immediately remove all contaminated clothing, including shoes. Flush skin or eyes with plenty of water for at least fifteen minutes. If liquid EOGas has gotten into your eyes, immediately see a physician for further treatment.

Effects of Overexposure to Ethylene Oxide: May be fatal if inhaled in high concentrations. May cause irritation of respiratory tract, chest tightness, headache, nausea, vomiting, diarrhea, light-headed feeling, dizziness, weakness, drowsiness, cyanosis, loss of coordination, convulsions, coma, delayed lung injury (fluid in lungs), immediate or delayed skin irritation and blisters, allergic skin.

Other Possible Delayed Health Effects: May cause nervous system injury, cataracts, adverse reproductive effects, chromosomal and mutagenicity changes, and cancer.

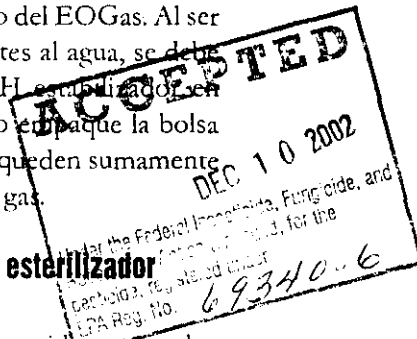
PEL: 1PPM-TWA Ethylene Oxide (OSHA - 29 CFR 1910.1047)

STEL: 5PPM-excursion limit, 15 minutes

ODOR: Ether-like in high concentrations. Exposure to toxic levels may occur without warning or detection by the user

See label or instructions for additional precautionary statements and statements of practical treatment.

Aeration: Failure to adequately air ethylene oxide absorbing materials may lead to contact chemical burns. The 16 hour sterilization cycle time at 50°C called for in these instructions includes an adequate amount of time to air most tubes, masks and the like. Large gas absorbent items like silicone breast implants and plastic extracorporeal blood filters may require additional aeration before they can be safely implanted or used. Aeration will progress more rapidly outside of the sterilizer bag. Andersen Products is prepared to determine aeration times for unusual products for our customers. Call customer service at 336-376-3000 for more information.

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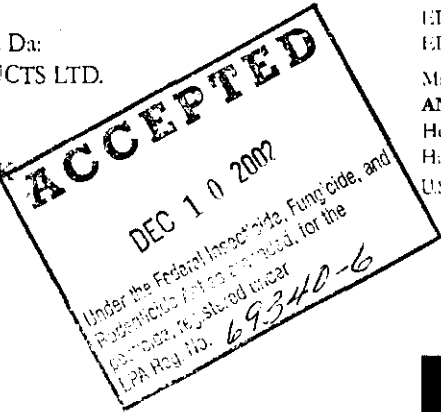
Packaging Products For Use With EOGas Sterilizers

- AN820** 2"x200' (5cm x 60m) Seal & Peel Packaging Roll Stock
- AN830** 3"x200' (8cm x 60m) Seal & Peel Packaging Roll Stock
- AN850** 5"x200' (12cm x 60m) Seal & Peel Packaging Roll Stock
- AN870** 7"x200' (18cm x 60m) Seal & Peel Packaging Roll Stock

Dosimeter®, Humidichip®, Seal and Peel®, and Steritest® are registered trademarks of Andersen Products, Inc. EOGas™ is a trademark of Andersen Products, Inc.

EOGas is protected by U.S. Patent 4,937,046 and 5,160,700 Humidichip is protected by U.S. Patent 5,082,636

Patents are pending on EOGas in: Canada, France, Great Britain, Germany, Italy, Japan, Spain

EPA REGISTRATION NO. 69340-5/6
EPA ESTABLISHMENT NO. 9417-NC-001

Manufactured and Distributed by:
ANDERSEN STERILIZERS, INC.
Health Science Park • 3154 & 3181 Caroline Drive
Haw River, North Carolina 27258-9789 USA

U.S. Distribution:

ANDERSEN PRODUCTS, INC.
Health Science Park • 3202 Caroline Drive
Haw River, North Carolina 27258-9789 USA

An ISO 9002 • EN 46002 - certified company 1-800-523-1276 • www.anpro.com

European Distributor:
H. W. ANDERSEN PRODUCTS LTD.
Davy Road • Clacton-On-Sea
Essex CO15 4XA UK

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