

Premoistened [14" x 14"] [8" x 10"] [3" x 3"] Towelette

CAT#SC110

# SUPER-CHLOR™

For wiping environmental surfaces and patient care equipment

Active Ingredient: Sodium hypochlorite.....0.525%  
Other Ingredients\*: .....99.475%  
\*Other ingredients do not include the weight of towelette.

## SUGGESTED AREAS OF USAGE:

Medical, Dental and Laboratory Counters, Exam Tables, Carts, Point-of-Care Equipment, Telephones, Sink Tops, Toilet Seats

## KEEP OUT OF REACH OF CHILDREN CAUTION

First Aid: If in eyes: Hold eye open and rinse slowly and gently with water for 15 – 20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice.

### STORAGE AND DISPOSAL

- Store product in a cool, dry area, away from heat and direct sunlight.
- Do not reuse empty packet and towelette.
- Discard used towelette in designated infectious waste container for autoclaving and disposal according to federal, state and local regulations.


(See panel on reverse for directions for use and additional precautionary statements.)

EPA Reg. No. 69687-1  
EPA Est. No. 69587-IL-001



©2003 Medtrol, Inc.

PH: 800-647-7180  
Net Contents [0.8 fl. oz.]  
[0.6 fl. oz.]  
[0.2 fl. oz.]

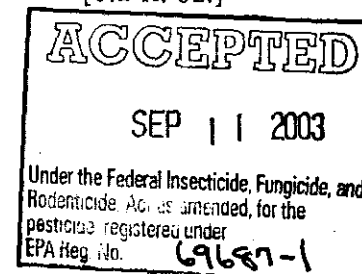

(Administrative Note: Bracketed text [ ] is optional versions of text for different packaging)

69687-1

9/11/2003

Page 182

#### Directions for use:

- It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.
- Always use personal protective equipment.
- Open [Super-Chlor™] [Product name] packet.
- Remove premoistened [14" x 14"] [8" x 10"] [3" x 3"] towelette.
- Apply towelette and wipe desired surface to be decontaminated.
- Allow surface to air dry and discard towelette and packet (see front panel).


#### Precautionary Statements

Hazards to humans and domestic animals


**CAUTION:** Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barriers or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

Complete product line information available by calling:

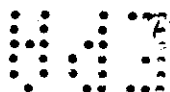

7157 N. Austin Avenue Niles, IL 60714 Ph: 800-647-7180 Fax: 800-255-3027  
E-mail: [sales@medtrol.com](mailto:sales@medtrol.com) Website: [www.medtrol.com](http://www.medtrol.com)

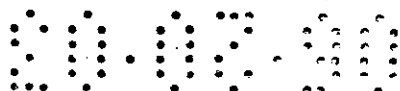

(Administrative Note. Bracketed text [ ] is optional versions of text for different packaging)

222