

69687-1

1/13/2014

1/18



UNITED STATES ENVIRONMENTAL PROTECTION AGENCY  
WASHINGTON, DC 20460

OFFICE OF CHEMICAL SAFETY  
AND POLLUTION PREVENTION

January 13, 2014

Abigail T. Downs  
Technology Sciences Group, Inc.  
c/o Medtrol, Inc.  
1150 18<sup>th</sup> Street, NW, Suite 1000  
Washington, DC 20036

Subject: **SUPER CHLOR**  
EPA Registration Number: 69687-1  
Application Date: October 18, 2013  
EPA Receipt Date: October 18, 2013

Dear Ms. Downs:

The following amendment, submitted in connection with registration under section of the Federal Insecticide, Fungicide, and Rodenticide Act (FIFRA), as amended, is accepted.

Label Amendment

- Add additional optional label claims and logos to the product label.


General comments:

Based on the review of the information submitted, the labeling amendment is acceptable. A stamped copy of the label is enclosed for your records. This labeling supersedes all previously accepted ones. The next label printing of this product must use this labeling unless subsequent changes have been approved. You must submit one (1) copy of the final printed labeling before you release this product for shipment with the new labeling. In accordance with *40 CFR § 152.130(c)*, you must distribute or sell this product under the previously approved labeling for 18 months from the date of this letter. After 18 months, you may only distribute or sell this product if it bears this new revised labeling or subsequently approved labeling. "To distribute or sell" is defined under FIFRA section 2(gg) and it's implementing regulation at *40 CFR § 152.3*.

2/18

Should you have any questions or comments concerning this letter, please contact Adam Heyward by email at [heyward.adam@epa.gov](mailto:heyward.adam@epa.gov) or by telephone at (703) 347-0274 during the hours of 6:00 am to 2:30 pm EST.

Sincerely,

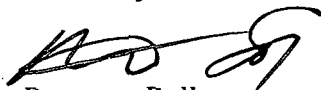

Demson Fuller  
Acting Product Manager (32)  
Regulatory Management Branch II  
Antimicrobials Division (7510P)

Enclosure: *Stamped labeling*

3/18

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

Premoistened [3" x 3"] [3" x 4"] [5" x 9"] [6" x 6"] [6" x 7"] [6" x 8"] [6" x 9"] [7" x 8"] [7" x 9"] [8" x 7"] [8" x 9"] [8" x 10"] [9" x 6"] [9" x 9"] [12" x 12"] [14" x 14"] Towelette [CAT#SC110]


# SUPER-CHLOR® [Product Name]

[For wiping] [Disinfecting wipe for] hard [,] non-porous [surfaces] [environmental surfaces] [healthcare surfaces][.]

[Effective against] [Kills] [Sporicidal] [for] [*Clostridium difficile*] [*C. difficile*] [*C. diff*] [*C DIFF*] spores] [ATCC 43598][,] [+Virucidal] [for] [+Hepatitis B Virus][,] [+HBV][,] [+Human] [+Hepatitis C Virus] [+Human Hepatitis C Virus][,] [+HCV][,] [+Human Immunodeficiency Virus Type 1][,] [+HIV-1][,] [Fungicidal] [for] [*Trichophyton mentagrophytes*] [ATCC 9533][,] [Bactericidal] [for] [*Staphylococcus aureus*-MRSA(MRSA)], [*Methicillin Resistant Staphylococcus aureus* [(MRSA)], [*Staphylococcus aureus* (MRSA)], [MRSA] [ATCC 33592] [,] [*Enterococcus faecalis*-VRE (VRE)], [*Vancomycin Resistant Enterococcus faecalis* (VRE)], [*Enterococcus faecalis* (VRE)], [VRE] [ATCC 51575][,] [*Staphylococcus aureus*] [ATCC 6538][,] [*Salmonella enterica*] [ATCC 10708][,] [and] [*Pseudomonas aeruginosa*] [ATCC 15442] [.]

Active Ingredient: Sodium hypochlorite.....0.65%  
Other Ingredients\*: .....99.35%  
Total.....100.00%

\*Other ingredients do not include the weight of towelette.

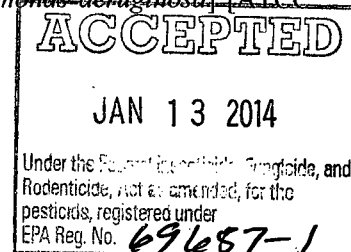

## SUGGESTED AREAS OF USAGE:

[On] [Medical] [,] [and] [Dental] [and] [Laboratory] [Counters] [,] [Exam Tables] [,] [Carts] [,] [Patient Care Equipment] [,] [and] [Point-of-Care Equipment] [,] [Telephones] [,] [Sink Tops] [,] [Toilet Seats][,] [Digital screens] [,] [on] [Plastics][,] [Glazed Porcelain] [,] [Glazed [ceramic] tiles] [,] [Stainless Steel] [,] [Formica] [,] [Vinyl] [,] [Rubber Surfaces] [,] [Hard] [Plastic Surfaces] [,] [Laminated [Plastic] Countertops] [,] [Glass] [,] [Plexiglas] [,] [Suitable For Use on Equipment Requiring Bleach Based Products] [,] [Bed Rails] [,] [IV Poles] [,] [Dialysis Machines] [,] [&] [and] [[Blood] Glucose Meters†] [Glucometers†] [.]

## KEEP OUT OF REACH OF CHILDREN CAUTION

**FIRST AID:** If in eyes: Hold eye open and rinse slowly and gently with water for 15 – 20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice.

Have the product container with you when calling a poison control center or doctor, or going for treatment.

(See [side] [back] panel[s] [on reverse] [of] [outer box] for directions for use and additional precautionary statements.)

OR

(See side panel of outer box for [suggested areas of usage][,] [first aid statement][,] [special instructions for cleaning and decontamination against +HIV-1, +HBV and +HCV][,] [precautionary statements][,] [storage and disposal])

7/18

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

EPA Reg. No. 69687-1  
EPA Est. No. 69687-IL-001  
[MEDTROL logo]  
[©XXXX Medtrol, Inc.]

Net Contents [0.1][0.15][0.2][0.25][0.3]  
[0.4][0.5][0.6][0.7][0.8][0.9][1.0] [xx.xx]oz.  
[DIN: {insert # as applicable}]

**DIRECTIONS FOR USE**

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

**[Hospital Disinfection] [Healthcare Facility Disinfection] [General Disinfection] [Disinfection]**

1. Always use personal protective equipment.+
2. Open [Super-Chlor®][Product name] [packet] [canister].
3. Remove premoistened [3" x 3"] [3" x 4"] [5" x 9"] [6" x 6"] [6" x 7"] [6" x 8"] [6" x 9"] [7" x 9"] [8" x 7"] [8" x 9"] [8" X 10"] [9" x 6"] [9" x 9"] [12" x 12"] [14" x 14"] [9" x 12"] [12" x 9"] towelette.
4. Apply towelette and wipe desired surface to be [decontaminated][disinfected].
5. {Note: one of the two statements will be on the label.} A 30 second contact time is required to kill HBV and HCV. A 3 minutes contact time is required [for efficacy against][to kill] [[Clostridium difficile] [C. difficile] [C. diff] [C DIFF] spores]. A 5 minute contact time is required [for effectiveness against][to kill] HIV and other organisms listed on the label. Reapply as necessary to ensure that the surface remains wet for the entire contact time.

OR

Allow treated surfaces to remain thoroughly wet for 5 minutes. Reapply as necessary to ensure that the surface remains wet for the entire contact time.

6. Allow surface to air dry and discard [used] towelette and [empty] [packet][canister] (see storage and disposal).


{Optional text for use with canister offerings}

**[To open canister:**

1. Always use personal protective equipment.+
2. Remove lid from canister
3. [Remove [foil] seal] [from mouth of canister] [if present][and discard]
4. Lift flap [in] [on] canister lid
5. [Pull up corner of center towelette/wipe [and][,] snap off [,] [and discard.] Take the next towelette/wipe, [twist to a point][and thread] [before][prior to][then][thread][threading] through dispenser hole in lid.]
6. DO NOT PUSH FINGER THROUGH OPENING
7. Replace lid. Pull out exposed towelette/wipe and snap off. The next towelette/wipe pops up automatically.
8. Snap center flap down when finished to retain moisture.
9. Do not flush towelette/wipe into toilet.]

{Optional text for use with canister}

**DIRECTIONS FOR DISPENSING:**

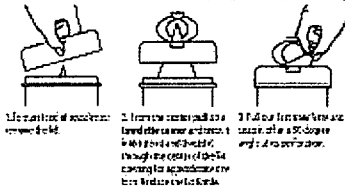

5/18

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

{Optional text for refills sold separately}

1. Always use personal protective equipment.+
2. To refill the wipes [towelette] cut open sealed pouch and place [the roll of wipes] into the canister.
3. [Pull up corner of center towelette/wipe [and][,] snap off [,] [and discard.] Take the next towelette/wipe, [twist to a point][and thread] [before][prior to][then][thread][threading] through dispenser hole in lid.]
4. DO NOT PUSH FINGER THROUGH OPENING
5. Replace lid. Pull out exposed towelette/wipe and snap off. The next towelette/wipe pops up automatically.
6. Snap center flap down when finished to retain moisture.
7. Do not flush towelette/wipe into toilet.]

{Optional text for single use packet}

[TO OPEN PACKAGE: Always use personal protective equipment.+ Tear open [along dotted line] [At tear notch] [along the top edge of pouch]. [Pull out] [Remove] towelette. [Unfold towelette before use.]] Dispose used wipes/towelette in trash. Do not flush wipes/towelette into toilet. Discard used wipes [towelette] and empty packet in designated infectious waste container for disposal according to Federal, State and Local regulations.

{Administrative Note: If a virucidal and/or HIV claim is made the following paragraph and special instructions are required}

**+KILLS HIV-1, HBV, AND HCV ON PRECLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood/body fluids and in which the surfaces/objects likely to be soiled with blood/body fluids can be associated with the potential for transmission of HIV-1 (associated with AIDS), HBV, and HCV.

**SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HBV, AND HCV ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.**

- PERSONAL PROTECTION: When handling items soiled with blood or body fluids, use disposable [protective] [latex] gloves, gown, masks [,] and eye protection.
- CLEANING PROCEDURES: Blood/body fluids must be thoroughly cleaned from surfaces/objects before application of [Super-Chlor][Product Name].
- CONTACT TIME: Allow surface(s) to remain wet for 30 seconds to kill HBV and HCV[.] [,] [for 3 minutes to kill [[*Clostridium difficile*] [*C. difficile*] [*C. diff*][*C DIFF*] spores]] [,] [and] [5 minutes to kill HIV and the other organisms listed on the label][.]
- DISPOSAL OF INFECTIOUS MATERIAL: Blood/body fluids must be autoclaved and disposed of according to local regulations for infectious waste disposal.

**PRECAUTIONARY STATEMENTS**

Hazards to humans and domestic animals

**CAUTION:** Causes moderate eye irritation. Avoid contact with eyes and clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco, or using the toilet. Remove and wash contaminated clothing before reuse.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the

6/18

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)," "towel(s)," "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

blood barriers or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

†This product is for the exterior surfaces of Glucometers and is not for use on the needle and/or the monitors.

**STORAGE AND DISPOSAL**

**Do not contaminate water, food or feed by storage or disposal**

**Pesticide Storage:** [For product sold in canisters] Keep this product (wipe/towelette) in a tightly closed container/canister, when not in use. Store canister in cool, dry, and well-ventilated area away from heat, open flame, direct sunlight and inaccessible to children. [For product sold in individual packets] Store in a cool, dry, and well-ventilated area away from heat, open flame, direct sunlight and inaccessible to children.

**Pesticide (towelette/wipe) Disposal:** Wastes resulting from the use of this product must be disposed of on site or at an approved waste disposal facility.

**Container/Canister Handling and Disposal:**

A. [For Household/Residential Products]

[For rigid nonrefillable container/canister less than 5 gallons]

**Container Handling:** Nonrefillable rigid container/canister. Do not re-use or refill this container/canister. Triple rinse (or equivalent), promptly after emptying. Offer for recycling, if available, or throw in trash.

B. [For Commercial/Industrial/Institutional Products]

[For rigid nonrefillable container/canister less than 5 gallons]

**Container Handling:** Nonrefillable rigid container/canister. Do not re-use or refill this container.

Triple rinse (or equivalent), promptly after emptying. Offer for recycling or reconditioning, or puncture and dispose of in a sanitary landfill, or incineration, or, if allowed by state and local authorities, by burning. If burned, stay out of smoke.

C. [For Refillable Containers]

**Container Handling:** Refillable container. Refill this container with pesticide only. Do not reuse this container for any other purpose. To clean the container before final disposal; Triple rinse (or equivalent), promptly after emptying. Offer for recycling or reconditioning, or puncture and dispose of in a sanitary landfill, or incineration, or, if allowed by state and local authorities, by burning. If burned, stay out of smoke.

D. [For individual packed wipes [towelettes]]

**Container Handling:** Wipes: Dispose of [wipe] [towelette] in trash after use. Do not flush. Offer empty packet for recycling if available or discard in trash.

Complete product line information available by calling:

7157 North Austin Avenue [MEDTROL, INC. logo]  
Niles, Illinois 60714  
Ph: 800-647-7180 Fx: 800-255-3027

e-mail: sales@medtrol.com  
Website: [www.medtrol.com](http://www.medtrol.com)  
[BARCODE LABEL]

7/18

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)," "towel(s)," "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

**{OPTIONAL ADDITIONAL TEXT FOR USE ON PRODUCT LABEL OR COLLATERAL LABELING}**

- [1:10 [dilution of] [stabilized] bleach [sodium hypochlorite] [dilution] [solution]]
- [1:10 dilution of [stabilized] [bleach] [sodium hypochlorite] [household bleach]
- [ 1:10 ][dilution of ] [solution of] [Sodium hypochlorite] [NaOCl] [bleach] [household bleach]
- [[3] [three] [minutes] [mins.] [contact time] [is effective] [is required] [efficacy] [against] [to kill] [[*Clostridium difficile*] [*C. difficile*] [*C. diff*] [*C DIFF*] spores] [in one-step]]
- [[5] [Five] [minutes] [mins.] [contact time] [is effective] [is required] [efficacy] [against] [to kill] [+HIV and other organisms listed on the label]]
- [[30] [Thirty] [seconds] [secs.] [contact time] [is effective] [is required] [efficacy] [against] [to kill] +HBC and +HCV ]
- [[1] [one] [-] step]]
- [1:10 [(6500 ppm)] [dilution of][sodium hypochlorite][NaOCl][bleach][equivalent to household bleach].
- [A one step cleaning and disinfecting sporicidal bleach wipe that meets CDC guidelines for killing *Clostridium difficile* spores.]
- [A potable water rinse is required for food contact surfaces]
- An effective way to [clean] [,] [and] [deodorize] [and] [disinfect] [against odor-causing organism] [on] hard, nonporous surfaces listed on this label]
- [Active] [sodium hypochlorite][0.65%][6500 ppm][[active][sodium hypochlorite]
- [Aids [in reducing] [in the reduction of] cross contamination between treated surfaces]
- [All of the [raw materials] [components] of [the] [this] [these] [packets] [,] [and] [liquid[s]] [,] [and] [canister[s]] [product[s] are made in the [U.S.A.][USA][.]]
- [All in one] [one-step] [cleaner and disinfectant][cleaner] [disinfectant]
- [Antibacterial]
- [An effective method to [clean] [and] [,] [deodorize] [and] disinfect hard, non-porous environmental [and] [healthcare] surfaces]
- [An effective way to clean and deodorize against odor-causing organisms and disinfect hard, non-porous surfaces] [Assembled for] [and] [distributed by] [:]
- [Available exclusively from] [:]
- [Avoid excessive heat]
- [Avoid prolonged freezing]
- [Avoid direct sunlight]
- [Available chlorine 0.65%, 6500 ppm]
- [Attached and kills] [Destroys] [Deactivates] [helps] [eliminate] [tough to kill] [*Clostridium difficile*][spores] in one-step on hard, nonporous healthcare surfaces]
- [Based on Federal EPA master labels and state registration reviews as of [date of most recently approved label]. Use as directed on pre-cleaned, hard, non-porous surfaces.]
- [Bathroom [disinfectant] wipes for hard, non-porous environmental surfaces]
- [Bleach [sodium hypochlorite] [disinfectant] wipe[s][.]]
- [Bleach wipe[s]]
- [Bleach disinfectants] [1:10 dilution] [hospital use]
- [[Blood] glucose meter[s]†]
- [Can be used to][Clean][s] [,] [deodorize][s] and disinfect][s] [in one step] [hard, non-porous] [healthcare] [and] [environmental] [surfaces] [in one step][.]
- [Can be used on] [Use on] hard [,] non-porous surfaces only.
- [Can be used on blood glucose meters†]

8/18

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

- [This product] [Product name] [cleans and disinfects] [kills] [*Clostridium difficile*] [*C. difficile*] [*C. diff*] [*C DIFF*] spores] in one-step]
- [Can be used] [For use] on hard [,] non-porous surfaces of] [Can be used on] [Can be] [Used to [clean and] disinfect] [[blood] glucose meters<sup>†</sup>][[blood] glucose monitors<sup>†</sup>] [glucometer[s]<sup>†</sup>] [,] [and] [blood pressure cuff[s]] [and] [,] [stethoscopes][in [blood] glucose meter<sup>†</sup> rental programs]
- [Check with your sales representative for updates in your state].
- [Complete product line information available by calling: xxx-xxx-xxxx]
- [Can be used to] [For use to][Disinfect[s] hard, non-porous [environmental] [and] [healthcare] surfaces]
- [Can be used to] [Used to][Disinfect[s] [headsets] [telephones] [[cell] [cellular] [wireless] [mobile] [digital] phone[s]]
- [Clean and disinfect surfaces with one [convenient] wipe]
- [Cleans, disinfects, and deodorizes]
- [Contains 6200 ppm [parts per million] [of] available chlorine]
- [Contains 0.62% available chlorine]
- [Contains 6500 [parts per million] [ppm] [of] active sodium hypochlorite]
- [Contains 0.65% sodium hypochlorite]
- [Contains] [0.65% [active] sodium hypochlorite]
- [Contains][6500][parts per million] [ppm] [0.65%][of][active][sodium hypochlorite] [NaOCl][bleach][equivalent to 1:10 dilution of household bleach]
- [Contains 6500 ppm NaOCl (yields 0.62% available chlorine)]
- [Contains 1:10 dilution (6500ppm) sodium hypochlorite]
- [Contains sodium hypochlorite in a 1:10 dilution that is recommended by CDC guidelines for effective disinfecting against infectious organisms.]
- [Can be used on] [For use on] [blood] [glucose meters<sup>†</sup>][Glucometers<sup>†</sup>]]
- [*Clostridium difficile*] [*C. difficile*] [*C. diff*] [*C DIFF*] spores]
- [[Convenient] [Premeasured] [Premoistened] [Disposable] [Pre-saturated] [towelette] [Disposable][germicidal] [sporicidal] wipes]
- [Disinfects, cleans, deodorizes]
- [Disinfectant wipe] [Can be used] [For use] [on][[Telephone[s]]][[cell] [cellular] [wireless] [mobile] [digital] phone[s]]
- [Disinfects hard, non-porous healthcare surfaces]
- [[Disinfectant] [Disinfecting] [bleach] [sodium hypochlorite] [wipe[s]]
- Disinfects [blood glucose meters<sup>†</sup>] [,] [and] [patient care equipment] [,] [and] [point-of-care equipment] and hard, non-porous healthcare surfaces.
- [Distributed by] [.]
- [Disposable]
- [Do not use on glassware, utensils or dishes]
- [Easy] [,] [this product] [Product name] [Can be used] [as a] [is a] [one-step] cleaning and disinfecting] [one-step cleaner and disinfectant] bleach wipes[s] [effective against] [that kills] [in the presence of] [*Clostridium difficile*] [*C. difficile*] [*C. diff*][*C DIFF*] spores] [in three minutes] [in 3 min.].]
- [Easy and convenience to use]
- [[Effective against] [Kills] [Bactericidal for] [*Staphylococcus aureus*] [ATCC 6538], [*Salmonella enterica*] [ATCC 10708][,] [and] [*Pseudomonas aeruginosa*] [ATCC 15442]] ] [in 5 minutes] [in 5 min.] [in the presence of organic soil (5% blood serum)] [and] [on [precleaned] hard, non-porous [environmental][and][healthcare] surfaces]



{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}  
 {Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

- [[Effective against] [Virucidal for] [Kills] [+Hepatitis B Virus] [+HBV] [and] [+Human] [+Hepatitis C Virus] [+HCV] [in 30 seconds] [in 30 sec.] [in the presence of organic soil (5% blood serum)] [and] [on [precleaned] hard, non-porous [environmental][and][healthcare] surfaces]
- [[Effective against] [Virucidal for] [Kills] [+Human Immunodeficiency Virus Type 1] [+HIV-1] [in 5 minutes] [in 5 min.] [in presence of organic soil (5% blood serum)] [and] [on [precleaned] hard, non-porous [environmental][and][healthcare] surfaces]
- [[Equivalent to] household bleach]]
- [This product][Product name][is a one-step cleaner and disinfectant][that][is][[Effective against] [Sporicidal for] [Kills] [[*Clostridium difficile*] [*C. difficile*] [*C. diff*][*C DIFF*] spores] [ATCC 63598] [in 3 minutes] [in 3 min.] in presence of 5% organic [soil] load [and] [on hard, non-porous [environmental] [and] [healthcare] surfaces]
- [[Effective against] [Fungicidal for][Kills] *Trichophyton mentagrophytes* [ATCC 9533]] ] [in 5 minutes] [in 5 min.] [in presence of organic soil (5% blood serum)] [and] [on [precleaned] hard, non-porous [environmental][and][healthcare] surfaces]
- [[Effective against] [Bactericidal for][Kills] [[*Staphylococcus aureus*-MRSA[(MRSA)], [[Methicillin Resistant *Staphylococcus aureus* [(MRSA)], [*Staphylococcus aureus* (MRSA)], [MRSA]] [ATCC 33592]] ] [in 5 minutes] [in 5 min.] [in presence of organic soil (5% blood serum)] [and] [on [precleaned] hard, non-porous [environmental][and][healthcare] surfaces]
- [[Effective against] [Bactericidal for][Kills] [[*Enterococcus faecalis*-VRE [(VRE)], [[Vancomycin Resistant *Enterococcus faecalis* [(VRE)], [*Enterococcus faecalis* (VRE)], [VRE] [ATCC 51575]] ] [in 5 minutes] [in 5 min.] [in presence of organic soil (5% blood serum)] [and] [on [precleaned] hard, non-porous [environmental][and][healthcare] surfaces]
- [[Effective against] [Kills] [Bactericidal for] [*Staphylococcus aureus*] [*S. aureus*] [ATCC 6538] [in 5 minutes] [in 5 min.] [in presence of organic soil (5% blood serum)] [and] [on [precleaned] hard, non-porous [environmental][and][healthcare] surfaces]
- [[Effective against][Bactericidal for] [Kills] *Salmonella enterica* ATCC 10708 [in 5 minutes] [in 5 min.] [in presence of organic soil (5% blood serum)] [and] [on [precleaned] hard, non-porous [environmental][and][healthcare] surfaces]
- [[Effective against] [Bactericidal for] [Kills] *Pseudomonas aeruginosa* ATCC 15442 ] [in 5 minutes] [in 5 min.] [in presence of organic soil (5% blood serum)] [and] [on [precleaned] hard, non-porous [environmental][and][healthcare] surfaces]
- [[Effective against] [Bactericidal for] [Kills] [*Staphylococcus aureus*] [ATCC 6538] [and][,] [*Salmonella enterica*] [ATCC 10708] and [*Pseudomonas aeruginosa*] [ATCC 15442]] ] [in 5 minutes] [in 5 min.] [in presence of organic soil (5% blood serum)] [and] [on [precleaned] hard, nonporous [environmental] [and] [healthcare] surfaces]
- [[Effective against][Virucidal for] [Kills] [+Human Immunodeficiency Virus Type 1] [[+HIV-1][,] [and][+Hepatitis B Virus][+HBV] [in presence of organic soil (5% blood serum)] [and] [+Human][+Hepatitis C virus] [+HCV] [in the presence of organic soil ((1% horse serum))][and][on [precleaned] hard, non-porous [environmental] [and] [healthcare] surfaces]
- [Effective against] [For use in [disinfecting] [and] [cleaning] [Other potentially infectious materials [(OPIM)]] [OPIM] [on hard non-porous surfaces]
- [Environmental Services][Renal Dialysis][Laboratories][Infection Control] [ICU/CCU][General Nursing] [Nursing][Home Healthcare] [Pediatrics] [Out Patient Care] [Surgery][Respiratory Care] [EMS] [Housekeeping] [Skilled long term care facility[ies]] [Skilled Nursing Facility] [SNF] [dental offices] [physical therapy] [physician offices][radiology][recovery room] [nursing homes] [nursing stations]
- [EPA registered to kill *Clostridium difficile* spores in 3 minutes][the fastest contact time available][based upon EPA master label [review][approval] [as of] [date of most recently approved label].

10/18

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

- [Expiration date] [EXP] [EXP DATE] [XXXXXXX]
- [Equivalent to the 1:10 dilution of [sodium hypochlorite][household bleach] following the [guidelines][of][the][CDC][,][and][OSHA] [For use in [disinfecting] [cleaning] [[Blood containing body fluids]] [blood and body fluid spills] [on hard non-porous surfaces]
- [Formulated] [for] [cleaning and disinfecting] [in [just] one step][.] [Eliminates the need to preclean surfaces prior to disinfecting.]
- [Following][Follows] the guidelines of [the] [CDC][,][and][OSHA]
- [Following][Follows] [the guidelines][regulations] of [the] [Federal] [and] [state] [guidelines] [regulations] [for] [disinfection of][disinfecting][blood][glucose meters<sup>†</sup>][Glucometers<sup>†</sup>]
- [For disinfection of][For disinfecting][Disinfects][blood][glucose meters<sup>†</sup>][Glucometers<sup>†</sup>][,][and][patient care equipment][,][and][point-of-care equipment][and] hard, non-porous healthcare surfaces]
- [For complete product line information, call Medtrol, Inc.]
- [For maximum product performance store at room temperature or cooler]
- [For one-step cleaning and disinfecting] [when][if] [[*Clostridium difficile*] [*C. difficile*] [*C. diff*][*C DIFF*] spores][are] [likely][to be present][of concern]
- [For refilling [Product Name]]
- [For surfaces that may come in contact with food, a potable water rinse in required. Do not use on glassware, utensils, or dishes.]
- [For product questions, call [name and phone number]]
- [For use in [cleaning and] disinfecting [fecal contamination] on[precleaned][hard, non-porous][high touch][objects] [and][environmental] [and] [healthcare] [surfaces]
- [Follows][Following] the guidelines of [the][CDC][and][APIC][for use in [cleaning and] disinfecting [possible][fecal contamination][on] [in][precleaned] patient care areas [on][hard, non-porous][high touch][objects][and][environmental] [and] [healthcare] [surfaces][such as] [bed rails][patient bed controls][phone and call button][chair[s]][light switches] [room temperature control knobs][sink tops][toilet seats][flush handles][bedside tables][drawer pull[s]][IV pole][door knobs][and][door levers] [bathroom handrails][bed pan[s]][tray table][toilet bedpan cleaner][multi-module monitor [controls] [touch screens][cables]][ventilator control panel][shower head and water control knobs] [bar code readers][infusion pumps][bedside commodes][wheel chairs][privacy curtains]
- [Fresh bleach scent]
- [Germicidal wipes]
- [Germicidal wipes] [Toallitas Germicidas]
- [Germicidal] [Germicide]
- [Glucometers<sup>†</sup>]
- [If streaking is observed after disinfecting for the appropriate contact time, wipe with a clean, damp cloth or paper towel.]
- [In the][area] [center] [station][department][s][listed][below][here][:] [Hospitals][Doctors's Offices][Nursing Homes][Schools] [Dental Offices] [Emergency Rooms]
- [Innovative healthcare products designed with a commitment to quality and excellence]
- [[Kitchen][and][bathroom][cleaner and disinfectant] [.] [For surfaces that may come in contact with food, a potable water rinse in required. Do not use on glassware, utensils, or dishes.]
- [Kills 99.99% of germs]
- [Kills] [[*Clostridium difficile*][*C. difficile*][*C. diff*][*C DIFF*] spores] [in [one] [1] step in [three] [3] [minutes] [min]
- [Kills +HBV [,] [and] +HCV in 30 [seconds] [sec.]
- [Kills] [is effective against] 99.9999% [[*Clostridium difficile*] [*C. difficile*] [*C. diff*] [*C DIFF*] spores] in 3 [minutes] [min] [in one step] on hard, nonporous surfaces][Lot Number] [Lot xxxxxx]

11/18

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

- [Made in [the] [U.S.A.] [USA]] [.]
- [Manufactured for] [.]
- [Manufactured] [and] [Distributed ] [Sold] in the [USA] [U.S.A.] [.]
- [Meets] OSHA blood borne pathogen standards][in][29 CFR 1910.1030] [and] [Guidelines of the CDC].
- [Meets efficacy standards for hospital disinfection]
- [Moist inanimate surface wipes]
- [Moist [bleach] [sodium hypochlorite] wipes for hard, non-porous [healthcare] [,] [and] [environmental] [inanimate] surfaces]
- [Must be][precleaned][preclean][prior to][before][disinfecting][disinfection][of] hard, non-porous [environmental][and][healthcare][surfaces] for [+Human][+Hepatitis C Virus][+HCV]
- [No mixing] [No measuring]
- [No need to pre-clean]
- [Not a skin wipe][.] (pictogram)
- [Not a baby wipe][.] (pictogram)
- [Now kills] {Note to reviewer: to be used with new efficacy claim – limited to 6 months of marketing.}
- [Now effective against] {Note to reviewer: to be used with new efficacy claim – limited to 6 months of marketing.}
- [One step [cleaner disinfectant] [disinfectant cleaner]
- [One-step] [cleaner] [and] [,] [disinfectant] [against][[Clostridium difficile] [C. difficile] [C. diff][C DIFF] spores] [ATCC 63598]
- [One step cleaning disinfectant that kills [Clostridium difficile] [C. difficile] [C. diff][C DIFF] spores] [ATCC 63598], [MRSA], [VRE], [Staphylococcus aureus], [Salmonella enterica], [Pseudomonas aeruginosa], [Trichophyton mentagrophytes],[ +HIV-1 and +HBV]. Preclean prior to disinfecting surfaces for +HCV.
- [OSHA [has approved] [recognizes] the use of [a][1:10 dilution of][bleach] [sodium hypochlorite] solutions as an effective means of protecting healthcare workers from the risk of contamination by bloodborne pathogens.]
- [Packaged] [Packed] for [XXXXXXXX]
- [Preclean prior to disinfecting surfaces for +HCV.]
- [[Product name][this product] [is a] [are] [for use on hard non-porous [environmental] surfaces in] [hotels][[kitchen][s] [and][bathroom[s]]][hospitality industry][pharmaceutical exhaust hoods][biological safety [cabinets][hoods]][janitorial surfaces] [computer[screen][and]keyboards][electronic digital equipment][telemetry][veterinary medicine][aviation][law enforcement][clean rooms][ISO9000 environments][extended care facilities][rehabilitation facilities][cafeterias][restaurants][emergency medical services][offices][[dental][and][medical][and][clinical][and][hospital] laboratory[ies][lab[s]]
- [[Product name][this product] [is a] [are] premoistened disposable towels for use on hard, non-porous [healthcare][and][environmental] surfaces where control of cross contamination is required]
- [[Product Name][These][The][towels] are saturated with [the] a solution [equivalent to] [a] [of] [prediluted] 1:10 [dilution of] [sodium hypochlorite] [bleach]] [household bleach (sodium hypochlorite)]]
- [Product Name] [These wipes] [Meets efficacy standards for hospital disinfection]
- [Product Name] [These wipes] [is a] [are] [Bleach disinfectant[s]] [at a] [1:10 dilution] [for] [hospital use]
- [Product name[s]][can be used to][help[s]][comply[lies] with] [meet[s]][the] [CDC guidelines] [and] [OSHA [blood borne pathogen standards in] [rule] [29 CFR 1910.1030] [to disinfect] [for] [disinfection of] [disinfecting] [blood] [glucose meters<sup>†</sup>] [Glucometers<sup>†</sup>]]

12/18

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

- [Product name[s]][can be used to][help[s]][comply[lies]with][meet[s]][the] [42 CFR 483.65][for] [rigorous cleaning the environment to remove [[*Clostridium difficile*][C.] *difficile*] [C. *diff*] [C *DIFF*] spores][,][and can help prevent transmission of the organism]
- [Product name[s]][can be used to][help[s]] [comply[lies] with] [meet[s]][the] [42 CFR 483.65] [for] [disinfection of] [disinfecting] [blood] [glucose meters<sup>†</sup>] [Glucometers<sup>†</sup>]
- [Product name[s]] meets [CDC] [and] [,] [OSHA] guidelines for [disinfection of] [disinfecting][blood][glucose meters<sup>†</sup>] [Glucometers<sup>†</sup>]
- [Product name] [this product] can be used to clean and disinfect in one step [;] [,] see directions for use [on this label] for cleaning and disinfecting in one step
- [Product name[s]] [can be used to] [help[s]] [comply[lies] with] [meet[s]] [the] [CDC] [,] [and] [OSHA] [guidelines for] [disinfection of] [disinfecting][blood][glucose meters<sup>†</sup>] [Glucometers<sup>†</sup>]
- [Protect from freezing]
- [Pre-saturated] [towel] [towelette] [wipe]
- [Pre-moistened towels with 1:10 dilution of bleach [(0.65%)]
- [Pre-moistened with sodium hypochlorite [(bleach)] in a 1:10 dilution (6500 ppm.) recommended by CDC guidelines for effective disinfecting against infectious organisms.]
- [Pre-moistened]with sodium hypochlorite in a 1:10 dilution [(6500 ppm)] as recommended by CDC.]
- [Ready to Use] [hospital [use] [grade] disinfectant]
- [Ready to use] [EPA registered hospital disinfectant]
- [Ready to use] [EPA registered hospital [use] [grade] [disinfectant] [bleach wipes]
- [Refills for [Product Name]
- [Rotate dated product[s] while in inventory]
- [Shield from direct sunlight]
- [Shelf-stable formula]
- [Saturated with a solution that] [Contains 6500 [parts per million] [ppm] [0.65%] [of active] sodium hypochlorite] [NaOCl][bleach]
- [See [side] [back][panel[s] [on reverse] [on outer box] for [First Aid][,][and][Directions for Use][,][and][Special instructions for cleaning and decontamination against +HIV-1, +HBV and +HCV][,] [and] [Precautionary Statements][and][Storage and Disposal].
- [Single Use Only]
- [Suitable for hospital use]
- [Suitable for use on hard, nonporpous [inanimate environmental] [and] [healthcare] surfaces]
- [Sodium hypochlorite] [0.65%] [active] [sodium hypochlorite] [NaOCl][bleach]
- [Sodium hypochlorite] [NaOCl] [bleach] [household bleach]
- [Sold by] [:]
- [Sporicidal] [Sporicide]
- [Sporicidal wipes]
- [Sporicidal] [Bactericidal] [Fungicidal] [Virucidal]
- [Store at room temperature or cooler for maximum product performance]
- [Store [Product Name] [these] [bleach] wipes in sealed packets] [when not in use] [Store [Product Name] [these] [bleach] wipes in tightly closed canister] [when not in use]
- [Store [this] [these] [Product Name] [product[s]] in a cool, dry, and well-ventilated area away from [heat] [,] [and] [open flame] [,] [and] [direct sunlight] [and] [inaccessible to children]
- [Suggested areas of usage][on][hard, non-porous][high touch] [objects] [and] [environmental] [and] [healthcare] [surfaces][in patient care areas][such as] [bed rails][patient bed controls][phone and call button][chair[s]][light switches] [room temperature control knobs][sink tops][toilet seats][flush handles] [bedside tables] [drawer pull[s]][IV pole][door knobs][and][door levers][bathroom handrails][bed pan[s]] [tray table][toilet bedpan cleaner][multi-module monitor [controls][touch

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

screens][cables]][ventilator control panel][shower head and water control knobs] [bar code readers] [infusion pumps] [bedside commodes] [wheel chairs][privacy curtains][PCV tubing][diagnostic equipment][stretchers][ambulance equipment] [patient monitoring equipment][trash cans][plastic mattress covers][computer keyboards] [instrument stands]

- [Test wipe on small inconspicuous area.]
- [The [Product Name] [bleach] wipes were specifically [designed] [created] to disinfect [blood] [glucose meters†] [Glucometers†] and [point of care] [bedside] [equipment] [and] [patient care equipment] [to meet] [to comply with] [the equipment manufacturers maintenance [requirements] [protocols] [procedures].
- EPA registered [product] [product name] [with a] [[*Clostridium difficile*][*C. difficile*] [*C. diff*] [*C DIFF*] spore kill claim with [a][one step] 3 minute contact time. This product may not yet be available in all 50 states [and Puerto Rico]
- EPA registered [product] [product name] [with a] [[+Human][+Hepatitis C Virus][+HCV] kill claim with 30 second contact time. This product may not yet be available in all 50 states [and Puerto Rico]
- EPA registered to kill [*Clostridium difficile*][*C. difficile*][*C DIFF*] spores [on hard, non-porous][environmental][and][,][healthcare][and][,] [medical equipment] [point-of-care equipment] [and] [,] [patient care equipment][surfaces] [in one-step] [in 3 minutes contact time].
- [This Product] [Product name] [kills [+HBV] [and] +HCV] in 30 secs.].
- [This product] [Product name] [eliminates the need to preclean [hard, non-porous] [surfaces] prior to disinfection]. Pre-cleaning is still required to kill +HIV-1, +HBV, and +HCV.
- [This product][Product name] [can be used][as a] [is a] [A] [One Step Cleaner [and] [,] [Deodorizer] [and] Disinfectant [effective against] [that kills]: [[*Clostridium difficile*] [*C. difficile*] [*C. diff*][*C DIFF*] spores] [,] [*Pseudomonas aeruginosa*] [,] [*Staphylococcus aureus*] [,] [*Salmonella enterica*] [,][[*Staphylococcus aureus*-MRSA[(MRSA)]], [[Methicillin Resistant *Staphylococcus aureus* [(MRSA)]], [*Staphylococcus aureus* (MRSA)], [MRSA]] [,] [[*Enterococcus faecalis*-VRE [(VRE)]], [[Vancomycin Resistant *Enterococcus faecalis* [(VRE)]], [*Enterococcus faecalis* (VRE)], [VRE] [,] [+ Hepatitis B Virus] [+HBV] [+Human Immunodeficiency Virus Type 1] [+HIV-1] [,] [and] [*Trichophyton mentagrophytes*] [in the presence of organic soil [5% blood serum]] [and] ] [on [precleaned] hard, non-porous [environmental] [and] [healthcare] surfaces]
- [This product][Product name] [can be used][as a] [is a] [A Cleaner [and] [,] [Deodorizer] [and] Disinfectant [effective against] [that kills]: [[*Clostridium difficile*] [*C. difficile*] [*C. diff*][*C DIFF*] spores] [,] [*Pseudomonas aeruginosa*] [,] [*Staphylococcus aureus*] [,] [*Salmonella enterica*] [,][[*Staphylococcus aureus*-MRSA[(MRSA)]], [[Methicillin Resistant *Staphylococcus aureus* [(MRSA)]], [*Staphylococcus aureus* (MRSA)], [MRSA]] [,] [[*Enterococcus faecalis*-VRE [(VRE)]], [[Vancomycin Resistant *Enterococcus faecalis* [(VRE)]], [*Enterococcus faecalis* (VRE)], [VRE] [,] [+ Hepatitis B Virus] [+HBV][,] [+Human][+Hepatitis C Virus] [+HCV] [,] [+Human Immunodeficiency Virus Type 1] [+HIV-1] [,] [and] [*Trichophyton mentagrophytes*] [in the presence of organic soil [5% blood serum]] [and] ] [on [precleaned] hard, non-porous [environmental][and][healthcare] surfaces]
- [This product][Product name] [can be used][as a] [is a] [A [One Step] Cleaner[,] [Deodorizer] [and] Disinfectant [effective against] [that kills]: [[*Clostridium difficile*] [*C. difficile*] [*C. diff*][*C DIFF*] spores] [in the presence of organic soil [5% blood serum]] [and] ] [on hard, non-porous [environmental][and][healthcare] surfaces]
- [This] [These] [Products[s]] [Product Name] [is] [a] [are] [Germicidal [bleach] wipe[s]]
- [This] [These] [Products[s]] [Product Name] [is] [a] [are] [Germicidal] [Germicide] [bleach] [wipe[s]]
- [This] [These] [Products[s]] [Product Name] [is] [a] [are] [stabilized] [bleach] [wipe[s]]
- [This] [These] [Products[s]] [Product Name] [is] [an] [are] [Antiviral] [bleach] [wipe[s]]
- [This] [These] [Products[s]] [Product Name] [is] [an] [are] [Antimicrobial] [bleach] [wipe[s]]

14/10

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout }

- [This] [These] [Products[s]] [Product Name] [is] [an] [are] [Antifungal] [bleach] [wipe[s]]
- [This] [These] [Products[s]] [Product Name] [is] [an] [are] [sporicidal] [bleach][wipe[s]]
- [This] [These] [Product Name] [wipe[s]] [is] [are] [contain] [a] [saturated] [with] [a] [Solution of] [a] [1:10 [dilution of] [activated] [bleach] [sodium hypochlorite]]
- [This] [These] [Product Name] [surface] [wipe[s]] [is] [are] [highly absorbent] [and] [tear-resistant]
- [This] [These] [Product Name] [is a] [are] [Bleach [sodium hypochlorite][disinfectant] [disinfecting] wipe[s]][.]
- [This] [These] [Product Name] [is a] [are] [Ready to Use] [EPA registered] [hospital grade disinfectant] [bleach] [wipes]
- [This][These][Product[s]][Product name[s]] [is][are][saturated with a solution][of][that contains] [6500 ppm] [0.65%] [of] [active] [sodium hypochlorite][available chlorine]
- [This][These] [Product[s]] [Product name[s]] [meet OSHA bloodborne pathogen standards in CFR 1910.1030 [and guidelines of [the] [CDC] ] [to disinfect][for][disinfection of][disinfecting][blood] [glucose meters<sup>†</sup>][Glucometers<sup>†</sup>].
- [This][These] [Product[s]] [Product name] [is a] [one-step] [cleaner] [,] [and][deodorizer][and] [disinfectant that kills *Clostridium difficile* spores, MRSA, VRE, *Staphylococcus aureus*, *Salmonella enterica*, *Pseudomonas aeruginosa*, *Trichophyton mentagrophytes*, +HIV-1 and +HBV.] [Preclean prior to disinfecting surfaces for +HCV.]
- [This] [These] [Product Name] [is] [a] [are] [for] [Can be used] [to] [for] [used to] [as a] [clean[s]] [cleaning] [cleaner] [,] [and] [deodorize[s]] [deodorizing] [deodorizer] [,] [and] [to] [decontaminate[s]] [decontaminating] [and disinfect[s]] [disinfecting] [disinfectant]
- [This] [These] [Product Name] [is] [are] included on EPA's [List C: EPA's Registered Antimicrobial Products Effective Against Human HIV-1 Virus] [,] [,] [[http://www.epa.gov/oppad001/list\\_c\\_hiv.pdf](http://www.epa.gov/oppad001/list_c_hiv.pdf)] [.]
- [This] [These] [Product Name] [is] [are] included on EPA's [List D: EPA's Registered Antimicrobial Products Effective Against Human HIV-1 and Hepatitis B Virus] [,] [,] [[http://www.epa.gov/oppad001/list\\_d\\_hepatitisbhiv.pdf](http://www.epa.gov/oppad001/list_d_hepatitisbhiv.pdf)] [.]
- [This] [These] [Product Name] [is] [are] included on EPA's [List F: EPA's Registered Antimicrobial Products Effective against Hepatitis C Virus] [,] [,] [[http://www.epa.gov/oppad001/list\\_f\\_hepatitisC.pdf](http://www.epa.gov/oppad001/list_f_hepatitisC.pdf)]
- [This] [These] [Product Name] [is] [are] included on EPA's [List H: EPA's Registered Antimicrobial Products Effective Against Methicillin Resistant *Staphylococcus aureus* (MRSA) and Vancomycin Resistant *Enterococcus faecalis* or *faecium* (VRE)] [,] [,] [[http://www.epa.gov/oppad001/list\\_h\\_mrsa\\_vre.pdf](http://www.epa.gov/oppad001/list_h_mrsa_vre.pdf)] [.]
- [This] [These] [Product Name] [is] [are] included on EPA's [List K: EPA's Registered Antimicrobial Products Effective against *Clostridium difficile* spores] [,] [,] [[http://www.epa.gov/oppad001/list\\_k\\_cdifffspores.pdf](http://www.epa.gov/oppad001/list_k_cdifffspores.pdf)] [.]
- [This product] [Product name] [is a cleaner disinfectant] [is] [Effective against] [that] [kills] [Blood borne pathogen[s] [(BBP)]] [BBP] [+HIV-1, +HBV and +HCV] [.]
- [This product] [Product name] [is a] [cleaner disinfectant] [disinfectant cleaner] [can be used for cleaning and disinfecting] [Effective against] [that kills] [For use in [disinfecting] [cleaning] [blood borne pathogens [(BBP)]] [BBP] [+HIV-1, +HBV and +HCV [on hard non-porous surfaces]
- [This product][Product name][is a][are][one-step] [[*Clostridium difficile* [*C. difficile*] [*C. diff*] [*C DIFF*] spores] [cleaner disinfectant][cleaning disinfectant][disinfecting cleaner]
- [This product][Product name][can be used to][clean[s] and disinfect[s]][hard, non-porous][medical equipment surfaces]
- [[This product][Product name] contains [towels saturated with] the power of Medtrol[']s] [bleach][sodium hypochlorite] solution]

CS/18

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

- [This product][is][are][a] [one step cleaner and disinfectant][one step cleaning and disinfecting] [sporicide][that][is] [sporicidal] [bleach wipes][that] [and][can be used to][help[s]][comply[ies]] [with] [meet[s]] the [CDC] [guidelines] [standards] [requirements] [recommendations] [for][killing] [[*Clostridium difficile*][C.] *difficile*][C *DIFF*] spores]
- [This product][is][a] [are][one step cleaner and disinfectant] [one step cleaning and disinfecting] [sporicide] [that is] [sporicidal][bleach wipes][that][and][can be used to][help[s]][comply[ies]] [with][meet[s]] the [CDC][guidelines] [standards] [requirements] [recommendations] [for][killing] [[*Clostridium difficile*][C.] *difficile*] [C. *diff*] [C *DIFF*] spores]
- [This Product][Product name] [cleans, deodorizes and disinfects in [1][one]step]
- [This Product [Product name] [kills [[*Clostridium difficile*][C. *difficile*][C. *diff*][C *DIFF*] spores in [1] [one] step [in [3][three][minutes][mins.]
- [This product][Product name][is][an][the only] EPA registered bleach wipe that kills *Clostridium difficile* spores [in] [one step][in] [3 minutes][with] [no precleaning required]
- [This product][Product name] has a one step, 3 minute kill claim for *Clostridium difficile* spores and has been registered by the Federal EPA and may not be available in all 50 states.] [Please check with your sales representative for updates in your state.]
- [This product][Product name] has a 30 second kill claim for [Hepatitis C Virus][HCV] and has been registered by the Federal EPA and may not be available in all 50 states.] [Please check with your sales representative for updates in your state.]
- [This product] [Product Name] [Can be used on blood glucose meters<sup>†</sup>]
- [These wipes contain 0.65% sodium hypochlorite [bleach]]
- [These] [large] [Product Name] [wipes] [can be used to] [help[s]] [to] [comply with] [make compliance with] [infection control procedures] [easier and more effective]
- [To clean [,] [and] disinfect [and] deodorize in one step: Wipe surface with towel until completely wet. Let stand for five minutes at room temperature. Wipe dry or allow to air dry. [Gross soil must be removed prior to disinfecting.] Gloves should be worn.]
- [This product] [Product name] [is][The first][The one][The only] [and only] [one step] [3 minute] [EPA registered bleach wipe that kills *Clostridium difficile* spores][in one step][in 3 minutes].
- [[This product][Product name] with the *Clostridium difficile* spore and HCV kill claims may not yet be available in all states.]
- [[This product][Product name][is registered federally with [*Clostridium difficile* spore][and][Hepatitis C Virus][HCV] kill claims, and [this product][Product name] will be coming soon to all 50 states.]
- [This product][Product name][is][a][hospital cleaner disinfectant [wipe] [saturated with a 1:10 dilution][of][with][ bleach][sodium hypochlorite].]
- [This product][Product name][that][is][a][one step][multipurpose, broad spectrum [bleach][sodium hypochlorite] disinfectant that kills pathogens rapidly.]
- [The][bleach][sodium hypochlorite] concentration of [Product name][this product] is equivalent to the CDC's recommended strength for effective disinfecting.]
- [This product][Product name] [contains][a 1:10 dilution][of][a][bleach][sodium hypochlorite] [solution][similar][equivalent][to][a 1:10 dilution][of][bleach][sodium hypochlorite][that is recommended by CDC guidelines for effective disinfecting [against infectious organisms].]
- [This product][Product name] meets OSHA standards for protecting healthcare workers from bloodborne pathogens.]
- [This product][Product name][is a] [fast acting, one step cleaner disinfectant].
- [This product][Product name][is a] [fast acting, one step cleaner disinfectant] [that][cleans and disinfects hard, non-porous surfaces quickly [in just one step].]

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)," "towel(s)," "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

- [The 3 minute] [*Clostridium difficile*][*C. difficile*][*C DIFF*][spores kill claim] [and][the][30 second] [Hepatitis C Virus][HCV][kill claim][have][has] [been registered by the Federal EPA and may not yet be available in all 50 states].
- [This product][Product name] [meets CDC recommendations for killing [*Clostridium difficile*][*C. difficile*][*C DIFF*] spores.]
- [This product][Product name] [can be used][Use as directed][as a one step disinfectant on [precleaned] hard, non-porous surfaces] [to disinfect][for disinfecting] against bloodborne pathogens.]
- [This product][Product name] [contains [bleach][sodium hypochlorite] in a 1:10 dilution and eliminates the need to preclean hard, non-porous [environmental] [and][healthcare] surfaces prior to disinfection.]
- [This product][Product name] [contains [bleach][sodium hypochlorite] in a 1:10 dilution [that is] recommended by CDC guidelines for effective disinfecting against infectious organisms.]
- [This product][Product name] [helps] [to][wipe[s]out] [(MRSA)], [[Methicillin Resistant *Staphylococcus aureus* [(MRSA)], [*Staphylococcus aureus* (MRSA)], [MRSA]] [ATCC 33592]] [in 5 minutes] [in 5 min.] in just [5][minutes][and [*Clostridium difficile*][*C. difficile*][*C DIFF*] spores][in 3 minutes][.][.][and] [with][wipe[s]] [saturated][with][that][contain[s]] [bleach][sodium hypochlorite] in a 1:10 dilution [that is] recommended by CDC guidelines for effective disinfecting against infectious organisms.]
- [Use[d]]
- [Use wipes to kill [*Salmonella enterica* [Salmonella]] [and] [*Staphylococcus aureus* [Staph]] on bathroom hard, non-porous surfaces]
- [Use[d] to disinfect hard, non-porous surfaces in hospitals, doctor's offices, nursing homes, schools and dental offices]
- [[Use to] [Effectively] [clean[s] and] disinfect[s] common hard, non-porous surfaces]
- [Use][This product][product name][as][is][a][can be used][as a][for][one-step cleaner and disinfectant][one-step cleaning and disinfecting] [effective against] [that kills] [in the presence of] [[*Clostridium difficile* [*C. difficile*][*C. diff*][*C DIFF*] spores][.]
- [Use][This product][can be used] [for][For use][is] [in][as][a][one-step] [cleaner and disinfectant] [cleaning and disinfecting][in one step] [disinfectant][disinfecting][of][fecal contamination][of][on][hard, non-porous][high touch][objects][and][environmental] [and] [healthcare][surfaces][when][if] [[*Clostridium difficile*] [*C. difficile*] [*C. diff*][*C DIFF*] spores] [are] [likely][to be present][of concern]
- [Use][This product][can be used] [for][For use][is] [in][as][a][one-step] [cleaner and disinfectant][cleaning and disinfecting][in one step] [disinfectant][disinfecting][of][fecal contamination][of][on][hard, non-porous][high touch][objects][and][environmental] [and] [healthcare] [surfaces]
- [Use][This product][can be used] [for][For use [in][as][a] [in one step] [disinfectant] [disinfection] [disinfecting] [for][of][fecal contamination][of][on][precleaned][hard, non-porous][high touch][objects] [and][environmental] [and] [healthcare] [surfaces] [when][if] [[*Clostridium difficile*][*C. difficile*] [*C. diff*] [*C DIFF*] spores][are] [likely][to be present][of concern]
- [Use][This product][can be used] [for][For use [in][as][a] [disinfectant] [disinfection] [disinfecting] [for] [of][fecal contamination][of][on][precleaned][hard, non-porous][high touch] [objects] [and] [environmental] [and] [healthcare] [surfaces]
- [Yields][[0.65%][6500 ppm] available chlorine [0.65%][6500 ppm]
- [Where control of the hazards of cross contamination between treated surfaces is of prime importance]



17/18

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}


{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}

The following pictograms are for optional inclusion to the Super-Chlor product label:


- Not a baby wipe pictogram (below)



- "To maintain product shelf life KEEP LID CLOSED after each use" pictogram (below)

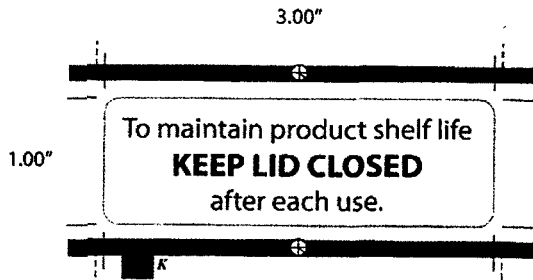


- Ship and store this end up pictogram (Below)

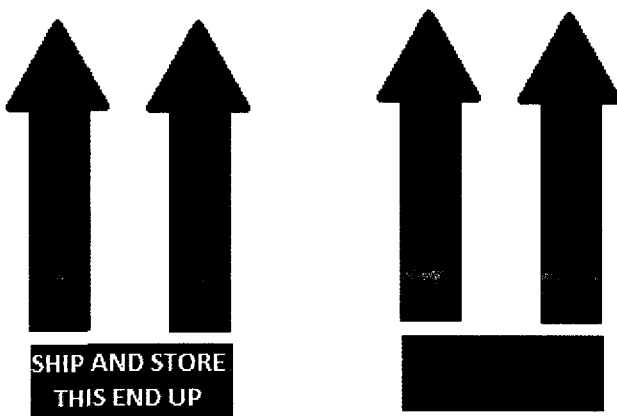

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}  
{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)", the terms "Premoistened", "Premeasured", "Convenient", "Disposable" and "Pre-saturated", and the terms "Glucometers", "Glucose Meters", and "Blood Glucose Meter" may be used interchangeably throughout.}


- 5-Minute Kill Time Clock pictogram (below)

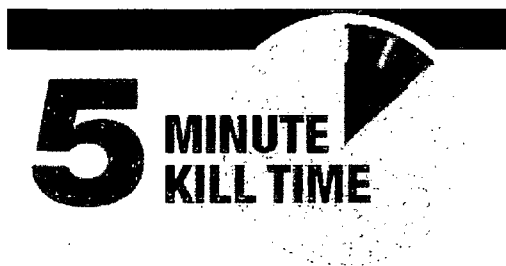

- 3-Minute Kill Time Clock pictogram



- Made in the USA

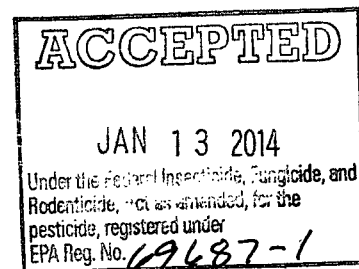

- Not for use on skin logo

