

Prediluted Sodium hypochlorite (Bleach) Solution

[CAT# CS114]

# CHLORO-SOL™ [Product Name]

For [Cleaning] [,] [Sanitization] [and] [Decontamination] [of] Hard Non-Porous [Surfaces] [Environmental Surfaces] [/] [,] [Patient Care Equipment Surfaces] Soiled with Blood & Body Fluids Prior to High Level Disinfectant and Sterilization.

Active Ingredient: Sodium hypochlorite..... 0.525%

Other Ingredients: .....99.475%

## SUGGESTED AREAS OF USAGE:

[[Medical] [,] [Dental] [and] [Laboratory] Counters] [,] [Exam Tables] [,] [Carts] [,] [Point-of-Care Equipment Surfaces] [,] [Telephones] [,] [Sink Tops] [,] [Toilet Seats] [Patient Care Equipment].

## KEEP OUT OF REACH OF CHILDREN CAUTION

**FIRST AID:** If in eyes: Hold eye open and rinse slowly and gently with water for 15 – 20 minutes.

Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.

Call a poison control center or doctor for treatment advice.

### STORAGE AND DISPOSAL

- Store product in a cool, dry area, away from heat and direct sunlight. • Do not reuse empty packet and [used] towelette.
- Discard used towelette [and] [empty] [packet] in designated infectious waste container for autoclaving and disposal according to federal, state and local regulations.

(See panel on reverse for directions for use and additional precautionary statements.)

EPA Reg. No. 69687-2

EPA Est. No. 69687-IL-001

[DIN: 02246210]


Net Contents [1 fl. oz.]

[16 fl. oz.]

[32 fl. oz.]

[128 fl. oz.]

[MEDTROL, INC. logo]  
[© 2004 Medtrol, Inc.]

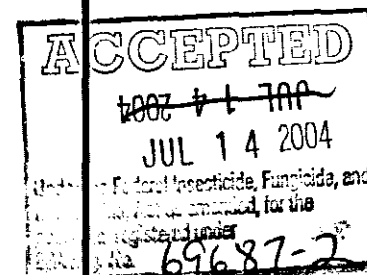

69687-2

6/14/2004

1/2

{Administrative Note: Bracketed text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)" may be used interchangeably throughout.}

{Administrative Note: The terms "packet(s)", "container(s)" and "bottle(s)" may be used interchangeably throughout.}

### Directions for use:

- It is a violation of Federal Law to use this product in a manner inconsistent with its labeling
- Always use personal protective equipment such as disposable gloves, gowns, masks or eye coverings when handling items soiled with blood and body fluids.
- Open [Chloro-Sol™] [Product Name] packet.
- [Pour] [Apply] [Spray] [Chloro-Sol] [Product Name] [liquid] [solution] on desired surface.
- Scrub and wipe with towel of your choice over desired surface to be [decontaminated] [disinfected].
- Allow a minimum contact time of 15 minutes and then allow to air dry.
- Discard towel and [empty] packet (see front panel).

### Precautionary Statements

Hazards to humans and domestic animals

**CAUTION:** Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling.


This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barriers or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

Complete product line information available by calling:

7157 North Austin Avenue  
Niles, Illinois 60714  
e-mail: [sales@medtrol.com](mailto:sales@medtrol.com)  
Website: [www.medtrol.com](http://www.medtrol.com)

[ MEDTROL, INC. LOGO ]

Ph: 800-647-7180  
Fx: 800-255-3027  
[BARCODE LABEL]

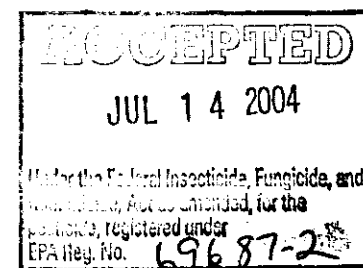

{Administrative Note: BracRêted text [ ] is optional version of text for different packaging.}

{Administrative Note: The terms "towelette(s)", "towel(s)", "wipe(s)", "wiper(s)" and "cloth(s)" may be used interchangeably throughout.}

{Administrative Note: The terms "packet(s)", "container(s)" and "bottle(s)" may be used interchangeably throughout.}