



U.S. ENVIRONMENTAL PROTECTION AGENCY
Office of Pesticide Programs
Antimicrobials Division (7510C)
401 "M" St., S.W.
Washington, D.C. 20460

EPA Reg. Number:

Date of Issuance:

70060-25

March 3, 2005

Term of Issuance: Conditional

Name of Pesticide Product: Aseptrol Dental
Water Treatment

NOTICE OF PESTICIDE:

- Registration
- Reregistration

(under FIFRA, as amended)

Name and Address of Registrant (include ZIP Code):

Englehard Corporation
101 Wood Avenue
Iselin, N.J. 08830-0770

Note: Changes in labeling differing in substance from that accepted in connection with this registration must be submitted to and accepted by the Antimicrobials Division prior to the use of the label in commerce. In any correspondence on this product always refer to the above EPA registration number.

On the basis of information furnished by the registrant, the above named pesticide is hereby registered/reregistered under the Federal Insecticide, Fungicide and Rodenticide Act. Registration is in no way to be construed as an endorsement or recommendation of this product by the Agency. In order to protect health and the environment, the Administrator, on his motion, may at any time suspend or cancel the registration of a pesticide in accordance with the Act. The acceptance of any name in connection with the registration of a product under this Act is not to be construed as giving the registrant a right to exclusive use of the name or to its use if it has been covered by others.

This product is conditionally registered in accordance with FIFRA sec. 3(c)(7)(A) provided you:

1. Submit and/or cite all data required for registration/reregistration of your product under FIFRA sec. 3(c)(5) when the Agency requires all registrants of similar products to submit such data.
2. Revise the EPA File symbol 70060-EL to EPA Registration Number 70060-25.
3. Obtain and add an EPA Establishment number.
4. Fix typo at Initial Treatment heading.
5. Submit two copies of the revised final printed label for our records.

If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA sec. 6(e).

Your release for shipment of the product constitutes acceptance of these conditions.

Copies of our reviews of data submitted in support of this registration are enclosed.

A stamped copy of the label is enclosed for your records.

Signature of Approving Official:

Emily H. Mitchell
Acting Product Manager 32

Date:

MAR - 3 2005

- All text in brackets [xxx] is optional and may or may not be included on a final label.
- All text in braces {xxx} is administrative and will not appear on a final label.

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

DANGER. Corrosive. Causes irreversible eye damage. Harmful if swallowed. Do not get in eyes or on clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco. Remove contaminated clothing before reuse.

FIRST AID

If in Eyes:	<ul style="list-style-type: none"> •Hold eye open and rinse slowly and gently with water for 15-20 minutes. •Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. •Call a poison control center or doctor for treatment advice.
If Swallowed:	<ul style="list-style-type: none"> •Call poison control center or doctor immediately for treatment advice. •Have person sip a glass of water if able to swallow. •Do not induce vomiting unless told to do so by the poison control center or doctor. •Do not give anything by mouth to an unconscious person.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

NOTE TO PHYSICIAN

Probable mucosal damage may contraindicate the use of gastric lavage.

ENVIRONMENTAL HAZARDS

This product is toxic to fish and other aquatic organisms.

PHYSICAL/CHEMICAL HAZARDS

Keep away from combustible, organic or other readily oxidizable materials. Do not mix with any other chemicals.

MAR - 3 2005

**ASEPTROL®
DENTAL WATER
TREATMENT**

**For the Cleaning and Control of
Microbial Contamination in Dental Unit
Water Lines**

Active Ingredients:
Sodium Chlorite 4.0%
Sodium Dichloroisocyanurate dihydrate ... 1.0%
Other Ingredients: 95.0%
Total: 100.0%

KEEP OUT OF REACH OF CHILDREN

DANGER

See [Side] {or} [Back] Panels for First Aid and Precautionary Statements

EPA Reg. No. 70060-
EPA Est. No.

Net Weight:

Manufactured By:
Engelhard Corporation
Iselin, NJ 08830

Sodium chlorite in this product is from Spain

[contains]



[Aseptrol and the Aseptrol logo are trademarks of Engelhard Corporation]

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

INITIAL TREATMENT:

1. Fill the 750 ml (3/4 gallon) dental unit water bottle with potable water and add two (2) 1.5 gram Aseptrol® Dental Water Treatment tablets. Allow the tablets to dissolve.
2. Flow about half of the treated water through the dental water lines and let the treated water sit within the lines for 5 minutes. Purge the remaining water through the lines.
3. Repeat steps 1 and 2.
4. Rinse and refill the dental unit water bottle with potable water and flush at least 500 ml (1/2 gallon) of water through the lines to remove any residual chemical.

ROUTINE TREATMENT:

1. Every 1 to 4 weeks, fill the 750 ml (3/4 gallon) dental unit water bottle with potable water and add one (1) 1.5 gram Aseptrol® Dental Water Treatment tablets. Allow the tablets to dissolve.
2. Flow about half of the treated water through the dental water lines and let the treated water sit within the lines for 5 minutes. Purge the remaining water through the lines.
3. Rinse and refill the dental unit water bottle with potable water and flush at least 500 ml (1/2 gallon) of water through the lines to remove any residual chemical.

Note: Periodic cleaning may be done on any day of the week.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal. Store in a cool, dry area away from sunlight and heat and in an area inaccessible to children. Keep product from contacting air prior to use. Place empty foil pouch in trash.

70060-25

e/a