

- All text in brackets [xxx] is optional and may or may not be included on a final label.
- All text in braces {xxx} is administrative and will not appear on a final label.

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

DANGER. Corrosive. Causes irreversible eye damage. Harmful if swallowed. Do not get in eyes or on clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco. Remove contaminated clothing before reuse.

FIRST AID

If in Eyes:	<ul style="list-style-type: none"> •Hold eye open and rinse slowly and gently with water for 15-20 minutes. •Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. •Call a poison control center or doctor for treatment advice.
If Swallowed:	<ul style="list-style-type: none"> •Call poison control center or doctor immediately for treatment advice. •Have person sip a glass of water if able to swallow. •Do not induce vomiting unless told to do so by the poison control center or doctor. •Do not give anything by mouth to an unconscious person.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

NOTE TO PHYSICIAN

Probable mucosal damage may contraindicate the use of gastric lavage.

ENVIRONMENTAL HAZARDS

This product is toxic to fish and other aquatic organisms.

PHYSICAL/CHEMICAL HAZARDS

Keep away from combustible, organic or other readily oxidizable materials. Do not mix with any other chemicals.

ACCEPTED

APR 26 2006

Under the Federal Insecticide, Fungicide, and Rodenticide Act as amended, for the pesticide, registered under EPA Reg. No. **70060-25**

**ASEPTROL®
DENTAL WATER TREATMENT**

For the Cleaning and Control of Microbial Contamination in Dental Unit Waterlines

Active Ingredients:

Sodium Chlorite	4.0%
Sodium Dichloroisocyanurate dihydrate ...	1.0%
Other Ingredients:	95.0%
Total:	100.0%

KEEP OUT OF REACH OF CHILDREN

DANGER

See [Side] {or} [Back] Panels for First Aid and Precautionary Statements

EPA Reg. No. 70060-25
EPA Est. No.

Net Weight:

Manufactured By:
Engelhard Corporation
Iselin, NJ 08830

Patent: 6,699,404

Sodium chlorite in this product is from Spain

[contains]



(Aseptrol and the Aseptrol logo are trademarks of Engelhard Corporation)

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

INITIAL TREATMENT:

1. Fill the 750 ml (3/4 liter) dental unit water bottle with potable water and add two (2) 1.7 gram Aseptrol® Dental Water Treatment tablets. Allow the tablets to dissolve.
2. Flow about half of the treated water through the dental waterlines and let the treated water sit within the lines for 5 minutes. Purge the remaining water through the lines.
3. Repeat steps 1 and 2.
4. Rinse and refill the dental unit water bottle with potable water and flush at least 500 ml (1/2 liter) of water through the lines to remove any residual chemical.

ROUTINE TREATMENT:

1. Every 1 to 4 weeks, fill the 750 ml (3/4 liter) dental unit water bottle with potable water and add one (1) 1.7 gram Aseptrol® Dental Water Treatment tablet. Allow the tablet to dissolve.
2. Flow about half of the treated water through the dental waterlines and let the treated water sit within the lines for 5 minutes. Purge the remaining water through the lines.
3. Rinse and refill the dental unit water bottle with potable water and flush at least 500 ml (1/2 liter) of water through the lines to remove any residual chemical.

Note: Periodic cleaning may be done on any day of the week.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal. Store in a cool, dry area away from sunlight and heat and in an area inaccessible to children. Keep product from contacting air prior to use. Place empty foil pouch in trash.

70060-25 4/26/2006 Page 1 of 1