

{Front Side of Pouch}

WHEN OPENING: AVOID CONTACT WITH EYES, SKIN, CLOTHING

←===== Tear Here =====>

Hype-Wipe®

DISINFECTING TOWEL WITH BLEACH

For One-Step Cleaning, Disinfecting Counters, Equipment and other Hard, Non-Porous Surfaces.

Guaranteed to contain at least a 1:10 dilution (0.55% by weight) of NaOCl (bleach) at expiration.


KILLS:

Staphylococcus aureus • *Pseudomonas aeruginosa* • *Salmonella choleraesuis*

Active Ingredient:

Sodium Hypochlorite	0.94%
Other Ingredients	99.06%
Total	100.00%

Towel: Folded 6" x 12" nonwoven polypropylene
EPA Reg. No. 70590-1 EPA Est. 70590-IN-1
Net Wt. 12.2 grams (excludes towel)

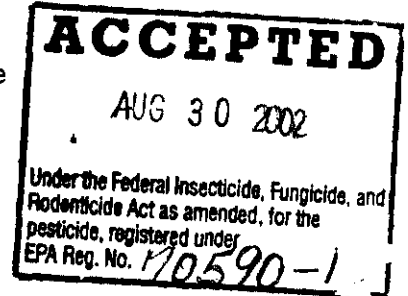

**Keep Out of Reach of Children
CAUTION**

Read complete directions and precautions on rear panel.

current technologies, inc.
P.O. Box 21
Crawfordsville, IN 47933 U.S.A.
Phone: 1-800-456-4022 Fax: 765-364-1607

Reorder # 9103

{Back Side of Pouch}

WHEN OPENING: AVOID CONTACT WITH EYES, SKIN, CLOTHING

←===== Tear Here =====>

- Meets AOAC Hard Surface Carrier Disinfection Test
- Guaranteed to contain at least a 1:10 dilution of household bleach for one year from manufacture date.
- Suitable for hard, non-porous surfaces including stainless steel, plastics, glass, ceramics.
- For use in clinical and industrial labs, clinics, hospitals, blood banks, physician and dental offices, wastewater facilities, emergency units.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

1. Open pouch, remove towel. Use towel and liquid to wipe surface.
2. Allow solution to contact surface for 5 minutes before wiping or allow to air dry. Dispose towel in appropriate waste container.
3. Personal Protection: Wear gloves when using.

Storage: Store in a cool, dry area away from sunlight and heat to inhibit deterioration of bleach solution. **Disposal:** Do not reuse empty pouch; wrap and put in trash.

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

CAUTION: Harmful if absorbed through skin. Avoid contact with skin, eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum or using tobacco.

First Aid

If in Eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice. **If on Skin**

Clothing: Take off contaminated clothing. Immediately rinse skin with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment advice. **If Swallowed:** Immediately call a poison control center or doctor for treatment advice. Have person sip a glass of water if able to swallow. Do not induce vomiting unless told to do so by the poison control center or doctor. Do not give anything by mouth to an unconscious person. Have the product container or label with you when calling a poison control center or doctor.

Physical or Chemical Hazards

Do not use this product with ammonia, acids such as vinegar, rust removers, toilet bowl cleaners. Will corrode aluminum.

Expiration Date: {MM/DD/YY}

{ } Denotes language that does not appear on the market label