



U.S. ENVIRONMENTAL PROTECTION AGENCY
Office of Pesticide Programs
Antimicrobials Division (7510C)
1200 Pennsylvania Avenue, N.W.
Washington, D.C. 20460

EPA Reg. Number:
75829-1

Date of Issuance:
SEP 09 2004

NOTICE OF PESTICIDE:

Registration
 Re-registration

(under FIFRA, as amended)

Term of Issuance:
CONDITIONAL

Name of Pesticide Product:
H₂Pro Clean Start Bottle A

Name and Address of Registrant (include ZIP code):

Garrison Dental Solutions
110 DeWitt Lane
Spring Lake, MI 49456

Note: Changes in labeling differing in substance from that accepted in connection with this registration must be submitted to and accepted by the Antimicrobials Division prior to use of the label in commerce. In any correspondence on this product always refer to the above EPA registration number.

On the basis of information furnished by the registrant, the above named pesticide is hereby registered/reregistered under the Federal Insecticide, Fungicide, and Rodenticide Act, as amended.

Registration is in no way to be construed as an endorsement or recommendation of this product by the Agency. In order to protect health and the environment, the Administrator, on his motion, may at any time suspend or cancel the registration of a pesticide in accordance with the Act. The acceptance of any name in connection with the registration of a product under this Act is not to be construed as giving the registrant a right to exclusive use of the name or to its use if it has been covered by others.

This product is conditionally registered in accordance with FIFRA Section 3(c)(7)(A) provided that you:

1. Submit and/or cite all data required for registration of your product under FIFRA Section 3(c)(5) when the Agency requires all registrants of similar products to submit such data; and submit acceptable responses required for re-registration of your product under FIFRA Section 4.

With this notice, the Agency is also approving the requested alternate brand name of "H₂Pro Bottle A Maintenance Treatment".

If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA Section 6(e). Your release for shipment of the product constitutes acceptance of these conditions.

A stamped copy of the label is enclosed for your records. If you have any questions concerning this letter, please contact myself at 703-308-6341, or Tony Kish at (703) 308-9443.

Signature of Approving Official:

Marshall Swindell, Product Manager,
Team 33, Regulatory Management Branch I,
Antimicrobials Division

Date:

SEP 09 2004

Garrison Dental Solutions
Spring Lake, Michigan 49456
EPA Company Number: 075829
EPA Establishment Number: 075829-MI-0

APPLICATION FOR REGISTRATION
DRAFT MASTER LABELING
H₂Pro Clean Start Bottle A /
H₂Pro Bottle A Maintenance

REVISED 8/26/04

2/2

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

PESTICIDE STORAGE

Store at room temperature out of direct sunlight. Do not contaminate water, food or feed by storage or disposal.

PESTICIDE DISPOSAL

Pesticide wastes are toxic. Improper disposal of excess pesticide is a violation of federal law. If these wastes cannot be disposed of by use according to label instructions, contact your state pesticide or environmental control agency, or hazardous waste representative at the nearest EPA regional office for guidance.

CONTAINER DISPOSAL

Securely wrap original container in several layers of newspaper and discard in trash.

ACCEPTED
with COMMENTS
in EPA Letter Dated:

SEP 0 9 2004

Under the Federal Insecticide,
Fungicide, and Rodenticide Act as
amended, for the pesticide,
registered under EPA Reg. No.

75 829-1

**H₂Pro™ Clean Start™
Bottle A**

Dental Water Line Treatment

H₂Pro has been specially formulated and clinically proven to control odor causing bacteria and clean dental unit water lines.

Active Ingredient: Silver 0.0015%
Inert Ingredients: 99.9985%
Total 100.0000%

This product contains 0.0001245 lbs of silver per gallon.

KEEP OUT OF REACH OF
CHILDREN

CAUTION

Manufactured by
Garrison Dental Solutions
Spring Lake, Michigan 49456

EPA Registration No. 75829-1
EPA Establishment No. 075829-MI-001

Net Contents: 4 US Ounces (118 ml)

DIRECTIONS FOR USE

FOR PROFESSIONAL USE ONLY

A general-purpose cleaner for use in self contained dental unit water line systems. Use only in combination with H₂Pro Clean Start Bottle B (EPA Reg. No. 75829-2).

INITIAL TREATMENT

Pour entire contents of both H₂Pro Clean Start Bottles A and B into empty dental chair water bottle.

Run all lines attached to the dental chair until red coloration appears exiting the lines. Allow lines to sit unused for a minimum of 12 hours.

Prior to chair use completely drain remaining Clean Start Treatment solution from the dental chair reservoir. Prepare and use H₂Pro Maintenance Treatment (alternate formula of EPA Reg. No. 75829-1) according to the Maintenance Treatment directions.

MAINTENANCE TREATMENT

After initial dental unit water line treatment using H₂Pro Clean Start Bottles A plus B, pour entire contents of H₂Pro Bottle A Maintenance Treatment (alternate formula of EPA Reg. No. 75829-1) into the 1 gallon dispensing container provided. Add 1 US gallon (3.785 liters) of distilled or deionized water (32:1 dilution). Dispense from container into self contained dental unit water bottle. Run all lines attached to the dental chair until blue coloration appears exiting the lines. Use continuously in dental water lines instead of distilled, deionized or tap water.