

79662-1

09/19/2008

1/3



U.S. ENVIRONMENTAL PROTECTION AGENCY

Office of Pesticide Programs
Antimicrobials Division (7510P)
1200 Pennsylvania Avenue NW
Washington, D.C. 20460

EPA Reg. Number:
79662-1

Date of Issuance:

SEP 19 2008

Term of Issuance:

Conditional

Name of Pesticide Product:

ICX®

NOTICE OF PESTICIDE:

Registration
 Reregistration

(under FIFRA, as amended)

Name and Address of Registrant (include ZIP Code):

Thomas Louisell
A-dec, Inc.
2601 Crestview Drive
Newberg, OR 97132

Note: Changes in labeling differing in substance from that accepted in connection with this registration must be submitted to and accepted by the Registration Division prior to use of the label in commerce. In any correspondence on this product always refer to the above EPA registration number.

On the basis of information furnished by the registrant, the above named pesticide is hereby registered/reregistered under the Federal Insecticide, Fungicide and Rodenticide Act.

Registration is in no way to be construed as an endorsement or recommendation of this product by the Agency. In order to protect health and the environment, the Administrator, on his motion, may at any time suspend or cancel the registration of a pesticide in accordance with the Act. The acceptance of any name in connection with the registration of a product under this Act is not to be construed as giving the registrant a right to exclusive use of the name or to its use if it has been covered by others.

This product is conditionally registered in accordance with FIFRA sec 3(c)(7)(a) provided that you:

1. Submit and/or cite all data required for registration of your product under FIFRA sec. 3(c)(5) when the Agency requires all registrants of similar products to submit such data; and submit acceptable responses required for re-registration of your product under FIFRA section 4.
2. Make the labeling changes listed below before you release the product for shipment:
 - a. Revise the "EPA Registration Number to read, EPA Reg. No. 79662-1"

Signature of Approving Official:

Marshall Swindell
Product Manager Team-33
Regulatory Management Branch I
Antimicrobials Division (7510P)

Date:

SEP 19 2008


Page 2

EPA Reg. No. 79662-1

If these conditions are not complied with, the registration will be subject to cancellation in accordance with FIFRA sec. 6(e). Your release for shipment of the product constitutes acceptance of these conditions.

A stamped copy of the label is enclosed for your records. Submit one (3) copy of your final printed labeling prior to release of this product for shipment. If you have any questions concerning this letter, please contact Demson Fuller at (703) 308-8062.

Sincerely,

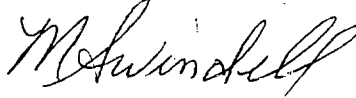

Marshall Swindell
Product Manager Team-33
Regulatory Management Branch I
Antimicrobials Division (7510P)

Enclosure: (Stamped Label)

A-dec®

ICX®

3
3

A-dec® ICX® tablets are specially formulated to maintain dental unit waterlines and prevent accumulation of odor and foul taste bacteria.

ACTIVE INGREDIENTS:

Sodium Percarbonate.....	6.96%
n-Alkyl(60% C ₁₄ 5% C ₁₂ 5% C ₁₈) dimethyl benzyl ammonium chloride	0.85%
n-Alkyl(68% C ₁₂ 32% C ₁₄) dimethyl ethylbenzyl ammonium chloride	0.85%
Silver Nitrate	0.14%
Other Ingredients.....	91.2%
<hr/>	
Total.....	100%

Net weight of tablet: 212.4 mg (2L application), 80.5 mg (0.7L application)
Net weight of product: 10.62 g (2L application), 4.03 g (0.7L application) – 50 tablets per box

KEEP OUT OF REACH OF CHILDREN

CAUTION

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

Causes moderate eye irritation, wear protective eyewear. Avoid contact with eyes, skin or clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco.

ACCEPTED
with COMMENTS
in EPA Letter Dated:

SEP 19 2008
Under the Federal Insecticide, Fungicide, and Rodenticide Act

79662-1

DIRECTIONS FOR USE:

ICX® is intended for use in general dental applications by providing water to the dental syringe, various handpiece ancillary devices, and cuspidor cupfill. When used according to these instructions for every bottle refill, optimum efficacy for control of odor and foul taste bacteria can be achieved.

1. Empty any remaining water left in the bottle.
2. Drop tablet into an empty dental unit water bottle (0.7L tablet in 0.7 liter bottle, 2 L tablet in 2 liter). Avoid touching the tablet with unprotected skin.
3. Fill bottle with water and connect to the dental unit.
4. Wait 60 seconds for the tablet to fully dissolve.

NOTE -This product is intended for use with potable water and in conjunction with regular dental unit water testing.

FIRST AID:

If solid product gets in eyes, hold eye open and rinse slowly and gently with water 15 to 20 minutes. Remove contact lenses, if present, after the first five minutes, then continue rinsing.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage disposal. Dispose of empty outer foil pouch in the trash. Store at room temperature out of direct sunlight; DO NOT store above 30°C (86°F).

A-dec Inc., 2601 Crestview Drive, Newberg, Oregon 97132, USA
Phone: 1-800-547-1883 Fax: 503-537-2843
EPA Establishment Number 84100
Made in USA

EPA Registration Number 79662-1