

79662-1

10/17/2008

10/6

OCT 17 2008

UNITED STATES ENVIRONMENTAL PROTECTION AGENCY
WASHINGTON, D.C. 20460



Office of Pesticide Programs

A-dec, Inc.
2601 Cresiview Drive
Newberg, OR 97132

Attention: Thomas H. Louisell

Subject: ICX
EPA Registration No. 79662-1
Amendment Dated October 2, 2008

The amendment, submitted in connection with registration under the Federal Insecticide, Fungicide and Rodenticide Act (FIFRA), as amended, is acceptable.

The Agency has no objection to the company request to change the signal word from "Caution" to "Warning" based upon the State of California review and recommendation for the Primary Dermal Irritation study.

A. stamped copy of the "accepted" labeling is enclosed for your records.

If you have any questions concerning this letter, please feel free to contact Demson Fuller at (703) 308-8062

Sincerely

Marshall Swindell
Product Manager 33
Regulatory Management Branch 1
Antimicrobials Division (7510P)

Enclosure

A-dec®

ICX®

A-dec® ICX® tablets are specially formulated to maintain dental unit waterlines and prevent accumulation of odor and foul taste bacteria.


ACTIVE INGREDIENTS:

Sodium Percarbonate.....	6.96%
n-Alkyl(60% C ₁₄ 30% C ₁₆ 5% C ₁₂ 5% C ₁₈) dimethyl benzyl ammonium chloride	0.85%
n-Alkyl(68% C ₁₂ 32% C ₁₄) dimethyl ethylbenzyl ammonium chloride	0.85%
Silver Nitrate	0.14%
Other Ingredients.....	91.2%
<hr/>	
Total.....	100%

Net weight of tablet: 212.4 mg (2L application), 80.5 mg (0.7L application)
 Net weight of product: 10.62 g (2L application), 4.03 g (0.7L application) 50 tablets per box

10.00202
10.31502

0.00302
0.14202

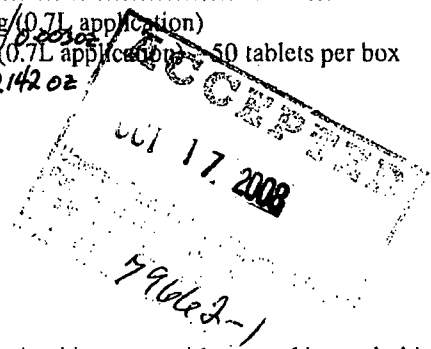

KEEP OUT OF REACH OF CHILDREN

WARNING

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

Causes moderate eye irritation, wear protective eyewear. Avoid contact with eyes, skin or clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco.

DIRECTIONS FOR USE:

ICX® is intended for use in general dental applications by providing water to the dental syringe, various handpiece ancillary devices, and cuspidor cupfill. When used according to these instructions for every bottle refill, optimum efficacy for control of odor and foul taste bacteria can be achieved.

1. Empty any remaining water left in the bottle.
2. Drop tablet into an empty dental unit water bottle (0.7L tablet in 0.7 liter bottle, 2 L tablet in 2 liter). Avoid touching the tablet with unprotected skin.
3. Fill bottle with water and connect to the dental unit.
4. Wait 60 seconds for the tablet to fully dissolve.

NOTE -This product is intended for use with potable water and in conjunction with regular dental unit water testing.

FIRST AID:


If solid product gets in eyes, hold eye open and rinse slowly and gently with water 15 to 20 minutes. Remove contact lenses, if present, after the first five minutes, then continue rinsing.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage disposal. Dispose of empty outer foil pouch in the trash. Store at room temperature out of direct sunlight; DO NOT store above 30°C (86°F).

A-dec Inc., 2601 Crestview Drive, Newberg, Oregon 97132, USA
 Phone: 1-800-547-1883 Fax: 503-537-2843
 EPA Establishment Number 84100
 Made in USA

EPA Registration Number 79662-1

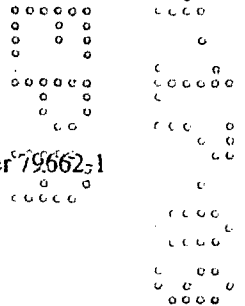

ICX®

A-dec® ICX® tablets are specially formulated to maintain dental unit waterlines and prevent accumulation of odor and foul taste bacteria.

ACTIVE INGREDIENTS:

Sodium Percarbonate.....	6.96%
n-Alkyl(60% C ₁₄ 30% C ₁₆ 5% C ₁₂ 5% C ₁₈) dimethyl benzyl ammonium chloride	0.85%
n-Alkyl(68% C ₁₂ 32% C ₁₄) dimethyl ethylbenzyl ammonium chloride	0.85%
Silver Nitrate	0.14%
Other Ingredients.....	91.2%

Total.....100%

Net weight of tablet: 212.4 mg / 0.008 oz (2L application), 80.5 mg / 0.003 oz (0.7L application)

Net weight of product: 10.62 g / 0.375 oz (2L application), 4.03 g / 0.142 oz (0.7L application) – 50 tablets per box

KEEP OUT OF REACH OF CHILDREN

WARNING

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

Causes moderate eye irritation, wear protective eyewear. Avoid contact with eyes, skin or clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco.

DIRECTIONS FOR USE:

ICX® is intended for use in general dental applications by providing water to the dental syringe, various handpiece ancillary devices, and cuspidor cupfill. When used according to these instructions for every bottle refill, optimum efficacy for control of odor and foul taste bacteria can be achieved.

1. Empty any remaining water left in the bottle.
2. Drop tablet into an empty dental unit water bottle (0.7L tablet in 0.7 liter bottle, 2 L tablet in 2 liter). Avoid touching the tablet with unprotected skin.
3. Fill bottle with water and connect to the dental unit.
4. Wait 60 seconds for the tablet to fully dissolve.

NOTE -This product is intended for use with potable water and in conjunction with regular dental unit water testing.

FIRST AID:

If solid product gets in eyes, hold eye open and rinse slowly and gently with water 15 to 20 minutes. Remove contact lenses, if present, after the first five minutes, then continue rinsing.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage disposal. Dispose of empty outer foil pouch in the trash. Store at room temperature out of direct sunlight; DO NOT store above 30°C (86°F).

A-dec Inc., 2601 Crestview Drive, Newberg, Oregon 97132, USA
 Phone: 1-800-547-1883 Fax: 503-537-2843
 EPA Establishment Number 84100
 Made in USA

EPA Registration Number 79662-i

Use with
2-liter bottle
50 TABLETS

a dec[®]
ICX[®]

A-dec [®] ICX [®] tablets are specially formulated to maintain dental unit waterlines and prevent accumulation of odor and foul taste bacteria.

Net weight of tablet: 212.4mg / 0.008 oz
Net weight of product: 10.62g / 0.375 oz
50 tablets/box

EPA Registration No. 79662-1
EPA Establishment No. 084100-CT-001

Use with
2-liter bottle
50 TABLETS

a dec[®]
ICX[®]

KEEP OUT OF REACH OF CHILDREN
WARNING

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

Causes moderate eye irritation, wear protective eyewear. Avoid contact with eyes, skin or clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco.

See insert for precautionary, first aid statements and storage and disposal.

LOT



+D7189010650117


Authorized EU Representative:
A-dec United Kingdom
Austin House, 11 Liberty Way
Nuneaton, Warwickshire CV11 6RZ England



U.S. Patent No. 7,033,511

Order no. 90.1065.01

4096

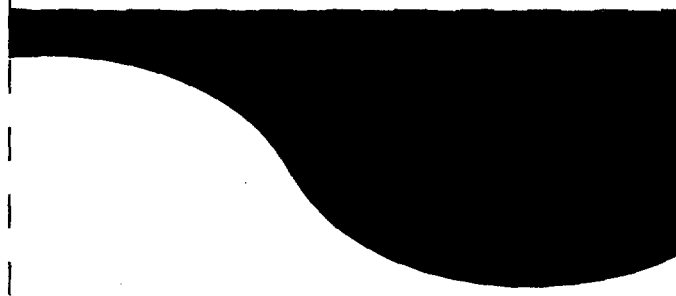

ICX[®]



Use with
2-liter bottle
50 TABLETS

Use with
2-liter bottle
50 TABLETS



ICX[®]

ACTIVE INGREDIENTS:

Sodium Percarbonate	6.96%
n-Alkyl (60% C ₁₄ , 30% C ₁₆ , 5% C ₁₂ , 5% C ₁₈) dimethyl benzyl ammonium chloride	0.85%
n-Alkyl (68% C ₁₂ , 32% C ₁₄) dimethyl ethylbenzyl ammonium chloride	0.85%
Silver Nitrate	0.14%
Other Ingredients	91.2%
Total	100%

DIRECTIONS FOR USE

It is a violation of federal law to use this product in a manner inconsistent with its labeling.

Before Refilling Bottle:


1. Empty any remaining water left in the 2 liter bottle.
2. Drop tablet into empty dental unit water bottle.
(Avoid touching the tablet with unprotected skin.)
3. Fill bottle with water, connect to dental unit.
4. Wait 60 seconds for tablet to fully dissolve.

NOTE -This product is intended for use with potable water and in conjunction with regular dental unit water testing.

A-dec Inc.
2601 Crestview Drive
Newberg, Oregon 97132 USA
Phone: 1-800-547-1883, 1-503-538-9471, Fax: 1-503-538-5911
©2008 A-dec Inc. All rights reserved.

www.a-dec.com

50f6

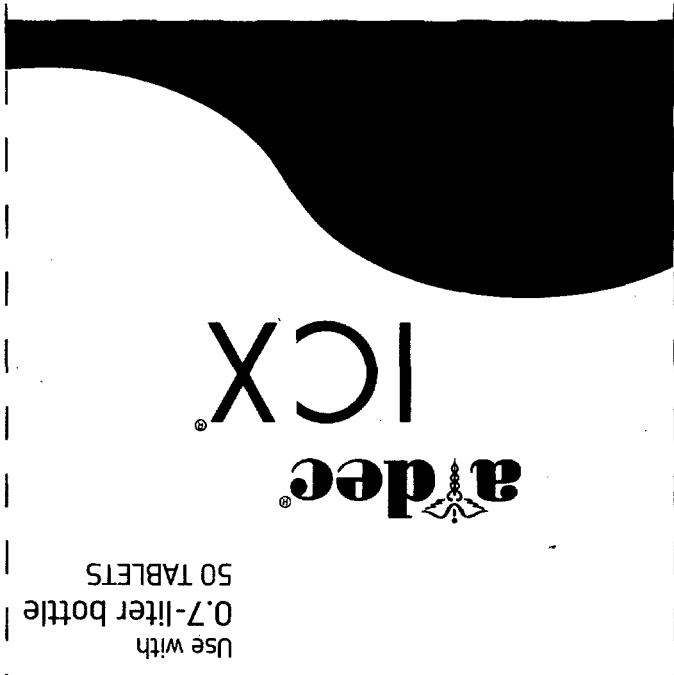

Use with
0.7-liter bottle
50 TABLETS



ICX[®]

A-dec Inc.
2601 Crestview Drive
Newberg, Oregon 97132 USA
Phone: 1-800-547-1883, 1-503-538-9471, Fax: 1-503-538-5911
© 2008 A-dec Inc. All rights reserved.

www.a-dec.com

ACTIVE INGREDIENTS:

Sodium Percarbonate	6.96%
n-Alkyl (60% C ₁₄ , 30% C ₁₆ , 5% C ₁₂ , 5% C ₁₈) dimethyl benzyl ammonium chloride	0.85%
n-Alkyl (68% C ₁₂ , 32% C ₁₄) dimethyl ethylbenzyl ammonium chloride	0.85%
Silver Nitrate	0.14%
Other Ingredients	91.2%
Total	100%

DIRECTIONS FOR USE

It is a violation of federal law to use this product in a manner inconsistent with its labeling.

Before Refilling Bottle:

- | | |
|---|--|
| 1. Empty any remaining water left in the 0.7 liter bottle. | 3. Fill bottle with water, connect to dental unit. |
| 2. Drop tablet into empty dental unit water bottle.
(Avoid touching the tablet with unprotected skin.) | 4. Wait 60 seconds for tablet to fully dissolve. |

NOTE - This product is intended for use with potable water and in conjunction with regular dental unit water testing.

6096

Use with
0.7-liter bottle
50 TABLETS

a dec[®]

ICX[®]

A-dec[™] ICX[®] tablets are specially formulated to maintain dental unit waterlines and prevent accumulation of odor and foul taste bacteria.

Net weight of tablet: 80.5mg / 0.003 oz
Net weight of product: 4.03g / 0.142 oz
50 tablets/box

EPA Registration No. 79662-1
EPA Establishment No. 084100-CT-001

Use with
0.7-liter bottle
50 TABLETS

a dec[®]

ICX[®]

KEEP OUT OF REACH OF CHILDREN
WARNING

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

Causes moderate eye irritation, wear protective eyewear. Avoid contact with eyes, skin or clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco.

See insert for precautionary, first aid statements and storage and disposal.

LOT



+D7189010640116

Authorized EU Representative:
A-dec United Kingdom
Austin House, 11 Liberty Way
Nuneaton, Warwickshire CV11 6RZ England



U.S. Patent No. 7,033,511

Order no. 90.1064.01