



UNITED STATES ENVIRONMENTAL PROTECTION AGENCY  
WASHINGTON D.C., 20460

OFFICE OF  
PREVENTION, PESTICIDES AND  
TOXIC SUBSTANCES

A-dec, Inc.  
2601 Crestview Drive  
Newberg, OR 97132

DEC 15 2009

Attention: Thomas Louisell  
Regulatory Manager

Subject: ICX  
EPA Reg. No. 79662-1  
Amendment Application Date: November 6, 2009

The amendment referred to above, submitted in connection with registration under the Federal Insecticide, Fungicide and Rodenticide Act, as amended, is acceptable, pending the following language is inserted:

- Storage Disposal language must be incorporated into the labeling per the guidance of 40 CFR part 165. In the case of your product, the following language must be incorporated: "Nonrefillable container. Do not reuse or refill this container. Offer for recycling if available or dispose of empty outer foil pouch in the trash"

Otherwise, the Agency has no objections to the changing the positioning of the ingredient statement and the change of the dissolution time for the product. A stamped label is included for your records.

If you have any questions concerning this letter, please contact Demson Fuller at (703) 308-8062.

Sincerely,

A handwritten signature in black ink that reads "M Swindell".

Marshall Swindell  
Product Manager (33)  
Regulatory Management Branch 1  
Antimicrobials Division (7510C)

DEC 15 2009

ACCEPTED  
 With COMMENTS  
 On EPA Form Dated:  
 DEC 15 2009

Under the Federal Insecticide,  
 Fungicide, and Rodenticide Act as  
 amended, for the pesticide,  
 registered under EPA Reg. No. 79662-1

Use with  
 0.7-liter bottle  
 50 TABLETS

**a dec**<sup>®</sup>  
**ICX**<sup>®</sup>

A-dec<sup>®</sup> ICX<sup>®</sup> tablets are specially formulated to maintain dental unit waterlines and prevent accumulation of odor and foul taste bacteria.

Net weight of tablet: 80.5mg / 0.003 oz  
 Net weight of product: 4.03g / 0.142 oz  
 50 tablets/box

EPA Registration No. 79662-1  
 EPA Establishment No. 084100-CT-001

ACTIVE INGREDIENTS:


Sodium Percarbonate .....	6.96%
n-Alkyl (60% C <sub>14</sub> , 30% C <sub>16</sub> , 5% C <sub>12</sub> , 5% C <sub>18</sub> ) dimethyl benzyl ammonium chloride .....	0.85%
n-Alkyl (68% C <sub>12</sub> , 32% C <sub>14</sub> ) dimethyl ethylbenzyl ammonium chloride .....	0.85%
Silver Nitrate .....	0.14%
Other Ingredients .....	91.2%
<b>Total .....</b>	<b>100%</b>

- DIRECTIONS FOR USE  
 It is a violation of federal law to use this product in a manner inconsistent with its labeling.
- Before Refilling Bottle:
- |   |  |
|---|--|
| 1. Empty any remaining water left in the 0.7 liter bottle.  | 3. Fill bottle with water, connect to dental unit. |
| 2. Drop tablet into empty dental unit water bottle.<br>(Avoid touching the tablet with unprotected skin.) | 4. Wait 120 seconds for tablet to fully dissolve.  |

NOTE -This product is intended for use with potable water and in conjunction with regular dental unit water testing.

LOT

PN: 1571F

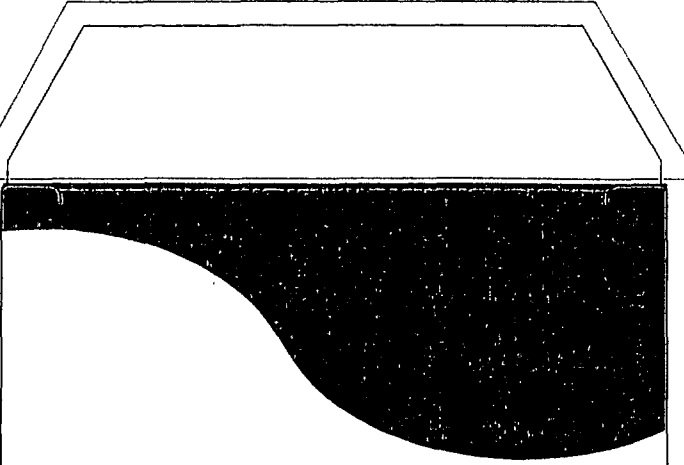

**ICX**  
**a dec**

Use with  
0.7-liter bottle  
50 TABLETS

Use with  
0.7-liter bottle  
50 TABLETS

**a dec**  
**ICX**

**KEEP OUT OF REACH OF CHILDREN**  
**WARNING**

**PRECAUTIONARY STATEMENTS**

**Hazards to Humans and Domestic Animals**

Causes moderate eye irritation, wear protective eyewear. Avoid contact with eyes, skin or clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco.

[www.a-dec.com](http://www.a-dec.com)

See insert for precautionary, first aid statements and storage and disposal.

*[Faint, illegible text]*

*[Faint, illegible text]*

79662-1