UNITED STATES ENVIRONMENTAL PROTECTION AGENCY WASHINGTON, D.C. 20460

JUN 10 2010



SEPA United States Environmental Protection Office of Pesticide Programs Office of Pesticide Programs

A-dec. Inc. 2601 Cresiview Drive Newberg, OR 97132

AGENT: Lewis and Harrison

122 C Street, N.W., Suite 740 Washington, D. C. 20001

Attention: Georgia Anastasious

Subject: ICX

EPA Registration No. 79662-1 Notification Dated May 17, 2010

This will acknowledge receipt of your notification of a new inert ingredient supplier, submitted under the provisions of FIFRA Section 3(c)(9).

Proposed Noatifiction

- Label change per PR Notice 2007-4

General Comments

Based on a review of the submitted material, the following comments apply.

The Notification is in compliance with PR Notice 98-10 and is acceptable. This information has been added to your file.

If you have any questions concerning this letter, please contact Martha Terry at (703) 308-6217.

Sincerely

Product Manager 33

Regulatory Management Branch 1 Antimicrobials Division (7510P)

3. Title

5. Date

Swania anastarious

both under applicable law.

2. Signature

4. Typed Name

Agent for A-dec

May 17, 2010

0000

0 0 0 0 0

(Stamped)



122 C Street, N.W., Suite 740 Washington, D.C. 20001 telephone 202.393.3903

fax 202.393.3906

Consultants in Government Affairs

May 17, 2010

HAND DELIVERED

Document Processing Desk (NOTIF) Office of Pesticide Programs (7504P) US Environmental Protection Agency Room S-4900, One Potomac Yard 2777 S. Crystal Drive Arlington, VA 22202-4501

ATTENTION:

Marshall Swindell

Product Manager (33)

SUBJECT:

A-dec, Inc

ICX (EPA Reg. No. 79662-1)

Application for Notification per PR Notice 2007-4

Dear Mr. Swindell:

On behalf of A-dec, I am submitting a notification for the product ICX (EPA Reg. No. 79662-1). The purpose of this notification is to incorporate the changes requested to the storage and disposal section in the EPA stamped label dated Dec 15, 2009 to the master label. The reason why I am submitting this as a notification is because A-dec encountered problems in the states and wanted to get confirmation that the storage and disposal language is now correct.

In support of this submission I have enclosed the following documents:

- 1) Application for Pesticide Notification.
- 2) One (1) copy of the label with changes highlighted.
- 3) Two (2) copies of the revised label.
- 4) Letter of authorization to allow Lewis & Harrison to act as company agent.

If you have any questions, or require any additional information, please feel free to contact me by phone at 202-393-3903 ext. 19, by fax at 202-393-3906, or by email at georgia@lewisharrison.com.

Sincerely,

Logia Anavasion

Georgia Anastasiou

Agent for,

A-dec, Inc

cc: A-dec (Phyton Corporation)

ICX[©]

A-dec[®] ICX[®] tablets are specially formulated to maintain dental unit waterlines and prevent accumulation of odor and foul taste bacteria.

ACTIVE INGREDIENTS:

Sodium Percarbonate	6.96%
n-Alkyl(60% C ₁₄ 30% C ₁₆ 5% C ₁₂ 5% C ₁₈) dimethyl benzyl ammonium chloride	0.85%
n-Alkyl(68% C ₁₂ 32% C ₁₄) dimethyl ethylbenzyl ammonium chloride	0.85%
Silver Nitrate	0.14%
Other Ingredients	91.2%
Total	100%

[Note to reviewer: EPA approved the location of the active ingredients statement on the back panel of the label on 12/15/09 in order to accommodate the small container size]

Net weight of tablet: 212.4 mg / 0.008 oz (2L application), 80.5 mg / 0.003 oz (0.7L application)

Net weight of product: 10.62 g / 0.375 oz (2L application), 4.03 g / 0.142 oz (0.7L application) – 50 tablets per box

KEEP OUT OF REACH OF CHILDREN

WARNING

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

Causes moderate eye irritation, wear protective eyewear. Avoid contact with eyes, skin or clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco.

DIRECTIONS FOR USE:

ICX® is intended for use in general dental applications by providing water to the dental syringe, various handpiece ancillary devices, and cuspidor cupfill. When used according to these instructions for every bottle refill, optimum efficacy for control of odor and foul taste bacteria can be achieved.

- 1. Empty any remaining water left in the bottle.
- 2. Drop tablet into an empty dental unit water bottle (0.7L tablet in 0.7 liter bottle, 2 L tablet in 2 liter). Avoid touching the tablet with unprotected skin.
- 3. Fill bottle with water and connect to the dental unit.
- 4. Wait 120 seconds for the tablet to fully dissolve.

NOTE -This product is intended for use with potable water and in conjunction with regular dental unit water testing.

FIRST AID:

If solid product gets in eyes, hold eye open and rinse slowly and gently with water 15 to 20 minutes. Remove contact lenses, if present, after the first five minutes, then continue rinsing.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage disposal. Store at room temperature out of direct sunlight; Do NOT store above 30°C (86°F). Nonrefillable container. Do not reuse or refill this container. Offer for recycling if available or dispose of empty outer foil pouch in the trash.

A-dec Inc., 2601 Crestview Drive, Newberg, Oregon 97132, USA

Phone: 1-800-547-1883 Fax: 503-537-2843

EPA Establishment Number 84100

Made in USA

EPA Registration Number 79662-1

000000

00000

00000

00