

79662-1

11/19/2011

1/2



UNITED STATES ENVIRONMENTAL PROTECTION AGENCY  
 WASHINGTON, D.C. 20460

OFFICE OF CHEMICAL SAFETY  
 AND POLLUTION PREVENTION

Lewis and Harrison  
 c/o for A-dec, Inc.  
 122 C Street, N.W. Suite 740  
 Washington, DC 20001

NOV 14 2011

Attention: Georgia Anastasiou  
 Regulatory Manager

Subject: ICX  
 EPA Reg. No. 79662-1  
 Amendment Application Date: August 15, 2011

The amendment referred to above, submitted in connection with registration under the Federal Insecticide, Fungicide and Rodenticide Act, as amended, is acceptable. The Agency has no objections to the changing the labeling to add an additional marketing statement. A stamped label has been included for your records.

If you have any questions concerning this letter, please contact Demson Fuller at (703) 308-8062.

Sincerely,

A handwritten signature in black ink, appearing to read "Marshall Swindell".

Marshall Swindell  
 Product Manager (33)  
 Regulatory Management Branch 1  
 Antimicrobials Division (7510C)

ardec®

ICX®

A-dec® ICX® tablets are specially formulated to maintain dental unit waterlines and prevent accumulation of odor and foul taste bacteria.

Maintains dental unit water line effluent ≤10 CFU/ml

**ACTIVE INGREDIENTS:**

Sodium Percarbonate.....	6.96%
n-Alkyl(60% C <sub>14</sub> 30% C <sub>16</sub> 5% C <sub>12</sub> 5% C <sub>18</sub> ) dimethyl benzyl ammonium chloride .....	0.85%
n-Alkyl(68% C <sub>12</sub> 32% C <sub>14</sub> ) dimethyl ethylbenzyl ammonium chloride .....	0.85%
Silver Nitrate .....	0.14%
Other Ingredients.....	91.2%
<b>Total.....</b>	<b>100%</b>

[Note to reviewer: EPA approved the location of the active ingredients statement on the back panel of the label on 12/15/09 in order to accommodate the small container size]

Net weight of tablet: 212.4 mg / 0.008 oz (2L application), 80.5 mg / 0.003 oz (0.7L application)  
Net weight of product: 10.62 g / 0.375 oz (2L application), 4.03 g / 0.142 oz (0.7L application) — 50 tablets per box


**KEEP OUT OF REACH OF CHILDREN**

**WARNING**

**PRECAUTIONARY STATEMENTS**

**Hazards to Humans and Domestic Animals**

Causes moderate eye irritation, wear protective eyewear. Avoid contact with eyes, skin or clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco.

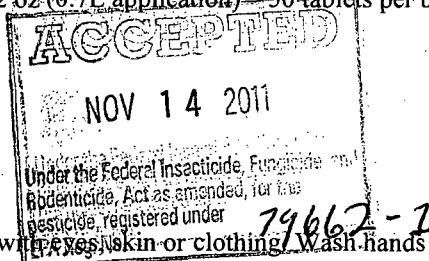

**DIRECTIONS FOR USE:**

ICX® is intended for use in general dental applications by providing water to the dental syringe, various handpiece ancillary devices, and cuspidor cupfill. When used according to these instructions for every bottle refill, optimum efficacy for control of odor and foul taste bacteria can be achieved.

1. Empty any remaining water left in the bottle.
2. Drop tablet into an empty dental unit water bottle (0.7L tablet in 0.7 liter bottle, 2 L tablet in 2 liter). Avoid touching the tablet with unprotected skin.
3. Fill bottle with water and connect to the dental unit.
4. Wait 120 seconds for the tablet to fully dissolve.

NOTE -This product is intended for use with potable water and in conjunction with regular dental unit water testing, following flush (shock) treatment with an antimicrobial product registered with the U.S.EPA and recommended by the manufacturer

**FIRST AID:**

If solid product gets in eyes, hold eye open and rinse slowly and gently with water 15 to 20 minutes. Remove contact lenses, if present, after the first five minutes, then continue rinsing.

**STORAGE AND DISPOSAL**

Do not contaminate water, food, or feed by storage disposal. Store at room temperature out of direct sunlight, DO NOT store above 30°C (86°F). Nonrefillable container. Do not reuse or refill this container. Offer for recycling if available or dispose of empty outer foil pouch in the trash.

A-dec Inc., 2601 Crestview Drive, Newberg, Oregon 97132, USA  
 Phone: 1-800-547-1883 Fax: 503-537-2843  
 EPA Registration Number 79662-1  
 EPA Establishment Number:  
 Expiration Date: 28 months from manufacturing date  
 Made in USA

